WIP'ANIOS

Cleaning and disinfectant wipes

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Cleaning and disinfectant wipes for medical devices which cannot be submerged in liquid, small equipment, incubators, work surfaces...

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Indications

Cleaning and disinfection of medical devices and equipment which cannot be submerged in liquids : ultrasonic probes, cables and connectors, physiotherapy probes, transducers, etc.

Wiping of endoscope sheaths during preliminary treatment prior to submersion for cleaning and disinfection.

Cleaning and disinfection of incubators.

Cleaning and disinfection of the outer surfaces of equipment (respirators, haemodialysis generators...).

Instruction for use

Ready to use product. Carefully wipe the surface to be treated. In case of heavy soiling, use a second WIP'ANIOS.

Let act from 5 to 15 minutes according to the required effectiveness.

It is not necessary to rinse the surface except in case of subsequent contact with mucous membrane, Well-close the packaging after each use.

WIP'ANIOS should not replace a submersion cleaning and disinfection stage where such a stage exists.

Qualitative composition

The impregnating solution of WIP'ANIOS contains : Didecylidimethylammonium chloride, Polyhexamethylene Biguanide hydrochloride, Isopropanol, non ionic surfactant, excipients.

Technical properties

Impregnating solution: 5 ml per wipe
Size: 200 x 200 mm

Microbiological properties

The impregnating solution is :
Bactericidal in 5 min. : EN 1040, EN 1276, NF T 72-190
Active against Mycobacterium tuberculosis (TB).
Fungicidal in 15 min : EN 1275 (C. albicans), EN 1650
Active in 5 min. against HIV-1, BVDV (surrogate of HCV), Rotavirus, Herpesvirus.

Precautions for use

Hazardous - follow the user precautions (Drafted in accordance with Directive 99/45/EC and adaptations).
Storage: between +5°C and +30°C.

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