Orthalgenat[®] Duo

MINT VANILLE

CE

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Instructions for use

Mode d'emploi

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Dear Customer

Thank you for having chosen a quality product from Dentaureum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaureum.de.

1. Manufacturer

Dentaureum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germany

2. With reference to our quality standards

Dentaureum ensures a faultless quality of the products manufactured by us. These nonobligatory recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

3. General notes

Orthalgenat® Duo is a fast setting impression material with clearly reduced generation of dust that is also suitable for taking impressions on multibracket patients due to its particularly high elasticity.

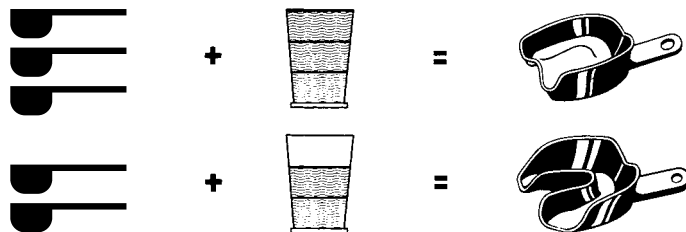
Safety note: Orthalgenat® Duo contains diatomite.
Please avoid inhalation of dust, that may be caused by improper handling.

Dosage: To achieve a consistent mixing ratio, invert the container **3–4 times daily** before removing the powder, or loosen the material with the spatula.

The measuring scoops provided guarantee the best possible dosage and make your job easier.

Overfill the scoop with powder. Tap scoop with spatula to remove any voids. Then level off scoop with dry spatula. Please do not compress the powder!

1 powder scoop (6 g) of **Orthalgenat® Duo** equals one graduation mark (12.6 ml) on the water measuring cup.

Mix powder and water in a clean mixing bowl.

Manual Mixing	Mix powder and water into a homogeneous mass for 30–45 seconds with the mixing spatula.
Automatic Mixing by Machine	Close the mixing container and shake briefly but vigorously. Then machine-mix for about 10–14 seconds. Follow the manufacturer's instructions.
Total Processing Time	Mixed by Machine: approx. 1 minute Mixed Manually: approx. 1.5 minutes
Setting Time	Mixed by Machine: approx. 1.5–2 minutes Mixed Manually: approx. 2–2.5 minutes (includes minimum manipulation time for mixing). The data refer to the use of deionized water at a temperature of 23 °C/73 °F. Lower temperatures lengthen the processing and setting time, while higher temperatures and increased water hardness shorten it. For a constant water temperature we recommend to keep the water in the dosing bottle REF 150-000-00.
Disinfection	Put the impression in a standard disinfection solution; please observe the manufacturer's instructions; e.g. Mucalgin® (Merz), Dentavon® (S&M).
Model Production	Rinse the hardened impression under running water (disinfect beforehand) and pour impression as soon as possible with plaster (e.g. Rapidur®, Moldano®). If the impression has to be stored or sent away for processing, it should be wrapped in a towel, that is moisturized with water and sealed in a plastic bag to minimize dimensional change.
Powder Storage	Orthalgenat® Duo should be stored dry and at room temperature (max. 25 °C/77 °F). Shelf life: 3 years from manufacturing date. Always reseal the container after each use. Is the material stored open over a longer period of time it gets damp creating a shorter setting time!

Color Indicator for Orthalgenat® Duo

- 1) Open the dropper bottle by tilting the white cap and cutting off the tip of the spout.
- 2) Dosage: Please add 5 drops of color indicator per three spoons of powder directly to the powder (**never add to the water!**).
Storage: Please store the color indicator at room temperature (max. 25 °C/77 °F).
Shelf life: 3 years from manufacturing date.
- 3) Now add water and mix as usual.

The color indicator change depends on the basic coloring of the **Orthalgenat® Duo** impression materials.

Orthalgenat® Duo VANILLE

pink	Color phase after mixing
orange	Make the impression
yellow	Remove (approx. 30–60 seconds after making the impression)

Orthalgenat® Duo MINT

purple	Color phase after mixing
blue	Make the impression
green	Remove (approx. 30–60 seconds after making the impression)

Note: Using the color indicator can cause temporary discoloration to the shine of the plaster models. However, the finished polished model has a normal appearance.

4. Availability

Orthalgenat® Duo VANILLE

REF 150-501-10	1 kg	Plastic container
REF 150-509-10	9 x 1 kg	Cardboard box
REF 150-520-10	20 x 1 kg	Cardboard box

Orthalgenat® Duo MINT

REF 150-501-20	1 kg	Plastic container
REF 150-509-20	9 x 1 kg	Cardboard box
REF 150-520-20	20 x 1 kg	Cardboard box

Color Indicator for Orthalgenat® Duo

REF 150-510-00	20 ml	Dosing bottle
----------------	-------	---------------

Dosing Bottle for Liquids

REF 150-000-00	500 ml	Dosing bottle
----------------	--------	---------------

Orthalgenat®-Ex

For removing residual alginate and cleaning the impression trays.

REF 150-010-00	1.5 kg	Plastic container
----------------	--------	-------------------

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