# smartfluorid<sup>®</sup> lacquer

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**smartbrush, blanco 02642**  
100 uds.

# smartfluorid<sup>®</sup> lacquer

Fluoride lacquer for long-term fluoridation and remineralisation, fast drying

Fig. 1

Fig. 2

Fig. 3

Fig. 4

## Range of indications

Intensive caries prophylaxis, desensitising oversensitive cervical areas and hypersensitive dentine by fluoridation. Forms a protective film against thermal and mechanical effects.

### 1. Preparation

Clean the surfaces to be treated (we recommend pretreat with **smartprotect<sup>®</sup>** Desensitizer) and dry. Clean and dry working areas. Use cotton wool rolls to keep the working area dry. Apply **smartprotect<sup>®</sup>** with a **smartbrush** or application brush and massage in for 10 sec. (Fig. 1). Dry completely in the air current (Fig. 2).

### 2. Application

Shake **smartfluorid<sup>®</sup> lacquer** well before use. The level of the lacquer and a metal ball facilitate mixing the lacquer in the bottle. Put a drop of **smartfluorid<sup>®</sup> lacquer** onto a flexible brush and apply with light touches to the surfaces (Fig. 3).

### 3. Drying

Allow the lacquer to take effect for 10 - 20 seconds and dry completely with the air jet until a polymer film forms (Fig. 4). Repeat the treatment two or three times at intervals of a few days with hypersensitive cervical areas and crown margins or si milar indications. Dry areas already treated and apply another layer of **smartfluorid<sup>®</sup> lacquer**.

## Important working hints

- Allow each application of **smartfluorid<sup>®</sup> lacquer** to take effect on the treated surface for as long as possible.
- It is essential to instruct the patient not to clean the tooth surface for 12 hours following application.
- The consistency of **smartfluorid<sup>®</sup> lacquer** should be thin enough after shaking that it flows easily from the drop dispenser. Thin with a few drops of 2-propanol if necessary.
- When used correctly, the lacquer matrix does not dissolve until it has been completely softened and after it has released the maximum amount of fluoride.
- Applying **smartprotect<sup>®</sup>** Desensitizer before hand has an added cleansing and desensitising effect.

## Note

Adverse events are not be expected if used as instructed. Immune reactions such as allergies or irritations can, however, not be excluded in principle. If in doubt, it is recommended to perform an allergy test prior to use.

Only to be used by trained specialist personnel.

DETX is not liable for any damage caused by improper application.

## Technical data:

- **Volume:**  
10 ml bottle
- **Colour:**  
clear
- **Drying time:**  
approx. 30 - 60 sec.

## Properties:

- Organic fluorides help to build up a fluoride depot for long-term fluoride release by amino fluoride and calcium fluoride.
- Forms a protective film that insulates the tooth structure mechanically and thermally.

## Contra-indication:

**smartfluorid<sup>®</sup> lacquer** may not be used in case of known allergies to one or more of the ingredients.

## Composition:

Fluoride lacquer in 2-propanol

- **Relevant ingredients:**  
Olaflur, calcium fluoride and acrylic compounds.

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