
Fascination[®] 2

CE 0483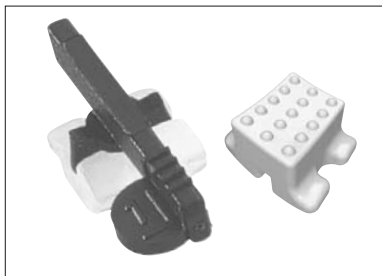

Gebrauchsanweisung	S. 3
Instructions for use	P. 8
Mode d'emploi	P. 13
Modo de empleo	P. 18
Modalità d'uso	P. 24

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Dear Customer

Thank you for having chosen a quality product from Dentaureum. In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at www.dentaureum.de.

1. Manufacturer

Dentaureum GmbH & Co. KG
Turnstraße 31 · 75228 Ispringen · Germany

2. With reference to our quality standards

Dentaureum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

3. Ceramic Orthodontic Brackets Instructions for use

Fascination®2 ceramic brackets are the aesthetic alternative to metal direct bond brackets. Their use corresponds in most respects with the use of metal brackets. Fascination®2 ceramic orthodontic brackets are made of polycrystalline aluminum oxide (Dentaureum patents EP 0337309, US 5242298 and US 4968459). They feature high fracture strength, low friction values and excellent bond strength. The Fascination®2 ceramic bracket bonding surface is coated with a silane layer and nub-like structures. This ensures the optimal adhesive

thickness for proper bonding and problem-free debonding. The shelf life of the silane coating is limited. **For information with reference to the durability, please refer to the product label.**

The „Roth Straight Arch Technique“ offers the Fascination®2 ceramic brackets with a positioning guide, which facilitates correct placement of the bracket. Information with reference to this feature is available in the directions for use section.

Use Fascination®2 ceramic brackets only according to the instructions!

4. Directions

4.1 One feature of the Fascination®2 ceramic brackets is their excellent bond strength. Best results are achieved with our ConTec and MIX adhesives.

Note: Using other than recommended adhesives may damage the enamel, when debonding the brackets.

4.2 Always use tweezers (eg REF 025-277-00), grasp the positioning guide, if necessary grasp mesial-distal, when removing Fascination®2 ceramic brackets from their package or when handling them. The bonding surface with nub-like structures is coated with a silane layer to assure adhesion. This layer should never be touched, damaged or contaminated in any way, otherwise the adhesion will be reduced considerably.

4.3 Tooth preparation

Please follow manufacturer's instructions for bracket adhesive being used.

4.4 Adhesive application

Prepare adhesive according to the manufacturer's instructions. Place only a very small amount of adhesive on to the center of the bracket base, **this will achieve optimal adhesion.**

4.5 Application of bracket

After application of the adhesive on the base, place the bracket on to the tooth with tweezers as usual, if necessary align with the positioning guide, and press it on **until the adhesive flows out a little at the sides.**

After having placed the bracket, immediately remove all excess adhesive **completely** from around the bracket before it hardens.

This is of special importance to ensure easy debonding at the end of the treatment period.

Please take care that the Fascination®2 ceramic brackets do not touch the antagonist teeth when placing the brackets (danger of abrasion). If necessary increase the vertical dimension.

4.6 Placing the arch wire

For the initial treatment we recommend rematitan® LITE- or Tensic® arch wires. Do not ligate wires with excessive force. Do not torque wires after ligating!

4.7 Ligating

Please observe setting times of the adhesive carefully according to the manufacturer's instructions. When adhesive has hardened and the positioning guide has been removed, ligate as usual with wire or plastic ligatures. We recommend our White Ligature Wires (REF 768-036-00 and 768-037-00) or the Dentalastics® plastic ligatures (eg REF 774-002-01).

Attention: Careful ligating avoids damage of brackets!

4.8 Bracket debonding

Use the Weingart pliers (REF 003-120-00). **Using any other instrument can lead to bracket damage.**

Remove arch wire from the bracket. To avoid damage to the bracket while debonding, place a blue separating ring (e.g. REF 774-200-01) around the bracket base for protection. Grasp the Fascination®2 ceramic bracket on the mesial-distal side of the bracket base, directly at the tooth surface, but without

applying pressure. Remove the bracket from the tooth with a short turn to the right or left side without pulling.

Note: When handled correctly, Fascination®2 ceramic brackets are not prone to breakage. Using other than recommended instruments can damage the brackets.

Should a breakage of the bracket occur due to incorrect handling, we recommend to scrape the bracket fragments and the adhesive carefully off the tooth enamel with bracket removing pliers (eg REF 003-349-00) or a scaler (eg REF 027-349-00).

Attention: Do not damage tooth enamel, danger of enamel fracture. Use proper evacuation procedures, protect everyone's eyes within the treatment area!

4.9 Additional facts

- In case of adhesion problems, due to improper preparation, please use a new bracket, as the active-bonding layer can only be applied by the manufacturer.
- Please review the technical information with reference to torque, angulation, size, etc. on the product label or in our catalog. Allergic reactions to Fascination®2 ceramic brackets are not known.
- Store the brackets in their original package.
- Do not expose to sunlight. Protect against heat and humidity.
- There are no particular regulations to dispose of Fascination®2 ceramic brackets after treatment.
- Should you have any questions, please contact your Dentaforum sales representative or our orthodontic fixed appliance hotline on +49 (0)7231 / 803-550.

5. Identification System for the »Roth Straight Arch Technique« (Positioning guide)

- 5.1** The color of the positioning guide and the tooth symbol identify the tooth/bracket combination.
- 5.2** Upper: right = red;
left = green
Lower: right = yellow;
left = blue;
- 5.3** The occlusal-gingival distance can be determined by means of the occlusal indicator. The distance between the occlusal end and the center of the slot is 6 mm.
- 5.4** The tooth symbols always point from occlusal to distal.
- 5.5 Removing the positioning guides**
When adhesive is hardened (according to manufacturer's instructions) remove the positioning guide at the mid section with a pair of tweezers, or with Weingart pliers (REF 003-120-00).

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