# SPORT-ELEC®

## INSTITUT

FR

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GB

### THE PRODUCT CAN BE USE IN THOSE CASE

- Dislocated shoulder
- Maintaining post operative
- Painful shoulder,
- Neurological shoulder
- Shoulder Immobilization
- Arm Support.
- Thoracic strap for immobilization of the elbow to the body.
- The splint has to fix the arm to the body, the forearm perpendicular to the arm to support it.

### MATERIALS

Neoprene group : 3 mm, micro elastic fibre, steel wire, plastic stick, velcro bands.

### IMPORTANT ADVISES AND WARNINGS

- This product is not burn protected. It will lose its characteristics by high temperature. Therefore do not use above 248°F.
- If you want to make changes for your comfort, please ask to your doctor or where you purchased the product.
- This product can not be used out of its own utility and cannot be combined with other products, only use with its original utilities.
- Usage of neoprene products is not allowed for more than 3-4 hours. In any case, do not use more than 8 hours (unless otherwise specified by your doctor).

### WASHING INSTRUCTIONS

- Please close the velcro bands if there are and take out the removable support (like joint parts...).
- Do not put in washing-machine. Only handwash it at 104°F.
- Do not use bleach.
- No dry-cleaning.
- Do not use dryer.
- No ironing.
- After the wash please be sure that it is totally dry before the use.

### UTILITY OF THE PRODUCT

- Please be sure the size of the product is exactly your size.
- Put on the jacket.
- Then place your arm with an angle of 90° below the chest and fold down the velcro strip on the jacket according to the size of your arms.
- Cross both velcro strips located in your back and swing forward to fix them on your arm. Make sure your arm is properly maintained at 90°. You should feel comfortable.
- For enhanced security and to avoid any moves of your arm, set the last velcro strip around your chest going underneath your unrestrained arm.

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