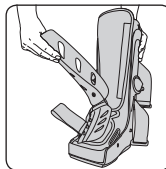
E

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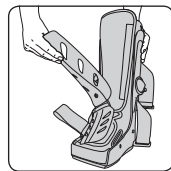
GB

Orthoses for immobilising the foot-leg joint area support damaged tissue structures. The painful ankle joint is immobilised, the load is removed from the muscles to aid the healing process.

Instructions for use:


Step 1

Open the circular Velcro fasteners and remove the front cover.

Step 2

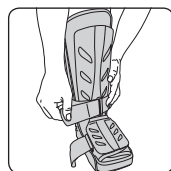Open the black padding and place the foot and leg in the walker brace. Make sure that the ankle is well supported. Now close the black padding, making sure there are no creases.


Step 3

Put back the gray front cover with the central Velcro fastener on the black padding. Close the 3 circular Velcro fasteners. Start with the fastener on the top of the foot and finish with the one on the leg. If necessary, increase the tension.

Step 4

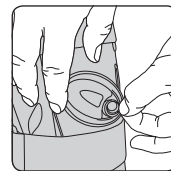Now use the blue side pumps to fill the attached air chambers. The air chambers can be filled independently. Start with the inner pump (medial).

Step 5

To reduce the air pressure or to remove the walker brace, let the air out. To do this, lift the grey covering hood next to the pump and press the silver button.

Accessories available:

Art. No. 100 370 understocking
Art. No. 100 380 protective cover against wetness
Art. No. 100 390 protective cover for the night
Art. No. 990 175 heel wedge set for Air Walker Achillo

Indications:

Pre- or postoperative/post-traumatic immobilisation, stable fractures of the foot and the malleolus, stable distal fractures of the leg, severe ankle sprains.

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

70 % Polyamide, 30 % Ester-based polyurethane

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only. In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

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