SANI-CLOTH® AF3

GERMICIDAL DISPOSABLE WIPE / TOALLITA GERMICIDA DESECHABLE

Bactericidal, Tuberculocidal, Virucidal** in 3 minutes / Bactericida, Tuberculocida, Antivírico** en 3 minutos

REORDER NO. P1450P

♦ **Bacteria / Bacteria**

■ *Bordetella bronchiseptica* ■ *Bordetella pertussis* ■ *Burkholderia cepacia*
■ *Campylobacter jejuni* ■ *Enterobacter aerogenes* ■ *Escherichia coli* O157:H7
■ *Escherichia coli* ■ *Klebsiella pneumoniae* ■ *Listeria monocytogenes*
■ *Proteus vulgaris* ■ *Pseudomonas aeruginosa* ■ *Salmonella enterica*
■ *Serratia marcescens* ■ *Shigella dysenteriae* ■ *Staphylococcus aureus*
■ *Streptococcus pyogenes* ■ *Vibrio cholera* ■ *Yersinia enterocolitica*

♦ **Multi-Drug Resistant Bacteria / Bacteria Resistente a Drogas Múltiples**

■ *Acinetobacter baumannii*, Multi-Drug Resistant ■ Community Acquired Methicillin Resistant *Staphylococcus aureus* (CA-MRSA) NARSA NRS123 (Genotype USA 400) ■ Community Acquired Methicillin Resistant *Staphylococcus aureus* (CA-MRSA) NARSA NRS384 (Genotype USA 300) ■ ESBL Resistant *Escherichia coli* ■ *Escherichia coli* - NDM-1 Positive CDC 1001728 ■ ESBL Resistant *Klebsiella pneumoniae* ■ *Klebsiella pneumoniae* - Carbapenem Resistant ■ *Klebsiella pneumoniae* - NDM-1 Positive CDC 1000527 ■ Methicillin Resistant *Staphylococcus aureus* (MRSA) ■ *Streptococcus pneumoniae* - Penicillin Resistant ■ Vancomycin Resistant *Staphylococcus aureus* (VRSA) NARSA VRS1 ■ Vancomycin Resistant *Enterococcus faecalis* (VRE)

♦ **Viruses** / Virus****

■ **Avian Influenza A H5N1 virus ■ **Cytomegalovirus, Strain AD-169
■ **Herpes simplex virus, type 2, Strain G ■ **Human Coronavirus, Strain 229E
■ **Influenza A virus/Hong Kong Strain ■ **Influenza B virus, Strain B/Hong Kong /5/72 ■ **Respiratory syncytial virus (RSV), Strain Long ■ **Rotavirus Strain WA ■ Kills Pandemic 2009 H1N1 Influenza A Virus

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Does not include the weight of the wipe / No incluye el peso de la toallita

**KEEP OUT OF REACH OF CHILDREN
CAUTION**

See back panel for additional
precautionary statements

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***KILLS HIV-1 (AIDS VIRUS), HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS AND PRE-CLEANED EXTERNAL SURFACES OF ULTRASOUND TRANSDUCERS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in three (3) minutes at room temperature (68°-77°F) in healthcare or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV).

Areas of use: Hospital and Healthcare Settings: Ambulances, Ambulatory Surgical Centers (ASC), Dental Offices, Dialysis Clinics, Healthcare Settings, Home Health Care, Hospitals, Laboratory, Nursing homes, Physician's offices and Rehabilitation.

Critical Care Areas: CCU, Emergency Rooms, E.R., ICU, Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU) and Surgery. **Hospital, Healthcare, and Critical Care Use Sites:** May be used on hard nonporous surfaces of: bed railings; blood glucose meters; cabinets; carts; chairs; counters; dental unit instrument trays; exam tables; gurneys; infant incubators; infant warmers; isolettes; IV poles; operating room tables and lights; physical therapy (PT) equipment surfaces; stethoscopes; stretchers; tables; telephones; toilet seats; and hard nonporous outside surfaces of ultrasound transducers and probes.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

To Dispense Wipes: Remove the lid from the pail. Press open the dispenser at center of the lid. Open the refill pack. Locate the first wipe at the center of the roll. Feed the first wipe through the dispensing orifice. Reapply the lid to the pail. Dispense the wipe.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

CLEANING AND DEODORIZING: Wipe surface with towelette until clean.

Let air dry.

For disinfecting hard non-porous surfaces only.

TO DISINFECT: Use a wipe to remove heavy soil. Unfold a clean wipe and thoroughly wet surface. Treated surface must remain visibly wet for a full three (3) minutes. Use additional wipe(s) if needed to assure continuous 3 minute wet contact time.

ULTRASOUND TRANSDUCERS: For use on external surfaces of Ultrasound Transducers only

To disinfect: Pre-clean ultrasound gel from external transducer to be disinfected by using a clean wash cloth. Then thoroughly wet the probe with a towelette. Treated surface must remain visibly wet for a full three (3) minutes. Use additional wipe(s) if needed to assure continuous 3 minute wet contact time. Let air dry.

***SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HEPATITIS B VIRUS AND HEPATITIS C VIRUS OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS**

Personal protection: When using this product, wear disposable protective

gloves, protective gowns, face masks, or eye coverings as appropriate when handling HIV-1 (AIDS Virus), HBV or HCV infected blood or body fluids.

Cleaning procedure: All blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfection by the germicidal cloth. Open, and unfold first germicidal cloth to remove heavy soil.

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PRECAUTIONARY STATEMENTS:

Hazards to humans and domestic animals

CAUTION: Causes moderate eye irritation. Avoid contact with eyes and clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using the toilet.

FIRST AID

If in eye: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

Storage: Store in a dry location. When not in use keep center cap of lid closed to prevent moisture loss. **Towelette Disposal:** Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet. **Refill Container Disposal:** Triple rinse cracked or broken containers and offer for recycling. If recycling is not available, discard in trash.

EPA REG. NO. 9480-9

EPA EST. NO. 9480-NY-1

MADE IN USA

NOT FOR USE ON SKIN

FOR USE ON HARD NONPOROUS SURFACES ONLY

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Professional Disposables International, Inc.

Two Nice-Pak Park,

Orangeburg, NY 10962-1376

1-800-999-6423

80P14501

(01)10310819001257

NOT FOR USE ON SKIN
FOR USE ON HARD
NONPOROUS
SURFACES ONLY

EPA REG NO. 9480-9
EPA EST. No.: 9840-NY-1

Manufactured for:
Professional Disposables International, Inc.
Two Nice-Pak Park,
Orangeburg, NY 10962-1376 • USA
1-800-999-6423

STORAGE AND DISPOSAL. Do not contaminate water, food, or feed by storage and disposal.
Storage: Store in dry location.
Towellette Disposal: Do not reuse towellette. Dispose of used towellette in trash. Do not flush in toilet. **Package Disposal:** Put empty packaging in trash collection.

PRECAUTIONARY STATEMENTS Hazards to humans and domestic animals
CAUTION: Causes moderate eye irritation. Avoid contact with eyes and clothing. Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using the toilet.
FIRST AID if in eye: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

[†]KILLS HIV-1 (AIDS VIRUS), HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS AND PRE-CLEANED EXTERNAL SURFACES OF ULTRASOUND TRANSDUCERS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in three (3) minutes at room temperature (68°-77°F) in healthcare or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV).
AREAS OF USE Hospital and Healthcare Settings: Ambulances, Ambulatory Surgical Centers (ASC), Healthcare Settings, Home Health Care, and Hospitals. **CRITICAL CARE AREAS:** CCU, Emergency Rooms, E.R., ICU, Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU) and Surgery. **Hospital, Healthcare, and Critical Care Use Sites:** May be used on hard nonporous surfaces of bed railings, blood glucose meters, cabinets, carts, counters, gurneys, IV poles, physical therapy (PT) equipment surfaces, stretchers, tables, telephones, and toilet seats.
This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

PDI® SANI-CLOTH® AF³
GERMICIDAL DISPOSABLE WIPE

Bactericidal, Tuberculocidal, and Virucidal** in 3 minutes
Fragrance Free

ACTIVE INGREDIENTS

n-Alkyl (68% C ₁₂ , 32% C ₁₄) dimethyl ethylbenzyl ammonium chlorides.....	0.14%
n-Alkyl (60% C ₁₄ , 30% C ₁₆ , 5% C ₁₂ , 5% C ₁₈) dimethyl benzyl ammonium chlorides.....	0.14%
OTHER INGREDIENTS.....	99.72%
TOTAL.....	100.00%

Does not include the weight of the wipe

20 WIPES 7.8 x 8.5 in. (19.81 x 21.59 cm)
Net Wt. 5.7 oz (162 g)

KEEP OUT OF REACH OF CHILDREN
CAUTION SEE BACK PANEL FOR ADDITIONAL PRECAUTIONARY STATEMENTS

TO DISPENSE: Open package lid. Completely remove inner seal. Pull out wipe at an angle. The next wipe pops up automatically. Snap lid closed after use to retain moisture. **DIRECTIONS FOR USE** It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

CLEANING AND DEODORIZING: Wipe surface with towellette until clean. Let air dry. For disinfecting hard non-porous surfaces only.

TO DISINFECT: Use a wipe to remove heavy soil. Unfold a clean wipe and thoroughly wet surface. Treated surface must remain visibly wet for a full three (3) minutes. Use additional wipe(s) if needed to assure continuous three (3) minute wet contact time.

ULTRASOUND TRANSDUCERS: For use on external surfaces of Ultrasound Transducers only. To disinfect: Pre-clean ultrasound gel from external transducer to be disinfected by using a clean wash cloth. Then thoroughly wet the probe with a towellette. Treated surface must remain visibly wet for a full three (3) minutes. Use additional wipes if needed to assure continuous 3 minute wet contact time. Let air dry.

[†]**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HEPATITIS B VIRUS AND HEPATITIS C VIRUS OF SURFACES/OBJECTS SOILED WITH BLOOD OR BODY FLUIDS**

Personal protection: When using this product, wear disposable protective gloves, protective gowns, face masks, or eye coverings as appropriate when handling HIV-1 (AIDS Virus), HBV or HCV infected blood or body fluids. **Cleaning procedure:** All blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfection by the germicidal cloth. Open and unfold first germicidal cloth to remove heavy soil. **Disposal of infectious materials:** Dispose of used towellette in accordance with local regulations for infectious waste disposal. **Contact time:** Use second germicidal cloth to thoroughly wet surface. Allow to remain wet three (3) minutes, let air dry. Although efficacy at a 30 second contact time has been shown to be adequate against HIV-1 (AIDS VIRUS), this time is not sufficient for the other non-HIV organisms listed on this label. Therefore, a 3 minute wet contact time must be used for all listed non-HIV organisms.

- ◆ **Bacteria**
 - *Pseudomonas aeruginosa*
 - *Staphylococcus aureus*
 - *Streptococcus pyogenes*
- ◆ **Multi-Drug Resistant Bacteria**
 - *Acinetobacter baumannii*, Multi-Drug Resistant
 - ESBL Resistant *Escherichia coli*
 - *Klebsiella pneumoniae* - Carbapenem Resistant
 - Methicillin Resistant *Staphylococcus aureus* (MRSA)
 - Vancomycin Resistant *Enterococcus faecalis* (VRE)
- ◆ **Viruses****
 - **Avian Influenza A H5N1 virus
 - **Influenza A virus/Hong Kong Strain
 - **Influenza B virus, Strain B/Hong Kong /5/72
 - **Respiratory syncytial virus (RSV), Strain Long
 - **Rotavirus Strain WA
- ◆ **Bloodborne Pathogens**
 - **Hepatitis B virus (HBV) - Duck HBV[†]
 - **Hepatitis C virus (Human) (HCV) - Bovine Diarrhea Virus[†]
 - **HIV-1 (AIDS virus)[†] Strain HTLV-III_B
- ◆ **Pathogenic Fungi**
 - *Candida albicans*
- ◆ **TB**
 - *Mycobacterium bovis* - BCG (TB)

PRECAUTIONARY STATEMENTS
Hazards to humans and domestic animals
CAUTION: Causes moderate eye irritation.
Avoid contact with eyes and clothing.
Wash hands thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using the toilet.

FIRST AID
If in eye: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

STORAGE AND DISPOSAL
Do not contaminate water, food, or feed by storage and disposal.
Storage: Store in dry location.

Towelette Disposal: Do not reuse towelette. Dispose of used towelette in trash. Do not flush in toilet.
Package Disposal: Put empty packaging in trash collection.

**NOT FOR USE ON SKIN
FOR USE ON HARD
NON POROUS
SURFACES ONLY**

EPA REG. NO.: 9480-9
EPA EST. NO.: A-9480-NY-1,
C-72956-AR-1
Alpha character will precede batch code on product

Manufactured for:
Professional Disposables International, Inc.
Two Nice-Pak Park,
Orangeburg, NY 10962-1376 • USA
1-800-999-6423

[†]KILLS HIV-1 (AIDS VIRUS), HEPATITIS B VIRUS (HBV) AND HEPATITIS C VIRUS (HCV) ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS AND PRE-CLEANED EXTERNAL SURFACES OF ULTRASOUND TRANSDUCERS PREVIOUSLY SOLED WITH BLOOD/BODY FLUIDS in three (3) minutes at room temperature (68°- 77°F) in healthcare or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV).

AREAS OF USE: Hospital and Healthcare Settings:
Ambulances, Ambulatory Surgical Centers (ASC), Dental Offices, Dialysis Clinics, Healthcare Settings, Home Health Care, Hospitals, Laboratory, Nursing homes, Physician's offices and Rehabilitation.

CRITICAL CARE AREAS: CCU, Emergency Rooms, E.R., ICU, Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU) and Surgery

Hospital, Healthcare, and Critical Care Use Sites:
May be used on hard nonporous surfaces of: bed railings; blood glucose meters; cabinets; carts; chairs; counters; dental unit instrument trays; exam tables; gurneys; infant incubators; interior and exterior surfaces of; infant warmers; isolettes; IV poles; operating room tables and lights; physical therapy (PT) equipment surfaces; stethoscopes; stretchers; tables; telephones; toilet seats; and hard nonporous outside surfaces of ultrasound transducers and probes.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

PDI SANI-CLOTH® AF3 GERMICIDAL DISPOSABLE WIPE

Bactericidal, Tuberculocidal, and Virucidal** in 3 minutes
Fragrance Free

ACTIVE INGREDIENTS
n-Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chlorides0.14%
n-Alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chlorides0.14%
OTHER INGREDIENTS99.72%
TOTAL100.00%

100 WIPES 8.2 x 9.8 in. (20.8 x 24.8 cm)
Net Wt. 2.0 lbs 4.6 oz (1037 g)

**KEEP OUT OF REACH OF CHILDREN
CAUTION** SEE BACK PANEL FOR ADDITIONAL PRECAUTIONARY STATEMENTS

- ◆ **Bacteria**
 - *Bordetella bronchiseptica*
 - *Bordetella pertussis*
 - *Burkholderia cepacia*
 - *Campylobacter jejuni*
 - *Enterobacter aerogenes*
 - *Escherichia coli* 0157:H7
 - *Escherichia coli*
 - *Klebsiella pneumoniae*
 - *Listeria monocytogenes*
 - *Proteus vulgaris*
 - *Pseudomonas aeruginosa*
 - *Salmonella enterica*
 - *Serratia marcescens*
 - *Shigella dysenteriae*
 - *Staphylococcus aureus*
 - *Streptococcus pyogenes*
 - *Vibrio cholera*
 - *Yersinia enterocolitica*
- ◆ **Multi-Drug Resistant Bacteria**
 - *Acinetobacter baumannii*, Multi-Drug Resistant
 - Community Acquired Methicillin Resistant *Staphylococcus aureus* (CA-MRSA) NARSA NRS123 (Genotype USA 400)
 - Community Acquired Methicillin Resistant *Staphylococcus aureus* (CA-MRSA) NARSA NRS384 (Genotype USA 300)
 - ESBL Resistant *Escherichia coli*
 - *Escherichia coli* - NDM-1 Positive CDC 1001728
 - ESBL Resistant *Klebsiella pneumoniae*
 - *Klebsiella pneumoniae* - Carbapenem Resistant
 - *Klebsiella pneumoniae* - NDM-1 Positive CDC 1000527
 - Methicillin Resistant *Staphylococcus aureus* (MRSA)
 - *Streptococcus pneumoniae* - Penicillin Resistant
 - Vancomycin Resistant *Staphylococcus aureus* (VISA) NARSA VRS1
 - Vancomycin Resistant *Enterococcus faecalis* (VRE)
- ◆ **Viruses****
 - **Avian Influenza A H5N1 virus
 - **Cytomegalovirus, Strain AD-169
 - **Herpes simplex virus, type 2, Strain G
 - **Human Coronavirus, Strain 229E
 - **Influenza A virus/Hong Kong Strain
 - **Influenza B virus, Strain B/Hong Kong /5/72
 - **Respiratory syncytial virus (RSV), Strain Long
 - **Rotavirus Strain WA
 - Kills Pandemic 2009 H1N1 Influenza A Virus
- ◆ **Bloodborne Pathogens**
 - **Hepatitis B virus (HBV) - Duck HBV†
 - **Hepatitis C virus (Human) (HCV) - Bovine Dermal Virus
 - **HIV-1 (AIDS virus)† Strain HTLV-IIIg
- ◆ **Pathogenic Fungi**
 - *Candida albicans*
- ◆ **TB**
 - *Mycobacterium bovis* - BCG (TB)

TO DISPENSE: Open package lid. Completely remove inner seal. Pull out wipe at an angle. The next wipe pops up automatically. Snap lid closed after use to retain moisture.

DIRECTIONS FOR USE It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

CLEANING AND DEODORIZING: Wipe surface with towelette until clean. Let air dry. For disinfecting hard non-porous surfaces only.

TO DISINFECT: Use a wipe to remove heavy soil. Unfold a clean wipe and thoroughly wet surface. Treated surface must remain visibly wet for a full three (3) minutes. Use additional wipe(s) if needed to assure continuous three (3) minute wet contact time.

ULTRASOUND TRANSDUCERS: For use on external surfaces of Ultrasound Transducers only. To disinfect: Pre-clean ultrasound gel from external transducer to be disinfected by using a clean wash cloth. Then thoroughly wet the probe with a towelette. Treated surface must remain visibly wet for a full three (3) minutes. Use additional wipes if needed to assure continuous 3 minute wet contact time. Let air dry.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HEPATITIS B VIRUS AND HEPATITIS C VIRUS OF SURFACES/OBJECTS SOILED WITH BLOOD OR BODY FLUIDS

Personal protection: When using this product, wear disposable protective gloves, protective gowns, face masks, or eye coverings as appropriate when handling HIV-1 (AIDS virus), HBV or HCV infected blood or body fluids.

Cleaning procedure: All blood and other body fluids must be thoroughly cleaned from surfaces and objects before disinfection by the germicidal cloth. Open and unfold first germicidal cloth to remove heavy soil.

Disposal of infectious materials: Dispose of used towelette in accordance with local regulations for infectious waste disposal.

Contact time: Use second germicidal cloth to thoroughly wet surface. Allow to remain wet three (3) minutes, let air dry. Although efficacy at a 30 second contact time has been shown to be adequate against HIV-1 (AIDS VIRUS), this time is not sufficient for the other non-HIV organisms listed on this label. Therefore, a 3 minute wet contact time must be used for all listed non-HIV organisms.

2OM928R