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ARTEX® BN

OPEN CENTRIC

Swivel ❶ locking bar upwards with the thumb; the top part of the Artex can be guided backwards freely. The locking bar limits travel in a posterior direction and prevents the top part of the Artex falling out.

Centric open ❷, condyle ball released.

CLOSE CENTRIC

Press locking ❸ bar down; the plastic tongue centres the condyle ball in the lowest position of the condyle path. Only rotation is possible with the top part.

Centric closed ❹: the condyle ball is fixed in position by the plastic tongue.

ZERO POSITION OF INCISAL GUIDANCE ACCORDING TO THE BONWILL TRIANGLE

The standard incisal guidance positioner 0° has a marking with two lines. When the point of the incisal guidance pin is placed centrally between the two lines, incisal guidance with the incisal pin is positioned according to the 110 mm Bonwill triangle (formed from the point of the incisal pin and the central points of the joint balls = inter-condylar distance).

MODEL PLATES, MAGNETIC

Model plates easily and quickly removable.

Scope of delivery:

Order No.:	
217360	Artex BN

Accessories/Components:

Order No.:	
217330	Base Plates blue, pair
217331	Base Plates blue, 50/Pkg.
215250	Retention Disk

WARNINGS AND SAFETY ASPECTS:

– How to check the centric adjustment of Artex BN, CN and Artex CT

If the upper part of the articulator has lateral play, this deviation to the side can be heard and felt by a „click“. In this case, it is necessary to readjust the centric.

Then, please proceed as follows:

- Close the centric lock.
- Unscrew the thread pins at the bottom of the rear part using an allen wrench.
- Adjust incisal guidance pin centrally in zero position.
- Hold the upper articulator part in this position. At the same time, pull the left semi-axis outwardly until reaching the stop, and use the allen wrench to tighten the thread pin. Now proceed in the same way with the right semi-axis.

– General information

Articulators are precision instruments.

Therefore, it is important to handle them with care.

Please read the following notes:

- Periodically check if the axis-joint system and the condylar paths are clean in order to maintain the smooth mobility of the condyle in the axis and thus maintain the functionality of the articulator.
- Remove dirt in this area with a steamcleaner, then blow dry.
- Do not place the articulator in a boil-out unit, a pressure pot or an ultrasonic bath.
- For cleaning and care, use only cleaning agents that are suitable for light metals.

Regarding the use of the cleaning agent, please follow the instructions of the according manufacturer. Do not use cleaning agents with a pH <5 or >8 since those might damage the anodized surfaces.

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Amann Girrbach AG
Amann Girrbach GmbH