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GB

Back Supports support injured tissue structures (bones, ligaments and cartilages). The compression hereby supports the holding function of the ligaments and is thus relieving the vertebrae.

The pad provides a massage effect which reduces swelling tendencies or which helps to an earlier restrain of swelling.

COOLMAX® wicks moisture away from the body, ensuring a pleasant and dry comfortable fit.

Instructions for use:

Place the support around the waist and – using the hand straps – evenly overlap the two Velcro fastening ends by tensioning slightly. Ensure that the support is centered and that the lower edge of the support is in the groin area (forward bend). This is important for optimal effect of the BORT StabiloBasic. The pad is velcroed to the support and can be positioned freely. Ensure that the wide side of the pad faces upward.

Indications: Pain in the lumbar spine, recurrent low back pain, osteochondrosis, spondylarthrosis, spondylolysis without spondylolisthesis, mild lumbar deformity (degenerative spondylolisthesis), ischialgia, dorsalgia.

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

46 % Spandex, 30 % polyamide, 24 % polyester

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

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Hand-wash the support in lukewarm water using a mild detergent. Do not use fabric softeners. Close the Velcro closure to avoid damaging other items being washed. Shape by hand and allow dry in the open air. Avoid heat.

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The **BORT StabiloBasic Sport** unites the stabilizing function of a back support with the moisture regulating properties of the COOLMAX® high-tech fiber. Innovative materials and the most modern of production technology give the **BORT StabiloBasic Sport** optimal properties for sports-related and therapeutic applications.

BORT-Quality products promote your healing process: in the fields of orthopedics, surgery, sports medicine and rehabilitation.