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LAVANID® - Wundgel

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en **INSTRUCTIONS FOR USE**

LAVANID® - Wound Gel

DESCRIPTION AND PRODUCT PROFILE
The surface of chronic wounds is generally made
up of devitalized and necrotic tissue, and is also
colonised with bacteria. For wound treatment to
be successful the wound surface needs to be re-
moved, e.g. through surgical debridement. Wound
irrigation and moistening are a necessary part of
this process. Chronic wounds exhibit dysfunctional
wound healing processes. This is why treatment
with preparations that are very well tolerated by
the tissues is indicated.
LAVANID® Wound Gel is a sterile, preserved
Ringer's solution-based wound gel which stands
out due to its very good tissue tolerance. In addi-
tion to cleaning and moistening wounds, LAVANID®
Wound Gel is also appropriate for preservative
dressings, e.g. for wounds colonised with multi-
resistant bacteria. Preservative dressings reduce the
risk of carrying the bacteria from the wound to the
patient's environment and they protect the wound
from bacteria getting in from the outside.

MECHANISMS OF ACTION
LAVANID® Wound Gel is a sterile, preserved
Ringer's solution-based wound gel which stands
out due to its very good tissue tolerance. In addi-
tion to cleaning and moistening wounds, LAVANID®
Wound Gel is also appropriate for preservative
dressings, e.g. for wounds colonised with multi-
resistant bacteria. Preservative dressings reduce the
risk of carrying the bacteria from the wound to the
patient's environment and they protect the wound
from bacteria getting in from the outside.

COMPOSITION
Ringer's solution (isotonic electrolyte solution
made up of sodium chloride, potassium chloride,
calcium chloride 2 H₂O, water for injection),
glycerin, macrogol, hydroxyethylcellulose, poly-
hexanide 0.04%.

INDICATIONS
LAVANID® Wound Gel is used to moisten and clean
wounds and to moisten bandages and wound
dressings with preservative.

TISSUE TOLERANCE
Experimental and clinical studies have shown
that the combination of Ringer's solution with
0.04% polyhexanide is very well tolerated by
tissue.
Clinical trials with LAVANID® Wound Gel have
shown that it does not negatively affect the for-
mation of healthy new tissue. There are no local
signs of irritant conditions. Use of this product
is pain-free for patients.
Because it is so well tolerated by tissues
LAVANID® Wound Gel is suitable for repeated
and long-term wound treatment.

INSTRUCTIONS FOR USE

- Prepare and clean the wound as usual.
- Apply LAVANID® Wound Gel to the wound or
a wound dressing (e.g. pads or compresses).
Dress wound as usual. LAVANID® Wound Gel
should remain on the wound until the next
dressing change. For optimal wound moisten-
ing we recommend increasing the amount of
LAVANID® Wound Gel and lengthening the
intervals between dressing changes. If the
dressing sticks to the wound while being
changed, moisten the dressing with a rinsing
solution (e.g. LAVANID® Wound Rinsing Solu-
tion) before removing it.
- If the LAVANID® Wound Gel is going to be
used more than once, please read the shelf-
life information.

CONTRAINDICATIONS
LAVANID® Wound Gel must not be used

- in cases of known hypersensitivity to any of
the ingredients,
- in the area of cartilage and joints.

There is insufficient data on its use during preg-
nancy and breast-feeding. Therefore, LAVANID®
Wound Gel should not be used unless urgently
indicated.

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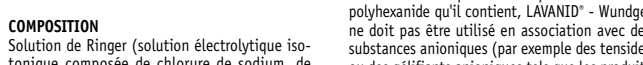
INTERACTIONS WITH OTHER DRUGS
Due to incompatibility problems with the ingredi-
ent polyhexanide, LAVANID® Wound Gel must not
be used in combination with anionic substances
(e.g. anionic tensides, anionic gel formers and
products containing PVP-iodine). In instances
where there is prior contact, these substances
must be rinsed liberally out of the wound.

SIDE EFFECTS
There are no known side effects.

INFORMATION ABOUT SHELF-LIFE
The expiry date can be found on the container.
Do not use after the expiry date!

LAVANID® Wound Gel is sterile as long as the
packaging is unopened and undamaged. Al-
though it is no longer sterile after the container
has been opened, because it contains a pre-
servative it can be used up to one month after
opening under certain conditions. These condi-
tions are that the contents be protected from
contamination. The container must not come in
contact with the wound. The opening should be
capped immediately after each use. Gel residue
around the opening should be wiped off with a
sterile compress.

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