periogenix™

Directions For Use

DESCRIPTION

Periogenix is a non-sterile periodontal wound dressing provided in a kit containing:

- One [1] Canister of Periogenix;
- Thirty [30] single use disposable Dental Trays.

Periogenix is a white emulsion with a consistency similar to toothpaste.

The single use disposable dental trays are designed to fit the maxillary (upper jaw) or mandibular (lower jaw) dental arches / gums of all patients.

USA Rx Only

INDICATIONS FOR USE

Periogenix is a periodontal wound dressing intended to protect injured periodontal tissue (gums) by forming a temporary physical barrier to avoid further irritation.

CONTRAINDICATIONS

- Not for use in patients with known hypersensitivity or allergy to any of the following **INGREDIENTS:** Perfluorodecalin, Glycerin, Hydrogenated Phosphatidylcholine, Cetearyl Alcohol, Polysorbate 60, Tocopheryl Acetate, Benzyl Alcohol, Methylparaben, Propylparaben.
- Not for use in patients who do not comply with instructions.
- Not for use in patients who are pregnant or are trying to become pregnant.

WARNINGS AND PRECAUTIONS

- Do not smoke, eat, or drink while the dental tray filled with Periogenix is applied to the dental arch / gums.
- Periogenix is not intended to stop bleeding.
- Intermittent and slight tingling or stinging may occur during the first 24 hours of Periogenix use.
- Discontinue treatment and contact your dentist or health care professional if inflammation of the dental arch / gum persists or the wound does not heal after 10 days of Periogenix use.

INSTRUCTIONS FOR USE

1. Dispense Periogenix from the canister into the trough of a dental tray in a line that is long enough to cover the treatment area of the dental arch / gum. The line of Periogenix should only fill the bottom half of the dental tray (see Figure 1).

Note: Do not overfill the dental tray with Periogenix (see Figure 2).

Figure 1
Dental tray with bottom half filled in treatment area.

Figure 2
Incorrectly filled (overfilled) dental tray.

Figure 3A
Dental tray with Periogenix placed over maxillary arch.

3. Apply dental tray filled with Periogenix as described in Step 2 above 3 times daily for up to 10 days.

 ≥ 00h:30m:00s
–
≤ 02h:00m:00s

 **3x / d x 10d**

2. Place the dental tray filled with Periogenix over the dental arch / gums, press into place around teeth, and leave in place for at least 30 minutes and up to 2 hours. The Periogenix will spread and cover the teeth and the soft tissue of the dental arch / gums at the treatment site (see Figure 3A and Figure 3B).

Figure 3B
Dental tray with Periogenix placed over mandibular arch.

4. After each treatment session place the used dental tray in a waste receptacle.
5. Excess Periogenix in the mouth may be wiped away or spit out. Swallowing small amounts of Periogenix is not harmful.

Note: It is normal for a small amount of Periogenix to expand out of the nozzle after use. For best results, wipe any excess material from the nozzle before each use.

periogenix™

LABELING

As required by US and international regulatory agencies, the following symbols are affixed to the device packaging.

SYMBOL KEY

FOR INSTRUCTIONS FOR USE AND PRODUCT LABELS

Attention, see instructions for use.

REF

Catalog number.

LOT

Lot number.

KIT

Components provided in kit.

[#]

Quantity.

Dental tray.

Canister of Periogenix.

Storage temperature; store product between 5 – 35°C (41 – 95°F).

EC REP

Authorized Representative in the European Community.

USA Rx Only

Prescription Use only in the USA: Caution: Federal law (U.S.A.) restricts this device to sale by or on the order of a dentist or other practitioner licensed by the law of the State in which he practices to use or order use of the device.

INGREDIENTS

Ingredients contained in Periogenix; Not for use in patients with known hypersensitivity or allergy to the listed ingredients.

Use by date: Year and Month

Do not reuse; single use only.

Non sterile.

Manufacturer.

Latex Free – Product does not contain Latex.

3 x /d x 10d

Treatment 3 times per day for up to 10 days.

≥ 00:30:00

≤ 02:00:00

Apply Periogenix in dental tray to dental arch/gums for at least 30 min. and up to 2 hrs.

Kit provides up to 10 days treatment (3 treatments per day).

Class I Device CE Mark.

This document is copyrighted with all rights reserved. Under copyright laws, it may not be copied in whole or in part or reproduced in any other media without the express written permission of OroScience, Inc. Permitted copies must carry the same proprietary and copyright notices as were affixed to the original. Under the law, copying includes translation into another language.

The information provided herein is subject to change without notice.

OroScience, Inc.™ and Periogenix™ are trademarks of OroScience, Inc.™

In various countries this product is covered by one or more of the following patents: US patent numbers: 5,834,519; 5,922,305; 5,958,377; 6,169,117; 6,197,279; 6,238,645; 7,198,179; 7,357,937. European patent numbers: 1 545 757; 1 599 392.

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OroScience, Inc.™ y PerioGenix™ son marcas comerciales de OroScience, Inc.™