Cement-It™ Universal C&B

Resin Cement

English

INSTRUCTIONS FOR USE

Français

French

MODE D'EMPLOI

Deutsch

German

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Italiano

Italian

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Español

Spanish

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Čeština

Czech

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English

INSTRUCTIONS FOR USE:

Indications: Cementation of porcelain crowns, inlays, and onlays and other dental restorations, including ceramics, metal alloys and composite restoratives.

Cementing Ceramic or Indirect Composite Crowns, Bridges, Inlays & Onlays

1. Apply 37% Phosphoric Acid Etching Gel* to enamel and dentin surfaces for 15 seconds. Rinse thoroughly for 15 seconds with water. It is important not to splash etch into eyes, onto soft tissues or skin.
2. Dry with clean, oil-free, air without desiccating the dentin. Do not allow etched surface to become contaminated with saliva or blood during the bonding procedure. **NOTE:** Never use excessive force on syringe plunger to dispense etch which may lead to over dispensing of the material. If resistance is felt replace current tip with a new tip and check flow extraorally.

Optional: Apply a generous coating of PrepEze™ Desensitizer to coat the tooth surface. Do not let it pool on the floor of the box. Air dry. Leave tooth slightly moist.

3. Apply two consecutive coats of Bond-1™ C&B™ Primer/ Adhesive within 20 seconds. Dry with a gentle stream of air to volatilize any residual ethanol solvent for approximately 10-15 seconds. Look for shiny appearance. If none present, reapply. Light cure for 10 seconds.
4. For ceramic restorations, apply silane coupler to internal fitting surfaces of ceramic restoration. Let stand for 60 seconds. Air dry. At this time, you may apply Bond-1 C&B Primer/ Adhesive to internal fitting surfaces. Air dry. For indirect composite restorations, apply Bond-1 C&B Primer/ Adhesive to internal fitting surfaces of indirect composite restorations. Air dry.
5. Apply Cement-It Universal C&B cement to inside surfaces of restoration. Seat restoration. Cement-It Universal C&B cement chemically cures within 4.5 minutes from time of mix. Alternatively, you can light cure for 40 seconds if the translucency of the restoration permits.

Cementing FibreKor™ Posts or Other Ceramic, Composite or Carbon-Fiber Posts

1. After try-in, adjust the occlusal height. Then, clean post with an alcohol wipe.

NOTE: The Silane Coupler may be added at this point to ceramic posts and allowed to air dry.

- Apply 37% Phosphoric Acid Etching Gel* to enamel and dentin surfaces for 15 seconds. Rinse thoroughly for 15 seconds with water. It is important not to splash etch into eyes, onto soft tissues or skin.
- Dry with clean, oil-free, air without desiccating the dentin. Do not allow etched surface to become contaminated with saliva or blood during the bonding procedure. **NOTE:** Never use excessive force on syringe plunger to dispense etch which may lead to over dispensing of the material. If resistance is felt replace current tip with a new tip and check flow extraorally.
- Saturate brush tip with Bond-1 C&B Primer/ Adhesive and flood canal. Use a paper point to wick-out excess: Lightly air dry for 10-15 seconds to volatilize residual ethanol.
- Using needle tip or lentulo spiral, fill canal with Cement-It Universal C&B cement. Apply additional cement to the post. Seat the post. Cement-It Universal C&B cement chemically cures within 4.5 minutes from time of mix.

Cementing a Metal Restoration to Enamel and Dentin

- Apply 37% Phosphoric Acid Etching Gel* to enamel and dentin surfaces for 15 seconds. Rinse thoroughly for 15 seconds with water. It is important not to splash etch into eyes, onto soft tissues or skin.
- Dry with clean, oil-free, air without desiccating the dentin. Do not allow etched surface to become contaminated with saliva or blood during the bonding procedure. **NOTE:** Never use excessive force on syringe plunger to dispense etch which may lead to over dispensing of the material. If resistance is felt replace current tip with a new tip and check flow extraorally.

Optional: Apply a generous coating of PrepEze Desensitizer to coat the dentin surface. Do not let it pool. Air dry. Leave tooth slightly moist.

- Apply two consecutive coats of Bond-1 C&B Primer/ Adhesive within 20 seconds. Dry with a gentle stream of air to volatilize any residual ethanol solvent for approximately 10-15 seconds. Tooth should have a shiny appearance. If not, apply an additional coat and gently air dry for 10 seconds. Light cure for 10 seconds.

Optional: Etch the inner surface of the sandblasted metal restoration. Rinse and air dry.

- Apply one coat of Bond-1 C&B Primer/ Adhesive to the inside of the restoration. Remove excess and residual ethanol with a jet of air for approximately

10-15 seconds.

- Place Cement-It Universal C&B cement mixture inside the restoration. Cement-It Universal C&B cement has a minimum working time of 45 seconds from mix and a maximum setting time of 4.5 minutes from mix.
- Seat restoration and apply even pressure with a bite stick, allowing excess cement to vent.
- Remove pre-polymerized excess with a small brush, cotton pledget, etc. After final set, remaining excess can be removed in a preferred manner.

NOTE: Tips and applicator brushes are for single patient use only, to prevent cross-contamination between patients.

NOTE: Refer to outer package for expiration date. Do not use after expiration date.

***CAUTION:** 37% Phosphoric Acid Etching Gel contains phosphoric acid. Avoid contact with skin, eyes, and soft tissue. In case of contact with skin or eyes, flush immediately with water; get medical attention for eyes. Do not take internally.

CAUTION: Bond-1 C&B Primer/ Adhesive contains ethanol and is flammable and volatile. Keep away from sources of ignition and excessive heat.

CAUTION: Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

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Resin Cement

Rx Only

Refrigeration required.
Store at 2-8°C. (36-46°F)

PENTRON

Manufactured by Pentron Clinical

1717 West Collins Ave. Orange, CA 92867 USA
800-551-0283 / 714-516-7557 / www.pentron.com

 European Union Representative
SpofaDental a.s.

Markova 238

CZ-506 01 Jičín

Phone: +420 493 583 111

LIMITED WARRANTY —

Pentron's technical advice, whether verbal or in writing, is designed to assist dentists in using Pentron's product. The dentist assumes all risk and liability for damages arising out of the improper use of Pentron's product.

In the event of a defect in material or workmanship, Pentron will replace the defective product or part thereof, or reimburse the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Pentron.

CAUTION: Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissues. Wash thoroughly with water after contact.

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