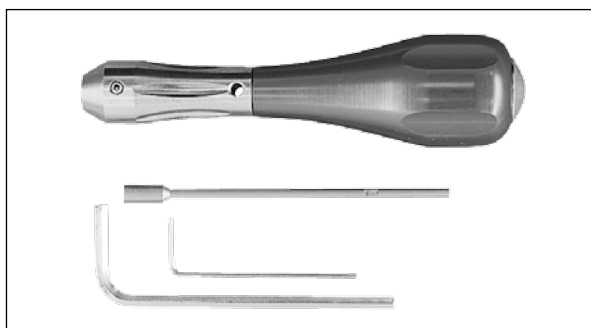
tomas®-screw driver

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Instructions for use

Mode d'emploi

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Stand der Information: 09/15

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Instructions for use tomas®-screw driver

Dear Customer

Thank you for choosing this high-quality product from Dentaaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

The written instructions cannot cover all possible situations that can occur during use. For this reason, our hotline is available to answer any other questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, that you study the enclosed current instructions for use or refer to our website at www.dentaaurum.com.

1. Manufacturer

Dentaaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germany

2. General product description

The tomas®-screw driver (REF 302-004-10) consists of the following parts (Fig. 1):

- A screw driver handle (REF 302-004-11)
- B screw driver tip (REF 302-004-12)
- C screw driver key 3.0 (REF 302-004-14)
- D screw driver key 1.3 (REF 302-004-16)
- E screw driver allen set screw (REF 302-004-17)

3. Use

The tomas®-screw driver must be disinfected, dismantled, cleaned, dried, reassembled and sterilized prior to each clinical use, see section 4 and 5.

The tomas®-pin is picked up with the screw driver tip and inserted into the bone. The dimension of the tomas®-screw driver permits very high torque to be exerted when screwing in the tomas®-pin. Therefore great care must be exercised when inserting the tomas®-pin to prevent fracturing.

If the tomas®-pin is inserted with the tomas®-screw driver a maximum torque of 20 Ncm must not be exceeded – also see the instructions for use tomas® (REF 989-593-00) as well as the insertion manual (REF 989-631-20).

4. Dismantling and reassembling

Use the 1.3 screw driver key (1.3 Allen key) to loosen the screw driver Allen set screw and remove the screw driver tip (Fig. 2).

Put the screw driver tip through the hole of the screw driver handle (Fig. 3).

Open the cap on the screw driver handle (Fig. 4).

Unscrew the cylindrical screw concealed beneath it with the 3.0 screw driver key (Fig. 5) and dismantle the screw driver tip (Fig. 6).

After cleaning the tomas®-screw driver is completely reassembled and prepared for sterilization.

5. Instrument preparation

The tomas®-screw driver is intended for repeated use. Before reuse the tomas®-screw driver must be disinfected, dismantled, cleaned, dried, reassembled and sterilized. For details please see the „Instructions for the preparation of pliers and instruments“ (REF 989-590-20).

6. With reference to our quality standards

Dentaaurum ensures a faultless quality of the products that we manufacture. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

7. Explanation of symbols

Please refer to the label. Additional information can be found at www.dentaaurum.com (Explanation of label symbols REF 989-313-00).

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Dentaurum GmbH & Co. KG | Turnstr. 31 | 75228 Ispringen | Germania