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Dentaplast KFO

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Instructions for use

Dentaplast KFO

Before using the product, please read these instructions for use carefully!

The manufacturer will not accept any liability for damage resulting from non compliance with the instructions for use.

1. Description of the product

Dentaplast KFO is a cold-curing resin. The product includes a liquid (monomer), which is available in bottles and a powder (polymer), which is available in jars.

2. Indication

The product was developed for the fabrication of orthodontic appliances using the sprinkle technique. The high stability of Dentaplast KFO allows precise working and provides superior accuracy of fit.

3. Contraindication

Dentaplast KFO is a resin based on methacrylates, inorganic fillers, initiators and catalysts. The product should not be used for patients with a suspected allergy until allergy tests have been completed to confirm that the patient is not allergic to the materials contained in the product.

4. Hazard and safety information

Signal word:

Danger

Hazard statements:

H225 Highly flammable liquid and vapour.
H315 Causes skin irritation.
H317 May cause an allergic skin reaction.
H335 May cause respiratory irritation.

Precautionary statements:

Prevention:

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

P233 Keep container tightly closed.

P261 Avoid breathing vapours.

P280 Wear protective gloves / protective clothing / eye protection / face protection.

Response:

P303+P361+P353 If on skin (or hair): Remove / Take off immediately all contaminated clothing. Rinse skin with water / shower.

Additional safety information about the product is included in the safety datasheet.

5. Storage and durability

Storage:

P403+P235 Store in a well-ventilated place. Keep cool.

Product contains: methyl methacrylate, ethylene dimethacrylate

6. Processing/Use

Preparation

Fasten wire elements and expansion screw in the usual way, for example using adhesive wax. Soak the model and apply a commercially available resin/acrylic separating liquid in accordance with the manufacturer’s instructions.

Processing

Several layers of powder are directly applied to the model from the dosing bottle. The thickness of a layer should not exceed 3 mm. Then use the dosing bottle for liquid to apply liquid onto the powder in a way to ensure that it is fully soaked but will not flow away. Apply powder once more to obtain a final layer. Bubbles can be soon recognized thanks to the high transparency of the material. Any bubbles can be removed by puncturing them with an instrument.

Polymerization

Dentaplast KFO must be hardened in the pressure pot no later than five minutes after it is applied. Polymerization: polymerize in a water bath for 20 minutes at 50°C and a pressure of 2.2 to 2.5 bar.

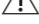
7. Technical data

Dentaplast KFO is a polymethyl methacrylate.

For additional technical data, please refer to the safety datasheet.

8. Symbols

8.1 Symbols according to DIN EN ISO 15223

	Reference number
	Lot number
	Can be used until YYYY-MM
	Attention!
	Maximum temperature 23° C
	Manufacturer

8.2 Symbols according to GHS regulation

	GHS07
	GHS02

8.3 Symbols according to NFPA (USA)

	HEALTH 1 FIRE 1 REACTIVITY 0	Hazard diamond
---	---	----------------

9. Additional information

These instructions for use are based on state-of-the-art methods and equipment and our own experience. The product may only be used for the indication described under item 2. The user himself is responsible for processing the product. Liability for incorrect results shall be excluded since the manufacturer does not have any influence on further processing. Any occurring claims for damages may only be made up to the value of our products.

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GmbH & Co. KG · Weissenhorner Str. 2 · 89250 Senden · Germany