General	Instructions for use
Henry Schein® Eurosil® Putty is a kneadable impression material based on condensation curing silicones. Henry Schein® Eurosil® Putty is hydrophilic and more compatible with moist surfaces than conventional condensation curing silicones. Henry Schein® Eurosil® Putty is easy to mix and not sticky.	
Classification Henry Schein® Eurosil® Putty meets the requirements of:	DIN 13913 2-a ISO 4823 Cat.B, Type 1 (high viscosity)

Indication
Primary impression material for the double-impression technique in conjunction with Henry Schein® Eurosil® Correction Paste.

Trays and adhesive
Perforated or non-perforated metal or plastic trays may be used with Henry Schein® Eurosil® Putty. In order to improve adhesion of Henry Schein® Eurosil® Putty to the tray, adhesive should be applied in a thin, even film. Allow to dry for 3 minutes before loading the impression tray.

Dispensing and mixing
Henry Schein® Eurosil® Putty and Henry Schein® Eurosil® Catalyst paste are dispensed in a ratio of 22:0.5 by weight or 30.5:0.44 by volume. This can easily be done by taking Henry Schein® Eurosil® Putty with the dosage spoon and adding 2 lengths of Henry Schein® Eurosil® Catalyst according to the diameter of the spoon. Knead until color is homogeneous (at least 45 seconds).

Impression taking
The ready kneaded material is loaded into the prepared tray. Insert loaded tray in the mouth not later than 1½ minutes after start of mix. Maintain position for at least 2 minutes.
Note: Times mentioned are valid for 23°C (77°F). Higher temperature shortens, lower temperature prolongs these times.

Casting
The impression should be stored under normal room conditions (max. 25°C/ 77°F). The impression should be poured within 2 - 4 hours after removal from mouth.

Electroplating
Impressions can be copper- or silver-plated.

Disinfection
Impressions can be disinfected with standard disinfection solutions.

Storage
The shelf life is at least 3 years, when stored at 23°C (73°F). Do not store above 25°C (77°F).

Additional notes
Avoid contact of Henry Schein® Eurosil® Catalyst with eyes. If contact occurs, wash the eyes with copious amounts of water and seek ophthalmological care.

Technical data	
Mixing time	30 sec.
Total working time (incl. mixing time)	1½ min.
Setting time (incl. mixing time).....	2½ min.
Recommended time in mouth	2 min.
Dimensional change after 24 hours	less than 0.6%
Compression set	less than 0.9%
Strain in compression	3.0%
Hardness Shore-A	ca. 70

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Hergesteld in den USA/Gefabriceerd in de U.S.A.
Melville, NY 11747 U.S.A.