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# EEC-Declaration of Conformity

Dentaurum J. P. Winkelstroeter KG  
Turnstraße 31 · 75228 Ispringen · Germany

hereby declares that the design and construction of the laboratory equipment described below, including the version marketed by us, comply with the basic regulations governing safety and health as stated in the EEC Guidelines. This declaration will become invalid if the laboratory equipment is modified or altered in any way without our prior consent.

|                                   |  |
|-----------------------------------|--|
| Description of the unit           | Electric Heating Ring H1 for Polyclav®   |
| REF                               | 070-900-00 / 070-901-00  |
| Start with unit No.               | 13-4032 / 24-4111  |
| EEC guidelines                    | 89/336/EEC                      Electromagnetic compatibility  |
|                                   | 92/031/EEC                      Modified Guideline   |
|                                   | 73/023/EEC                      Electrical equipment used within<br>certain voltage limits   |
|                                   | 93/068/EEC                      Modifications of the above<br>guidelines   |
| Applied unified standards         | EN 60335   |
|                                   | EN 55011/03.91   |
|                                   | EN 50082-1/01.92   |
|                                   | EN 61326:1997+A1:1998  |
| Date and manufacturer's signature | 28.07.05   |
| Signatory                         | <br>.....<br>- Dipl.Ing.(FH) K. Merkle -<br>Production Manager Mechanics |

DENTAURUM  
J. P. WINKELSTROETER KG  
Turnstraße 31 · Tel. (0 72 31) 8 03 - 0  
75228 Ispringen

## Dear Customer

Thank you for having chosen a quality product from Dentaurum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Due to a constant development we recommend, even when you use the same products frequently, to study the enclosed latest issue of the instructions for use or refer to our website at [www.dentaurum.de](http://www.dentaurum.de).

## 1. Manufacturer

Dentaurum J. P. Winkelstroeter KG · Turnstraße 31 · 75228 Ispringen · Germany

## 2. With reference to our quality standards

Dentaurum ensures a faultless quality of the products manufactured by us. These nonobligatory recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

## 3. General notes

### 3.1 Notes on Safety

- The Heating Plate should only be used for heating the Polyclav® polymerization pressure vessel.
- Do not place inflammable substances such as cardboard or paper on the Heating Plate.
- Never use the Heating Plate without the Polyclav® pressure vessel.
- If repair is required: remove the electric supply plug before opening the unit.

### 3.2 Application and Description

The thermostatically controlled Heating Plate for the Polyclav® retains the constant temperature that is necessary for the polymerization process. The temperature of the water in the Polyclav® pressure vessel is kept at approx. 43 °C / 109 °F.

### 3.3 Technical Data

|                           |  |          |                |
|---------------------------|--|----------|----------------|
| Voltage                   | 230 V                                    | 50/60 Hz | REF 070-900-00 |
|                           | 115 V                                    | 50/60 Hz | REF 070-901-00 |
| Average power consumption | 125 W                                    |          |                |
| Size                      | 240 x 240 x 75 mm / 9 1/2" x 9 1/2" x 3" |          |                |

### 3.4 Operation of the Unit

- The Heating Plate is connected to the power supply via a grounded plug.

**Important:** The power supply voltage must correspond to the voltage stated on the plate of the unit.

- The unit is switched on using the power supply switch.
- During the heating phase, the red light glows.
- Switch the unit off after use.

**Important:** Never use the unit without the Polyclav® pressure vessel.

### 3.5 Care and Maintenance

The Heating Plate requires very little care or maintenance.  
Use a damp cloth to clean the unit.

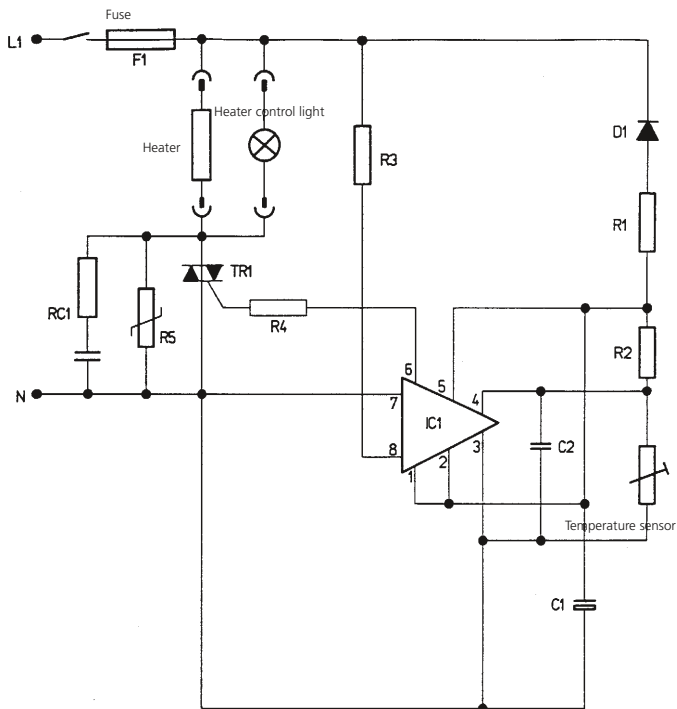
**Note:** Never use abrasive or corrosive cleaning agents to clean the unit.

### 3.6 Spare Parts

|                       | <b>Unit, 230 V</b> | <b>Unit, 115 V</b>                     |
|-----------------------|--------------------|--|
|                       | 070-900-00         | 070-901-00                             |
| Description           | REF                | REF                                    |
| Rocker switch         | 907-129-10         | 907-143-10                             |
| Light (red)           | 908-831-00         | 908-831-00                             |
| Electric supply cable | 908-832-00         | 908-832-00                             |
| Control module        | 978-120-10         | 978-120-20 (circuit board inside unit) |
| Temperature sensor    | 907-070-10         | 907-070-10                             |

### 3.7 Circuit Diagram

### 3.8 Disposal

When a Dentaurem product is finally no longer in use the disposal regulations of that particular country apply. Dentaurem or the dental trade is available to answer questions regarding the correct disposal of any specific product.

**Note:**

This symbol indicates, products marked with it should not be disposed off together with garbage of households. The legislator disallows commercial customers to return electronic waste via municipal collection points. Further information is available through Dentaurem or the dental trade.

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