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orthopädie
chirurgie
sportmedizin
rehabilitation

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Cervical Support
Soutien cervical
Soporte cervical
Collare cervicale

Art.-No.
127 260 weiß
127 360 haut
127 660 blau
127 460
127 560
StabiloNec®
127 570
StabiloNec®Eco

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GB

Injuries in the cervical vertebra area are affecting a complex structure of ligaments, muscles, sinews, nerves and vessels. Immobilisation support recovery of these tissue structures and helps at the same time avoiding to a high degree mismovements.

Instruction for use:

Open up cervical support and grip it with both hands horizontally. The Velcro closure strap is placed at the back.

Put on cervical supports in a way that the recess fits directly below the chin. Then wrap the support around the neck and lock the Velcro closure at the rear side. Place the cervical support in a way that it supports but does not strangle. The support is made of soft foam and therefore can also be worn at night.

Indications:

For whiplash injuries to the cervical spine, after operations on the cervical spine and after torticollis operations. Cervical spine syndrome.

Contraindications:

Tracheostomy, severe swellings, large-area skin injuries, oversensitivity against one of the materials used.

Composition:

PUR foam, cover 100% cotton

Important Instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove it if necessary. If complaints continue, please contact the doctor.

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