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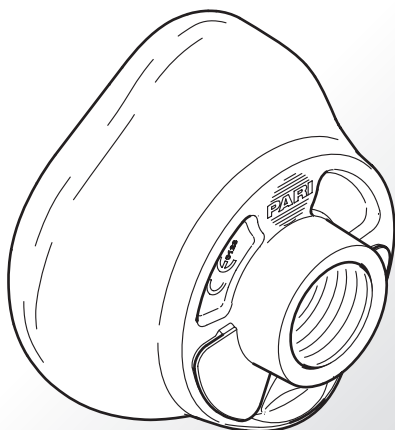
EN Instructions for use

- Read these instructions carefully and completely before using for the first time. Keep these instructions for use in a safe place.
- The PARI SMARTMASK® Kids must always be handled as a component of or accessory to the PARI nebulisers; accordingly, the instructions for use for these products must also be followed. In particular, the chapter entitled „Hygienic Re-use“ in the instructions for the respective instrument also applies for the PARI SMARTMASK® Kids.

Components

Item no. 078G5000 (PARI nebuliser not included)

Intended use

The PARI SMARTMASK® Kids provides a mask inhalation system for children aged 2 and older who are unable or unwilling to inhale through a mouthpiece.

The PARI SMARTMASK® Kids is designed for repeated use.

Caution:

To avoid the risk of the child not getting enough oxygen, this mask must not be used for children less than 2 years of age.

The PARI SMARTMASK® Kids must only be used with the following PARI nebulisers: the LC PLUS® family, LC STAR®, LL, the LC® SPRINT family, and eFlow®rapid.

Using the mask

Image A:

- Place the mask on the nebuliser.

Image B:

- For the most efficient treatment, hold the PARI nebuliser horizontally.

- The PARI SMARTMASK® Kids must not be used on children who are lying down or asleep.
- To avoid aerosol losses, make sure that the mask seals tightly.

Image C:

Make sure that the mask valves are lying flush against the outside of the openings! They must not be pressed inwards under any circumstances!

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