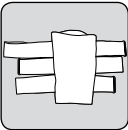
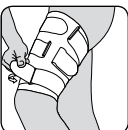
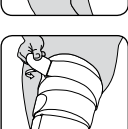
E

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
GB

The Thigh Support with Velcro system regulates the muscular tone and prevents relapse injuries. The support functions as a muscle suspension support (tape support), is individually and pressure adjustable. The anatomically formed knit prevents the support from slipping.

User instructions: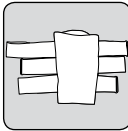

1. The strapping system is already fitted when you receive your Thigh Support. Please ensure that the Velcro tapes are **fully** open when applying the support.


2. Apply the Thigh Support whilst standing, leg relaxed and knee slightly bent.

3. Starting with the lower strap, pull the strap forward by tugging slightly, feed smoothly through the eyelets and close. Attaching the straps to the Velcro surfaces prevents the Velcro tapes from sliding down.

4. The remaining upper straps are then closed in the same way.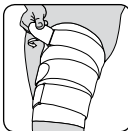

5. Please adjust the strap pressure with knee slightly bent, sensing compression, but no pain.

Indications:

Muscular injuries to the thigh (rupture of muscle fibre, muscle contusions), treatment of sports injuries, to prevent recurring injuries.

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

Composition:

65 % Polyamide, 28 % elastodies, 7 % viscose

Important instructions:

No reutilisation – this medical device is meant for the care of one patient only.

In case of doctor's order to wear support during night, avoid hindering blood circulation. In case of numbness loosen support and remove if necessary. If complaints continue, please contact the doctor.

While wearing the support do not use creams or ointments. This can damage the material.

The product contains latex and can trigger allergic reactions.

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