NAVI[®] Blef

ESPIUMA LIMPIADORA DE PARPADOS

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UK

INTRODUCTION:

The daily hygiene of eyelids and eyelashes is very important for the contact lens wearers that suffer from dry eye

and blepharitis.

NAVI[®]BLEF is a solution with camomile extracts and Melaleuca, simple to use and adapted for frequent use.

NAVI[®]BLEF promotes proper eyelid and eyelashes hygiene, is efficacious and improves user compliance. NAVI[®]BLEF is recommended for daily hygiene of the eyelids of contact lens wearers, before eye surgery and to cleanse eyelids and eyelashes in case of blepharitis or dry eyes.

WHAT IS BLEPHARITIS?

It is a chronic inflammation of the eyelids and is one of the most common disorders of the eye, often the underlying reason for eye discomfort, redness and tearing. Other eye symptoms of Blepharitis include, burning, itching, light sensitivity and irritation, sandy and gritty sensation that is more accentuated upon awakening.

Blepharitis is a chronic disease for which there is no cure and requires a long-term treatment and maintenance to keep it under control and prevent subsequent related complications linked to these type of conditions.

WHAT IS DRY EYE?

It is known as disorder of the tear film due to tear deficiency or excessive tear evaporation which can cause damage to the interpalpebral ocular surface. Dry eye is associated with symptoms of ocular discomfort that can cause an increased bacteria load of the eyelids and eyelashes.

EYE SURGERY:

The use of NAVI[®]BLEF can help to reduce the risk of eye infections caused by the bacteria present on the eyelids and eyelashes during and after eye surgery.

NAVI[®]BLEF removes the eyelid debris, which can be colonized by bacteria, reduces the bacterial load due to its detergent action, by scrubbing mechanically at the base of the eyelashes. It stabilizes the tear film and reduces tear evaporation and therefore brings relief to dryness conditions of the eye.

DIRECTIONS:

- 1 - Prior to use wash your hands and then clean the fingertips with NAVI[®]BLEF foam.
- 2 - Press the delivery pump and place the foam on the clean fingertip.
When used before or after surgery it is recommended to put the foam on a sterile gauze.
- 3 - Close the eyes and massage the eyelids and eyelashes with NAVI[®]BLEF foam. Do not touch or open the eyes during the application. Leave the foam for approximately 60 to 80 seconds and used it 2 to 3 times a day for a period of 7 days, for maximum efficacy.
- 4 - Rinse the eyelashes and eyelids with a warm water avoiding home made solutions.

SPECIAL RECOMMENDATIONS:

- Wash your hands prior to use.
- Remove contact lenses prior to use.
- Keep the eyes closed during the application.
- Use hypoallergenic eye makeup and delicate skin lotions around your eyes during the treatment.
- For external use only.
- Keep the product away from the children.
- Don't use for children under 3 years.
- Store product in fresh and dry place and under 35°C.
- Don't use the product after 6 months from opening.

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