E

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Your physician has prescribed you with a high-quality therapeutic shoe, which is suited for early mobilisation in the form of this aid.

Please observe the following points in order to ensure that wearing this shoe is as comfortable as possible for you.

Instruction for use:

Protect the open foot wound by means of a bandage or similar product. Use crutches when wearing the BORT forefoot relief shoe. Always wear a non-slip shoe with the same heel height on your healthy foot.

You are not permitted to drive a vehicle whilst wearing the BORT forefoot relief shoe.

The supplied toe cap can be adjusted using the Velcro closure located on the shoe in order to protect against the cold or dust if necessary.

Indications

Post-operative, wounds/injuries of the forefoot and the sole of the foot, post-operative following corrective osteotomy in the forefoot area.

Contraindications:

Peripheral arterial disease (PAD), lymphatic drainage disorders and indistinct soft tissue swellings remote from the applied aid, sensory loss and blood flow disorders in the affected body part, skin disorders in the treated part of the body.

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