# Kit Endoperox polvo + glicerina

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# Kit Endoperox powder + glycerol

## Dentine whitener for devitalized teeth

### Formula Endoperox powder

100% carbamide peroxide.

### Properties

Oxygen is released from the Hydrogen Peroxide by the presence of the oral fluids. This release of oxygen occurs in a practically neutral medium. The principal effects resulting from the release of the latent oxygen are :

- bleaching,
- killing bacteria,
- mechanical cleansing by gas bubbles.

### Indications

Whitening of pulpless teeth, stained for example, by blood.

### Directions for use

- Before proceeding to the whitening make sure of the tightness of the canal filling.
- Clear the pulp chamber.
- Open the canal on about 3 mm.
- Cover the floor of the cavity obtained with either a glass ionomer cement or a carboxylate cement.
- Etch the pulp chamber with phosphoric acid for 1 minute.
- Abundantly rinse with water. Dry.
- Clear the pulp chamber with alcohol. Dry.
- Make a paste by mixing two spoonfuls of powder with 1 drop of glycerol to a firm consistency.
- Cover the vestibular side with the paste.
- Cover with a pledget of cotton wool.
- Seal with a cement.
- Make a second appointment with the patient 4 or 5 days later.
- If sufficiently white, fill.
- If not repeat the treatment which is completely harmless to the dentine.

It is preferable to obtain the bleaching over a few visits, rather than in one single visit to avoid obtaining too great a discolouration.

### Warnings

Avoid contact with the mucosa and the eyes as the product contains a high percentage of carbamide peroxide. Take protective measures as necessary.

### Storage

Keep away from light in its original packaging and do not store above 25°C.  
Close the container tightly after use.

### Cautions

For professional use only in the practice of dentistry.

THIS PRODUCT SHOULD NOT BE INGESTED.  
In case of accidental ingestion, induce vomiting and contact a Poison Control Center immediately.  
In case product comes in contact with the eyes, rinse well with water and consult an ophthalmologist immediately.

### Presentation

Kit containing:  
1 x 0.18 oz (5 g) jar of Endoperox powder (100% carbamide peroxide),  
1 x 5 ml flask of glycerol,  
1 measuring spoon.

NDC : 0362-1701-00

Made in France

Distributor: SEPTODONT, Inc.  
245 Quigley Boulevard - Suite C  
NEW CASTLE, Delaware 19720 U.S.A.  
Made in France by:  
Septodont