VDW 6:1

Contra-angle handpiece

Winkelstück

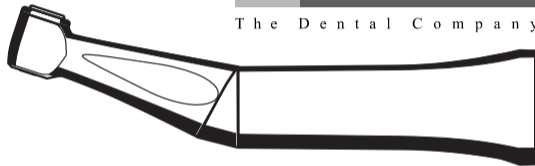
Contrangolo

Contrángulo

Contre-angle

sirona.

The Dental Company

Operating Instructions

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Istruzioni d'uso

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Sirona Dental Systems GmbH

VDW 6:1 – Operating Instructions

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VDW 6:1 – Istruzioni d'uso

61 34 378 D 3451

English

Deutsch

Italiano

Sirona Dental Systems GmbH

VDW 6:1 – Instrucciones de uso

VDW 6:1 – Notice d'utilisation


Español

Français

61 34 378 D 3451

VDW 6:1

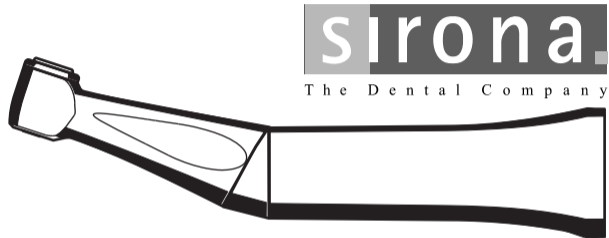Contra-angle handpiece

English

Operating Instructions

For dental use only

We reserve the right to make any alterations which may be required due to technical improvements.

Dear customer,

The VDW 6:1 contra-angle handpiece is intended only for dental treatment at the dental treatment center.

To prevent any personal injury or material damage, please observe the warning and safety information provided in the present operating instructions. All such information is highlighted by the captions CAUTION, ATTENTION or NOTE.

Your Instruments Team

Contents

1	Indications for use	5	6	Technical description	10
2	Contraindications	5	6.1	The VDW 6:1 contra-angle handpiece complying with ISO 7785 - 2	10
3	Warnings	6	6.2	Service life of Sirona instruments	11
4	Precautions	8	6.3	VDW 6:1 contra-angle specifications	12
5	Adverse reactions	9			

7	Step-by-step instructions	13	10 Disposal	25
7.1	Putting the contra-angle handpiece into operation	13		
7.2	Inserting/removing the endo file	14		
7.3	Care with spray	15		
7.4	Cleaning the pushbutton chuck	17		
7.5	Cleaning and disinfection	18		
7.6	Sterilization instructions	22		
8	Spare parts and consumables	24		
9	Storage and transport conditions	24		

1 Indications for use

The **VDW 6:1** handpiece is for the mechanical root canal preparation of tooth disease and dental traumas through processing of tooth hard and soft substance.

The **VDW 6:1** contra-angle handpiece is intended only for dental treatment at the dental treatment center.

2 Contraindications

None

3 Warnings

This product is intended only for use by trained dental personnel. Read these operating instructions carefully prior to startup of the contra-angle handpiece and keep them handy for further reference.

Observe all safety information and warnings, including the following:

- Prior to putting it into operation, the new contra-angle handpiece must be sterilized.
- The contra-angle handpiece should be sterilized after each patient to prevent any transmission of infective agents such as e.g. HIV (the AIDS virus) or hepatitis B

which might cause serious illnesses or even lead to death.

- Suitable hygienic measures must be taken to prevent cross contamination among patients, users and other persons.
- Use the contra-angle handpiece for removal (e.g. of caries) only when the instrument is clamped securely in place. Do not pull the instrument out of the contra-angle handpiece head. A loose or partially removed instrument could detach itself from the head or break off and cause injuries.
- **After sterilization:**
Immediately remove the contra-angle handpiece from the steam sterilizer.

- Sterilize again once the storage period has elapsed according to the manufacturer of the sterilizing container.
- **Prior to each use**, test whether the fitted instrument is firmly seated by pulling and turning it.

⚠ CAUTION! Make sure not to inadvertently press the pushbutton when pulling back the patient's cheek with the contra-angle handpiece while the motor is running.
Risk of burning the oral mucosa!

- Do not repair the contra-angle handpiece yourself.
- If the contra-angle handpiece is defective, the area around the instrument head may heat up, thus creating a risk of burning the patient's oral mucosa.
- Fit or withdraw only with motor at standstill.

- Use only faultless instruments. These must be clean to prevent dirt from being carried into the clamping system. They should be checked for sharpness from time to time.
- For application details and operating data on the different instruments (e.g. speed and setting the preparation depth), please refer to the information provided by the manufacturer. Use only instruments whose shanks comply with the requirements of **DIN EN ISO 1797-1**.
- Do not use any instruments whose specifications differ from the ones indicated!
- Use only faultless instruments in order to prevent fatigue fractures of the instruments as far as possible.
- Test whether the file is firmly seated by pulling and turning it.

4 Precautions

- Do not immerse the contra-angle handpieces in cold water to speed up the cooling process. This would permanently damage your contra-angle handpiece!
- Do not leave any contra-angle handpieces on the motor overnight, in order to prevent oil from penetrating into the electric motor. Never lubricate the electric motor!

4 Precautions

It is recommended to place a dental dam when preparing a root canal.

In case of prolonged **non-use**, the contra-angle handpiece must be cleaned and cared for in accordance with the instructions. It must be stored in a dry, heated room.

Use only approved or original replacement and accessory parts.

If the contra-angle handpiece is defective, the area around the instrument head may heat up, thus creating a risk of burning the patient's oral mucosa.

In case of noticeable malfunctions or damage, stop using the instrument immediately and notify your dental depot or the manufacturer.

Sirona Dental Systems GmbH

If you have any questions, please also contact your dental depot or the manufacturer.

5 Adverse reactions

5 Adverse reactions

None known.

6 Technical description

6.1 The VDW 6:1 contra-angle handpiece complying with ISO 7785 - 2

The **VDW 6:1** contra-angle handpiece complies with the latest regulations reflecting the current state of the art.

According to these regulations, contra-angle handpieces may only be used for the described application by trained personnel, in compliance with the applicable hygiene standards, occupational safety regulations and accident prevention measures as well as the present operating instructions.

Furthermore, the user is obliged to use only faultless materials, to pay attention to the correct application as well as to

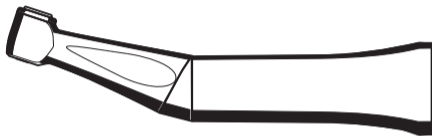
protect himself or herself, the patient and other persons against hazards.

The VDW 6:1 contra-angle handpiece is part of the technical equipment of the dental practice.

The contra-angle handpiece serves for transmitting the driving power and speed of the electric motor (handpiece coupling according to ISO 3964) to the preparation tool.

The required operating speed is attained by performing the presettings at the dental treatment center.

A motor working speed of up to 40,000 rpm and a gear ratio of 6:1 yields a maximum speed of 6,700 rpm on the file.

6.2 Service life of Sirona instruments

When used as intended:


- Non-moving parts of Sirona instruments have a typical service life of 5 years*
- Moving parts of Sirona instruments have a typical service life of 3 years*

Elastomers, e.g. O-rings, must be replaced depending on wear as described in the operating instructions.

For safety reasons, however, we recommend an annual check of the instrument holder system.


* No warranty claim can be inferred here, as wear may occur earlier or later than indicated above depending on use, frequency of sterilization and frequency of maintenance.

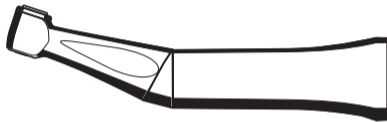VDW 6:1

6.3 VDW 6:1 contra-angle specifications

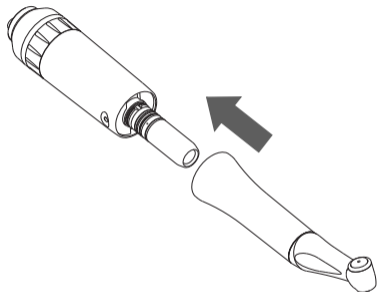- Compatible with INTRAmatic system®
- Maximum motor speed 40,000 rpm
- Gear ratio 6:1
- Maximum operating speed 6,700 rpm


7 Step-by-step instructions

7.1 Putting the contra-angle handpiece into operation

Fitting: Fit the contra-angle handpiece **up to the stop**. Make sure that it is firmly seated on the motor.

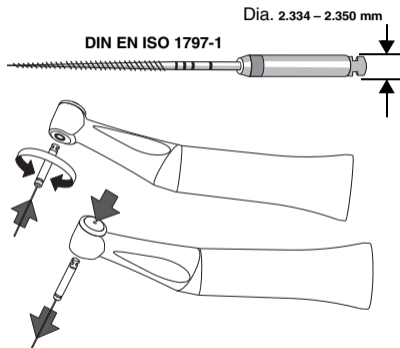Removing: Simply pull off the contra-angle handpiece. Do not hold or pull on the supply hose!


14

7.2 Inserting/removing the endo file

Select the required file from the assortment available.

Fitting: Insert the file without pressing the pushbutton cover and while the drive is at standstill, turning it slightly forwards and backwards **until it snaps into place.**

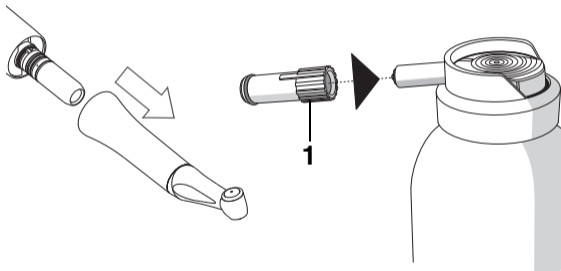Removing: After it has come to a stop, withdraw the file while pressing and holding down the pushbutton.


7.3 Care with spray

at least every **evening** and **prior to sterilization**.

- Remove the file from the treatment head.
- Separate the contra-angle handpiece from the motor.
- Fit the spray adapter (ISO) (1) onto the nozzle of the spray can.

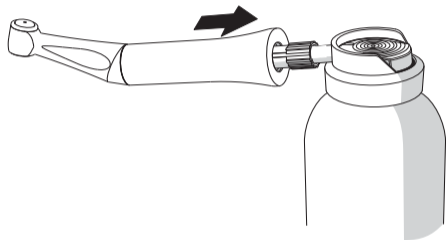i NOTE *We recommend using T1 Spray from Sirona Dental Systems GmbH.*


- Plug in the contra-angle handpiece **up to the stop** and hold it firmly.
- Apply spray for **1 – 2 seconds**.

Repeat the spraying procedure until the liquid which comes out of the head is no longer dirty.

Let the drive run briefly between applications.

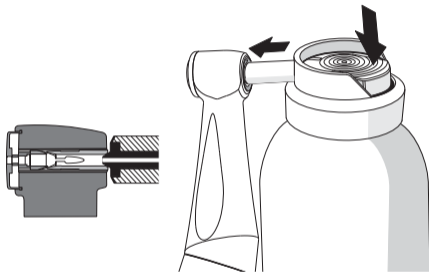- Wipe off any leaking oil with a dry cloth **while the drive is at standstill**.
- Prior to use, briefly run the contra-angle handpiece at no load on the dental unit and wipe it with a dry cloth **while the drive is at standstill**.

61 34 378 D 3451

7.4 Cleaning the pushbutton chuck

To remove deposits and ensure proper functioning of the clamping system, the chuck must be cleaned with **oil spray**.

At least once a week

- Press the contra-angle handpiece head with the chuck firmly against the spray can nozzle and spray the chuck for 1 – 2 seconds.
- Always keep the spray can upright!

Wipe off any protective oil that leaks with a lint-free cloth.

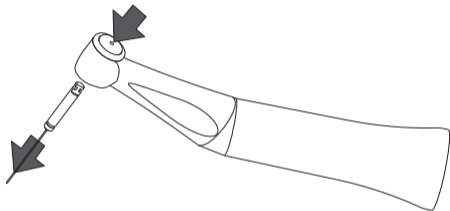
⚠ ATTENTION! *Always clean and oil the chucks of treatment instruments as specified by the manufacturer, even when they are being processed automatically!*

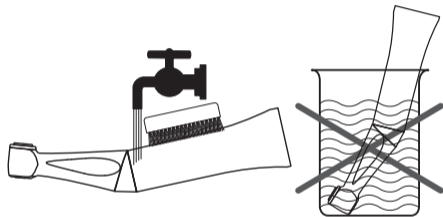
7.5 Cleaning and disinfection

Wear the appropriate protective clothing.

Remove the rotating instrument.

Automated processing is preferable over manual.

7.5.1 Manual cleaning

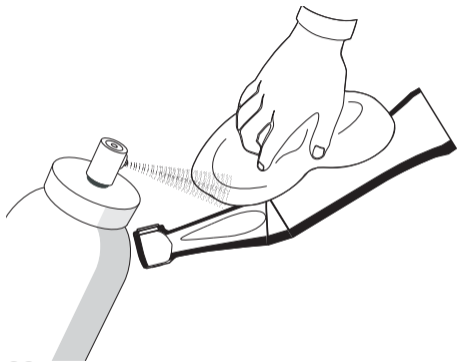
Manual cleaning must always be combined with disinfection.

⚠ ATTENTION! *Never immerse in disinfectants! Never clean in an ultrasonic bath!*

- Clean the contra-angle handpiece by brushing it off under running water (< 38 °C, < 100 °F, at least drinking water quality) and then blowing it out.
- Spray with T1 spray, see section 7.3.

Clean and disinfect spray channels with suitable agents and adapters according to manufacturer's instructions, e.g. with WL-cid made by Alpro.

All disinfectants have to be approved in your country and be proven to have bactericide, fungicide and viricide properties.


20

Outer **spray and wipe disinfection** is permissible using instrument disinfectants such as **MinutenSpray classic** by Alpro **MinutenWipes** by Alpro

In the USA and Canada:
CAVICIDE® or **CAVIWIPES™**

Please observe the manufacturer's instructions for using the instrument disinfectants.

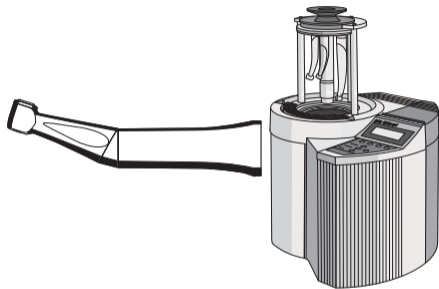Subsequently, sterilization is to be carried out according to section 7.6.


7.5.2 Autoclaving

⚠ ATTENTION! Always manually process the chucks of the instruments, even when automated processing is used, as described in the section "**Cleaning the pushbutton chuck**".

We recommend **DAC UNIVERSAL** by **Sirona** for automated cleaning, disinfection and care.

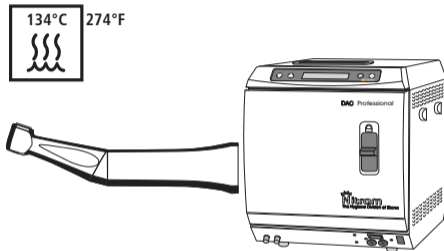For further details, please refer to the operating instructions supplied with the unit.

7.6 Sterilization instructions

Prior to sterilization:

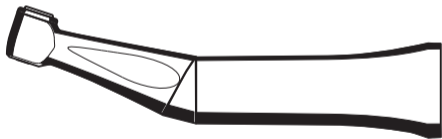
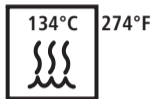
Clean and disinfect contra-angle handpiece (7.5).

Protect contra-angle handpiece by applying T1 Spray (7.3).

Contra-angle handpieces may be sterilized only in the steam sterilizer with saturated water vapor at **134 °C (274 °F), 3 min. holding time (2.04 bar; 29.59 psi overpressure)**.

Steam sterilizers that fulfill the requirements of EN 13060, class B or S and are suitable for the sterilization of contra-angle handpieces: e.g. **DAC PROFESSIONAL** by **Sirona**.

⚠ ATTENTION! Do not exceed 140 °C (284 °F) even during the drying phase!

Contra-angle handpieces must be sterilized in a packaging which is suitable for sterilization and storage, e.g. a paper/plastic composite packaging.

All instruments must be stored so that they are protected from contamination.

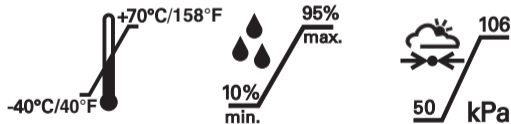
8 Spare parts and consumables

T1 Spray (6 x 250 ml cans)
Spray adapter (ISO) for spray can
O-ring for spray adapter

Order No.:
59 01 665
89 17 858
70 36 353

Use only approved or original replacement and accessory parts.

9 Storage and transport conditions

10 Disposal

To the best of our current knowledge, the product does not contain any substances which are hazardous to the environment.

The product must be disinfected prior to disposal.

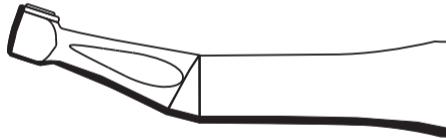
Please observe the applicable national regulations regarding disposal.

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