3225 Deming Way, Suite 190, Middleton, WI 53562, USA

TEL. (800) 537-7123 • International TEL. (714) 516-7400
 FAX (800) 537-7345 • International FAX (714) 516-7581

OptiMix is internationally distributed through Kerr Corporation

European Union Representative
 Kerr Italia, S.p.A.
 Via Passanti, 332
 I-84018 Scafati (SA)
 ITALIA
 +39-081-850-8311

Kerr U.K. Limited
 1 Mallard Business Centre
 Mallard Road, Bretton
 Peterborough, Cambridgeshire
 PE3 8YP
 ENGLAND
 +44-1733-260-998

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Kerr Australia Pty, Limited
 Inc.
 Unit 11
 112-118 Talavera Road
 North Ryde 2113
 New South Wales
 AUSTRALIA
 +61-29-870-7500

Sybron Dental Specialties Japan,
 Kitamura Bldg.
 29-24 Honkomagome
 Bunkyo-Ku, Tokyo
 113-0021
 JAPAN
 +81-3-5977-3126

OPERATING INSTRUCTIONS OPERATING
 INSTRUCTIONS OPERATING INSTRUCTIO
 OPERATING INSTRUCTIONS OPERATING
 TIONS **OPERATING INSTRUCTIONS** OPER
 INSTRUCTIONS OPERATING INSTRUCTIO
 OPERATING INSTRUCTIONS OPERATING
 INSTRUCTIONS OPERATING INSTRUCTIO

OPTIMIX

- ◆ OPERATOR'S MANUAL
- ◆ MANUEL D'UTILISATION
- ◆ MANUAL DO OPERADOR
- ◆ MANUAL DEL USUARIO

TECHNICAL DESCRIPTION AND INSTRUCTIONS FOR USE

Page

1. DESCRIPTION/FEATURES	3
2. SAFETY PRECAUTIONS	4
3. OPERATION	5

- A) Starting/Stopping Operation
- B) Programming Instructions
- C) Setting Mixing Times & Speeds
- D) Saving Options
- E) Invalid Entries
- F) Power Down
- G) Display Screen Messages
- H) International Language Options

◆ ENGLISH ◆

4. TROUBLESHOOTING GUIDE	10
5. FUSE REPLACEMENT	11
6. CLEANING, CARE AND MAINTENANCE	11
7. SPECIFICATIONS AND EQUIPMENT CLASS	12
8. WARRANTY	13
9. LIABILITY	13
10. REPLACEMENT PARTS & ADDITIONAL PRODUCTS	14
11. FIGURES/PHOTOGRAPHS	73

TECHNICAL DESCRIPTION

1. DESCRIPTION/FEATURES

The OptiMix is a computerized mixing system intended for dental office amalgam mixing.

Save these instructions for future use. Save Shipping Container and Packing Inserts. (See Troubleshooting for Service Information).

Box contains:

<u>Qty.</u>	<u>Description</u>
1	OptiMix Computerized Mixing System
1	Operator's Manual
1	OptiMix Reference and Quick Save Card
1	OptiMix Reference Card for Kerr capsules, tablets and generic products

Many features are incorporated into the OptiMix as defined below:

- Keypad entry programming for accurate setting of mixing speed and time as shown in (Figure 1-1).
- Computerized operating system with digital technology that adjusts motor function automatically.
- LCD display of programmed settings as shown in (Figure 1-2), and displaying error information in six different languages as shown in (Figures 1-3 and 1-4).
- Pre-programmed settings for standard capsules as shown in (Figure 1-5).
- Capsule holder designed for easy capsule loading and removal as shown in (Figure 1-6).

2. SAFETY PRECAUTIONS

An exclamation point within a triangle  is intended to alert the user to the presence of important operating, maintenance, and precautionary instructions in the literature accompanying the product.

a) IMPORTANT SAFEGUARDS:

 **When using electrical devices, basic safety precautions should always be followed to reduce the risks of fire, electric shock, and/or injury to persons including:**

- Read all instructions before using the machine.
- Use the product only as directed, positioned on a hard, flat surface with sufficient space from countertop drop off.
- To protect against electric shock, do not immerse cord, plugs, or machine in water or other liquid.
- Do not operate machine with a damaged cord or plug, or after damage in any other manner. Return this unit only to the authorized service center for repair.
- To reduce the risks of fire or shock, do not remove any service covers. No user serviceable parts inside.

Repair of unit is done by Kerr located at 21 Commerce Drive, Danbury, CT 06810. Please check with your dealer before sending in your unit.

 **CAUTION: This unit has a safety interlock door switch. Do not tamper with switch and/or operate machine with door open. If unit continues to operate with door open, return entire unit for repair.**

 **CAUTION: Earth grounding circuit continuity is vital for safe operation of OptiMix. Never operate unit with Earth grounding disconnected.**

INSTRUCTIONS FOR USE

3. OPERATION

We recommend the operator (s) become familiar with the unit before use.

A) STARTING/STOPPING OPERATION:

 **CAUTION: Proper input voltage is indicated on the serial number label on back of unit!**

Connect the power cord to the OptiMix and the AC (Mains) supply outlet. Turn on power switch, following initialization of the Kerr OptiMix, the last used saved program will be displayed (Figure 3A-1). This last program displayed can be run by pressing START or another program can be used by entering a program # from the provided pullout card (Figure 3A-2) and pressing START.

To stop the mixing cycle, before the preset time has elapsed:

- a) Press any key or
- b) Open the lid to the mixing chamber

B) PROGRAMMING INSTRUCTIONS:

1. PROGRAMS 01-09

These program assignments are for doctor entered and saved programs that are regularly used and can be recalled by pressing keys (01-09) and then START.

Program locations 01-09 could contain **ALLOY INFORMATION**, time and speed, if they were copied from programs in location 10-40 or saved as custom entries.

2. PROGRAMS 10-40

These program assignments are for use with the capsule products listed in **Section 10, REPLACEMENT PARTS AND ADDITIONAL PRODUCTS**.

NOTE: These programs may be modified by changing speed and/or time, as described in Section 3, Operation listed under 3C and 3D. The changed program may be RUN but not saved. If the modified program is to be retained it can be saved to a program number in the 1 to 9 group. Programs in this group are recalled by entering the two-digit program number and then START.

3. PROGRAM 00

The program location 00 is used to clear a program location from 01-09 and replace the information with a 0000 CPM and 0.0 second generic entry.

4. PULL-OUT CARDS

There are two pullout cards located at the back of the OptiMix unit. One card has a writing surface to allow the program information e.g. material, speed and time to be entered for reference. A second card shows program locations 10-40 containing a listing of the Kerr capsules identified by color, and the program # together with the material, speed and time.

C) SETTING MIXING TIMES AND SPEEDS USING MANUAL KEYBOARD ENTRY:

1. To manually set speed:

- a) Press SPEED key.
- b) Enter speed from 3000 to 4800 CPM.

Note: The last digit is automatically set to zero, so enter first three digits only.

2. To manually set time:
 - a) Press TIME key.
 - b) Enter the time from 05.0 to 30.0 seconds. The decimal point will be automatically assigned.

Note: The third digit must be a 0 or 5 (0.5-sec. increments).

3. Use + (plus) and – (minus) keys to change speed and time:

Speed and time changes may also be made by using the + (plus) and – (minus) keys to increment 50 CPM and 0.5 seconds.

- a) Press SPEED key.
- b) Hold down + (plus) or – (minus) key until the required speed is displayed.
- c) Press TIME key.
- d) Hold down + (plus) or – (minus) key until the required time is displayed.

D) SAVING OPTIONS:

To save the settings, press the SAVE key, a beep sounds and the two digit program number flashes. If the new settings are to be saved to the displayed program number, then the SAVE key should be pressed again. If the new settings are to be saved to another program number, then that program number should be entered and SAVED.

1. Saving preset programs 10-40 to location programs 01-09:
 - a) Enter desired program number 10-40.
 - b) Press the SAVE key, a beep sounds and the two digit program number flashes.
 - c) Enter the desired program number 01-09.
 - d) Press the SAVE key again.

2. Saving modified (speed, time) programs 10-40 to location programs 01-09:
 - a) Enter desired program number 10-40.
 - b) Change speed /time using manual entry or +/- (plus/minus) keys.
 - c) Press the SAVE key, a beep sounds and the two digit program number flashes.
 - d) Enter the desired program number 01-09.
 - e) Press the SAVE key again.
3. Saving a custom setting (speed, time) to location programs 01-09.
 - a) Enter desired program number 01-09.
 - b) Change speed/time using manual entry or +/- (plus/minus) keys.
 - c) Press the SAVE key, a beep sounds and the two digit program number flashes.
 - d) Press the SAVE key again.
4. Clearing saved programs 01-09:
 - a) Enter 00.
 - b) Press the SAVE key, a beep sounds and the two digit program number flashes.
 - c) Enter desired program 01-09 to be cleared.
 - d) Press the SAVE key again.

E) INVALID SETTINGS:

1. The OptiMix control is programmed to not allow invalid entries to be saved. For time entries, the entered values must be between 5 and 30 seconds and be of the format XX.X and 0X.X seconds. For speed entries, the entered values must be between 3000 and 4800 CPM and 3 digits.
2. If an INVALID entry is made, the display will show “invalid setting” and the incorrect time or speed will not be saved. It will be necessary to push the TIME (or SPEED) key and re-enter the required setting.

F) POWER DOWN:

When the OptiMix is powered down the last used SAVED program will be recalled when the unit is turned on. If a modified program is being used it should be SAVED before the OptiMix is turned off. Failure to SAVE a program will result in the program being deleted on power down.

G) DISPLAY SCREEN MESSAGES:

Lid Opened

Appears if the lid is opened while the machine is running and then the mixing cycle will stop as shown in (Figure 3G-1).

Mix Aborted

Appears if any key is pressed while the machine is running and then the mixing cycle will stop as shown in (Figure 3G-2).

Invalid Setting

Appears if the entered values for time or speed, or the format, are incorrect or outside of the allowed range. This message will also be displayed if an attempt is made to change a setting on a hard coded program location 10-thru 40 as shown in (Figure 3G-3).

Sensor Error

Appears when there is a loss of speed sensor signal as shown in (Figure 3G-4) and the mixing cycle stops. The machine should be sent in for service.

H) INTERNATIONAL LANGUAGE OPTIONS

To select another language, press the “Key #” listed below while turning the power switch on.

<u>Key #</u>	<u>Language</u>
0	English
1	French
2	German
3	Spanish
4	Italian
5	Portuguese

Note: The chosen language display is recalled each time the OptiMix is turned on.

4. TROUBLESHOOTING GUIDE

PROBLEM	SERVICE PROCEDURE
1. No display appears.	1a. Activate power switch on rear panel. 1b. Check for correct voltage setting. 1c. Check for fuse overload on rear panel. 1d. Check to see power cord is properly connected at machine and the outlet.
2. Unit does not run.	2a. Close door to mixing chamber. 2b. Incorrect power setting, check voltage setting. 2c. Microprocessor lock-up. Turn machine “OFF”, then turn “ON” power switch.
3. Unit runs at irregular speeds.	3a. Unit will regulate speed over a range of input voltage variation, if speed is irregular check line voltage setting. 3b. Lock up, turn machine “OFF”, then turn “ON” using switch on rear panel. 3c. If mix results become unstable, unit requires servicing.
4. Display freezes.	4a. Check for correct voltage settings. 4b. Microprocessor lock-up. Turn machine “OFF”, then turn “ON” switch on rear panel.

Kerr will, upon request, make available circuit diagrams, part lists, calibration instructions and other information which will aid appropriately qualified technical personnel to repair parts of the equipment classified as repairable.

5. FUSE REPLACEMENT

- a)  **CAUTION: Before attempting to change fuses turn the master power switch S1 “OFF” (O) and disconnect amalgamator unit from AC outlet (Mains).**
- b) Switch off master switch **S1** and unplug the line cord. Pry the fuse holder open with a small screwdriver as shown in Figure 5b-1. Remove the fuses from the holder as shown in Figure 5b-2 and check to see which fuse is blown. The thin wire in the center of the fuse will be broken or melted away if it is a defective fuse. Replace the fuse in the fuse holder and firmly push the fuse holder back into the power entry block.
- c) Replace fuses with same type and value:
- 115/120V: T 0.63A/250V, part no. 930847
230V: T 0.315A/250V, part no. 930846

6. CLEANING, CARE AND MAINTENANCE

 **CAUTION: Before cleaning unit turn master power switch “OFF” (O) and disconnect unit from AC outlet (Mains)!**

 **CAUTION: Do not allow cleaning solution to run into unit through the power switch, fuse door, or capsule arm opening. this may damage unit and/or cause electrical shock.**

 **CAUTION: Do not use a bleach solution to clean case, as this will yellow the surface finish.**

Clean the OptiMix case using a soft cloth and Cavicide cleaning solution. Follow recommendations on Cavicide container.

- Use care while wiping windows to avoid scratches.
- Your OptiMix has been lubricated at the factory and will not require lubrication for the life of the machine.
- Calibration of unit is not required as on some amalgamators.

7. SPECIFICATIONS AND EQUIPMENT CLASS

- | | |
|---|---|
| a) AC supply connection: | 115/120V~/0.5 A, 50/60 Hz
230V~/0.25A, 50/60 Hz |
| | Note: Mains supply voltage fluctuations are not to exceed 10% of nominal rated supply voltage |
| b) Power input: | 100 VA, max. |
| c) Equipment class: | I |
| d) ISO 7488 Amalgamator Class: | Type 2 Variable Frequency |
| e) Protection from ingress of liquids: | None |
| f) Operational Environment: | |
| • Ambient temperature | 59°F to 104°F (15° C to 40°C) |
| • Relative humidity | 10% to 100%, including condensation |
| • Atmospheric pressure | .5 atm to 1.0 atm (500 hPa to 1060 hPa) |
| | Table-top equipment for indoor use only |
| | Equipment not suitable for use in the presence of flammable anaesthetic mixture with air or nitrous oxide |
| g) IEC 1010 Installation category: | II |
| h) IEC 1010 Pollution Degree: | 2 |
| i) Transport and storage: | |
| • Ambient temperature | -40° F to 158° F (-40° C to 70° C) |
| • Relative humidity | 10% to 100%, including condensation |
| • Atmospheric pressure | .5 atm to 1.0 atm (500 hPa to 1060 hPa) |
| j) Operation: | Continuous operation with intermittent loading, duty cycle of 0.5 minutes on/2.0 minutes off |
| k) Fuses (2 per unit): | 115/120V: T 0.63A/250V, part no. 930847
230V: T 0.315A/250V, part no. 930846  |
| l) Unit overall dimensions: | |
| • Height | 5" |
| • Length | 10" |
| • Width | 7 5/8" |
| m) Unit weight: | 8.25 lbs. |
| n) Additional symbols: | |
| • Attention, consult accompanying documents |  |
| • OFF |  |
| • Protective Earth |  |
| • Alternating Current |  |
| • ON |  |

8. WARRANTY

Kerr hereby warrants that for a period of one year from the date of purchase this instrument shall be free from defects in material and workmanship and will perform satisfactorily under normal use and service.

THE WARRANTY STATED HEREIN IS THE SOLE WARRANTY APPLICABLE TO KERR PRODUCTS. KERR EXPRESSLY DISCLAIMS ANY AND ALL OTHER WARRANTIES EXPRESSED OR IMPLIED, INCLUDING WARRANTIES OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR USE. KERR'S LIABILITY WITH RESPECT TO ITS PRODUCTS IS EXPRESSLY LIMITED TO THE REMEDIES SET FORTH ABOVE. THE REMEDIES ARE THE BUYER'S EXCLUSIVE REMEDIES. KERR SHALL UNDER NO CIRCUMSTANCES BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES.

9. LIABILITY

Kerr considers itself responsible for the effects on safety, reliability and performance of this product only if:

- assembly operations, extensions, re-adjustments, modifications or repairs are carried out by persons authorized by Kerr.
- the electrical installation of the relevant room complies with the National and Local requirements.
- the equipment is used in accordance with these instructions for use.

10. REPLACEMENT PARTS AND ADDITIONAL PRODUCTS

Replacement Parts

930847 Fuse, 115/120V: T 0.63A/250V

930846 Fuse, 230V: T 0.315A/250V

Additional Products

Part Number	Tylin Package Description	Qty. per Pkg.	Capsule Color Coding	PROG. NO.
18962	Tylin Regular Set Single Spill (400 mg)	50 Jar	Blue Body/Lt. Blue Cap	#31
18965	Tylin Slow Set Single Spill (400 mg)	50 Jar	Lt. Blue Body/Lt. Blue Cap	#28
18968	Tylin Regular Set Single Spill (400 mg)	500 Jar	Blue Body/Lt. Blue Cap	#31
18971	Tylin Slow Set Single Spill (400 mg)	500 Jar	Lt. Blue Body/Lt. Blue Cap	#28
18963	Tylin Regular Set Double Spill (600 mg)	50 Jar	Blue Body/Gray Cap	#32
18966	Tylin Slow Set Double Spill (600 mg)	50 Jar	Lt. Blue Body/Gray Cap	#29
18969	Tylin Regular Set Double Spill (600 mg)	500 Jar	Blue Body/Gray Cap	#32
18972	Tylin Slow Set Double Spill (600 mg)	500 Jar	Lt. Blue Body/Gray Cap	#29
18964	Tylin Regular Set Triple Spill (800 mg)	50 Jar	Blue Body/White Cap	#33
18967	Tylin Slow Set Triple Spill (800 mg)	50 Jar	Lt. Blue Body/White Cap	#30
18970	Tylin Regular Set Triple Spill (800 mg)	500 Jar	Blue Body/White Cap	#33
18973	Tylin Slow Set Triple Spill (800 mg)	500 Jar	Lt. Blue Body/White Cap	#30
20391	Tylin Regular Set Double Spill (600 mg) Trial Package (with Activator)	6 Capsules	Blue Body/Gray Cap	#32

Additional Products

Part Number	Contour Package Description	Qty. per Pkg.	Capsule Color Coding	PROG. NO.
16574	Contour Fast Set Single Spill (400 mg)	50 Jar	Green Body/Lt. Blue Cap	#10
16568	Contour Regular Set Single Spill (400 mg)	50 Jar	Rust Body/Lt. Blue Cap	#13
16565	Contour Fast Set Single Spill (400 mg)	500 Jar	Green Body/Lt. Blue Cap	#10
16559	Contour Regular Set Single Spill (400 mg)	500 Jar	Rust Body/Lt. Blue Cap	#13
16573	Contour Fast Set Double Spill (600 mg)	50 Jar	Green Body/Grey Cap	#11
16571	Contour Regular Set Double Spill (600 mg)	50 Jar	Rust Body/Grey Cap	#14
16567	Contour Fast Set Double Spill (600 mg)	500 Jar	Green Body/Grey Cap	#11
16563	Contour Regular Set Double Spill (600 mg)	500 Jar	Rust Body/Grey Cap	#14
18239	Contour Fast Set Triple Spill (800 mg)	50 Jar	Green Body/White Cap	#12
18238	Contour Regular Set Triple Spill (800 mg)	50 Jar	Rust Body/White Cap	#15
18241	Contour Fast Set Triple Spill (800 mg)	500 Jar	Green Body/White Cap	#12
18240	Contour Regular Set Triple Spill (800 mg)	500 Jar	Rust Body/White Cap	#15
20390	Contour Fast & Regular Set Double Spill (600 mg) Trial Package (with Activator)	3 Capsules	Fast Set - Green Body/Grey Cap	#11
		3 Capsules	Regular Set - Rust Body/Grey Cap	#14

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