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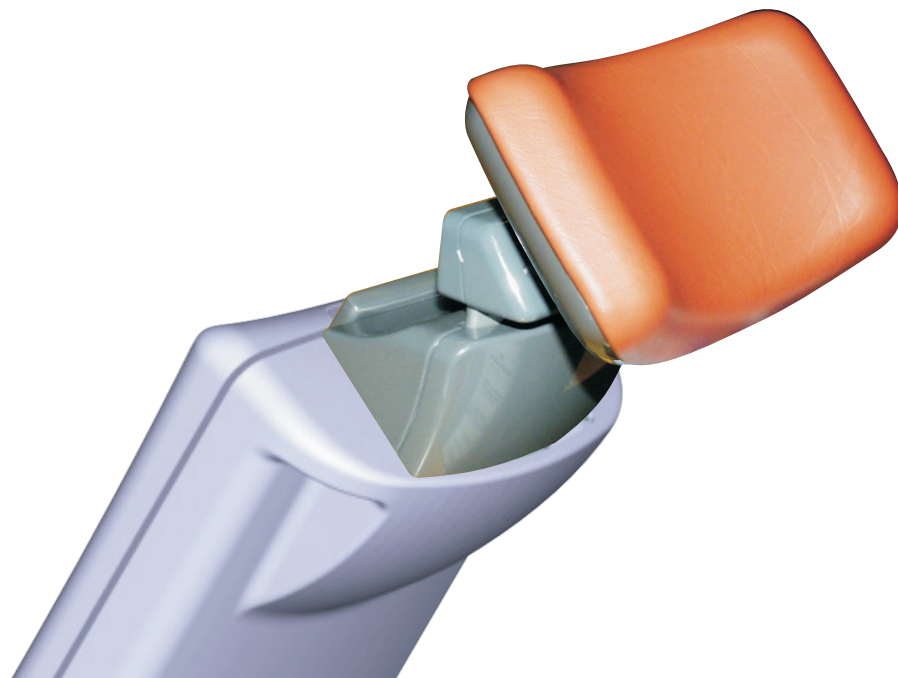
English

# Contents

|          |                                                                          |          |
|----------|--------------------------------------------------------------------------|----------|
| <b>1</b> | <b>Warning and safety information .....</b>                              | <b>4</b> |
| <b>2</b> | <b>Technical description .....</b>                                       | <b>5</b> |
| <b>3</b> | <b>Installing the Neckrest Insert for MultiMotion .....</b>              | <b>6</b> |
|          | 3.1 Replacing MultiMotion with the neckrest insert for MultiMotion ..... | 6        |
| <b>4</b> | <b>Operation .....</b>                                                   | <b>7</b> |
|          | 4.1 Patient positioning .....                                            | 7        |
| <b>5</b> | <b>Care and cleaning instructions for the practice team .....</b>        | <b>8</b> |

# 1

## Warning and safety information

---

### Intended use

The MultiMotion neckrest insert is used for positioning the head of a patient sitting in a wheelchair during diagnosis, therapy and dental treatment by properly trained personnel.

The neckrest is designed exclusively for use on Sirona treatment centers.

### Identification of warning and safety information

To avoid personal injury and material damage, you must also observe the warning and safety information provided in the present operating instructions. All such information is highlighted by the signal words  **NOTE**,  **CAUTION** or  **WARNING**.

### Maintenance and repair

As manufacturers of dental medical equipment and in the interest of the operational safety of your system, we stress the importance of having **maintenance and repair** of your treatment center performed only by ourselves or by agencies expressly authorized by us. Furthermore, safety-critical system components must always be replaced with **original spare parts** upon failure.

When having such work done, we suggest that you request a certificate stating the type and extent of work performed, including information about any modifications of the rated parameters or of the operating ranges (if applicable), as well as the date, name of organization and signature.

### Modifications to the system

Modifications to this system which might affect the safety of the system owner or are prohibited by law!

For reasons of product safety, this product may be operated only with original Sirona accessories or third-party accessories expressly approved by Sirona. The user assumes the risk of using non-approved accessories.

If any equipment not approved by Sirona is connected, it must comply with the applicable standards.

## 2 Technical description

---

### Model designation

Neckrest Insert for MultiMotion

### Operating conditions

Ambient temperature: 10 °C – 40 °C (50 °F – 104 °F)  
Relative humidity: 10 % – 85 %  
Air pressure: 700 hPa – 1060 hPa

### Transport and storage conditions

Temperature: -40 °C – +70 °C (-40 °F – 158 °F)  
Relative humidity: 10 % – 95 %  
Air pressure: 500 hPa – 1060 hPa

### Tests/approvals

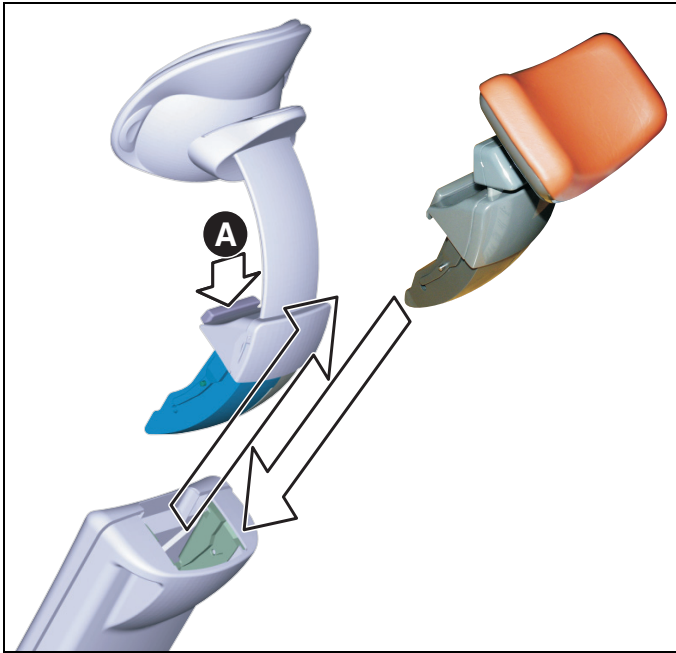
Neckrest insert for MultiMotion complies with the requirements of  
**IEC 60601-1: 1998 + A1: 1991 + A2: 1995 and the ISO 6875: 1995**

In connection with Sirona treatment centers, the Neckrest Insert for MultiMotion complies with the provisions of the Council Directive 93/42/EEC of June 14, 1993 concerning medical devices.

# 3 Installing the Neckrest Insert for MultiMotion

## 3.1 Replacing MultiMotion with the neckrest insert for MultiMotion

### Removing the MultiMotion

For certain treatments (e. g. of wheelchair patients), it is expedient for better patient positioning to remove the MultiMotion completely and insert the neckrest.

To remove the MultiMotion, proceed as follows:

1. Press release button **A**.
2. Pull the complete headrest out of the headrest extension.

### **i** NOTE

*Deposit the removed MultiMotion in a safe place where it cannot fall onto the floor.*

### Installing the neckrest insert for MultiMotion

- Check the guide of the removed headrest for contamination. Remove any contamination.
- Insert the neckrest in the guide from above until it audibly locks in place.

**Pull on the neckrest insert for MultiMotion again to make sure that it is locked securely in place.**

# 4 Operation

## 4.1 Patient positioning

Position the patient from the back side of the patient chair so that the back of the head lies on the pad of the neckrest.

Adjust the patient chair correspondingly to allow this.

### **CAUTION**

*Make sure that the patient lies securely on the neckrest during the treatment, e.g. by applying the locking brakes on the wheelchair.*

# 5

## Care and cleaning instructions for the practice team

---

---

### Care and cleaning agents

#### **CAUTION**

*Use only cleaning and care agents which are approved by Sirona!*

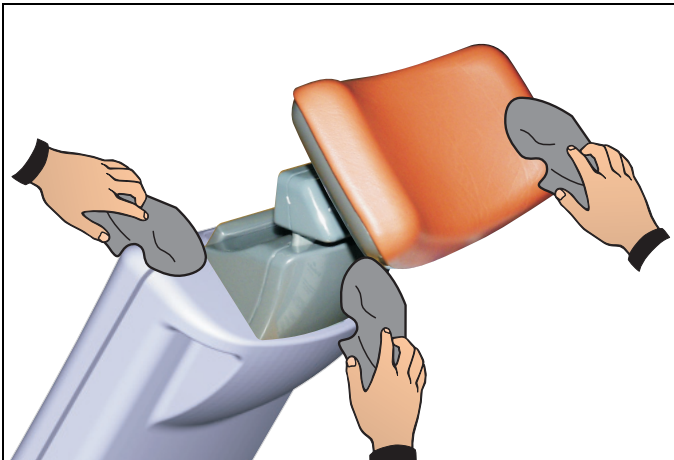
---

You will find a list of all agents approved at the time of delivery in the documentation folder supplied with your treatment center.

A continuously updated list of approved agents can be downloaded from the Sirona Internet homepage under **REF 59 70 905**: ⇒ **www.sirona.com**

If you do not have Internet access, you can use one of the following two ways to order the list:

- Order from your local dental depot
- Order from Sirona  
Tel: ++49 (0) 62 51/16-16 16  
FAX: ++49 (0) 62 51/16-18 18

---

### Disinfecting the surfaces

The surfaces can be spray and wipe disinfected with surface disinfectants.

---

We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH 2007
**Sirona Dental Systems GmbH**
Sirona Dental Systems LLC