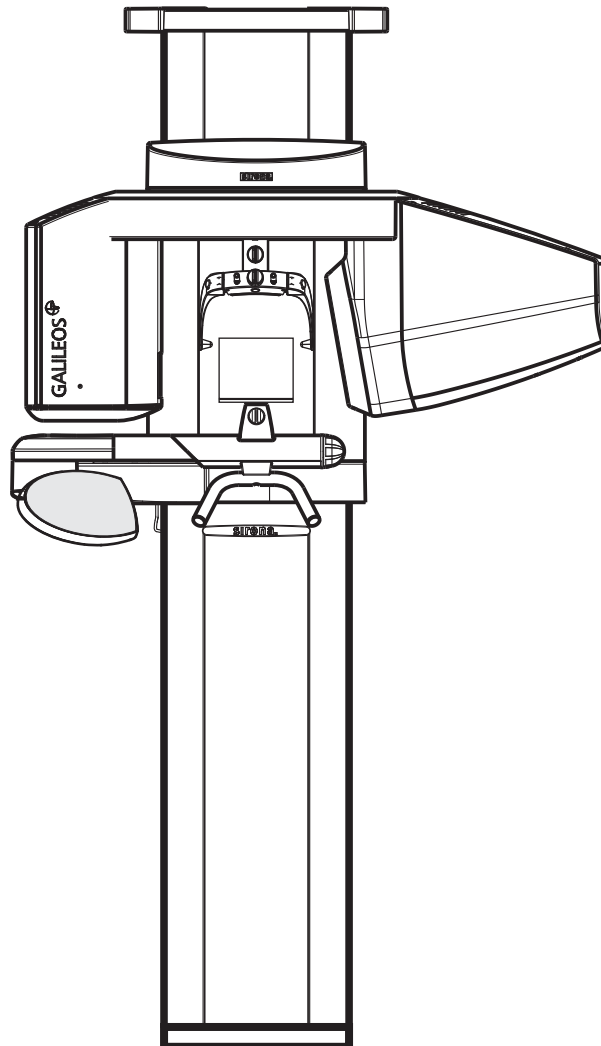
GALILEOS

Gebrauchsanweisung Schablonenhalter
Operating Instructions for the Template Holder
Notice d'utilisation du porte-guide
Manual del operador del portaplantillas
Istruzioni d'uso per supporto maschera

Neu ab:

03.2013

sirona
The Dental Company

GALILEOS

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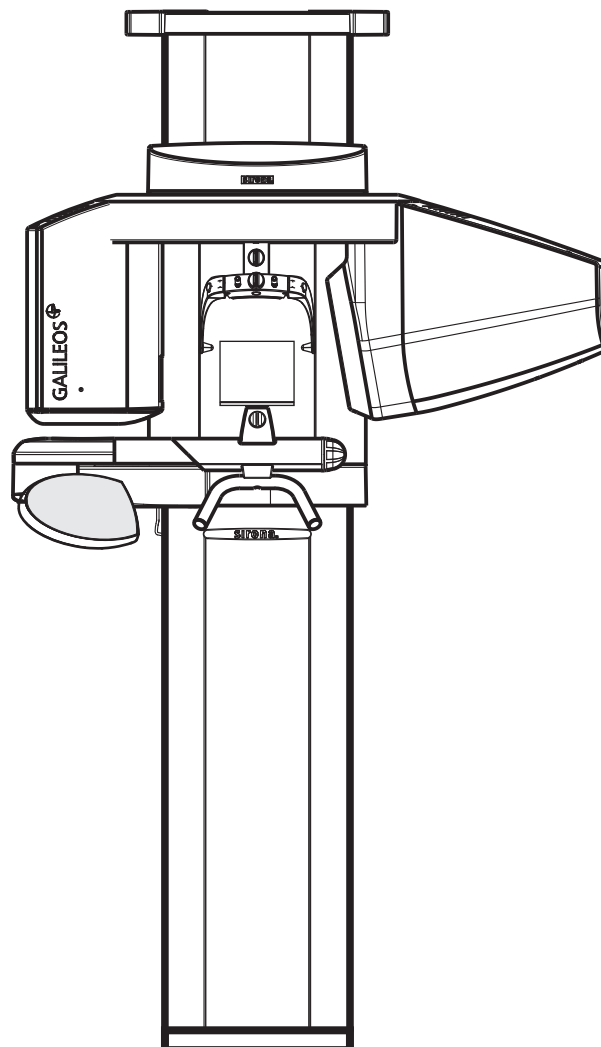
03.2013

GALILEOS

Operating Instructions for the Template Holder

English

1 General Information on the Operating Instructions for the Template Holder

Dear Customer,

Thank you for purchasing your **GALILEOS** X-ray system and the **template holder**.

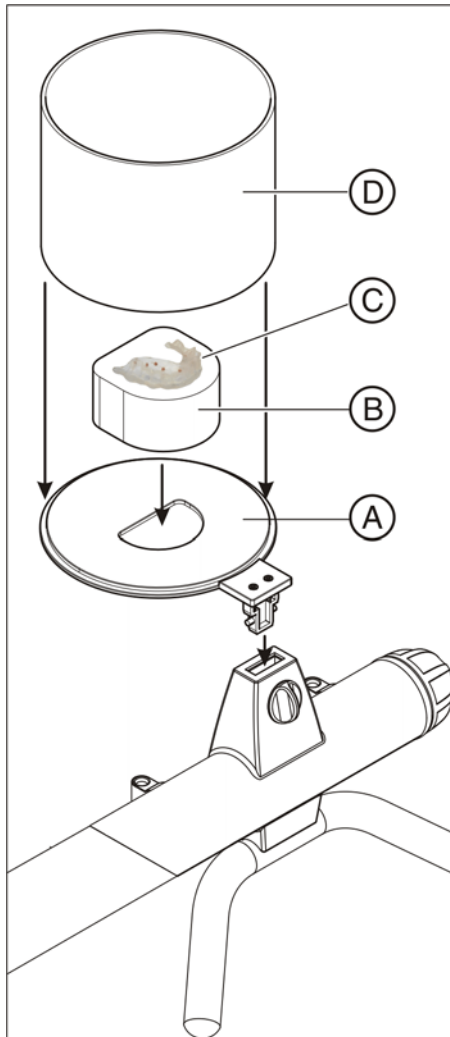
The **technical documentation** is also part of the product. Always keep this documentation within easy reach.

The present document contains the "**Operating Instructions for the Template Holder**".

Please also refer to the "**GALILEOS Operating Instructions**", as well as any warning and safety information contained therein.

Your **GALILEOS** Team

2 Exposures using the Nobel Template Holder

A	Support plate (included in delivery)
B	Foam support (included in delivery)
C	X-ray template, customized for each patient
D	Aluminum cylinder (included in delivery)

- ✓ To perform implant planning using the NobelGuide software from Biocare, two exposures are required.
- 1. Take an exposure of the patient using the X-ray template provided for this purpose (see the GALILEOS Operating Instructions and NobelGuide documentation).
We recommend the following acquisition parameters when working with NobelGuide:
Patient scan
GALILEOS Compact: V04 HC, 35mAs or 42mAs for adults
GALILEOS Comfort: V01 HC, 35mAs or 42mAs for adults
GALILEOS Comfort PLUS: V05 HD HC, 30mAs for adults
- 2. The template holder is required for the second exposure, during which only the X-ray template should be scanned.

Install the template holder as follows:

- ✎ Push the patient fixing system fully upward and remove the head band or the entire patient fixing system, so that you have enough space to position the template holder.
- ✎ Insert the support plate (**A**) of the template holder up to the stop and lock it using the control knob, as shown in the figure.
- ✎ Position the foam support (**B**) on the cutout of the support plate.
- ✎ Position the X-ray template (**C**) on the foam support, in line with the position of the patient's jaw.
- ✎ For GALILEOS Compact/Comfort:
Place the aluminum cylinder **D** over the X-ray template and the foam support.
NOTICE! The aluminum cylinder must be situated in the groove of the support plate.
- ✎ For GALILEOS Comfort PLUS:
The aluminum cylinder (**D**) must not be used.

3. Now perform the second scan.
We recommend the following acquisition parameters when working with NobelGuide:
Template scan:
GALILEOS Compact: V04 HC, 42mAs
GALILEOS Comfort: V01 HC, 42mAs
GALILEOS Comfort PLUS: V0G (volume program for template scan)
 4. Export both data sets for NobelGuide using the SIDEXIS XG "Volumes for third-party manufacturers" function and the Nobel profile.
- ↪ Both scans are now in DICOM format for further processing in the NobelGuide software from Biocare.

IMPORTANT

The holder can and should also be used, when you wish to take exposures of similar objects (e.g. bone structures) using GALILEOS.

We reserve the right to make any alterations which may be required due to technical improvements.

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© Sirona Dental Systems GmbH 2013
Sirona Dental Systems GmbH