Angelcare®

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AC401 DELUXE PLUS

OWNER'S MANUAL

KEEP INSTRUCTIONS FOR FUTURE REFERENCE.

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






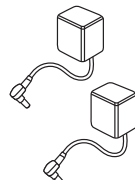
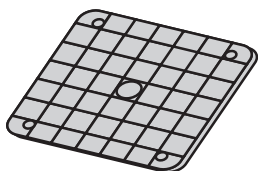
AC401-2P DELUXE

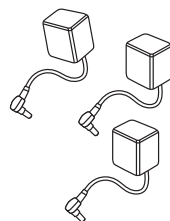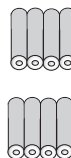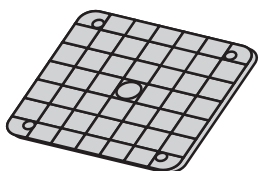AC401 DELUXE PLUS

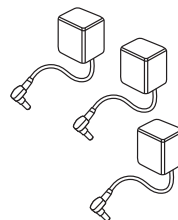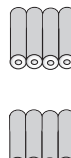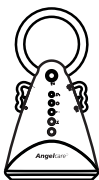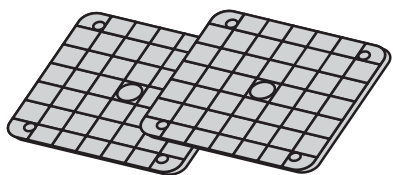If your monitor includes 2 parents units, you only need to program one parents unit. The second will automatically pick up the same settings when turned ON.

If your monitor includes 2 sensor pads, you can use either one or both sensor pads. Please refer to Step 1 for installation.

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**E
N**

Owner's manuel

Keep instructions for future reference

WARNING

Failure to follow these warnings and the assembly instructions could result in serious injury or death.

This product is designed to aid in the monitoring of your child. When using the monitor, you must still provide the supervision necessary for the continued safety of your child. Always be sure that both the transmitter and receiver are working properly and are within range of each other.

- **STRANGULATION HAZARD.** Keep the adapter cords out of the reach of children. Do not use with extension cords.

- **DO NOT** place the Nursery Unit or its cord within your child's reach. Do not put the Nursery Unit inside a crib or playard.

- **Use ONLY** with 110-120 volt AC electrical outlets.

- Protect the AC adapter cords. Place them so they are not walked on or pinched by furniture or other items.

- Allow for proper ventilation when units are in use. Do not cover the Nursery Unit or Parent Units with any object such as a blanket. Do not place it in a drawer or in any location which would muffle the sound or interfere with the normal flow of air.

- **DO NOT** immerse any part of the Nursery Unit, Parent Units, or docking station in water. Clean only with a dry cloth.

- **DO NOT** place unit near water and moisture. Do not use the Nursery Unit outdoors. Do not use the Nursery Unit near possible wet areas, such as a bathtub, shower, wash basin, sink, laundry tub, swimming pool, wet basement, etc.

- Keep Nursery Unit away from heat sources, such as fireplaces, radiators, stoves, and TVs or other appliances. Heat can damage the case or electrical parts.

WARNING: This product is not a toy.

- Do not use the Nursery Unit if:
 - The adapter cords or plugs have been damaged.
 - The unit has been exposed to liquids.
 - Unit has been dropped or is damaged.

- **DO NOT OPEN THE NURSERY UNIT, PARENT UNIT, PARENT UNIT BASE OR ADAPTERS.** No user-serviceable parts inside. Risk of electrical shock, fire or death.

- The antenna used for this transmitter must be installed to provide a separation distance of at least 20 cm (7.9") from all persons and must not be co-located or operating in conjunction with any other antenna or transmitter.

IMPORTANT: You must use only rechargeable batteries in the Parents' Unit. **DO NOT** use alkaline batteries or you will severely damage the unit. Charge the batteries for 15 hours prior to first use. Be sure to have the monitor in the OFF position during this period. Only use the AC adapters included with your monitor.

DO NOT use these adapters with any other equipment.

- When unplugging the AC adapter from the nursery unit, be sure it is switched to the off position. If it isn't, the unit will revert to battery mode if batteries are installed.

- You can use rechargeable batteries in the Nursery Unit, however, they will not be recharged by plugging in the AC adapter. To recharge, remove the batteries and use an external charger.

- This product is **NOT** a substitute for direct supervision of your baby. Check your baby's activities at regular intervals. Monitoring of premature babies, or those considered to be at risk, should always be carried out under the supervision of a health professional.

- Always place both the Nursery Unit and Parents' Unit on a flat surface in an upright position and allow for proper ventilation.

- Avoid damage to the Sensor Pad cord and to the adapter cords. Route cords so that they are not likely to be walked on or pinched by items placed on or against them.

- Route sensor pad cord only below crib mattress and out of reach of children.

- Use **ONLY** AAA (LR03) rechargeable batteries and/or the AC adapters provided for the Parents' Units.

- During long periods of non-use remove all batteries from their compartments and unplug the AC adapters.

- This monitor uses public airwaves to transmit signals. The Parents' Unit may pick up signals or interference from other nursery monitors in your area. Your monitor may even be picked up by other households. To protect your privacy, make sure both units are turned off when not in use. If you experience interference, change the channel on both units.

- When the monitor is in use, do not use a crib mobile or other crib accessories that may produce vibrations. Other sources of vibrations may include fans, washing machines, loud music, etc. These sources of vibration may interfere with the performance of your Angelcare® monitor and the alarm will not sound if it continues to detect any movement other than your baby's. Be sure all vibration sources are eliminated before using the monitor.

- You may experience false alarms for several reasons. The most common reason is the Nursery Unit has not been turned off after you have removed your baby from the crib. Another reason is the Sensor Pad has not been placed under the mattress properly. Be sure the Sensor Pad is on a flat, firm surface with the printed side facing up. The sensitivity level may also need to be adjusted. Refer to the Movement Sensitivity Adjustment section on P 9.

- Always be sure to immediately check on baby whenever an alarm sounds.

- Your monitor can be used in any crib that is stationary and designed to safely hold a baby. If your crib has a spring or slat base you will need to place a piece of Masonite board under the Sensor Pad. This support must be at least 13 inches long on all four sides and 1/4 inch thick. Only use your Angelcare® monitor in a stationary sleeping environment such as a crib where the Sensor Pad can rest on a completely flat, firm surface.

FC WARNING

Modifications not authorized by the manufacturer may void users authority to operate this device.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

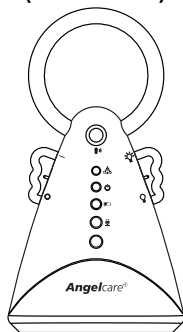
- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:


- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

YOUR ANGELCARE® MONITOR COMPONENTS

NURSERY UNIT (TRANSMITTER)

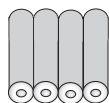
RECHARGEABLE PARENTS UNIT (RECEIVER) & CHARGER


4 AAA RECHARGEABLE
BATTERIES FOR
PARENTS UNIT

YOU WILL ALSO NEED :

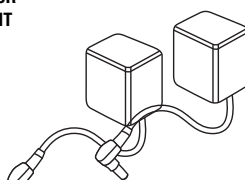4 AAA non-rechargeable batteries(not included) as back up for the nursery unit in case of power failure.


2 (Two) 7.5 volt AC adapters

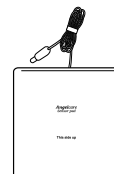
SENSOR PAD

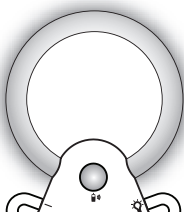
FEATURES


Practical soothing
night light.

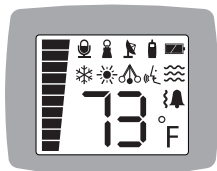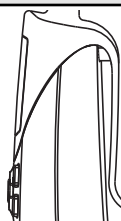Choice of voice activated or
continuous sound reception.


Convenient belt clip.

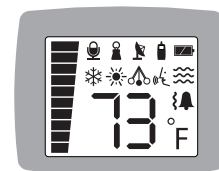
Nursery room temperature
display on the parents unit.

ALARM AND WARNING

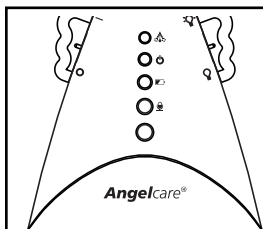Your Angelcare® monitor will let you know if no movement is detected by alarms, vibration and visual effects. The practical portable parents unit keeps you informed at all time.

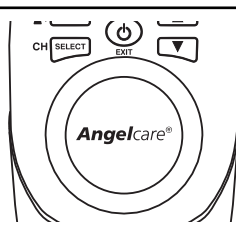
An alarm is triggered on the
nursery unit and the flashing
pendulum green light
stops flashing.

An alarm is triggered on the
parents unit.

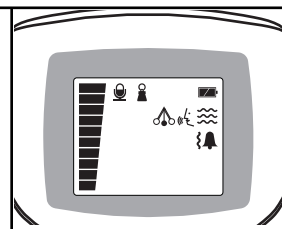Vibration on the parents
unit (not available in
some countries).

The parents unit lights up
and the nursery unit icon
appears.

Understanding the Symbols

Symbol	Name	Meaning
	Nursery Unit	Appears when options are being set for the Nursery Unit. Appears when no movement is detected. Flashes when Nursery Unit is paging Parents' Unit.
	Nursery Unit and No Channel	Nursery Unit flashes and "--" is displayed when Parents' Unit is on but Nursery Unit is off.
C3	Channel	Indicates to which channel the monitor is set.
73°F	Temperature Display	Indicates temperature of nursery.
	Parents' Unit	Appears when options are being set for the Parents' Unit.
	Sound	Appears when Sound feature is selected. Flashes when the Parents' Unit is temporarily on mute.
	Movement	Appears when Movement feature is selected.
	• "Tic" • Movement Indicator	Appears when "Tic" feature is selected. Swings when movement is detected regardless of whether or not "Tic" is on.
	Out-of-Range	If Activated • Out-of-Range symbol appears. • Appears when Nursery and Parents' Units are within range. • Flashes when they are out of range. • Flashes when Nursery Unit shuts down due to low battery. • Flashes when the channel is being changed. • Flashes when Nursery and Parents' Units are reconnecting. Symbol will flash even if feature is not activated when: • The channel is being changed. • The Nursery and Parents' Units are reconnecting.

Symbol	Name	Meaning
	Voice Activation	Appears when Voice Activation is on. Flashes when its sensitivity level is being set.
	Sound Alarm	Appears when Sound alarm is on.
	Vibration Alarm	Appears when Vibration alarm is on.
	Low Temperature	Appears when Low Temperature alert is on. Flashes when nursery temperature is below set temperature range.
	High Temperature	Appears when High Temperature alert is on. Flashes when the nursery temperature is above set temperature range.
	Battery Level	Indicates battery level of Parents' Unit. Flashes when Parents' Unit is being charged.
	Battery and Nursery Unit	Nursery Unit symbol appears and Battery symbol flashes when the battery level in the Nursery Unit is low.
	Level Bar	Sound Lights indicating baby's movement. Displays volume level of speaker in Parents' Unit when setting up. Displays Voice Activation sensitivity level when setting up.



Summary of Alarms

Display	Alarm	What It Means
	Continuous beeping and/or vibration	Sensor Pad does not detect movement.
	Two beeps sound every 8 seconds.	Parents' Unit is out of range of the Nursery Unit or the Nursery Unit is off.
	Three beeps sound every 4 seconds.	Nursery temperature is below the set temperature range.
	Three beeps sound every 4 seconds.	Nursery temperature is above the set temperature range.
	A short beep sounds every 15 seconds.	Battery level is low on Parents' Unit.
	A short beep sounds every 15 seconds.	Battery level is low on Nursery Unit.
	Continuous beeping.	Nursery Unit is paging the Parents' Unit.

HOW TO OPERATE YOUR ANGELCARE® MONITOR

STEP 1

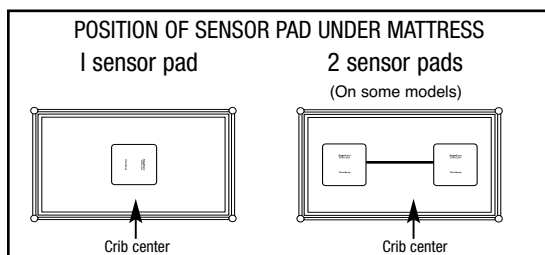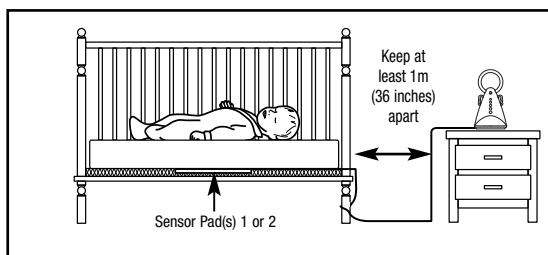INSTALLATION OF THE SENSOR PAD UNDER THE MATTRESS


- Position the sensor pad as shown above. Make sure the printed side faces up. Do place any bedding between the mattress and the sensor pad.
- Secure the sensor pad cord to crib using a tie wrap. **Run the cord on the floor to ensure it is out of the baby's reach.**
- The sensor pad must rest on a completely flat and rigid surface. Place a 6mm (1/4") thick plywood board to cover the full size of the crib or with minimum dimensions of 28cm (11") X 28cm (11") X 6mm (2") between the crib base and mattress, when using one sensor pad. The plywood will push the sensor pad into the mattress and keeps the contact between the sensor pad and the mattress, should the baby move around. Make sure the sensor pad's cord passes underneath the mattress as shown above.
- The sensor pad works with all baby bed sizes and all mattresses, except memory foam mattresses.. You will have to test your sensor pad on STEP 13.

STRANGULATION HAZARD. Keep the adapter cord and the sensor pad cord out of the reach of children. Because of the serious strangulation risk to infants and toddlers, parents and caregivers **should never place the nursery unit within 3 feet (1 metre) of a crib.**

STEP 2

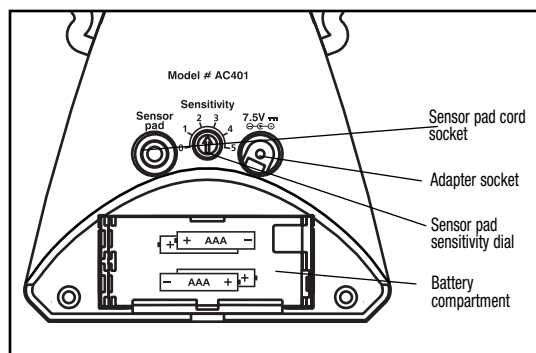
INSTALLATION OF THE NURSERY UNIT


- Connect the long sensor pad cord into the nursery unit.
- Insert 4 AAA regular batteries (not included) into nursery unit (for battery backup in case of power failure).
- Connect AC adapter to nursery unit and plug adapter into an electrical outlet in the baby's room.
- After both units are properly installed you may adjust the sensor pad sensitivity level. The sensitivity dial shows the level 1 to 5, where 1 is less sensitive and 5 very sensitive. It is preset between level 2,5 and 3. Should there be a false alarm, increase the sensitivity by a half level during testing until it works perfectly for your mattress and bed size.

STEP 3

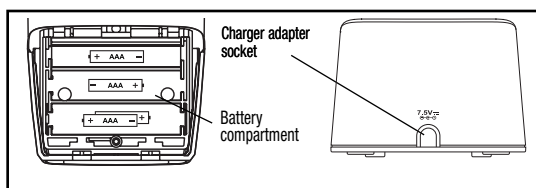INSTALLATION OF THE PARENTS UNIT

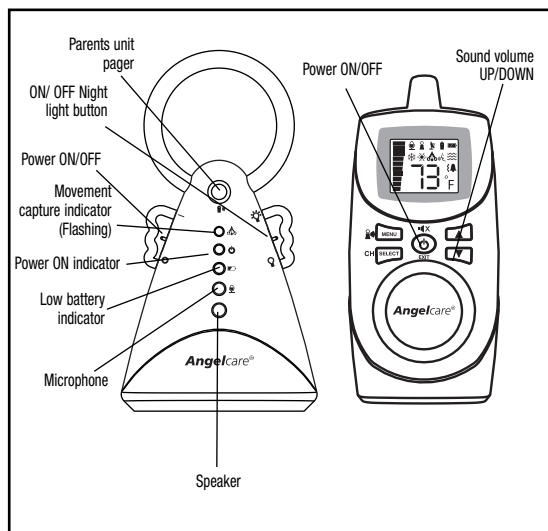- Insert 4 AAA NIMH rechargeable batteries (included) into parents unit. NEVER use regular alkaline batteries, they may explode.
- Connect AC adapter to parents unit charger and plug adapter into an electrical outlet. Let charge for 15 continuous hours before using the monitor for the first time.


STEP 4**PAIRING THE PARENTS UNIT AND THE NURSERY UNIT**

- Once the parents unit is fully charged (after 15 hours) turn the unit ON and turn the nursery unit ON.


If the parents unit is turned ON and the nursery unit is OFF, the parents unit will start beeping after 1 minutes. After 4 seconds, the unit will shut itself off.

Note: If you lost connection refer to Step 12 "Connecting the Parents unit to the Nursery unit"

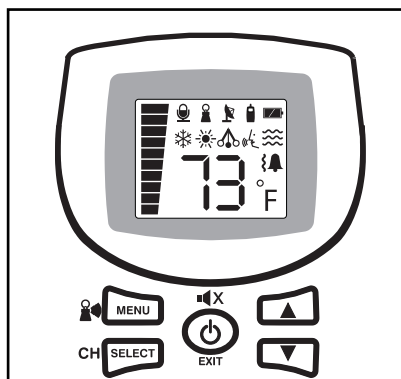- The available options are programmable but you can use the units right away, with the following factory settings :**
 - Sound, movement and tic activated
 - Out of range notification ON
 - Voice activated OFF
(This function is ON in Switzerland, Germany and Austria)
 - Temperature display ON, Room temperature control OFF
 - Parents unit alarm ON, vibration OFF
 - Speaker volume at level 3
 - Voice sensitivity at level 2
- To avoid feedback (whistling), keep both units at least 3 meters apart (10 feet).
- You can adjust the sound volume with the arrow buttons.
- To turn the night light ON, move the right wing button UP.

STEP 5**SCREEN AND BUTTONS**

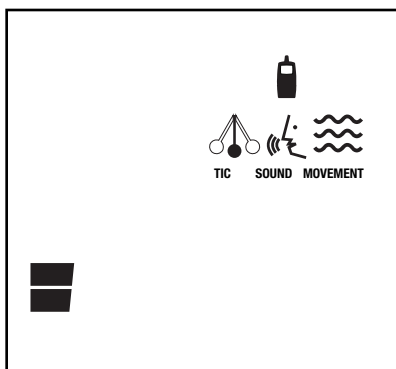- All programmable functions of your Angelcare® monitor are available through the MENU button.
- Note:** Press the MENU button briefly to access the menu. If you press the MENU button too long you will get in the pairing mode and the out of range indicator will start flashing. If this happens, wait 20 seconds and then press MENU to start again.
- When the desired option appears on your screen, press SELECT.
The different choices of each option is available through the UP and DOWN arrows.
- AT ANY TIME after programming a function, if you wish to exit the menu selection, press on the EXIT button. If you wish to perform additional programming press MENU to view the next option.**

STEP 6**TICS, SOUNDS AND MOVEMENTS**

Your Angelcare® monitor gives you the possibility to chose what you wish to monitor.

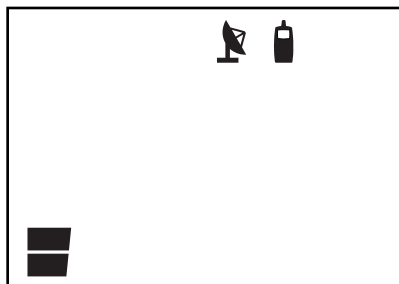
- The "TIC" is a sound that tells you that your sensor pad is detecting movements.
- The "SOUND" function is activated by the baby's voice or is transmitted continuously (see STEP 8)
- The "MOVEMENT" function is activated when movement is detected by the sensor pads. It detects all baby's movements and warns you when there is no movement.

A CONTINUOUS BEEPING SOUND is heard when no movement is detected for 20 seconds.

If you are receiving false alarms, adjust the sensor pad sensitivity level (STEP 2).

- Press MENU, the 3 flashing icons will appear.
- Using the arrows, select each icon, and decide if you want it ON or not.
- When your selection is made press SELECT, and it will bring you to the next icon.
- Press MENU to view the next option, or EXIT if you wish to stop programming now.

STEP 7 OUT OF RANGE INDICATOR


Your monitor is equipped with an optional "Out of range" indicator function, which will advise you when the two units are not communicating with each other. Out of range situations can be caused by distance or by physical interferences.

Range: up to 250 meters/ 820 ft (open field); between 80 meters/ 262 ft and 100 meters/328 ft (indoors).

When the units are out of range you will hear a DOUBLE BEEP every 8 seconds and the "out of range" icon will flash on the screen of the parents unit.

- Press MENU, the OUT OF RANGE icon and the word ON will appear on the screen.
- Turn this function ON or OFF by using the arrows.
- Press SELECT
- Press MENU to view the next option, or EXIT if you wish to stop programming now.

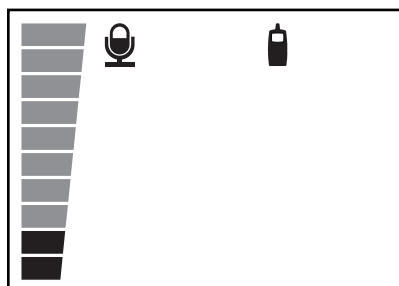
STEP 8 MICROPHONE SENSITIVITY

The sound monitoring can be "voice activated" or "continuous". If you choose the continuous transmission mode, you will hear your baby's every little sounds. If you use "voice activated" you will hear sounds depending on the level of sensitivity that you choose.

- Press MENU, the MICROPHONE and the word ON will appear on the screen.
- Leave it ON to keep it "voice activated" or OFF to be in "continuous" mode. Use the arrows to select On or OFF.
- Press SELECT.
- If you wish to use the voice activated monitoring function, Press MENU again to choose the level of sensitivity of the microphone (4 levels), using the arrows.
- Press SELECT when the desired level appears on the screen.
- Press MENU to view the next option, or EXIT if you wish to stop programming now.

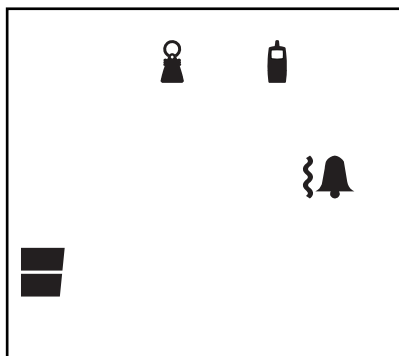STEP 9 ALARMS AND VIBRATION (VIBRATION NOT AVAILABLE IN SOME COUNTRIES)

This menu will appear ONLY if the vibration option is available on your unit. It allows you to choose between the following notification options :

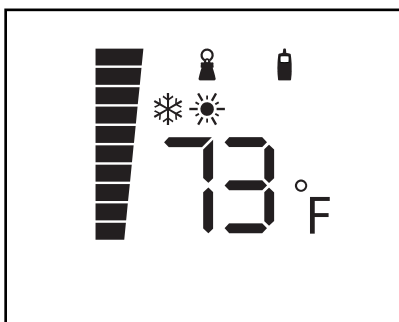
1. Alarm only
2. Vibration only
3. Alarm and vibration

If selected, the vibration will work for the following alarm functions: 1) when the units are out of range (if out of range function activated); 2) when minimum or maximum temperature level is reached (if this function is activated); 3) when there is no movement for 20 seconds (if movement function is activated)

NOTE : The vibration mode will not work when the batteries are low or if the parents unit is on the charger.

- Press MENU and choose the desired option. The wave icon is for the vibration mode and the bell icon is for the alarm mode.
- Select the desired option, using the arrows. Press SELECT.
- Press MENU to view the next option, or EXIT if you wish to stop programming now.


STEP 10 ROOM TEMPERATURE CONTROL

Your Angelcare® monitor can also monitor your baby's room temperature and let you know when the temperature is too high or too low. A triple BEEP will be heard if the temperature reaches levels out of the chosen range.

- Press MENU, the snowflake and sun icons and the word OFF will appear on the screen.
 - Turn it ON or OFF using the arrows.
 - Press SELECT.
 - If you select to have the room temperature control ON, Press MENU.
- The SNOW FLAKE icon will appear, with the temperature. Select the desired lowest temperature with the arrows and press SELECT.
- Press MENU. The SUN icon will appear, with the temperature. Select the desired highest temperature with the arrows and press SELECT.
 - Press EXIT as this was the last available programmable function.

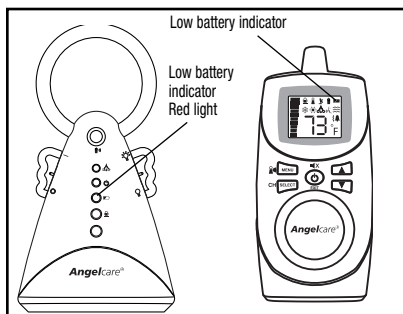STEP 11 BATTERY MODE AND WARNINGS


- The monitor will revert to battery mode during a power failure only if batteries are inserted in **both** units.
- It is also possible to use the Angelcare® monitor in places where electricity is not available.
- When the nursery unit's low battery RED light is flashing, its batteries need to be replaced. On the screen of the parents unit, the battery icon flashes and the nursery unit icon appears. A short BEEP will also be heard every 15 seconds for about 30 minutes, until the unit shuts itself off.
- Parents unit should be recharged when icon is clear (will not flash) and a beep sound is heard. A short BEEP sound will be heard every 15 seconds until the unit shuts itself down.

Note: Do not use Alkaline batteries in the parents unit.

IMPORTANT : If the "Out of range" function is ON, when the nursery unit shuts itself off, the "Out of range" signal will be heard and displayed on the parents unit.

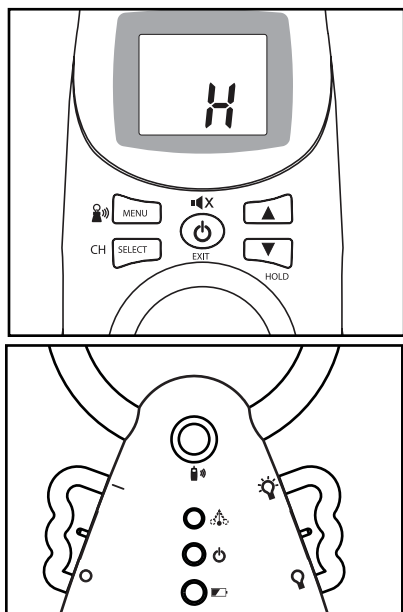STEP 12 ADDITIONAL FUNCTIONS

• CHANNEL SELECTION:

If the transmission is not clear, or if you hear parasite sounds, you should change the channel. Press and hold the SELECT button to change the channel (with the arrows). Then press the SELECT button again to confirm.

• TEMPORARY MUTE:

If you want to mute the sound for 2 minutes, press the center EXIT button. To reactivate the sound before the 2 minutes have elapsed, press EXIT again. You can also turn the alarm off by pressing the same button (this does not apply to the low battery alarm).

• HOLD:

Press and hold the down volume arrow for 2 second. This symbol "H" will appear on the parents unit display. This function allows you to put the monitor on Hold when you need to pick the baby out of the crib for a short period of time. During this period, the movement alarm will not go off, nor the sound will be transmitted. A double beep will be heard every minute to let you know the monitor is on Hold. To void this function, press and hold the down volume arrow for 2 second. The display will revert to normal.

Note: When there is a movement alarm, press any key to put the monitor on HOLD. During SOUND ONLY mode, the HOLD function is not active.

• CONNECTING THE PARENTS UNIT TO THE NURSERY UNIT:

If you lose the connection between the 2 units, turn them both off. Turn ON the parents unit and press and hold the MENU button, the "out of range" indicator will flash and you will hear a beep. Turn ON the nursery unit and press and hold the top button of the nursery unit to re-connect. The Tic light flashes. This normally happens within a few seconds and you will hear a beep to confirm it. If successful, the parents unit display will first show the channel of transmission (C1 to C8), which will be replaced by the room temperature and icons for all activated functions.

Note : The connecting process should be done within 30 seconds. If longer, the operation will be ignored and you will have to start over.

• PAGING:

If you want to locate the parents unit, press the top button of the nursery unit.

STEP 13 TESTING YOUR ANGELCARE® MONITOR

Note: Test your Angelcare® monitor BEFORE using it for the first time and then on a regular basis to make sure that it is functioning properly. The Angelcare® monitor can be used outside of the crib in a different sleeping area. Test your monitor when moving the sensor pad to a new location to ensure sensitivity is adjusted correctly to the new padding or mattress. Consult the movement sensitivity adjustment section (step 6).

1. TEST THE SOUND Switch the parents unit to the "Sound Only" function. Have someone else speak into the nursery unit or place a radio near the nursery unit. You should hear sound from the parents unit. If there is no sound, refer to the Troubleshooting section.

2. TEST YOUR MONITOR TO DETECT MOVEMENT Make sure the sensor pad's cord is plugged into the nursery unit then turn both units ON. Set the parents unit on the "Movement only" function. Gently move your hand on the mattress. The top green light (with pendulum symbol) will flash on the nursery unit with your every movement. Remove your hand from the mattress. Due to the lack of movement, a "pre-alarm" "Tic" sound will be emitted after 15 seconds and 5 seconds later, the alarm will sound and the green light on the nursery unit will turn off.

3. TEST THE STOPPING OF THE ALARM To stop the alarm, place your hand lightly on the mattress so that the sensor pad starts detecting movement again. The green light with the pendulum icon will start flashing again, as movement is detected by the sensor pad. You may also shut the nursery unit OFF and turn it ON again to stop the alarm.

NOTE: If the alarm does not sound, the sensor pad may be detecting a continuous vibration from the floor, a strong draft or the crib is being touched. Avoid contact with crib when monitor is in use. Place crib near a solid supporting wall and away from strong air currents. The sensitivity of the sensor pad may need to be reduced, consult the movement sensitivity adjustment STEP 2.

4. VERIFYING THE BATTERY MODE Verify that the battery mode is functional by operating both units without the AC adapters. If the green light on the nursery unit does not light up, the batteries need to be replaced. If the parents unit doesn't light up and the screen does not appear, it needs to be recharged.

Maintenance of your Angelcare® monitor

RETAIN INSTRUCTIONS: Keep the user's manual for future reference.

WATER, MOISTURE AND HEAT DAMAGE: Keep all components away from water (sink, bath tub, swimming pool) and places that emit heat (stove, radiator).

VENTILATION: ALWAYS place both units on a flat surface in the upright position so that air can flow freely around them. The sensor pad is made of plastic and does not permit the air to circulate. Therefore, mildew build up could develop underneath the mattress in the sensor pad area. We suggest rotating the mattress regularly to prevent any mildew.

AC POWER SOURCE: Use only the AC adapters supplied with the Angelcare® monitor.

CORD PROTECTION: Avoid damage to the cords of the sensor pad and adapters. Route cords so that they are not likely to be walked on or pinched by items placed upon or against them.

CLEANING: Disconnect all units before cleaning. **DO NOT** immerse in water. Use a dry cloth to keep monitor free from dust. **DO NOT** spray with cleaning products or solvents. Wipe sensor pad with a cloth moistened very slightly with an antiseptic or mild detergent.

OBJECT AND LIQUID ENTRY: Care should be taken so that objects do not fall onto units and that liquids are not spilled into either unit or sensor pad through its openings.

NON-USE PERIODS: When not using the monitor for any long period, remove the batteries from both units and disconnect the AC adapters from electrical outlet.

TECHNICAL SPECIFICATIONS

No. of Channels: 8

Range: up to 820 ft. (250m)

Power Supply: Four AAA (LR03) alkaline batteries (not included) or 7.5V AC adapter for Nursery Unit. Four AAA (LR03) rechargeable batteries for each Parents' Unit and 7.5V AC adapter for Charging Base for each Parents' Unit.

CAUTION:

To reduce the risk of electric shock, do not expose this product to rain or moisture.

Danger of explosion if battery is incorrectly replaced. The included batteries are 4 AAA NIMH rechargeable batteries manufactured by Sanik Battery Co. Ltd. SN-AAA60HJ 1.2V 600mAh and by GPI International Limited, GN60AAAHC, 1.2V 600mAh. **They may only be replaced by 4 AAA NIMH rechargeable batteries.**

Batteries:

- Do not mix old and new batteries
- Do not mix alkaline, standard (carbon-zinc), or rechargeable (ni-cad, ni-mh, etc) batteries

Transmission:

927 MHz (USA and Canada)

Data transmission between the parents unit and nursery unit is made on the 2.4GHz frequency.

TROUBLESHOOTING

Problem	Possible Cause	Solution
False alarms.	- Baby was removed from crib and nursery unit was not switched OFF. - Sensor Pad cord not connected properly to nursery unit. - Sensor pad is not in full contact with crib mattress, or crib mattress is not resting on a completely flat or rigid surface. - Baby is in a very deep sleep or has moved away from sensor pad, to the edge of the crib.	- Turn off the nursery unit. - Verify the connection between the sensor pad and nursery unit. Unplug and reconnect the cord. - Make sure there is no bedding between mattress and sensor pad. Sensor pad must rest on a completely flat and rigid surface. Place a piece of 6 mm (1/4") plywood to cover the full size of the crib base. - Make sure sensor pad is positioned as described under the section How to operate your Angelcare® monitor. Sensitivity may need to be adjusted, consult the section on testing your Angelcare® monitor.
Alarm will not sound.	- Nursery unit is detecting movement when crib is being touched. - Nursery unit is detecting movement from outside the crib such as motorized devices, strong drafts, etc... - Sensitivity is set too high.	- Avoid contact with the crib when monitor is ON. - Place crib near a solid supporting wall to avoid sensor pad from detecting movement from outside crib. - You may need to decrease the sensitivity, consult the section on Testing your Angelcare® monitor and the section How to operate your Angelcare® monitor.
No sound is being transmitted.	- Units are too far apart (out of range indicator is off on the parent Unit). - Batteries are low or improperly installed. - AC adapters are not connected properly or electrical outlet is out of order. - One of the units is not ON.	- Bring parents unit closer to nursery unit. - Check/replace batteries or recharge parents unit - Check connections or change to another electrical outlet. - Ensure both units are ON.
Nursery unit power ON indicator (second green light) does not light up.	- AC adapters are not connected properly or electrical outlet is out of order. - Nursery unit is not ON.	- Check connections or change to another electrical outlet. - Move the left wing up to the ON position.
Parents unit is ON but the digital screen shows "--" and the nursery unit icon is flashing	The nursery unit is not ON.	Turn the nursery unit ON.
Very weak reception	- Units are too far apart and the "out of range" indicator is OFF. - One or both units are placed near a large metal mass. - One or both units are not in the upright position. - Batteries are low. - The link between the parent and nursery unit is lost.	- Turn "out of range" indicator ON as shown in STEP 7 in the How to operate your Angelcare® Monitor section. - Bring units closer together. - Change the position of one or both units. - Place units on a flat surface away from baby's reach. - Replace the batteries or recharge parents unit . - Connect the parent unit to the nursery unit as shown on STEP 12 of the How to operate your Angelcare monitor section.
Static, distortion, interference (interference from other monitors, wireless phones, walkie-talkies, etc...).	- Selected channel is not free of interference. - Parents Unit located near motorized devices, fluorescent lights, television, etc...	- Select a different channel as shown on STEP 12 of the How to operate your Angelcare® monitor section. - Rotate Parents unit or move it away from source of interference.
Feedback (loud noises emitted from Parents Unit).	Units are too close to each other.	- Move units farther apart (at least 3 meters/10 feet apart). - Decrease volume on parents unit.

LIMITED WARRANTY

Angelcare® warrants to the original owner of this product that Angelcare® rechargeable movement and sound monitor (model AC401) is free from any material and workmanship defects, for a period of one (1) year from the date of the original purchase. If this monitoring system fails to function properly when used as directed under normal conditions within the one-year period, Angelcare® will either repair or replace the product, at our discretion, free of charge. Please contact Angelcare.

The product must be accompanied by a proof of purchase, either a bill of sale or other proof that the monitoring system is within the warranty period. Angelcare® will bear the cost of repairing or replacing the product and shipping it back to you.

This warranty does not apply to a product which has been damaged as a result of improper maintenance, an accident, improper voltage supply or any other form of misuse. The warranty is also void if the owner repairs or modifies the product in any way. Angelcare® is not liable for any incidental or consequential damages with regard to this product. The warranty also excludes any liability other than what is stated above. No other warranty is given.

LEGAL RIGHTS VARY FROM COUNTRY TO COUNTRY. SOME COUNTRIES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

WE ARE NOT IN ANY WAY RESPONSIBLE FOR ANY OUTCOME RESULTING FROM THE USE OF THIS PRODUCT.

IMPORTANT! This Angelcare® monitor is a personal care product. Please do not return this product to the store. If you have questions or need help, please call Angelcare before returning this product.

Protect the environment!

This electronic product contains reusable material and must not be disposed with domestic waste. Please take it to a collecting point for electric and electronic equipment (ask your local authorities for details).

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*The ANGELCARE trademark is a registered trade mark of the Angelcare Development Inc.
La marque ANGELCARE est une marque enregistrée propriété de Les Développements Angelcare Inc.
La marca ANGELCARE es una marca registrada de Angelcare Development Inc.*