Medtronic

5873W

Wrench kit

Kit de clés

Kit de llaves

Technical manual • Manuel technique
Manual técnico

! USA

Caution: Federal Law (USA) restricts this device to sale by or on the order of a physician.

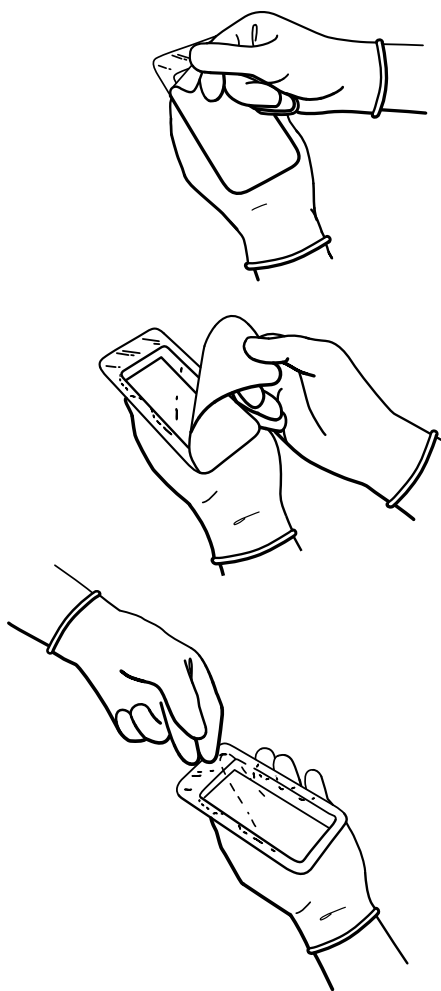
CE
0123
1994

The following are trademarks or registered trademarks of Medtronic in the United States and possibly in other countries:

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Medtronic

Sterile package opening instructions

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Explanation of symbols

Explication des symboles

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Conformité Européenne (European Conformity). This symbol means that the device fully complies with European Directive AIMD 90/385/EEC.

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Manufacturer
Fabricant
Fabricante

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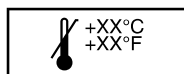
Description/intended use

The 5873W Wrench kit can be used to connect any Medtronic lead system to any Medtronic pulse generator. The larger, white-handled torque wrench is designed to slip when overtorqued to prevent damage to the socket or the setscrew(s). Sufficient tightening of the setscrew is indicated by a clicking sound as the torque wrench ratchets on itself. The white-handled hex wrench is designed to twist when overtorqued to prevent damage to the socket or the setscrew(s).

Note: The wrenches have the same tip dimensions; so, it is up to the individual's preference as to which wrench to use.

The Model 5873W is intended for single use only.

Contraindications

To date, there are no known contraindications to the use of the 5873W Wrench Kit.

Contents of package

One #2 White-Handled Torque Wrench
One #2 White-Handled Hex Wrench

Precautions

Carefully inspect the package before opening. If the container, the package, or the wrenches appear to have been damaged, return the wrench kit to your local Medtronic representative.

Sterilization

Medtronic has sterilized the package contents with ethylene oxide before shipment. This device is for single use only and is not intended to be resterilized.

Instructions for use

Note: The following three steps apply to both the larger, white-handled torque wrench and to the smaller, white-handled hex wrench. Please refer to the technical manual provided with the pulse generator for complete details on connecting the lead(s) to the pulse generator.

1. Insert the torque/hex wrench into the bisected grommet as shown in Figure 1 and Figure 2. The torque/hex wrench should be left inserted in each distal grommet until the lead is secure. This eliminates the “piston effect” (difficulty in inserting the lead due to compression of trapped air in

the connector port) by allowing a pathway for venting trapped air when the lead is inserted. The rubber connector seal maintains the torque/hex wrench in position.

- A. Check that the setscrew(s) are retracted from the connector port by visual inspection of the opening. If a setscrew(s) is obstructing the pathway, rotate the setscrew(s) counterclockwise one or two revolutions until the opening is no longer obstructed.

Caution: Counterclockwise rotation beyond this point may disengage the setscrew from the connector port.

- B. Eliminate the “piston effect” when the lead is inserted by leaving the torque/hex wrench in the distal grommet.

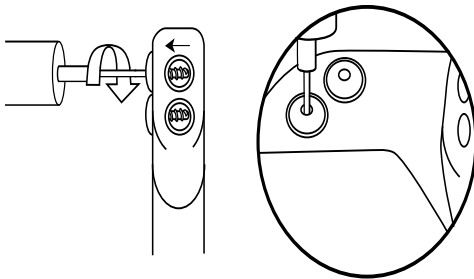
Figure 1.

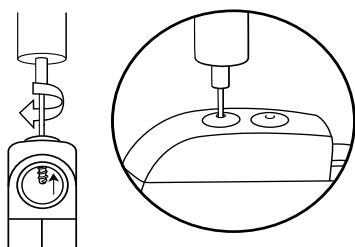
Figure 2.

Connector port setscrew(s) preparation using the torque wrench

2. To insert the lead, push the lead into the connector port until the lead pin is clearly visible in the pin viewing area.

Note: Lubricate the lead connector end, to ease lead insertion, with sterile water only.

3. To secure the lead, tighten the setscrew(s) by turning clockwise as indicated in Figure 3 and Figure 4. When using the larger, white-handled torque wrench, complete tightening will be indicated by a clicking sound. When using the smaller, white-handled hex wrench, tighten the setscrew(s) clockwise, until you feel resistance. You should be able to turn the setscrews at least 1/2 turn before resistance is felt. If the setscrew turns more than one

complete revolution without resistance, the lead is not properly inserted. If this occurs, back the setscrew out and reinsert the lead. The setscrew(s) must be engaged for pacing to begin. Do not pull on the lead until all setscrews have been tightened.

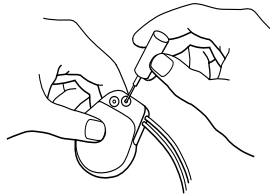
Figure 3.

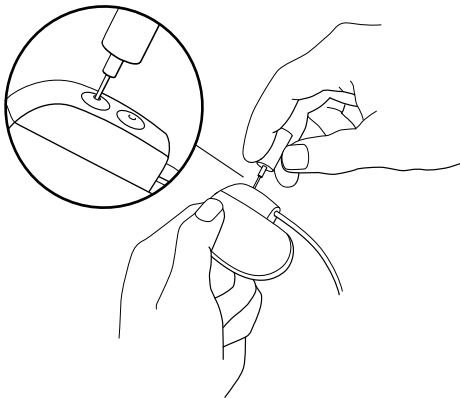
Figure 4.

Tightening the setscrew(s) using the torque wrench

Service

For supplemental information, contact your local Medtronic representative, or call or write Medtronic at the appropriate address or telephone number.

Special Notice

Medtronic Wrench Kits (the “Wrench Kits”) consist of Tools (the “Tools”) to connect a Medtronic lead system to a Medtronic pulse generator. The Tools may easily be damaged by improper handling or use. Consequently, no representation or warranty is made that failure or cessation of function will not occur.

Disclaimer of warranty

The following Disclaimer of warranty applies to United States customers only:

ALTHOUGH THE MEDTRONIC MODEL 5873W WRENCH KIT HEREAFTER REFERRED TO AS “PRODUCT” HAS BEEN MANUFACTURED UNDER CAREFULLY CONTROLLED CONDITIONS, MEDTRONIC HAS NO CONTROL OVER THE CONDITIONS UNDER

WHICH THIS PRODUCT IS USED. MEDTRONIC, THEREFORE DISCLAIMS ALL WARRANTIES, BOTH EXPRESS AND IMPLIED, WITH RESPECT TO THE PRODUCT, INCLUDING, BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. MEDTRONIC SHALL NOT BE LIABLE TO ANY PERSON OR ENTITY FOR ANY MEDICAL EXPENSES OR ANY DIRECT, INCIDENTAL OR CONSEQUENTIAL DAMAGES CAUSED BY ANY USE, DEFECT, FAILURE OR MALFUNCTION OF THE PRODUCT, WHETHER A CLAIM FOR SUCH DAMAGES IS BASED UPON WARRANTY, CONTRACT, TORT OR OTHERWISE. NO PERSON HAS ANY AUTHORITY TO BIND MEDTRONIC TO ANY REPRESENTATION OR WARRANTY WITH RESPECT TO THE PRODUCT.

The exclusions and limitations set out above are not intended to, and should not be construed so as to contravene mandatory provisions of applicable law. If any part or term of this Disclaimer of Warranty is held to be illegal, unenforceable or in conflict with

applicable law by a court of competent jurisdiction, the validity of the remaining portions of this Disclaimer of Warranty shall not be affected, and all rights and obligations shall be construed and enforced as if this Disclaimer of Warranty did not contain the particular part or term held to be invalid.

The following Disclaimer of Warranty Applies to Customers Outside the United States:

Although the Medtronic Model 5873W Wrench Kit, hereafter referred to as “Product” has been carefully designed, manufactured and tested prior to sale, the Product may fail to perform its intended function satisfactorily for a variety of reasons. The warnings contained in the Product labeling provide more detailed information and are considered an integral part of this **DISCLAIMER OF WARRANTY**.

Medtronic, therefore, disclaims all warranties, both express and implied, with respect to the Product. Medtronic shall not be liable for any incidental or consequential damages caused by any use, defect or failure of the Product, whether the claim is based on warranty, contract, tort or otherwise.

The exclusions and limitations set out above are not intended to, and should not be construed so as to, contravene mandatory provisions of applicable law. If any part or term of this DISCLAIMER OF WARRANTY is held by any court of competent jurisdiction to be illegal, unenforceable or in conflict with applicable law, the validity of the remaining portion of the DISCLAIMER OF WARRANTY shall not be affected, and all rights and obligations shall be construed and enforced as if this DISCLAIMER OF WARRANTY did not contain the particular part or term held to be invalid.

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© Medtronic, Inc. 2011
Medtronic B.V.
Medtronic Australasia Pty Ltd