
ENGLISH

*DeVilbiss® PulseDose® Extended Quick Reference Card*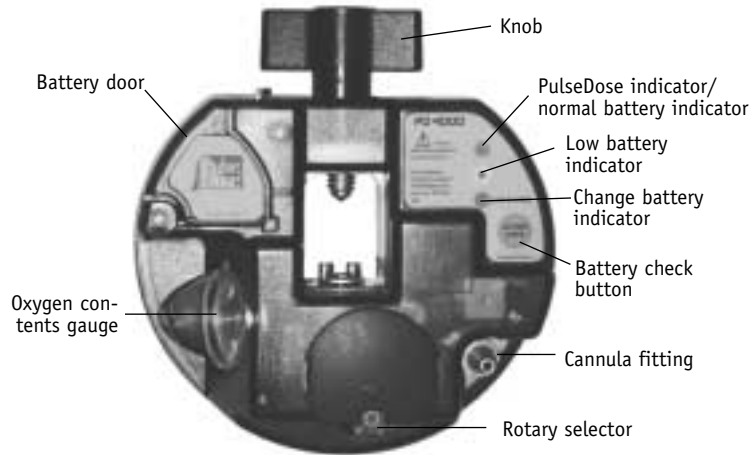

**USING YOUR PULSEDLOSE CONSERVING DEVICE**

1. Attach the conserving device to the oxygen cylinder.
2. Open the cylinder.
3. Turn the rotary selector to the correct prescription flow setting.
4. Attach the standard nasal cannula to the conserving device and to your nose and face.
5. Breathe normally, the conserving device will deliver a bolus of oxygen at the leading edge of inspiration on every breath up to 40 BPM.
6. When you are finished using the PulseDose Conserving Device, turn your cylinder to the closed position and rotate the rotary selector to the "O" position.

**NOTE-** Further explanation found on page 11 in instruction guide.

Sunrise Medical  
Respiratory Products Division  
100 DeVilbiss Dr.  
Somerset, PA 15501-2125

Phone: (814) 443-4881  
Toll Free Phone: (800) 333-4000

ENGLISH

*DeVilbiss® PulseDose® Troubleshooting Quick Reference*

*When you have a problem with the operation of your DeVilbiss PulseDose Extended Conserving Device, please take a few moments to check for these possible causes before calling your home medical equipment provider.*

**PROBLEM- UNIT WON'T PULSE**

- Be sure the rotary selector is set to the appropriate flow setting.
- Check the battery level by pressing the "battery check" button. The green light should illuminate.
- Take cannula off nose and ensure tubing isn't kinked.
- Attach cannula to nose and breathe.
- If unit still does not pulse, call your HME provider.

**PROBLEM- UNIT ALARMS**

- Were you breathing on the unit at the time of the alarm?  
The unit has a no-use alarm that will trigger if you haven't breathed on it after 3 minutes. It will then alarm every 30 seconds until you breathe on it or turn it off.
- If you are not using the unit, you must turn it off by turning the rotary selector to "O".
- The unit will alarm on each breath if the cylinder is closed or empty. What does the pressure gauge read?  
- Empty- Change tank.  
- Not Open- Be sure the tank regulator is all the way open. This is especially important on tanks that are close to empty.
- If "Low Battery" LED is illuminated, change battery.
- If none of the above, call your HME provider.

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