INTERPUPILLOMETRO DIGITALE

DIGITAL PD METER / INTERPUPILÓMETRO DIGITAL / PD-MESSGERÄT

Ref. **04969**


MANUALE D'USO E MANUTENZIONE - GARANZIA
OPERATION AND MAINTENANCE MANUAL - WARRANTY
MANUAL DE USO Y MANTENIMIENTO - GARANTÍA
GEBRAUCHS - UND WARTUNGSHANDBUCH

SERVICE CENTRE

OPTICS & SERVICE
centrostyle®

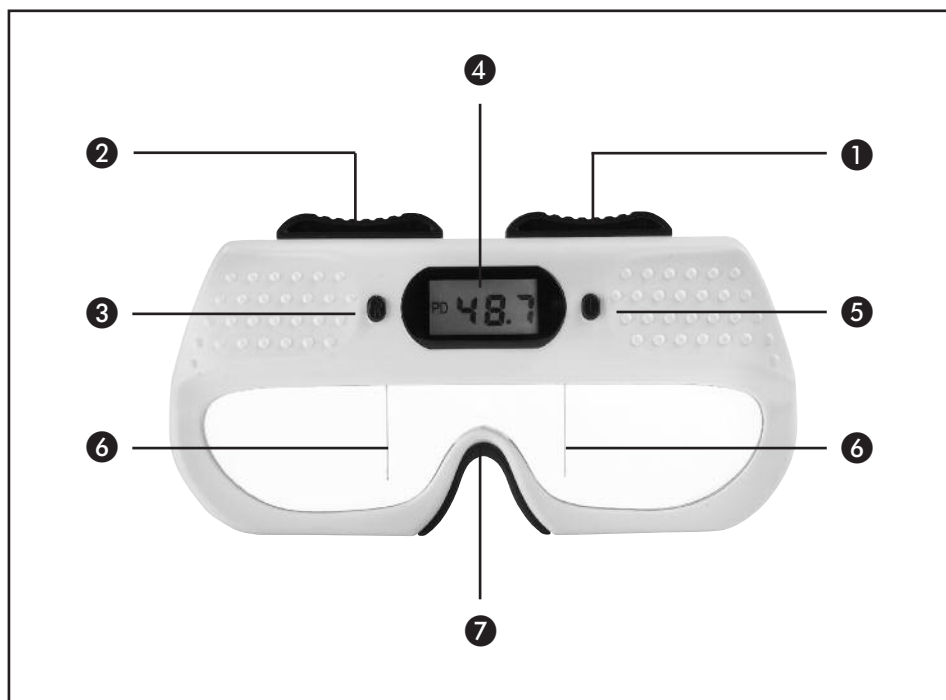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GB Description of controls

- ① PD adjustment slide (right).
- ② PD adjustment slide (left).
- ③ Right eye measuring display
- ④ PD distance
- ⑤ Left eye measuring display
- ⑥ Reference lines.
- ⑦ Nose bearer

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IMPORTANT NOTICE

GB

WARNING! Read this manual carefully before using the appliance.

Keep this manual safe for reference throughout the life of the appliance.

Copyright Centro Style S.p.A.. All rights reserved. Total or partial reproduction of this manual is strictly forbidden without the prior written consent of Centro Style S.p.A.

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Servizio clienti: Tel. 0332 270270 (orario: 8.30/12.30 - 14.00/18.00)

Page 1	TRANSPORT AND STORAGE.
Page 1	STANDARD ACCESSORIES.
Page 1	DESCRIPTION OF THE INSTRUMENT.
Page 2	SETTING UP, START-UP AND OPERATION.
Page 2	SAFETY PRECAUTION.
Page 3	MAINTENANCE AND CLEANING.
Page 3	PUTTING OUT OF SERVICE.
Page 3	TECHNICAL SPECIFICATION.
Page 4	WARRANTY AND SERVICE.

TRANSPORT AND STORAGE

Always ship and store the instrument and accessories in their original packaging.

- Do not expose the instrument to temperatures below 0° or above 60° C, to relative humidity above 85% or to a very dusty surrounding.
- Do not charge the PD-meter's box with more than 15 kg. Do not subject to hard knocks or mechanical strain.

STANDARD ACCESSORIES

1 operation manual
1 digital PD-meter Ref.04969.
1 battery 12 V (23AE).

DESCRIPTION OF THE INSTRUMENT

This instrument is a digital PD-meter, both for binocular and monocular measurements. It is an ancillary instrument in the determination of the optical centre of ophthalmic lenses for correction glasses.

TARGET USERS: professional, only for use by adequately trained staff.

APPLICATION FIELD: optical shops, optical laboratories. Instrument for external measurement, not a medical diagnostical instrument.

SETTING UP, START-UP AND OPERATION

Open the packaging and make sure it is undamaged, check that it contains all parts listed on page 1 and are in perfect condition.

If damage has incurred during transport, contact Centro Style or its distributor.

- Open the battery compartment and insert the enclosed battery with correct polarity.
1. The instrument switches on automatically when placed in vertical position and switches off again when placed in horizontal position.
 2. Place the PD-meter on the patient's nose, with both oculars in correspondence to the patient's eyes.
The PD-meter can be placed onto the patient's nose even if the patient is wearing glasses.
 3. The reference lines must be brought into correspondence with the patient's pupils. To achieve this, move the PD adjustment slides ❶ and ❷.
 4. Take the PD-meter off the patient's nose and read the patient's pupilar distance and the semi-distance for the left ❸ and the right ❹ eye.

SAFETY PRECAUTIONS

- Replace exhausted batteries with new ones. Do not use batteries of different makes or not completely charged.
- This instrument is not water-tight nor resistant to splashes. Never immerse it in water or expose it to splashes
- Do not try to repair it yourself or by an unauthorized person. Always send it to Centro Style or their distributor for repair.
- The PD-meter does not generate electromagnetic disturbance which could interfere with other equipment.

This instrument is designed and manufactured for use only as explicitly stated in this manual. All other uses are considered improper.

Centro Style declines all responsibility for injury to persons or damage to property resulting from:

- Improper use and associated risks.
- Unauthorized modifications or the pare parts.
- uses of non original accessories.

MAINTENANCE AND CLEANING

- If not used for longer periods, remove the batteries from the instrument.
- As there are no mechanical parts, no regular maintenance is necessary. The reference lamp inside is designed for long term use.
- Clean the external surfaces with a clean cloth slightly dampened in water. Do not use solvents or potentially aggressive substances. Do not spray liquids directly onto the instrument. Do not use abrasive cleaning material.

PUTTING OUT OF SERVICE

At the end of the instrument's working life, or if it becomes irreparably damaged, remove the batteries and dispose of them separately.

Notes of the disposal of different materials. Separation and recycling of waste material.

Refer to European Waste Disposal Directive 160200 governing the disposal of equipment or decommissioned equipment parts to dispose of or recycle.

Description of materials used in the PD-meter:

- The external structure of the body is made from ABS plastic.
- The printed circuit board with its components and wiring is inside.

TECHNICAL DATA

Power supply	1 Battery 12V (23AE)
Total PD range	45 – 80 mm
Partial PD range	22,5 – 40 mm
Resolution	0,5 mm
Precision	+/- 0,5 mm
Dimensions (mm) l x w x h	173 x 90 x 30 mm
Weight	150 g (with battery)
Surrounding conditions whilst in use	Temperature between 0° e 60° C. relative humidity < 85%
	EN61000-6-3:2007 EN61000-6-1:2007

WARRANTY AND SERVICE

- 1 CHECK THE CONTENTS OF ALL THE BOXES
- 2 NOTIFY CUSTOMER SERVICE OF ANY PROBLEMS WITHIN **8** DAYS

If any fault develops within the warranty period, do not open the cleaner but contact Technical Assistance immediately. All parts recognised as having manufacturing faults will be replaced under the terms of the warranty.

The warranty covers faults and breakages not caused by normal wear, provided the cleaner has not been tampered with. Consumables are not covered by the warranty.

* Within one year of purchase

full warranty

* from the date of invoice.

IN CASE OF MALFUNCTIONING

Never return the cleaner or parts without prior notice!

Contact your nearest authorised Service Centre before returning the cleaner or any parts, or to request repairs under the warranty.

Centro Style S.p.A. Via G.D. Martinengo, 21040 Vedano Olona, Varese.

Fax: + 39 0332 270 405

E-mail: export.dpt@centrostyle.it

Call to clarify the following points:

- 1 - Cleaner code and serial number
- 1 - Type of fault
- 3 - Quick over-the-phone troubleshooting and problem solving
- 4 - Quick over-the-phone troubleshooting with delivery of spare parts
- 5 - Always state the purchase date, the number of the invoice or delivery note and your customer code.

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Centro Style S.p.A. Via G.D. Martinengo, 7 - 21040 Veduggio Olona - Varese.
Anschrift: Centrostyle S.p.A. Postfach 669, 6855 Stabio TI
CENTROSTYLE S.p.A. - Via G.D.Martinengo,7 - 21040 Vedano Olona - Varese - Italy