CPAP 20e

CLICK 2

CPAP unit
Aparato CPAP

CPAP 20e

WM 27320

CLICK 2

WM 27330

Description of the equipment and instructions for use
Descripción del aparato e instrucciones de manejo

English




4

Español

52

Overview

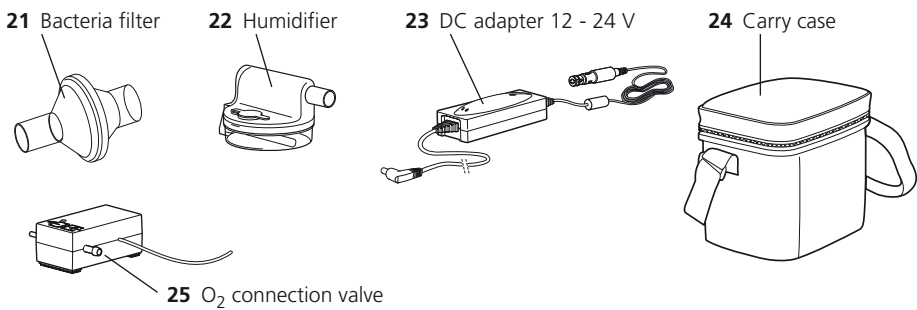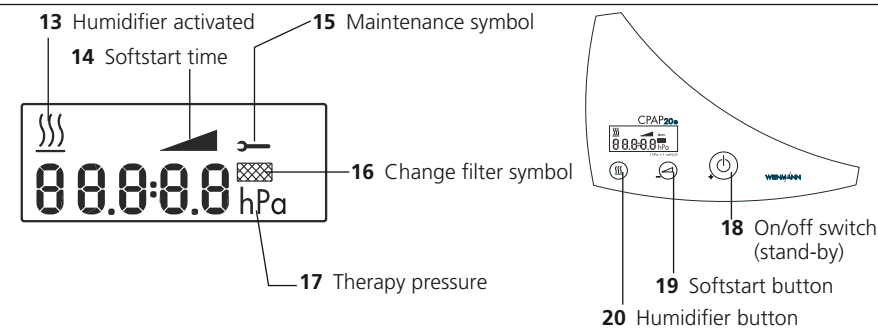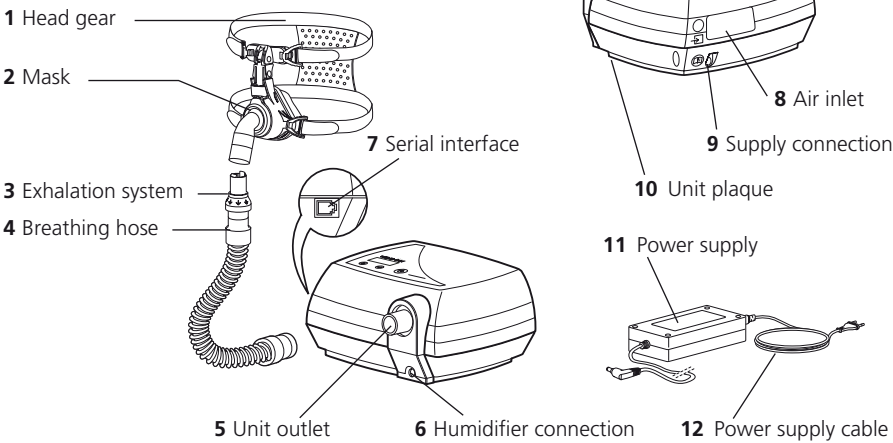CPAP 20e

Legend

1 Head gear (Accessories)

Helps locate the mask correctly and securely.

2 Mask (Accessories)

The mask is used to provide the patient with air at the required therapy pressure.

3 Exhalation system (Accessories)

Exhalation air containing carbon dioxide is diverted here during the therapy.

4 Breathing hose

The air flows through the breathing hose to the mask.

5 Unit outlet

The air flows from here via the breathing hose and mask to the patient.

6 Humidifier connection

Used to connect the CLICK 2 humidifier (supplied as an accessory).

7 Serial interface

Used exclusively to connect to devices for visualization and evaluation of therapy data.

8 Air inlet

Securely positions the coarse and fine dust filters.

9 Supply connection

This is the connection for the external power supply or DC adapter.

10 Unit plaque

Provides information about the unit, e.g. serial number and year built.

11 Power supply

Used to supply power to the unit.

12 Power supply cable

Connects the power supply to the power supply.

13 Humidifier activated

This symbol appears when the humidifier is in operation. The current humidity level is displayed.

14 Softstart time

This symbol appears when the softstart time is set or the softstart is activated.

15 Maintenance symbol

This symbol appears every 10,000 operating hours. The unit must then be serviced.

16 Change filter symbol

This symbol appears every 250 operating hours. The fine filter must then be changed.

17 Therapy pressure

The therapy pressure is shown in hPa (hectoPascal). $1 \text{ hPa} = 1 \text{ mbar} \approx 1 \text{ cm H}_2\text{O}$.

18 On/off switch (stand-by)

Switches the therapy unit on and off.

19 Softstart button

Activates the softstart or sets the softstart time (5 - 30 minutes in 5 minute increments)

20 Humidifier button

Activates the humidifier or sets the humidifier level (6 levels).

21 Bacteria filter (Accessories)

Used to protect the patient from bacteria, particularly if the unit is being used by a number of patients.

22 Humidifier (Optional)

Used to heat and humidify the air flow created by the therapy unit.

23 DC adapter 12 - 24 V (Accessories)

For operating the therapy unit via a DC socket (12 - 24 V).

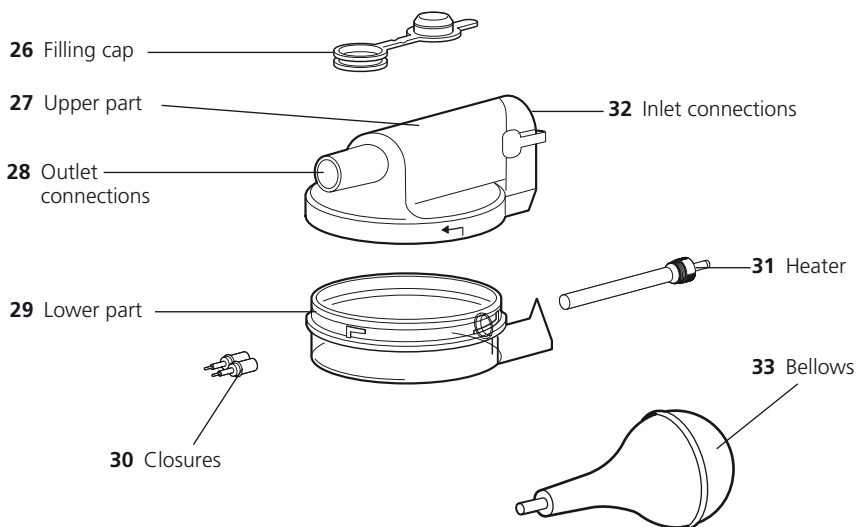
24 Carry case (Accessories)

For carrying the therapy unit.

25 O₂ connection valve (Accessories)

For introducing oxygen into the mask.

CLICK 2 (optional)

Legend

26 Filling cap

For filling the humidifier.

27 Upper part

Closes the humidifier, contains the filling cap.

28 Outlet connections

From here the moistened air flows into the breathing hose.

29 Lower part

Contains approximately 280 ml water.

30 Closures

Used to shut off the pressure gland during preparatory hygienic operations.

31 Heater

Heats the water in the humidifier. This humidifies the air.

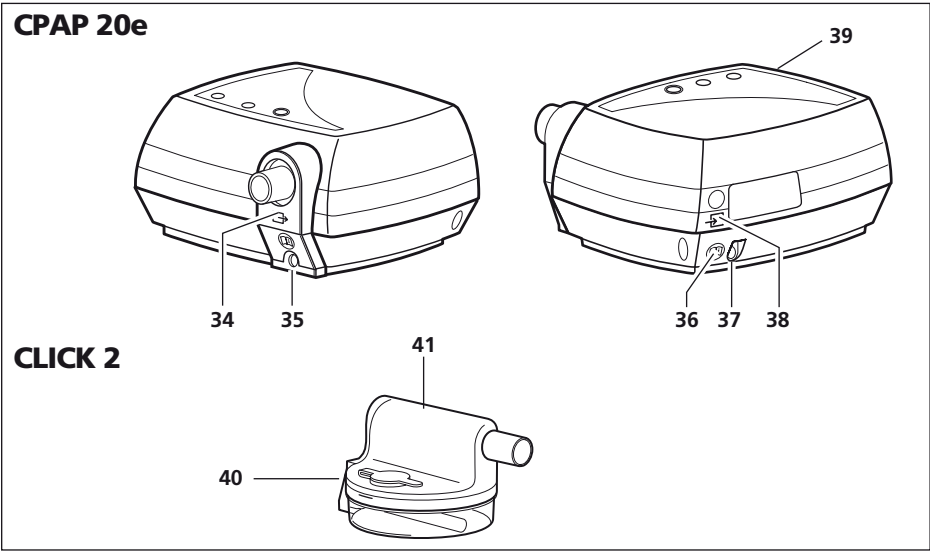
32 Inlet connections

The air flows from the therapy unit into the humidifier.

33 Bellows

Dries the CLICK 2.

Special markings on the unit

	Symbol	Meaning
CPAP 20e:		
34		Unit outlet: air escapes with 4 - 20 hPa
35		Please use the appropriate User Guide! Connector for respiratory humidifier CLICK 2 WM 27330.
36		Please use the appropriate User Guide! Connection for power supply WM 24480 or DC adapter WM 24469, to operate the therapy units using a DC socket (12 - 24 V)
37		DC voltage
38		Unit inlet: allows air at room temperature to enter
39		Please use the appropriate User Guide! Connection for setting the therapy parameters with SOMNOadjust WM 23930 and SOMNOsupport WM 23975 by specialist staff or to control the O ₂ connection valve, WM 24042. Max. current consumption 163 mA.
CLICK 2 (optional):		
40		Unit is hot! Do not touch the heater rod.

	Symbol	Meaning
41		Do not use the humidifier on patients whose airways have been bypassed

Unit plaques

	Year built
	Level of protection from electric shock: Unit type B
	Do not dispose of the unit as household waste
	Please use the appropriate User Guide!
SN	Unit serial number
	Type of protection from electric shock: Unit of protection class II

Markings on the packaging

Symbol	Meaning
	Permitted temperature for storage: -20°C to + 70°C
% 0-95	Permitted humidity for storage: max. 95 % relative humidity.

Contents

1. Description of unit	10	5.6 Disinfection CLICK 2	30
1.1 Intended use CPAP 20e	10	5.7 Multiple patient use CPAP 20e	30
1.2 Intended use CLICK 2	10	5.8 Multiple patient use CLICK 2	31
1.3 Functional description CPAP 20e	11	6. Functional check	32
1.4 Functional description CLICK 2	11	6.1 Intervals	32
2. Safety Instructions	12	6.2 Function check of CPAP 20e	32
2.1 Safety instructions	12	6.3 Function check of CLICK 2	33
2.2 Contraindications	14	7. Malfunctions and Rectification	34
2.3 Side effects	15	7.1 CPAP 20e	34
3. Set up of equipment	16	7.2 CLICK 2	35
3.1 Setting up and connecting the CPAP 20e	16	8. Maintenance	37
3.2 Filling and connecting the CLICK 2	17	8.1 Intervals	37
3.3 Breathing mask	18	8.2 Maintenance includes the following:	37
3.4 Connect other accessories	19	9. Disposal	37
4. Operation	20	10. Scope of supply	38
4.1 CPAP 20e startup	20	10.1 Standard scope of supply	38
4.2 CLICK 2 setting	21	10.2 Accessories	38
4.3 Softstart setting	22	10.3 Replacement parts CPAP 20e	41
4.4 After usage	23	10.4 Replacement parts CLICK 2	41
4.5 Read off therapy duration	23	11. Technical data	42
4.6 Traveling with therapy units	24	11.1 Specifications	42
5. Hygienic preparation	25	11.2 Pneumatic diagram	45
5.1 Intervals	25	11.3 Safety distances	46
5.2 Cleaning CPAP 20e	25	12. Warranty	47
5.3 Cleaning CLICK 2	27	13. Index	48
5.4 Descaling CLICK 2	29	14. Declaration of Conformity	49
5.5 Disinfection CPAP 20e	29		

1. Description of unit

1.1 Intended use CPAP 20e

CPAP 20e is a CPAP unit (CPAP = Continuous Positive Airway Pressure) used to treat sleep-related breathing disorders in patients from the age of 12 years.

No special knowledge or training is required in order to use the unit. All that is required is instruction by a clinic or a medical practitioner.

Caution!

CPAP 20e can only reliably prevent airway closure when the CPAP pressure prescribed by a doctor for the specific patient has been calculated, e.g. in a sleep laboratory.

CPAP 20e is **not** designed for life support.

Only use the unit for the purposes described below.

1.2 Intended use CLICK 2

The CLICK 2 humidifier is an optional accessory for the Weinmann CPAP 20e therapy unit.

The CLICK 2 humidifier is used to heat the air flow created by the CPAP 20e and to increase its humidity. This will prevent the patient's upper airways drying out during the therapy.

CLICK 2 can only be used with the CPAP 20e therapy unit. Operation with third-party devices is not permitted.

Only use the unit for the purposes described below.

1.3 Functional description CPAP 20e

CPAP 20e operates according to the principle of an electrically driven flow generator, with the pressure level being controlled electronically.

- A fan draws in ambient air via a filter and conveys it to the unit outlet. From here the air flows through the breathing hose and the mask to the patient.
- A pressure sensor measures the upcoming pressure at the unit outlet and shows this on the display. The microprocessor-controlled flow generator controls the pressure to the set value and reduces fluctuations in pressure caused by breathing.
- A softstart automatic system is installed to help the patient fall asleep more easily. When the unit is switched on, the CPAP pressure starts at the initial pressure selected by the doctor and is then slowly increased to the setpoint pressure selected. The time of the pressure increase can be adjusted between 5 and 30 minutes in 5-minute increments.
- The therapy parameters are adjusted using the *SOMNOadjust* remote control system or the *SOMNOsupport* evaluation program.
- CPAP 20e automatically saves all settings in the event of a power failure. This means that the unit does not need to be reset when power is restored.

1.4 Functional description CLICK 2

The humidifier works on the so-called overflow principle. The air coming from the therapy unit is conveyed across the surface of a heated water tank, increasing the relative humidity and the temperature of the air flow. The amount of humidity released can be adjusted using the buttons on the Weinmann therapy unit. The output of the heater rods and consequently the temperature of the water in the humidifier chamber is controlled electronically by the Weinmann unit.

The water level can be checked at any time through the window in the humidifier chamber.

2. Safety Instructions

Please read these operating instructions through carefully. They form part of the equipment and should be available at all times.

For your own safety, the safety of your patients, and to comply with the requirements of EU Directive 93/42 EEC, please observe the following points.

2.1 Safety instructions

Operating the unit

Caution!

- The power supply supplied allows the unit to be operated at voltages between 115 V and 230 V. Check that your supply voltage falls within this range. With a 12 V DC or 24 V DC supply, use the WM 24469 DC adapter available as an accessory.
- CPAP 20e must be connected to an easily accessible socket so that the mains plug can be quickly removed in the event of a power failure.
- Do not cover the unit or power supply. Do not use in the vicinity of a heater or in direct sunlight. The unit may overheat, resulting in insufficient therapy delivery and damage to the unit.
- Maintain a safe distance between the CPAP 20e and equipment that emits HF radiation (e.g. mobile phones). This unit may otherwise malfunction (see page 46). If CPAP 20e interferes with other equipment (e.g. a radio), put the CPAP 20e further away from the equipment.

Caution!

- Do not use the therapy unit or humidifier if the unit is malfunctioning, parts are damaged and/or the contact between the humidifier and the heater rod is damp.
- The performance of the humidifier can be affected if it is used outside the permitted temperature range.
- Observe the operating instructions for your mask.
- Please observe the section „5. Hygienic preparation" on page 25 to avoid infection or bacterial contamination.

Transport/Accessories/Spare Parts/Repair

Caution!

- Do not transport (or tilt) the CPAP 20e with the humidifier installed. Tilting might cause water remaining in the humidifier to run into the CPAP 20e and damage it.
- Only have checks and repair work carried out by Weinmann or specialist staff.
- Malfunctions and a lack of biocompatibility may result if third-party articles are used. Please bear in mind that in these cases any warranty entitlement and liability shall lapse where the accessories recommended in the instructions for use or original spare parts are not utilized.

Caution!

- Modifications to the CPAP 20e and CLICK 2 are not permitted.
- Third-party masks may only be used after being approved by Weinmann. Using unapproved masks may result in unsuccessful therapy.
- Please make sure that the accessories you use are complete and are suitable for the therapy. This particularly applies to the exhalation system. The CO₂ content of the exhaled air may otherwise adversely affect your breathing.

Oxygen supply

Warning!

- **There is a risk of fire!** Do not smoke or use open fires if oxygen is being used during therapy. The oxygen may be deposited in clothing, bed linen or in your hair and can only be removed by thorough ventilation.

Caution!

- An oxygen supply is only permitted if the WM 24042 O₂ connection valve is used.
- As regards oxygen, please make sure you observe the safety instructions included in the instructions for your oxygen system.

2.2 Contraindications

The CPAP 20e should not be used with certain diseases or only with particular caution. In each case the decision whether to use CPAP 20e for therapy lies with the doctor in charge.

These include:

- cardiac decompensation
- severe heart arrhythmia
- severe hypotension, particularly in conjunction with intravascular volume depletion
- severe epistaxis
- high risk of barotrauma
- ventilatory insufficiency due to a cause other than OSA
- nocturnal hypoxemia due to a cause other than OSA (obesity hypoventilation syndrome)
- pneumothorax or pneumomediastinum
- pneumo-encephalus
- head injury
- status after brain operation and surgical procedure performed on hypophysis or on middle or inner ear
- acute inflammation of the sinuses (sinusitis), middle ear inflammation (otitis media) or perforated eardrum
- dehydration

Threatening situations were not observed with CPAP 20e.

2.3 Side effects

When using CPAP 20e the following undesirable side effects may occur during short-term and long-term usage:

- pressure points on the face caused by the mask and the forehead padding
- reddening of facial skin
- nasal congestion
- dry nose
- dry mouth in morning
- feeling of pressure in the sinuses
- irritation of conjunctiva of eyes
- gastrointestinal insufflation (“wind”)
- nosebleeds

3. Set up of equipment

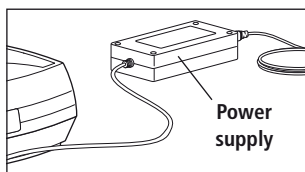
3.1 Setting up and connecting the CPAP 20e

1. Set up the unit on a flat surface, e.g. a bedside cabinet or on the floor next to the bed.


Caution!

Do not cover the unit or power supply. Do not use in the vicinity of a heater or in direct sunlight. The unit may overheat, resulting in insufficient therapy delivery and damage to the unit.

2. Plug the power supply cable into the supply connection of the CPAP 20e. Then plug the power supply into a socket outlet (using the power supply cable).

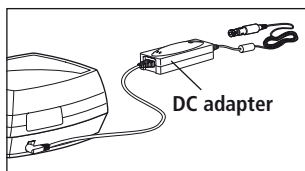
The power supply automatically determines the appropriate mains voltage (115 V - 230 V).

If the CPAP 20e is to be operated with a voltage of 12 or 24 V DC, connect the WM 24469 DC adapter (available as an option) instead of the power supply.

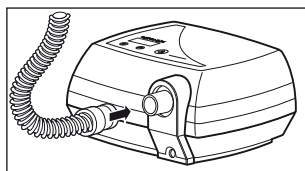On the display „“ appears to show it is ready for use.

3. Plug the breathing hose into the unit outlet.

Note:

The device automatically compensates for differences in air pressure (e.g. as a result of great altitude).

3.2 Filling and connecting the CLICK 2

Before using for the first time

Before using for the first time, rinse the humidifier with copious amounts of water (see "5. Hygienic preparation" on page 25).

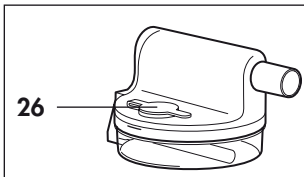
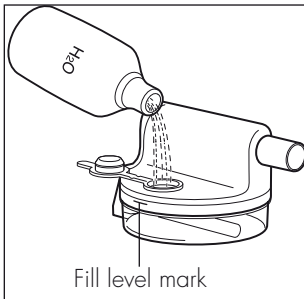
Filling the humidifier

Caution!

- Do not fill the humidifier when it is connected to the therapy unit and do not overfill above the maximum level, otherwise water may enter the therapy unit. This can damage the unit.
- Do not fill the humidifier with hot water as this might damage the housing and heating rod resulting in incorrect operation and insufficient therapy delivery.


Note

- The use of aromatic additives, such as eucalyptus oil, may damage the plastic parts of the humidifier.

1. Open the filling cap of the humidifier.
2. Fill the humidifier up to the mark with distilled water. If no distilled water is available, low calcium boiled water can be used instead (but not as a rule!). Check that the heater rod does not scale up (see "5.4 Descaling CLICK 2" on page 29).
3. Firmly close the humidifier with the filling cap.
4. After filling, check that the humidifier is not leaking. Run your finger over the bottom of the humidifier or place the humidifier on a piece of kitchen paper. Any escaping water will then be easy to spot.

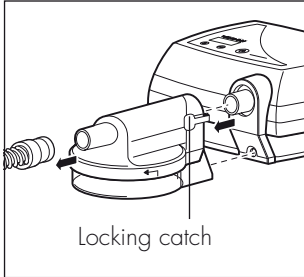Adapting the humidifier

1. If necessary, release the corrugated tubing from the therapy unit.
2. Click the humidifier into position from the front onto the sleep apnea unit. The inlet connection and the heating rod must be directed towards the sleep apnea unit. Take care to engage the lock.
3. Connect the corrugated tube to the outlet connections of the humidifier.
4. To release the humidifier, press the locking catch and withdraw it off the unit.

3.3 Breathing mask

Fitting the mask

1. Adjust the forehead support of the mask (if present).
2. Connect the headgear to the mask.
3. Position the mask.
4. Adjust the headgear so that the bulge of the mask only results in low pressure to prevent pressure points on the face.

Note

For further instructions, please refer to the operating instructions for the relevant mask.

Warning!

Full-face masks must be equipped with an emergency exhalation valve so that you can breathe through nose or mouth in the event of any device failure. Full-face masks without an emergency exhalation valve may not be used!

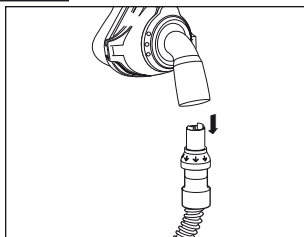
External exhalation system

No exhalation system is integrated in some masks. In order for the used, carbon dioxide (CO₂)-enriched air to be able to escape, an external exhalation system is required on these masks. Follow the instructions for use of your exhaled air system.

Warning!


Without an exhalation system the CO₂ concentration would rise to critical values in the mask and hose, thus hindering your breathing. Risk of suffocation!

- Fit the external exhalation system between the breathing mask and the breathing tube.


3.4 Connect other accessories

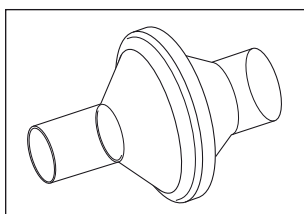Bacteria filter

If the CPAP 20e is to be used by several patients (e.g. in a hospital), a bacteria filter should be used to protect against infection. It is inserted between the breathing tube and the device outlet port. Using a bacteria filter can reduce the pressure and flow rate.

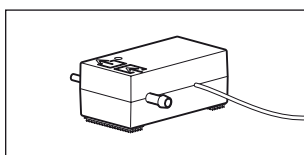Oxygen supply

For safety reasons (risk of fire) the direct introduction of oxygen into the breathing hose or the mask without any special protective device is not permitted. However, the supply of up to 4 l/min oxygen to the breathing mask is possible when using the WM 24042 oxygen connection valve.

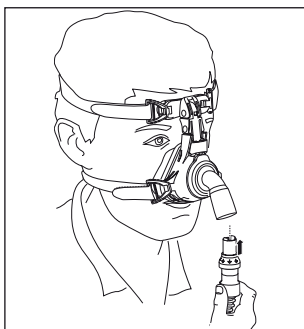
An oxygen concentrator (e.g. Oxymat), liquid oxygen or an oxygen cylinder with a suitable pressure reducer may be used. This application must be prescribed by the patient's doctor.

To prevent oxygen entering the therapy unit, the CPAP 20e automatically generates a therapy pressure of at least 5 hPa when an oxygen connection valve is attached. You must always follow the safety instructions for handling oxygen as well as the operating instructions for the oxygen valve and the oxygen unit used.

4. Operation

4.1 CPAP 20e startup

Put on the mask

1. Put on the mask as described in Section “3.3 Breathing mask” on page 18.
2. Connect the breathing hose to the mask (plug cone).
3. Feed the breathing hose away from your head.

Warning! Risk of injury!

Never place the breathing hose around your neck.

Switch on the device

1. Press the on/off switch.

The therapy period will appear for approx. 3 seconds in the display. The flow generator starts to convey air through the breathing hose.

If softstart has been preselected, the softstart display will then light up and the softstart time will appear in the display.

CPAP 20e is now ready for operation.

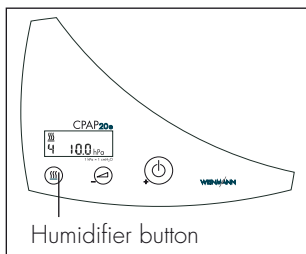
Note:

If the device is not used for 15 minutes, it will automatically switch off (only if the therapy pressure is higher than 6hPa).

4.2 CLICK 2 setting

Switching on the humidifier


1. Fill up the humidifier and set up as described in "3.2 Filling and connecting the CLICK 2" on page 17.
2. Switch on the therapy unit.

3. Press the humidifier button  of the therapy unit. The humidifier symbol  and the humidification level display (level 4 in this case) are shown in the display of the therapy unit.

Setting the heating level

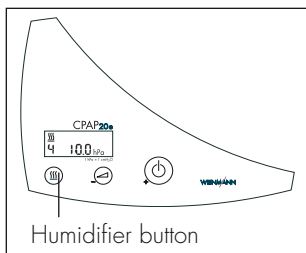1. Make sure your therapy unit is switched on.

2. Press the humidifier button  for approx. 3 seconds until the number and the humidifier symbol  start to flash.
3. Release the humidifier button .
4. Now select the heating level (1-6) using the on/off switch (+) or the softstart button (-).

The displayed value is saved after 4 seconds.

Which is the best setting for you will depend on the room temperature and the air humidity. The optimum setting will therefore vary depending on the time of year. Under normal conditions heating level 3 is sufficient.

If the temperature is 23 °C and air humidity 70 %, a humidity of 100 % can be reached with heating level 6 and a flow rate of 20 l/min.

If your airways are dry when you wake up, the heating level is too low. Select a higher setting.

If condensation forms in the breathing hose during a night therapy session, the heating level is too high. Choose a lower heating level.

Switching off the humidifier

There are two ways of switching off the humidifier:

- Press the humidifier button  of the sleep apnea unit. The humidifier symbol  in the display of the sleep apnea unit goes out. The therapy unit remains switched on.
- Press the on/off switch on your therapy unit. The humidifier and the therapy unit are both switched off.
When you switch the therapy unit on again, the humidifier is also switched on automatically and runs at the heating level of the previous therapy session.

Automatic switch-off at low levels

If the therapy unit detects that there is not enough liquid in the humidifier, it switches the humidifier off within 15 minutes. The humidifier symbol  in the display goes out.

4.3 Softstart setting

The softstart time can be selected in 5-minute increments up to a maximum of 30 minutes. Proceed as follows:

1. Switch on CPAP 20e and press the softstart button . Hold the button down until the display for the softstart time flashes.
2. Now select the softstart time (5-30 minutes in 5-minute increments) using the on/off switch (+) or the softstart button (-).

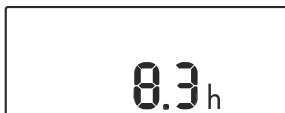
If a button is not pressed for 4 seconds, the displayed softstart time will be saved and the display stops flashing.

The softstart settings are saved when the unit is switched off. Whenever CPAP 20e is started up, it automatically starts in softstart mode if this was switched on.


Softstart can be switched on and off at any time by pressing the softstart button .

4.4 After usage

1. Remove the headgear with the mask.
2. Push the on/off switch to switch the device off.

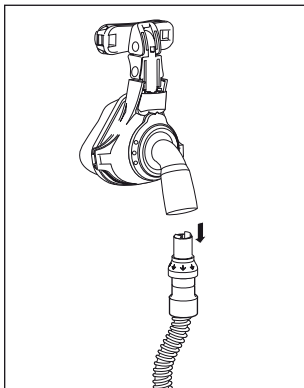The daily therapy duration of the last session is shown on the display. Then "0" appears.

3. Remove the tube connection and the exhalation system from the mask.
4. Clean the mask and the exhalation system (see "5. Hygienic preparation" on page 25).

Note:

CPAP 20e and the power supply consume approx. 4 watt in standby mode. The unit does not have a mains switch. To save electricity you can remove the power plug from the outlet socket during the day. It is therefore advisable to connect the unit to a readily accessible socket. The stored values and settings will not be lost.

5. Clean the humidifier (see "5. Hygienic preparation" on page 25).

4.5 Read off therapy duration

CPAP 20e saves the data from 366 days and the associated therapy duration. You can display the following values:

- Total therapy duration in hours
- Average therapy duration per day
- Therapy duration of current day including date

Note:

A **therapy day** always starts and ends at 12 noon. Data recorded between midnight and 12 noon is allocated to the previous **day**.

The total therapy duration is shown for a few seconds when the unit is **switched on**. When **switching off** the therapy duration for the current day is displayed.

To read off further data, proceed as follows:

1. When switching off the unit hold down the on/off switch for approx. 4 seconds until the average therapy duration per day is displayed.
2. Release the on/off switch. The average therapy duration per day and the total therapy duration are each displayed for 5 seconds. The current date and the therapy duration for the day then flash up alternately.
3. The daily therapy duration for previous days can now be displayed.

Press the softstart button until the required date appears in the display.

You can also simply press and hold the softstart button, releasing it when the desired date is displayed.

Use the on/off switch in the same way to return to the current date.

If a button is not pressed for 10 seconds, the total operating time will appear for 3 seconds. Then "0" appears.

4.6 Traveling with therapy units

To transport the therapy unit over long distances, it should be packed in the carry case (available as an accessory).

1. Place the unit, the power supply, the power cable, the breathing hose, the mask incl. exhalation system and, if necessary, the humidifier in the carry case.
2. Close the zips of the carry case and adjust the shoulder strap to the correct length.

Caution!

Please do not forget to take spare filters and the operating instructions with you, as well as the DC adapter if necessary. If you want to take the therapy unit as hand luggage in an aircraft, you should find out any formalities necessary from the airline before departure.

5. Hygienic preparation

5.1 Intervals

- Clean the mask and exhalation system every day in accordance with the operating instructions.
- Check the filters regularly:
 - clean the coarse dust filter once a week and replace it at least every 6 months.
 - change the fine filter depending on the level of soiling, however at least every 250 hours or once a month (filter change indicator).
- Regularly wipe clean the housing of the CPAP 20e and the power supply.
- The breathing hose should be cleaned according to the level of soiling, however at least once a month. Replace it every 12 months.
- Wash the headgear in accordance with the accompanying instructions for usage.
- Clean the humidifier every day.

5.2 Cleaning CPAP 20e

Cleaning the breathing hose

1. Detach the breathing hose from the unit and from the exhalation system.
2. Clean the corrugated hose so no residues remain with a small amount of detergent in warm water. Rinse the inside of the hose thoroughly.
3. Then rinse thoroughly both inside and outside using clean, warm water.

Caution!

4. Shake the hose thoroughly.

5. Hang up the hose and allow to drip to prevent moisture from getting into the CPAP 20e. The breathing hose is dried completely with the help of the CPAP 20e .
6. Plug the breathing hose into the unit outlet.
7. To start the drying process, press at the same time the softstart button (CPAP 20e switched off) and the on/off switch.

The unit will now switch on and dry the breathing hose (this takes 30 minutes). The remaining time is displayed. It switches off automatically at the end of the drying process.

You can interrupt the process at any time by pressing the on/off switch.
If the breathing hose is still damp in places after drying, start the drying process again.

Cleaning the CPAP 20e housing

Warning!

- Electric shock! Before cleaning, remove the power supply cable from the supply connection of the therapy unit.
- Make sure no liquid gets into the housing. Never immerse the unit in disinfectant or any other liquid. This can damage the unit and consequently endanger users and patients.

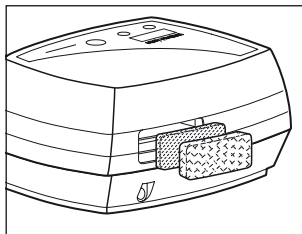
Caution!

- Make sure no liquid gets into the pressure glands. This may cause the unit to malfunction.

Wipe down the unit, the power supply and the power supply cable with a damp, soft cloth.

The CPAP 20e must be completely dry before starting up.

Cleaning the coarse dust filter/changing the fine filter

1. Remove the coarse dust filter and clean under fresh running water until no residues remain.
2. Replace the fine filter if necessary, as it cannot be cleaned.
3. Let the coarse dust filter dry.

Before start up, the coarse dust filter must be completely dry.

4. Replace the coarse dust filter.

Clear the filter change indicator

When switching on the unit, press and hold the on/off switch.

- The total therapy duration is displayed.
- The change filter sign appears.
- Do not release the button until the filter change indicator goes blank.

Reset the hours meter

If the fine filter was changed due to soiling before 250 operating hours, the hours meter must be reset to zero. Proceed as follows:

1. When switching on the unit, press and hold the on/off switch.

The filter change indicator will come on after about five seconds before going off again after another three seconds.

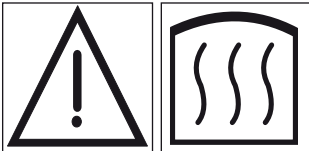
2. Release the on/off switch.

Accessories


To clean the exhalation system, the mask and the headgear, please see the section "Cleaning" in the relevant operating instructions.

To clean the humidifier CLICK 2, please see the section "5.3 Cleaning CLICK 2" on page 27 of these operating instructions.


5.3 Cleaning CLICK 2

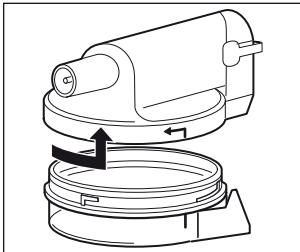Do **not** touch the heater rod. The metal takes a long time to cool down.

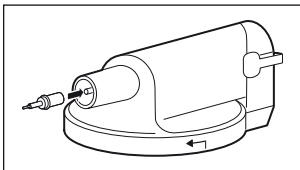
1. Open the humidifier by twisting the upper and lower parts against each other in a counter-clockwise direction.

2. No water must penetrate into the pressure line during cleaning or this could lead to incorrect pressure readings. Therefore the pressure line must remain sealed.

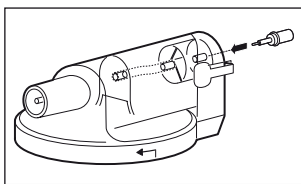
To seal the pressure line at the outlet-connections push the silicone tube of the closure (supplied as standard CLICK 2) over the end of the pressure line.

To seal the pressure line at the inlet connections, push the thin end of the closure by a slight rotational movement into the seal of the pressure line.

Clean the upper and lower parts with lukewarm water to which a little detergent has been added. We recommend cleaning the housing parts once a week in the upper tray of the dishwasher at a maximum temperature of 65 °C.

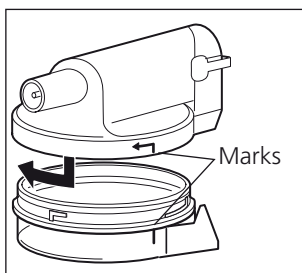3. Then rinse the parts in clean water.
4. Vigorously shake the lower part to remove any water that may have entered the heater rod contact. Dry the parts thoroughly with a soft cloth. The heater rod contact must be dry.
5. Check whether the pressure measurement tube is still correctly fitted in the upper part. If necessary reinsert it.


6. Fit the upper and lower parts together such that the marks are aligned. Turn the upper and lower parts against each other clockwise as far as they will go.
7. Remove the closure. The humidifier is now ready for use again.

For hygiene reasons, we recommend that the plastic parts are replaced after a maximum of two years use.

CLICK 2 drying

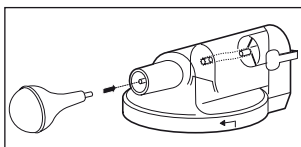If water penetrates into the pressure line, then dry CLICK 2 as follows:

1. Push the bellows included in the delivery, as illustrated, onto the end of the pressure line at the outlet connectors.

Caution!

The bellows must **only** be used for drying the humidifier. During drying CLICK 2 must not be connected to the therapy unit. Otherwise the unit could be damaged.

2. Press the bellows together so that any moisture is blown out.

Caution!

Keep the bellows pressed in so that moisture is not sucked back in again.

3. Remove the bellows.
4. Repeat steps **1.** to **3.** two or three times.

5.4 Descaling CLICK 2

If the lower part of the housing and the heater rod are covered in scale:

1. Pour about 150 ml of normal vinegar (5% solution without additives) into the lower part of the humidifier and leave to stand for one hour.
2. Pour the vinegar solution away.
3. Wipe the housing parts with a clean cloth and rinse thoroughly several times with water.
4. Then carefully wipe all parts.

5.5 Disinfection CPAP 20e

If necessary, for example after an infectious illness or if the unit has become unusually contaminated, you can also disinfect the housing, the power supply cable, the breathing hose, and the humidifier. Please observe the operating instructions for the disinfectant used. We recommend that you wear suitable gloves when disinfecting the equipment (e.g. household or disposable gloves).

Unit

Warning!

- Electric shock! Before cleaning, remove the power supply cable from the supply connection of the therapy unit.
- Make sure no liquid gets into the housing. Never immerse the unit in disinfectant or any other liquid. This can damage the unit and consequently endanger users and patients.

Caution!

- Make sure no liquid gets into the pressure glands. This may cause the unit to malfunction.

The housing, power supply and power supply cable of the CPAP 20e are cleaned by being simply wiped over with disinfectant. We recommend TERRALIN.

We recommend that you wear suitable gloves when disinfecting the equipment (e.g. household or disposable gloves).

Breathing hose

- We recommend GIGASEPT FF as a disinfectant.
- When disinfecting use the same procedure as for cleaning.
- The breathing hose must not be heated to temperatures in excess of 70 °C. Sterilization is not permitted.

Accessories

To disinfect or sterilize the exhalation systems and the mask, please see the section "Hygienic preparation" in the relevant operating instructions.

When there is a change of patient please refer to section "5.6 Disinfection CLICK 2" on page 30 and "5.8 Multiple patient use CLICK 2" on page 31 for instructions on how to disinfect the CLICK 2 humidifier.

5.6 Disinfection CLICK 2

The humidifier can be disinfected. Please observe the operating instructions for the disinfectant used. We recommend that you wear suitable gloves when disinfecting the equipment (e.g. household or disposable gloves).

We recommend GIGASEPT FF.

- The procedure for dismantling and assembling the humidifier is the same as for cleaning.
- To ensure full disinfection, the filling cap should also be removed and disinfected separately.

For hygiene reasons, we recommend that the plastic parts are replaced after a maximum of two years use.

5.7 Multiple patient use CPAP 20e

If the unit is to be utilized for another patient without using a bacteria filter, it must undergo a hygiene treatment beforehand. This has to be carried out by Weinmann or a dealer.

The procedure used for the hygiene treatment is described in the inspection sheet and the service and repair instructions for CPAP 20e.

5.8 Multiple patient use CLICK 2

If CLICK 2 is to be utilized for another patient without using a bacteria filter, it must undergo a hygiene treatment beforehand. This should be carried out by an authorized dealer.

The procedure used for the hygiene treatment when there is a change in patient is described in the service and repair instructions for the CPAP 20e.

6. Functional check

6.1 Intervals

Carry out a functional check on the CPAP 20e and the CLICK 2 at least every 6 months.

If you find a defect during the functional check, do not use the unit until the defect has been remedied.

6.2 Function check of CPAP 20e

1. Assemble CPAP 20e with the breathing hose, exhalation system, mask and power supply cable ready for operation.
2. Push the on/off switch to switch the device on. If Softstart is switched on, switch it off with the softstart button. The fan conveys air through the breathing hose to the mask and the current pressure in hPa is shown in the display.
3. Close the opening of the mask, e.g. using your thumb or hand.

Compare the pressure shown on the display with the CPAP pressure prescribed. There should only be a divergence of max. ± 1.0 hPa.

4. Press the softstart button to switch on softstart. The softstart display appears and the display shows the softstart time.

The pressure increases slowly and reaches the CPAP pressure after the time selected. The CPAP pressure is then shown on the display.

5. Switch off the unit using the on/off switch.

If the values or functions are not satisfactory, please send in the unit for repair to the dealer or the manufacturer.

6.3 Function check of CLICK 2

1. Carry out a visual inspection of the plastic housing.

If cracks/damage or severe soiling is evident, the plastic parts or seals must be replaced.

2. Fill the humidifier up to the mark with water.

3. Check whether the humidifier is leaking.

4. Empty the water.

5. Now pour in about 50 ml of water.

6. Connect the humidifier to the therapy unit.

7. Switch on the therapy unit.

8. Switch the humidifier on by pressing the humidifier button  on the therapy unit.

9. Set heater level 6 on the therapy unit.

10. Check whether the humidifier is getting warmer.

If the water has not heated up after 15 minutes, send the humidifier and the therapy unit for repair to the dealer or the manufacturer.

7. Malfunctions and Rectification

If there are defects that cannot be rectified at once, please contact Weinmann or your dealer immediately to have the unit repaired. To prevent greater damage, stop using the unit.


7.1 CPAP 20e

Malfunction	Cause	Rectification
No running noise, standby/status display not illuminated.	No supply voltage available	Check power supply cable is securely connected. Check function of outlet by connecting another piece of equipment (e.g. a lamp).
Unit is running but does not reach CPAP pressure level selected.	Filter soiled	Clean/change filter (5.2, page 25).
	Mask not airtight	Adjust headgear so that mask is airtight. Replace mask if defective.
Filter change indicator  lights up.	Filter soiled	Clean/change filter (5.2, page 25).
Error message "Err E", "Err b", "Err c", "Err A", "Err H" or "Err S" in display.	Problems with electronics or program "crashed"	Disconnect power and reconnect.
Service indicator  lights up.		Unit must be checked or serviced by Weinmann or a dealer as soon as possible.
There is no pressure or incorrect pressure to the mask.	Water in the pressure measurement port	Dry the pressure measurement line, see page 28.
	The pressure measurement tube is incorrectly fitted in the upper part.	Push the pressure measurement tube on to the correct connecting piece, see 5.3.
	No mains voltage available at the therapy unit.	Check mains cable is securely connected. Check function of outlet by connecting another piece of equipment (e.g. a lamp).

7.2 CLICK 2



Malfunction	Cause	Rectification
Humidifier does not warm up.	Humidifier is not activated.	Switch on the air humidifier, see 4.2, page 21.
	Electronics defect	Send the humidifier and the therapy unit to your dealer for repair.
The humidifier is leaking.	The heater rod seal is defective.	Replace the seal, see page 35.
	The seal on the underside of the housing is leaking.	Replace the seal, see page 35.
	Cracks in lower part	Replace the lower part.

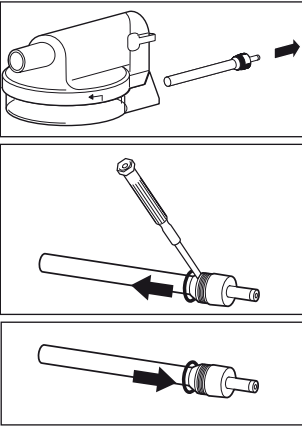Replacing the heater rod seal

1. Remove the heater rod from the lower part of the housing by turning counter-clockwise.
2. Using a small, clean screwdriver, carefully prise out the seal.
Important!
Take care to ensure that the groove in which the seal fits is not damaged.
3. Carefully press the new sealing ring into the groove on the heater rod. Under no circumstances use a tool.
4. Fit the new heater rod.

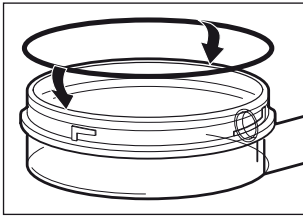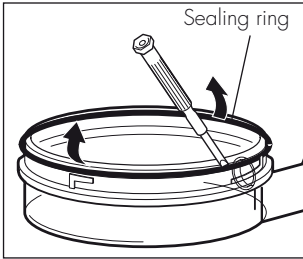Replacing seal of housing base

1. Open the breath humidifier by turning the base and top section anticlockwise against each other.

2. Take a small clean screwdriver and carefully remove the sealing ring from the groove of the housing base.

Important!

Make absolutely sure that the groove containing the sealing ring is not damaged when doing so.

3. Carefully press the new sealing ring into the groove of the housing base. When doing so, do not use tools under any circumstances.

4. Reassemble base and top section.

8. Maintenance

Regular maintenance should be carried out as a preventive measure. Please adhere to the stipulated intervals.

8.1 Intervals

- Every 10,000 operating hours (maintenance symbol appears in display)
- Every 4 years at the latest (see maintenance sticker on rear of unit)

8.2 Maintenance includes the following:

- Filter change
- Hygienic preparation of the unit and fan unit
- Complete function test

9. Disposal

Do not dispose of the unit with domestic waste. For proper disposal of the device and its components, please contact a certified waste disposal site for electronic goods. Ask your Environmental Officer or local council for the address. Appliance packaging (cardboard and inserts) may be disposed of in the paper recycling bin.

10. Scope of supply

10.1 Standard scope of supply

CPAP 20e

WM 27320

Parts	Order number
CPAP 20e basic unit	WM 27321
Breathing hose	WM 24445
Power supply	WM 24480
Power supply cable	WM 24133
Instructions for use CPAP 20e + CLICK 2 GB	WM 66101

CLICK 2

WM 27330

Parts	Order number
CLICK 2 basic unit	WM 27331
Closures, complete and packed	WM 24020
Bellows	WM 24394

10.2 Accessories

The following accessories can be ordered separately when required.

Parts	Order number
CLICK 2 humidifier	WM 27330
SOMNO <i>adjust</i> remote adjust device (only for trained medical and engineering staff)	WM 23930
SOMNO <i>support</i> evaluation software (only for trained medical and engineering staff)	WM 23975
Oxygen valve	WM 24042
DC adapter (12-24 V)	WM 24469
Bacteria filter	WM 24476
Carry case	WM 27335
Set of 10 breathing hoses	WM 15689
Silentflow	WM 23600

Parts	Order number
Sound absorber, complete (alternative exhalation system)	WM 23685
SOMNOmask, size S	WM 25110
SOMNOmask, size M	WM 25120
SOMNOmask, size L	WM 25130
SOMNOmask, size S, blue	WM 25210
SOMNOmask, size M, blue	WM 25220
SOMNOmask, size L, blue	WM 25230
SOMNOplus, size S	WM 25710
SOMNOplus, size M	WM 25720
SOMNOplus, size L	WM 25730
SOMNOcap, size M, blue	WM 23673
SOMNOcap, size L, blue	WM 23674
SOMNOcap, size M, white	WM 23686
SOMNOcap, size L, white	WM 23687
SOMNOstrap, blue	WM 23695
SOMNOstrap, white	WM 23177
SOMNOstrap extra	WM 25250
JOYCEstrap	WM 26290
JOYCEstrap EXTRA, 3 pcs.	WM 26291
JOYCEstrap EXTRA, 4 pcs.	WM 26292
JOYCEcap	WM 26293
JOYCE vented, size S	WM 26110
JOYCE vented, size M	WM 26120
JOYCE vented, size L	WM 26130
JOYCE vented, size XL	WM 26140
JOYCE vented 40 hPa, size S	WM 26111
JOYCE vented 40 hPa, size M	WM 26121
JOYCE vented 40 hPa, size L	WM 26131
JOYCE vented 40 hPa, size XL	WM 26141
JOYCE GEL vented, size S	WM 26112
JOYCE GEL vented, size M	WM 26122
JOYCE GEL vented, size L	WM 26132
JOYCE non vented, size S	WM 26160

Parts	Order number
JOYCE non vented, size M	WM 26170
JOYCE non vented, size L	WM 26180
JOYCE non vented, size XL	WM 26190
JOYCE non vented 40 hPa, size S	WM 26161
JOYCE non vented 40 hPa, size M	WM 26171
JOYCE non vented 40 hPa, size L	WM 26181
JOYCE non vented 40 hPa, size XL	WM 26191
JOYCE GEL non vented, size S	WM 26162
JOYCE GEL non vented, size M	WM 26172
JOYCE GEL non vented, size L	WM 26182
JOYCE Full Face vented, size S	WM 26410
JOYCE Full Face vented, size M	WM 26420
JOYCE Full Face vented, size L	WM 26430
JOYCE Full Face vented, size XL	WM 26440
JOYCE Full Face vented 40 hPa, size S	WM 26411
JOYCE Full Face vented 40 hPa, size M	WM 26421
JOYCE Full Face vented 40 hPa, size L	WM 26431
JOYCE Full Face vented 40 hPa, size XL	WM 26441
JOYCE Full Face GEL vented, size S	WM 26412
JOYCE Full Face GEL vented, size M	WM 26422
JOYCE Full Face GEL vented, size L	WM 26432
JOYCE Full Face _{plus} vented 40 hPa, size S	WM 26413
JOYCE Full Face _{plus} vented 40 hPa, size M	WM 26423
JOYCE Full Face _{plus} vented 40 hPa, size L	WM 26433
JOYCE Full Face _{plus} vented 40 hPa, size XL	WM 26443
JOYCE Full Face non vented 40 hPa, size S	WM 26461
JOYCE Full Face non vented 40 hPa, size M	WM 26471
JOYCE Full Face non vented 40 hPa, size L	WM 26481
JOYCE Full Face non vented 40 hPa, size XL	WM 26491
JOYCE Full Face GEL non vented, size S	WM 26462
JOYCE Full Face GEL non vented, size M	WM 26472
JOYCE Full Face GEL non vented, size L	WM 26482

When using other mask systems, please observe the relevant operating instructions.

10.3 Replacement parts CPAP 20e

Parts	Order number
Breathing hose	WM 24445
Set of 10 breathing hoses	WM 15689
Coarse dust filter	WM 24481
Set of 2 coarse dust filters	WM 15321
Fine dust filter, packed	WM 24401
Set of 12 fine filters	WM 15668
Power supply	WM 24480
Power supply cable	WM 24133

10.4 Replacement parts CLICK 2

Parts	Order number
Housing top section, complete	WM 27339
Lower part of housing	WM 27333
Seal for lower part of housing	WM1145/143
Heater	WM 24264
Filling cap	WM 24378
Seal, heater	WM 1145/19
Closures, complete and packed	WM 24020
Bellows	WM 24394

11. Technical data

11.1 Specifications

	CPAP 20e	CPAP 20e with CLICK 2						
Product class as per 93/42/EEC	IIa							
Dimensions WxHxD in cm	21 x 9.8 x 18.5				21 x 14 x 31.3			
Weight	approx. 1.25 kg				approx. 1.5 kg (without water)			
Temperature range – operation – storage	+5 °C to +35 °C –20 °C to +70 °C If the unit runs at +40 °C , the air output can heat up to 42 °C.							
Adm. humidity for operation and storage	≤ 95 % rF (no condensation)							
Air pressure range	600 - 1100 hPa (automatic altitude compensation) (allows for operation in an altitude of up to approximately 4000 meters)							
Diameter of breathing hose connection (mask) in mm	19.5 (suitable for 22 mm standard cone)							
Electrical connection	115-230 V AC +10/-15 %, 50–60 Hz with WM 24480 power supply or 12-24 V DC -15/+25 % with WM 24469 DC adapter							
Power consumption during – operation – standby	230 V 0.1 A 0.02 A	115 V 0.2 A 0.03 A	24 V 0.8 A 0.2 A	12 V 1.6 A 0.24 A	230 V 0.14 A 0.02 A	115 V 0.28 A 0.03 A	24 V 1.3 A 0.2 A	12 V 4.0 A 0.24 A
Classification as per prEN 60601-1:2004 – type of protection from electric shock – level of protection from electric shock – protection against damaging ingress of water – operating mode	Protection class II Type B IPX1 Constant operation							

	CPAP 20e	CPAP 20e with CLICK 2
Electromagnetic compatibility (EMC) as per EN 60601-1-2 – radio shielding – radio interference suppression	Test parameters and limit values can be obtained from the manufacturer if required. EN 55011 B EN 61000-4 parts 2 to 6, part 11	
Mean sound pressure level/operation at distance of 1m from unit in patient position	≤ approx. 30 dB (A) at 10 hPa (corresponds to an acoustic power level of 38 dB (A))	
CPAP operating pressure range Pressure accuracy	4 to 20 hPa ± 0.8 hPa (1mbar = 1 hPa ≈ 1cm H ₂ O)	
Max. CPAP pressure with malfunction	< 30 hPa	
Flow at max. speed at: 20 hPa 15 hPa 10 hPa 5 hPa 0 hPa Tolerance	85 l/min 100 l/min 120 l/min 140 l/min 155 l/min ± 10 l/min	80 l/min 95 l/min 115 l/min 135 l/min 150 l/min ± 10 l/min
Heating of respiratory air	2.5 °C	depending on heating level

	CPAP 20e	CPAP 20e with CLICK 2
Short-term constant pressure level measured as per prEN ISO 17510:2004 when therapeutic pressure is applied: 20 hPa 15 hPa 10 hPa 5 hPa Long-term constant pressure level prEN 17510-1:2004	$\Delta p \leq 1.3 \text{ hPa}$ $\Delta p \leq 1.1 \text{ hPa}$ $\Delta p \leq 0.8 \text{ hPa}$ $\Delta p \leq 0.6 \text{ hPa}$ $\Delta p = 0.2 \text{ hPa}$	
Fine filter separation level to 1 μm to 0.3 μm	$\geq 99.5 \%$ $\geq 85 \%$	
Fine filter service life	≥ 250 hours with normal room air	

Subject to design modifications.

	CLICK 2
Product class as per 93/42/EEC	II a
Dimensions WxHxD in mm	120 x 97 x 157
Weight (without water)	250 g
Temperature range – operation – storage	+5 °C to +35 °C –20 °C to +70 °C
Humid operation and storage	$\leq 95 \%$ relative humidity
Ambient pressure range	600 to 1100 hPa
Electrical connection	30 V DC
Power consumption	20 VA
Classification as per EN 60601-1 – level of protection from electric shock	Unit type B
Electromagnetic compatibility (EMC) as per EN 60601-1-2 – Radio shielding – radio interference suppression	EN 55011 B IEC 1000-4, parts 2 to 6, part 11
Max. permitted fill quantity	280 ml
Max. permitted operating pressure	30 hPa


	CLICK 2
Max. permitted flow (free-flowing)	160 l/min
Max. mask temperature	37 °C
Gas leakage at 30 hPa	negligible
Humidifier output at heating level 6: Flowrate = 20 l/min Flowrate = 30 l/min Flowrate = 40 l/min	14.1 mg/l 10.3 mg/l 8.0 mg/l
at 23 °C and 65 % relative humidity	

Subject to design modifications.

All values established subject to ATPD conditions (ambient temperature and pressure, dry).

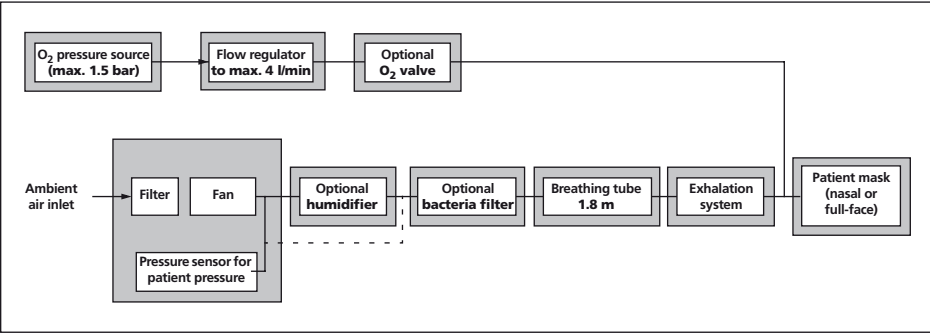11.2 Pneumatic diagram

11.3 Safety distances

Recommended safety distances between portable and mobile HF telecommunications equipment (e.g. mobile phones) and CPAP 20e or CLICK 2			
Nominal output of HF equipment in W	Safety distance dependent on transmission frequency		
	in m		
	150 kHz - 80 MHz	80 MHz - 800 MHz	800 MHz – 2.5 GHz
0.01	0.12	0.12	0.24
0.1	0.37	0.37	0.74
1	1.17	1.17	2.34
10	3.69	3.69	7.38
100	11.67	11.67	23.33

12. Warranty

- Weinmann offers a warranty that the product, when used in accordance with requirements, will remain free from defects for a period of two years from date of purchase. For products whose durability is clearly indicated as less than two years, the warranty expires on the expiration date indicated on the packaging or in the user's manual.
- Claims against the warranty can be made only when accompanied by the sales receipt, which must show salesperson and date of purchase.
- We offer no warranty in the case of:
 - Disregard of usage instructions
 - Operating errors
 - Improper use or handling
 - Third-party intervention by non-authorized persons for the purpose of device repair
 - Acts of God, e.g., lightning strikes, etc.
 - Transport damage as a result of improper packaging of returned items
 - Lack of maintenance
 - Operational and normal wear and tear, which includes, for example, the following components.
 - Filter
 - Batteries and rechargeable batteries
 - Articles for one-time usage, etc.
 - Failure to use original spare parts.
- Weinmann is not liable for consequential harm caused by a defect if it is not based on intention or gross negligence. Weinmann is also not liable for minor physical injury to life or limb resulting from negligence.
- Weinmann reserves the right to decide whether to eliminate defects, to deliver a defect-free item or to reduce the purchase price by a reasonable amount.
- If Weinmann rejects a claim against the warranty, it assumes no expense for transport between customer and manufacturer.
- Implied warranty claims remain unaffected by these changes.

13. Index

Keyword:	page:
A	
Accessories	38
C	
Cleaning	25
CLICK 2	17, 21, 42
D	
Disinfection	25, 29
Drying	25
E	
Exhalation system:	18, 23, 25, 32, 39
F	
Filter	25, 34, 41, 44
Functional check	32
H	
Humidifier	13, 19, 29, 38, 42
I	
Inspection	13
M	
Malfunctions	34
Mask	18, 20, 23
R	
Replacement parts	41
S	
Safety instructions	12, 13
Side effects	15
Softstart	11, 20, 22, 32
Standby mode	23
T	
Traveling	24
W	
Warranty	47

14. Declaration of Conformity

EEC Letter of Conformity on Medical Devices

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declare in sole
responsibility the

Product Name: **CPAP Sleep Apnea Device**
Type/Model: **CPAP 20e**

being in conformity
with the respective
regulations of the
following guideline:

Medical Device Directive 93/42/EWG

Classification:

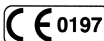
Ila

standards used

in particular:
IEC 60601-1: 2005
EN 60601-1-2: 2001 + A1: 2006
prEN ISO 17510-1: 2005

Mark:

The product has the mark by
TÜV-Rheinland Product Safety GmbH
51105 Cologne – Germany

Hamburg, 10/04/2006

Dr. K.-A. Feldhahn
President
Chief Technology Officer

WEINMANN

WM 70131 10.06

EC Declaration of Conformity on Medical Devices

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declare in sole,
responsibility the

Product Name: Respiratory Humidifier
Type/Model: Click 2

being in conformity
with the respective
regulations of the
following guideline:

Medical Device Directive 93/42/EWG

Classification:

Ila

standards used :

in particular:
IEC 60601-1: 2005
EN 60601-1-2: 2001 + A1: 2006
prEN ISO 8185: 2005

Mark:

The product has the mark by
TÜV-Rheinland Product Safety GmbH
51105 Cologne – Germany

Hamburg, 08/07/2007

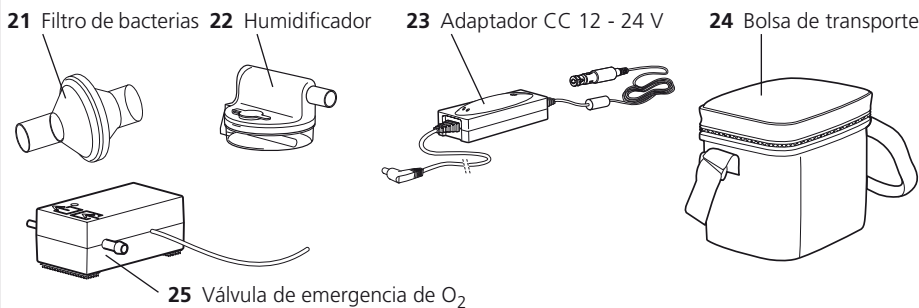
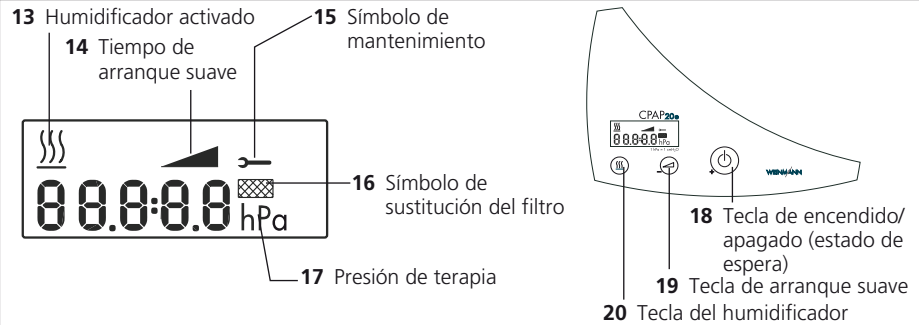
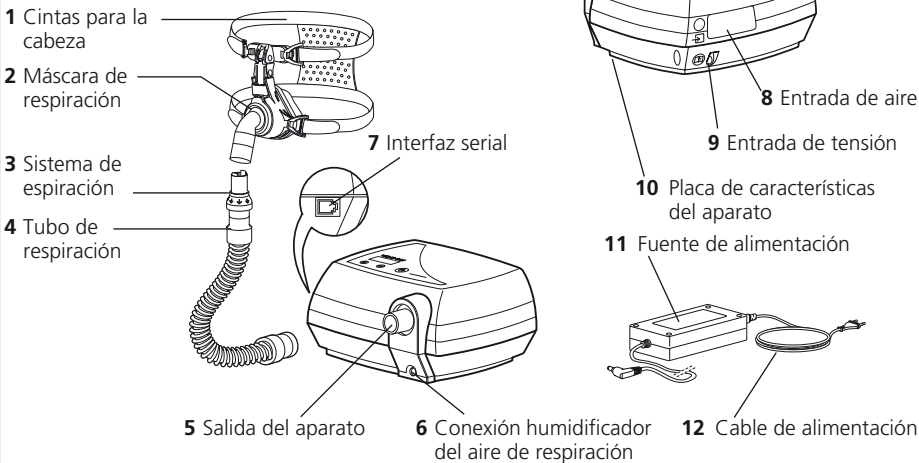
K.-A. Feldhahn
President
Chief Technology Officer

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Safety GmbH, 51105 Köln.
Geräte für Medizin GmbH+Co.KG