- ES Tobillera
- IT MT Calze con tallone rinforzato
- PT Meia para calcanhar
- GB MT Heel sleeve

Delta-Sport-Nr.: BG-1416

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GB MT

Read through the instruction manual carefully before use.

Make sure you keep the instructions for future reference!

The manufacturer and importer shall accept no liability arising from the improper use of this item or from failure to comply with the advice set out in these directions for use.

## Shipment

1 x Directions for use

1 x Heel sleeve

## Symbols Used

Follow instruction manual

Manufacturing date

Lot identification

Manufacturer

## Technical Data

Material: 100% TPE gel; 82% polyamide, 18% spandex

## Designated Use

This product is intended for private use.

### Intended use:

- Alleviates shoe pressure discomfort on the Achilles and discomfort from Haglund's deformity or ganglion cysts.

### Do not use if you have any

- Circulatory disorders
- Open wounds
- Skin diseases

## **Safety Advice**

### **Risk of injury!**

- If you experience any problems when wearing the sleeve, remove it immediately and seek the advice of a doctor.
- Ensure that the sleeve fits snugly but is not too tight. Ask your doctor if you are unsure.
- Check the sleeve for tears or damage before every use.
- This sleeve should only be used when in perfect condition.
- Do not wear this sleeve during sleep.

### **Risk of skin reaction and irritation!**

- In some cases, this bandage can cause skin to become irritated or cause an allergic reaction.
- Wash the areas of the skin which are to be covered by the bandage regularly in order to minimise skin reaction and irritation.
- In order to minimise skin reaction and irritation, the skin must be free of gel, grease, creams, balms, oils or similar. If problems persist, seek the advice of a doctor immediately.

## **Using this article**

**WARNING!** Ensure that the sleeve fits snugly but is not too tight. Ensure that the blood circulation in the forefoot and toes is not affected. Pull the bandage over the heel as shown.

## Cleaning and Care

- Before using the bandage for the first time, wash it thoroughly.
- It is advisable to carry out a further wash after 2 or 3 times of use.
- After washing, you should rinse your bandage thoroughly.
- Wash with similar colours only.

## Disposal

Please dispose of the product and packaging in an environmentally friendly manner, separating different materials for recycling as required!

Dispose of this item through an authorised disposal company or through your local waste disposal facilities - ask your local authority for information if further guidance is required. Ensure that you comply with all regulations currently in force.

---

<b>ES</b>	<b>☎</b> Línea directa de producto 00800-88080808 (gratuita de lunes a viernes 10 - 16 horas)
<b>IT</b> <b>MT</b>	<b>☎</b> Numero verde 00800-88080808 (Lu-Ve dalle 10 alle 16, gratuito)
<b>PT</b>	<b>☎</b> Linha de apoio ao cliente 00800-88080808 (2.ª a 6.ª. 10:00 - 16:00 horas, chamada gratuita)
<b>GB</b> <b>MT</b>	<b>☎</b> Product hotline 00800-88080808 (Mon.-Fri. 10am - 4pm, free phone)

---