
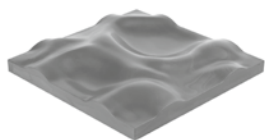
terrasensa®

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Operating Manual //
Manual de instrucciones //
取扱説明書

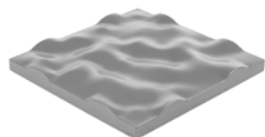
terrasensa® classic


terrasensa® flat

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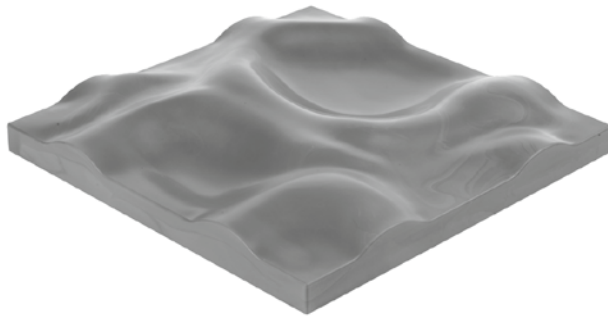terrasensa® classic

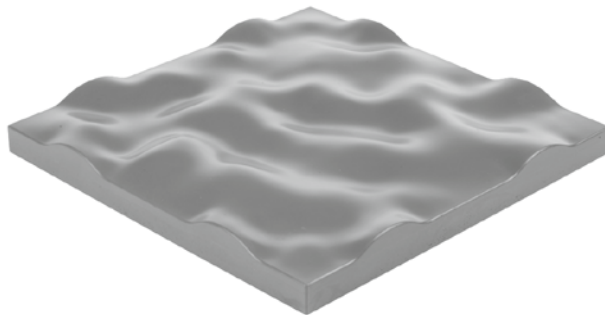terrasensa® flat

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is according to class I and correspondes to the basic requirements of the MDD 93/42/EEC in consideration of the change of the directive 2007/47/EC

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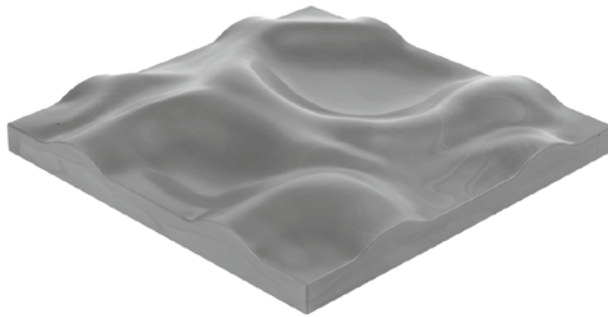
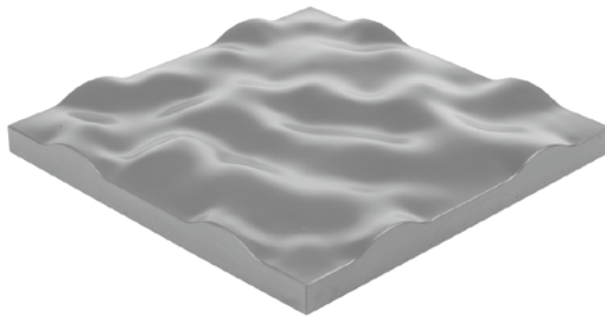
terrasensa® flat

Contents

1.	Legal Note	18
2.	Foreword	18
3.	Safety Instructions	19
4.	Package Contents	19
5.	Intended Use	19
5.1	Users	20
5.2	Organisational Measures	21
5.3	General Safety Instructions	21
5.3.1	Safety Precautions for the User	21
5.3.2	Safety Precautions for the Operator	22
6.	Installation	22
7.	Use	23
8.	Cleaning	24
9.	Transport and Storage	25
10.	Disposal	26
11.	Dimensions and Weight	26
12.	Symbols on the Product	26
13.	Optional Modules	27
14.	Declaration of Conformity	28
15.	Contact	29

1. Legal Note

This manual is part of the health product delivered by Hübner GmbH and describes the proper use of terrasensa® classic or terrasensa® flat.

As for the rest, the copyright law is applicable (Copyright®). Hübner GmbH owns the copyright of this operating manual. Duplication or editing by a third person is prohibited.

terrasensa® classic or terrasensa® flat are products of the brand sensa® – by Hübner Group. terrasensa® and sensa® are internationally registered trademarks. Unauthorised use will be prosecuted.

We will be thankful for suggestions with respect to the improvement of our products and relevant documentations. For any correspondence, please use the following addresses.

Document/output: Operating manual terrasensa® classic/flat / As of: 08/2012

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www.teo-kassel.de

2. Foreword

Before using terrasensa® classic or terrasensa® flat, please familiarise yourself with the entire manual; especially, the safety, cleaning and usage instructions. Make sure that every person who uses terrasensa® classic or terrasensa® flat is also aware of this information and follows the same.

The products are articles of daily use, whose safety in use and usability can only be maintained with regular checking and cleaning. Therefore, the user(s) must pay attention and adhere to this operating manual. The manufacturer will not be liable for personal injuries or material damage resulting from an improper use or non-adherence to this operating manual or other kinds of misuse of the product.

The latest valid editions of the user information, especially of the operating manual, can be found at www.sensa-hubner.com. On the website, you will also find further information and examples for the use of the purchased product.

3. Safety Instructions

Safe use of terrasensa® classic or terrasensa® flat depends mainly on factors that are under your control. These factors are specified in the safety instructions of this and the following chapters.

NOTE!

... refers to information that serves for better understanding.

CAUTION!

... indicates a danger with a low risk level, which, if not avoided, can result in minor or moderate injuries.

ATTENTION!

... indicates a danger to physical health of the user, the product, the material or the environment.

4. Package Contents

One or multiple terrasensa® classic or terrasensa® flat
(depending on the packaging unit: single package or set of six) -

terrasensa® classic / HÜBNER – product number: 01850906000;
terrasensa® flat / HÜBNER – product number: 01859981500

operating manual and user information (e.g. brochure or training poster).

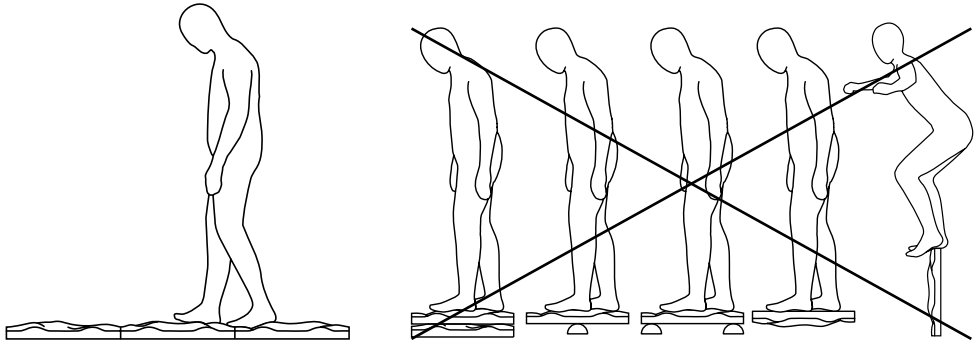
Before using the product, please check terrasensa® classic or terrasensa® flat and the operating manual for damage and verify that all contents are included! In case of any complaints, please contact the dealer.

5. Intended Use

terrasensa® classic or terrasensa® flat is a health product and training device for exercise therapy and sport-oriented use. Areas of application for terrasensa® classic and terrasensa® flat are sensomotoric and proprioceptive training, coordination training as well as physio- or ergotherapy e.g. in rehabilitation.

The exercises are designed (by an expert) depending on the motor skills and performance targets of the user (examples of exercises can be found at www.sensa-hubner.com).

As directed, terrasensa® classic or terrasensa® flat is exclusively used for standing, walking, jumping, crawling, sitting or lying exercises. The usage area is exclusively the relief-like shaped upper side. The tile should be used with the usage side pointing upwards only (not vertical or bent or laid out on an uneven surface). The under side of terrasensa® classic or terrasensa® flat, recognisable by the deepened technical label and four anti-slip supports, serves for secure positioning on suitable surfaces.

5.1. Users

An important part of the safety concept of the device is the operating manual. Thus, all persons, who:

- prepare,
- set up,
- use,
- dismantle,
- clean and disinfect,

the terrasensa® classic or terrasensa® flat, must read the operating manual and user information. Please pay special attention to the safety instructions in every chapter.

The device may only be used by persons, who have been instructed for proper handling or modular installation of the elements under consideration of this operating manual. The instructions may only be given by persons, who are eligible to give them based on their knowledge and experience.

5.2. Organisational Measures

For the intended use, remove terrasensa® classic or terrasensa® flat from the packaging. To open the packaging, do not use tools that can damage terrasensa® classic or terrasensa® flat by cutting, pricking or in another manner.

terrasensa® classic or terrasensa® flat may only be used on floor coverings that are suitable for it. These must be even, clean (free from dust and grease), tightly fixed or anti-slip so that the tile cannot slip or tilt. Choose an exercise area that offers optimal free space and best possible safety on all sides, since the use of terrasensa® classic or terrasensa® flat as directed can lead to a fall. A safe distance of 1.50 m – 2 m from objects (and other persons exercising) must be maintained around the exercise area.

terrasensa® classic or terrasensa® flat must only be used at ambient and surface temperatures ranging between -10°C and +50°C; pay attention to direct sunlight through windows.

5.3. General Safety Instructions

terrasensa® classic or terrasensa® flat is entirely made of polyurethane plastic. This plastic is free from softeners and dermatologically safe. In the tests conducted by independent testing institutes (LLNA-Local Lymph Node Assay and Acute Dermal Irritation), an allergic effect on skin and respiratory tracts was not proven. The integral foam system used corresponds to the requirements of the directive 2002/95/EC and fulfils the REACH requirements. It is free from arsenic compounds, asbestos, chlorofluorocarbon compounds, chromium compounds, formaldehyde and fibre-like elements.

terrasensa® classic or terrasensa® flat is waterproof in case of an intact surface (no cracks in the outer surface) thanks to the material characteristics. If the outer surface is damaged due to improper handling, then the open-cell foam core can absorb, store and release liquids.

At damaged points, breakage of foam parts cannot be ruled out (choking hazard for babies and small children). Damaged tiles can no longer be fully cleaned and disinfected.

5.3.1. Safety Precautions for the User

- Consult your doctor, therapist or an expert trainer before starting the training for the first time;
- Stop the training immediately if weakness, dizziness or pain occurs during the training;
- Always be careful when stepping on and getting down from terrasensa® classic or terrasensa® flat;

- Read the operating manual before using terrasensa® classic or terrasensa® flat;
- Verify the proper condition of terrasensa® classic or terrasensa® flat;
- Exercises especially designed for the user may only be done under guidance or observation.

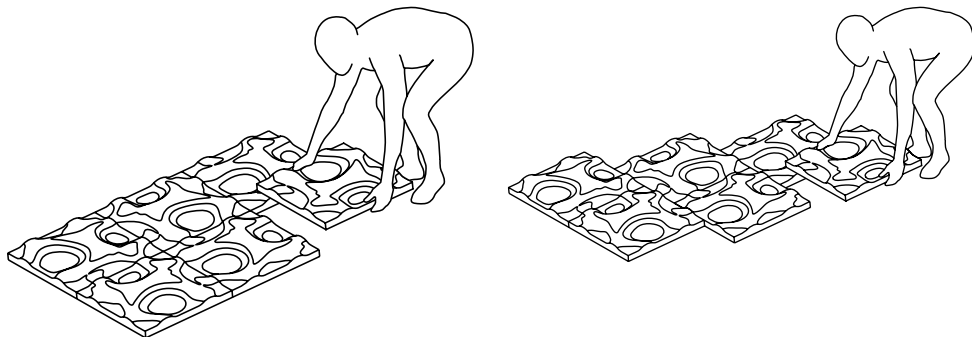
5.3.2. Safety Precautions for the Operator

- Make sure that the user has taken all safety precautions;
- Read the operating manual carefully before installation, maintenance or use;
- Make sure that all users are instructed for the proper use of the device;
- Make sure that terrasensa® classic or terrasensa® flat is installed and operated on a stable surface. Do not use terrasensa® classic or terrasensa® flat on an uneven surface;
- Check surface temperature;
- Make sure that there is sufficient space and free area;
- Pay attention to cleaning and hygiene instructions before re-use;
- Do not clean the product with corrosive substances, strong alkaline solutions or other strong solvents;
- Replace worn and damaged terrasensa® classic or terrasensa® flat;
- Exercises especially designed for the user may only be done under guidance or observation.

6. Installation

terrasensa® classic or terrasensa® flat is, as illustrated in the figure below, laid flat on suitable surfaces. terrasensa® classic or terrasensa® flat can be safely used on all DIN-conform floor coverings. Before every use, the adhesion of the lower side (anti-slip supports) on the base should be checked and if necessary, the base and lower side (anti-slip supports) should be wiped clean to restore the adhesion. The protective foil on the four anti-slip supports must be removed before using terrasensa® classic or terrasensa® flat for the first time.

When working with more than one terrasensa® classic or terrasensa® flat elements, these must be laid without gaps, in a full or half interlock pattern.

terrasensa® classic or terrasensa® flat can inadvertently move on uneven surfaces, hindrances or objects.

➔ Ensure a suitable surface to lay the tiles.

If terrasensa® classic or terrasensa® flat is used on asphalt surfaces or surfaces with stony or pointed bases, this can result in damage.

➔ Avoid use on such surfaces.

terrasensa® classic or terrasensa® flat can slip on smooth floors.

➔ Before using for the first time, the protective foil on the anti-slip supports must be removed. Before use, the adhesion of the anti-slip supports should be checked and if necessary, they should be wiped clean to restore the adhesion.

7. Use

As directed, terrasensa® classic or terrasensa® flat is exclusively used for standing, walking, jumping, crawling, sitting or lying exercises. The usage area is exclusively the relief-like shaped upper side.

terrasensa® classic or terrasensa® flat can be used barefoot or with suitable footwear such as gym shoes.

The exercises are designed for the user by an expert (therapist or trainer) depending on the motor skills and performance targets of the user.

Examples of exercises can be found online at www.sensa-hubner.com

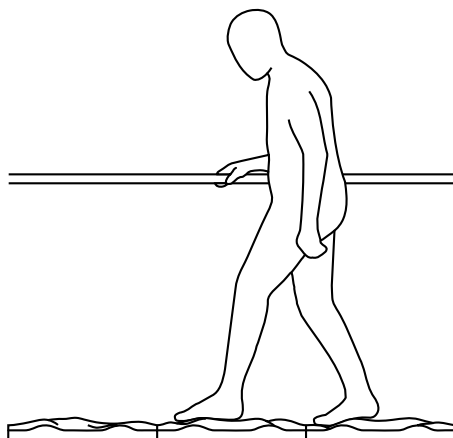
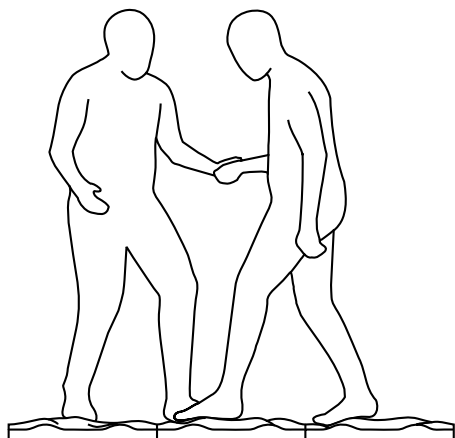
When using terrasensa® classic or terrasensa® flat for the first time or when stepping on it, there is a risk of falling due to the unusual shape or the weak sensorimotor function of the user.

➔ The free space around the exercise area must be between 1.5 and 2 metres.

Exercises on terrasensa® classic or terrasensa® flat by persons with considerable health restrictions, with accessories such as balls, rings, strings, etc. or with a special training programme may only be done under the guidance and with the help of a therapist or trainer.

There is a risk of falling in case of a wet, frozen, sticky surface or moist feet, slippery socks or slippery soles.

- ➔ Maintain surfaces in a dry and hygienically clean condition.
- ➔ Make sure that the user's feet are dry or that shoe soles are slip resistant.

Before using, check terrasensa® classic or terrasensa® flat for damage.

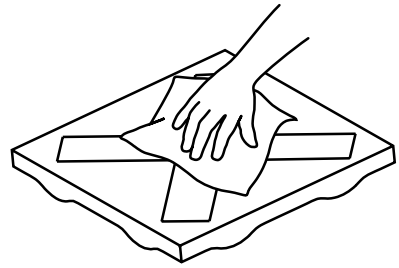
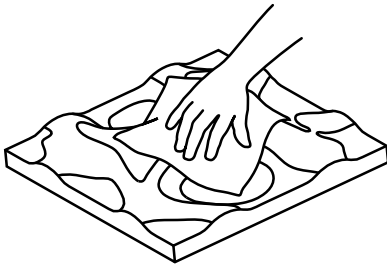
- ➔ Damaged terrasensa® classic or terrasensa® flat elements must be replaced.

8. Cleaning

The terrasensa® classic or terrasensa® flat elements get contaminated due to different types of use or due to continuous change of users. Therefore, clean and disinfect them intensively and thoroughly after every use.

If surface adhesion of the anti-slip supports reduces considerably, then these must be wiped clean. In case of dry deposits, terrasensa® classic or terrasensa® flat can be cleaned with a damp cloth and in case of moist or greasy deposits, with suitable cleaning agents. Cleaning and wipe disinfection as per clinical standards and norms can be done by accordingly qualified persons.

terrasensa® classic or terrasensa® flat can be used again only after restoring the slip resistance.

Clean and disinfect terrasensa® classic or terrasensa® flat with a damp cloth or as per the applicable cleaning and disinfection schedule.

Use of cleaning agents and disinfectants can result in skin irritations.

→ Please pay attention to the relevant safety instructions.

The use of solvents, strong alkaline solutions or other corrosive chemicals can damage terrasensa® classic or terrasensa® flat.

→ When using a cleaning agent and/or disinfectant, pay attention to its properties.

9. Transport and Storage

When terrasensa® classic or terrasensa® flat is not in use, it can be stored with or without the packaging. Preferably, store it vertically instead of horizontally, because this makes it easy to handle the elements and terrasensa® classic or terrasensa® flat is thereby compressed (squeezed) only to a limited extent. Due to the relief-like structure, terrasensa® classic or terrasensa® flat cannot be stacked horizontally in a tilt-proof way.

terrasensa® classic or terrasensa® flat should be stored in clean and well-ventilated rooms with average humidity.

Ultraviolet and infrared radiation (UV-IR radiation – e.g. from sunlight or fluorescent lamps) affects the characteristics and adhesion of terrasensa® classic or terrasensa® flat.

→ Avoid regular and long periods of storage (and continuous use) in sunlight or artificial lighting.

10. Disposal

Dispose of terrasensa® classic or terrasensa® flat through a communal or authorised waste management company. Thereby, pay attention to the applicable regulations.

11. Dimensions and Weight

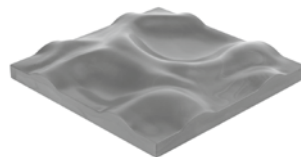
Dimensions: approx. 500 x 500 x (up to) 77 mm

Weight: approx. 3,200 g

Packaging units:

Single package: 507 x 507 x 80 mm – approx. 3.5 kg





Set of six: 505 x 505 x 430 mm – approx. 20 kg

12. Symbols on the Product

Symbol	Meaning
	CE symbol as per the European directives 93/42/EEC and EC 2007/47
	Pay attention to operating manual
	Manufacturer's product number (identification number)
	Material code for polyurethane

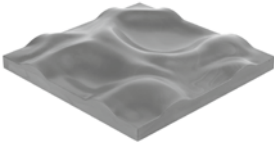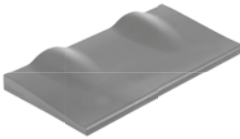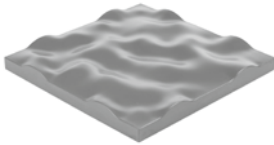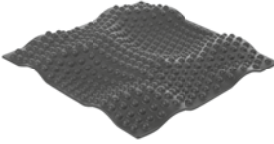13. Optional Moduls

Picture	Name	Meaning
	terrasensa® classic	The structure tiles can be laid on a surface in full or half interlock pattern (refer to point 6). terrasensa® classic tiles interlock on the face side.
	terrasensa® rise	terrasensa® rise serves as an ascending aid for terrasensa® classic or terrasensa® flat. terrasensa® rise interlocks with terrasensa® classic or terrasensa® flat on the face side.
	terrasensa® flat	terrasensa® flat is a structure tile similar to terrasensa® classic, only with a flatter and less demanding relief structure. terrasensa® flat interlocks with terrasensa® classic or terrasensa® rise on the face side.
	terrasensa® 3D-reflex	terrasensa® 3D-reflex is a support tile for terrasensa® classic with foot reflex zone profile. terrasensa® 3D-reflex interlocks only with terrasensa® classic.

More products of the brand sensa® – by Hübner Group can be found at:
www.sensa-hubner.com

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Name und Anschrift des Herstellers: HÜBNER GmbH
HÜBNER GmbH