CAREPLAN® VpH

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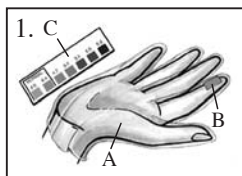
CAREPLAN[®] VpH

Test glove for measuring the vaginal pH, as developed by Prof. Saling and Dr. Heimann.

Please read the entire instruction leaflet carefully.

PACK CONTENTS

This pack contains 50 hygienically packed test gloves coated with a pH indicator, a instruction leaflet and a pH colour scale.

A. Glove

B. pH indicator area
on the index finger

C. Colour scale (on the
back of the pack)

INDICATIONS

The CarePlan[®] VpH test is designed for measuring the acidity of the vaginal environment (vaginal pH). In most cases, the vaginal pH provides an early indication of any abnormality in the vaginal environment. These abnormalities often encourage the development of infections. Infections of this kind are a frequent cause of premature births (= birth before the week 37 of pregnancy), low-birth-weight babies, and reduced fertility.¹⁻⁹

The micro-organisms in the vagina usually keep each other in balance; Lactobacilli predominate and are responsible for the acidic pH (the normal pH is between 4.0 and 4.4). This natural protection ensures that disease-causing micro-organisms are not so easily able to spread, to ascend into the womb and to trigger labour pain there or to cause premature bursting of the waters and therefore premature birth. The increasing colonisation of the vagina by infectious bacteria often goes unnoticed by the mother-to-be. If an infection of this kind is detected early, prompt medical treatment can be given and the risk of premature birth reduced. This is why it is so important to start measuring the pH of the vagina regularly from as early as possible in the pregnancy.

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