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DIGITAL AUTOMÁTICO DE BRAÇO

MG-150f

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Características:

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Fabricante

Rossmax International Ltd
 12 F, NO189, Kang Chien Road, 11
 Taipei, Taiwan/China

Blood Pressure Monitor


DIGITAL AUTOMATIC ARM TYPE

MG-150f

Instruction Manual

Features:

- Automatic
- Fuzzy Technology
- 90 memory entries
- One Touch Operation

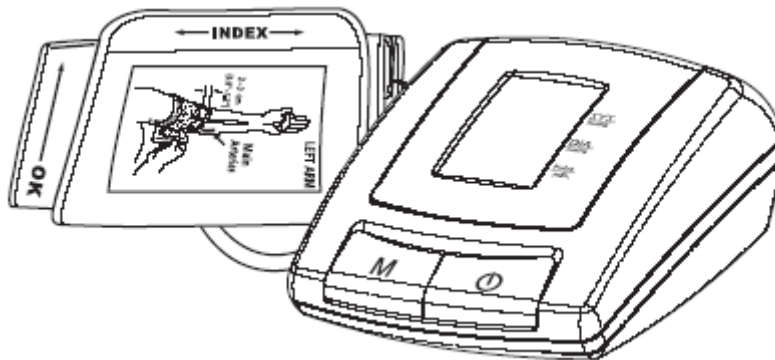CONSULT YOUR PHYSICIAN IF YOU FEEL VARIATION IN YOUR PRESSURE RESULT. THE USE OF THIS PRODUCT DOES NOT DISPENSE THE MEDICAL ADVICE.

DO NOT USE THE PRODUCT FOR INVASIVE MEASUREMENT

Introduction

Blood pressure measurements determined with MG-150f are equivalent to those obtained by a trained observer using cuff/stethoscope auscultation method, within the limits prescribed by the American National Standard, Electronic or Automated Sphygmomanometers. MG150f is to be used only by adults for personal use.

Attention: Please read this manual carefully before use; for specific information on your own blood pressure, contact your physician. Please be sure to keep this manual.

Real Fuzzy Technology

The MG-150f uses oscillometric method to detect your blood pressure. Before the cuff starts inflating, the device will establish a baseline cuff pressure equivalent to the air pressure. Your MG-150f will determine the appropriate inflation level based on one's pressure oscillations, followed by cuff deflation. During the deflation, the device will detect the amplitude and slope of the pressure oscillations and thereby determine the systolic, diastolic and pulse for you.

About Blood Pressure

What is Blood Pressure?

Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest). The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.


Why Measure Your Blood Pressure?

Among the various health problems afflicting modern people, problems associated with high blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressure a necessity of identifying those at risk.

Blood Pressure Standard

The World Health Organization (WHO) and National Blood Pressure Education Program Coordinating Committee have developed a blood pressure standard, according to which areas of low- and high- risk blood pressure are identified. This standard, however, is a general guideline as individual's blood pressure varies among different people and different age groups, etc.


It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk. For reliable monitoring and reference of blood pressure it is recommended to keep long term records.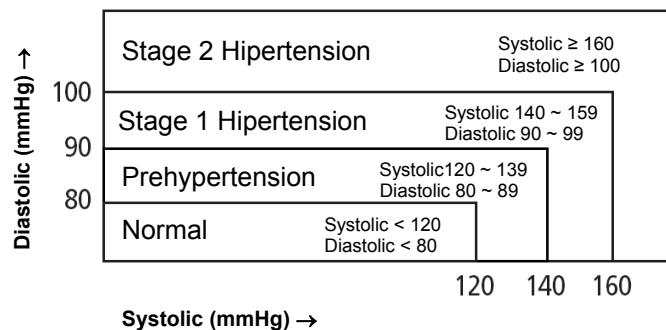


Blood Pressure Fluctuation

Blood pressure fluctuates all the time!


You should not be over worried if you encountered two or three readings at high levels. Blood pressure changes over the month and even throughout the day. It is also influenced by season and temperature.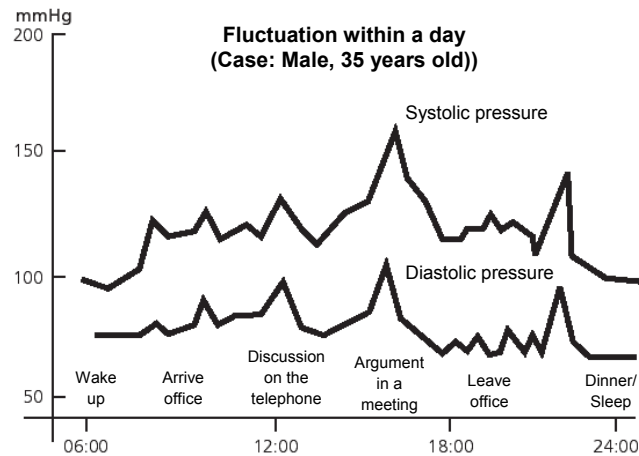

Name/Function of Each Part

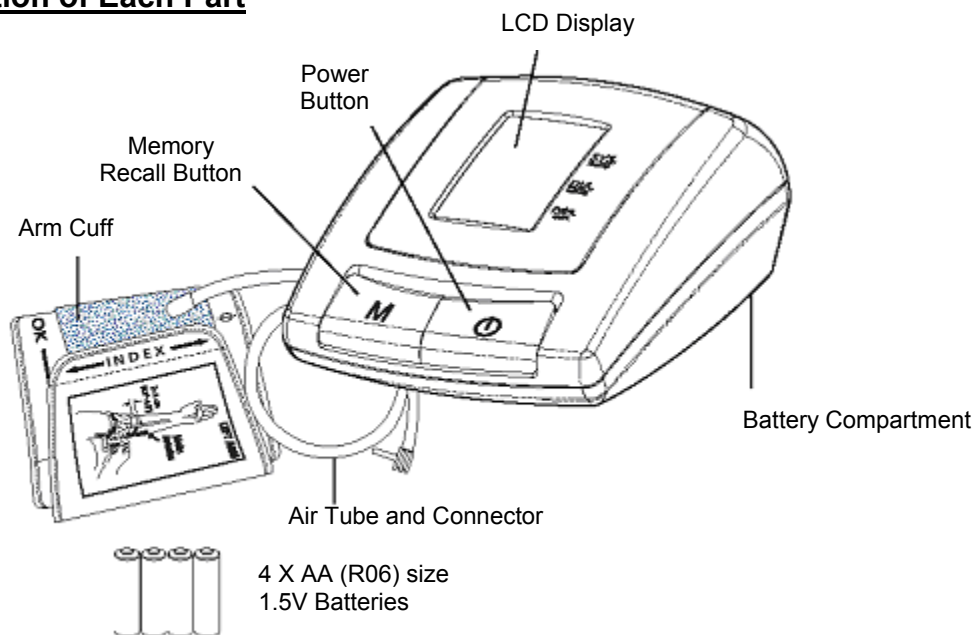Display Explanation

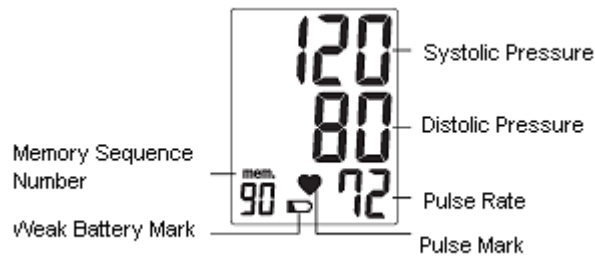Symbols:

Weak Battery Mark: Appears when batteries should be replaced

Pulse Mark: Shows the pulse rate per minute

mem.

Memory Mark: Shows the number of stored measurements

Measurements Error: Make sure the L-plug is securely connected to the air socket and measure again quietly. Wrap the cuff correctly and keep arm steady during measurement. If error keeps occurring, contact the service center.

Air Circuit Abnormality: Make sure the L-plug is securely connected to the air socket and measure again quietly. If error keeps occurring, contact the service center.


Pressure Exceeding 300 mmHg: Switch the unit off and measure again quietly. If error keeps occurring, contact the service center.

Data Error: Remove and reload the batteries. If error keeps occurring, contact the service center.

Batteries Install

1. Press down and lift the battery cover in direction of the arrow to open the battery compartment.
2. Install or replace 4 "AA" sized batteries in the battery compartment according to the indications inside the compartment.
3. Replace the battery cover by clicking in the bottom hooks first, then push in the top end of the battery cover.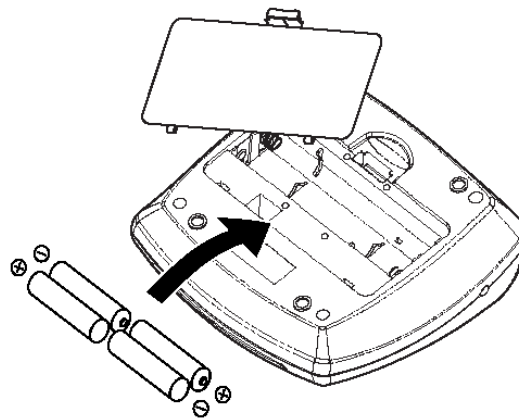

You need to replace batteries when:

1. The weak battery mark appears on display.
2. The  button is pushed and nothing appears on display.


Note:



Batteries are hazardous waste.
Do not dispose them together with the household garbage.

Applying the arm cuff


1. Plug in the cuff connecting tube into the unit.
2. Unwrap the arm cuff, leaving the "Index" end of the cuff through the D-ring of the cuff. Do not dispose them together with household garbage.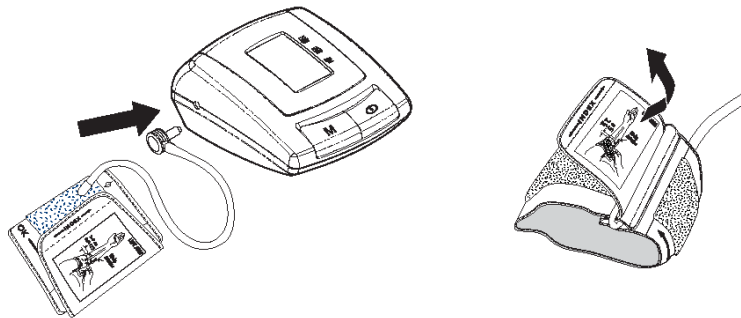


3. Put your left arm through the cuff loop. The "OK" range indication should be positioned closer to yourself with the rubber tube pointing in the direction of your arm. **Position the artery mark (ϕ) over the main artery (on the inside of your arm) in the upper arm.**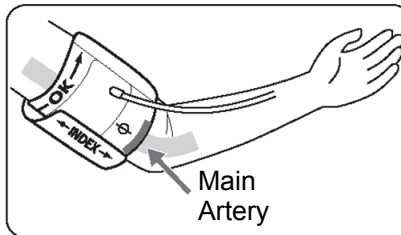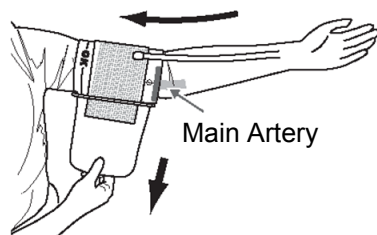

4. Turn your left palm upward and place the edge of the arm cuff at approximately 2 to 3 cm above the inner side of the elbow joint. Tighten the cuff by pulling the "Index" end of the cuff.

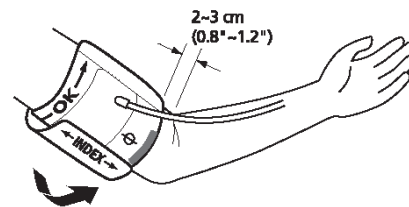
5. **If the Index line falls within the OK range, indicated at the edge of the cuff, this cuff is suitable for your use.** If the Index line falls out of the OK range, you may need a special cuff for measurement.
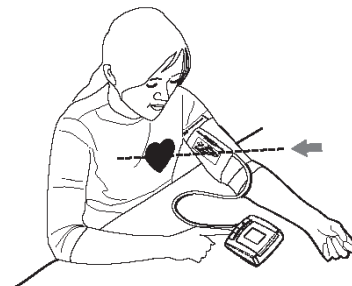


Taking a Measurement

1. Press the POWER button. All display will appear for approximately one second before returning to "0".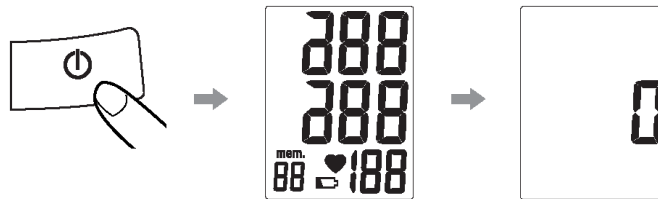


2. The unit will **automatically inflate to the appropriate inflation level based on the user's pulse oscillations**. Measurement will then begin. **It is important to remain still and quiet during measurement. Any significant movement may affect measurement results.**
3. When the measurement is completed, systolic pressure, diastolic pressure and pulse rate will be **shown simultaneously** and be **saved automatically** in memory system. Up to **90 memories** can be saved.
4. Measurement is now completed. Press the  button to turn off the power. If no button is pressed, the unit will shut off automatically in 1 minute.

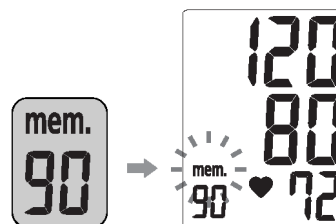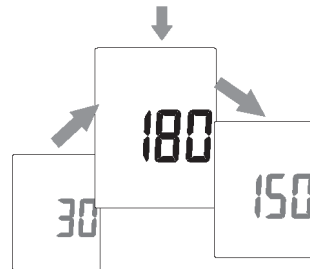
5. **Measurement readings:** The display will automatically show the systolic and diastolic pressure in mmHg after each measurement and the pulse rate/min too.

Please see the example below: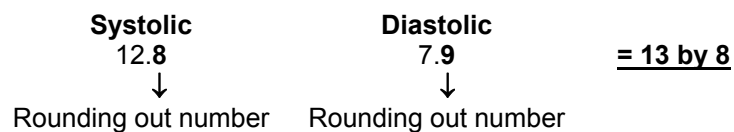

Memory

1. To recall stored blood pressure readings from memory, simply press the **"M"** button. The last set of memorized readings will be displayed.
2. Another press of the **"M"** button will recall the previous set of readings.

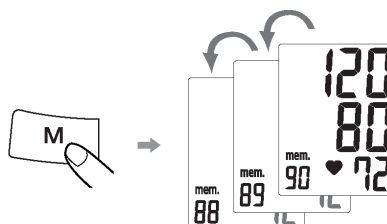3. You can erase the memories stored by:
 - Remove the batteries.
 - Press and hold the memory recall key for more than 5 seconds.
4. The data in the memory zone can be erased if any of the batteries is moved away. (We suggest the users to first record the data before replace or move away the batteries.)

Helpful Tips

Here are few helpful tips to help you obtain more accurate readings:

- Do not measure your blood pressure immediately after consuming a large meal. To obtain more accurate readings, please wait one hour before measuring.
- Do not smoke or drink alcohol before measuring your blood pressure.
- It is important that you relax during measurement. Try to take a 20-minute rest before a reading.
- Do not take measurements if you are under stress or under tension.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- **Wait about 20 minutes before taking the next pressure measurement.**

Troubleshooting

If any abnormality should arise during use, please check the following points.

Symptoms	Check Points	Correction
No display when the POWER button is pressed	Have the batteries run down?	Replace them with two new batteries.
	Have the batteries' polarities been positioned incorrectly?	Re-insert the batteries in the correct positions.
EE mark shown on display or the blood pressure value is displayed excessively low (high)	Is the cuff placed correctly?	Wrap the cuff properly so that it is positioned correctly.
	Is the cuff pressure pumped enough? (50~60 mmHg above normal systolic value?)	Measure it again and pump up 50~60 mmHg more than normal systolic value.
	Did you talk or move during measurement?	Measure it again quietly. Keep arm steady during measurement.
	Did you shake the arm with the cuff on?	

Note: If the unit still does not work, return it to your dealer. Under no circumstance should you disassemble the unit and attempt repairs yourself.

Cautionary Notes

1. The unit contains high-precision components. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
2. Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
3. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
4. The unit should not be operated by children so to avoid hazardous situations.
5. If the unit is stored near freezing, allow it to acclimate at room temperature before use.

6. For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device may have difficulty in determining the proper blood pressure.
7. To stop operation at any time, press the air release valve, and the air in the cuff will be rapidly exhausted.

Specifications

Measurement Method	: Oscillometric
Measurement Range	: Pressure: 40~250 mmHg; Pulse: 40~180 beats/ minute
Pressure Sensor	: Semi conductor
Accuracy	: Pressure: ± 3 mmHg; Pulse: $\pm 5\%$ of reading
Inflation	: Pump Driven
Deflation	: Automatic Pressure Release Valve
Memory capacity	: 90-set memory
Auto-shut-off	: 1 minute after last button operation
Operation Environment	: 10°C~40°C (50°F~104°F); 40%~85% RH max.
Storage Environment	: -10°C~60°C (14°F~140°F); 10%~95% RH max.
Power Source	: 6VDC, four R06 (AA) batteries
Dimensions	: 137 (L) X 140 (W) X 64 (H) mm
Weight	: 400 g (G.W) (w/o Batteries)
Arm circumference	: Adult: 24~36 cm (9,4"~14,2")
Limited users	: Adult users
K.B.	: Short time operation 2 minutes.
	: Type BF Device and cuff are designed to provide special protection against electrical shocks.

*Specifications are subject to change without notice.

Warranty

1. This product is guaranteed for 2 years from the date of acquisition and this guarantee is effective only by presenting the **RECEIPT OF PURCHASE** stating the buyer's name, date of purchase, name, and address of the distributor.
2. Only defective products with manufacturing problems are covered by this guarantee. Devices with malfunctioning problems caused by misuse or improper handling even if not intentionally, are not covered by guarantee.
3. The guarantee covers the functioning of the monitor with the exception of the cuff because it is a fragile piece and if handled improperly it can blow up or to be punctured, etc.
4. This guarantee does not cover damages by dropping down the device, direct exposition to high humidity, water, battery leakage, and the batteries included in the device, repairs executed by unauthorized person, violation of the device by the customer or by unauthorized person.
5. The guarantee does not cover the cost of round-trip transport for repair, the result or damage caused by improper use of the device and other expenses not specified here.
6. The guarantee is good only in countries where this product is officially commercialized by the manufacturer or the local distributor and authorized by the manufacturer, always with the presentation of the receipt of the purchase.
7. This is a non-transferable guarantee; it means it is good for the first consumer only.
8. During the period of this warranty the producer is obligatory to repair the product 30 days from the date of receiving the device for repair.
9. The manufacturer reserves the right to replace the defective product by a new one in case of need and decision of the manufacturer.
10. All repairs executed during the period of warranty do not extend the deadline of the guarantee.
11. After the guarantee is expired, the manufacturer has to manage to reserve components and to supply technical assistance during 5 years in order to offer paid-services to users.

12. During the warranty period all services of maintenance and of consumed components under bad manipulation are not acceptable.
13. The periodical revision/calibration of the product is not covered by the guarantee.
14. The technical assistance and warranty are restricted to the Brazilian territory only.
15. This guarantee does not cover damages by misuse or improper handling, even if not intentional, dropping down the device, direct exposure to high humidity, water, and sunshine.

Customer Service Center

From Monday to Friday (08:00 a.m. to 06:00 p.m.).

In Brazil: São Paulo and Grande São Paulo: Phone: (11) 3813-1092.

Other regions: 0800-555-750.

Other countries: please contact your local distributor.

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