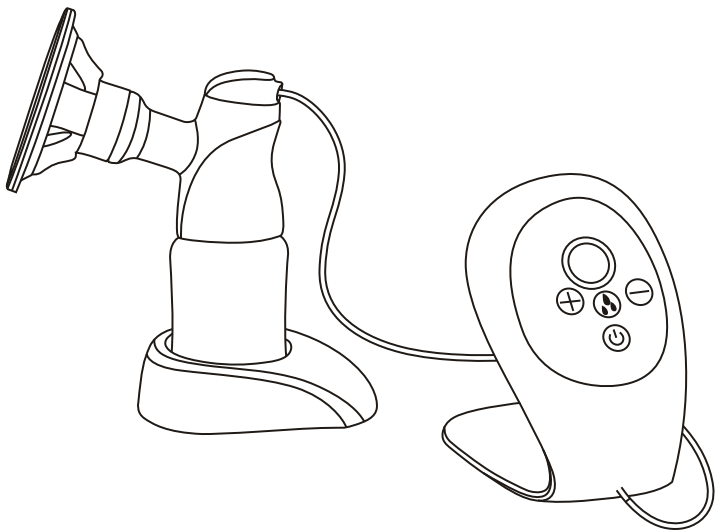
# SACALECHES ELECTRICO

## MANUAL DE INSTRUCCIONES

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A large rectangular area containing horizontal lines for writing. Each line set consists of a solid top line, a dashed midline, and a solid bottom line. There are 10 such sets of lines filling the page.

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**INNOVACIONES M.S. S.L.**

Pol. Ind. La Cava.

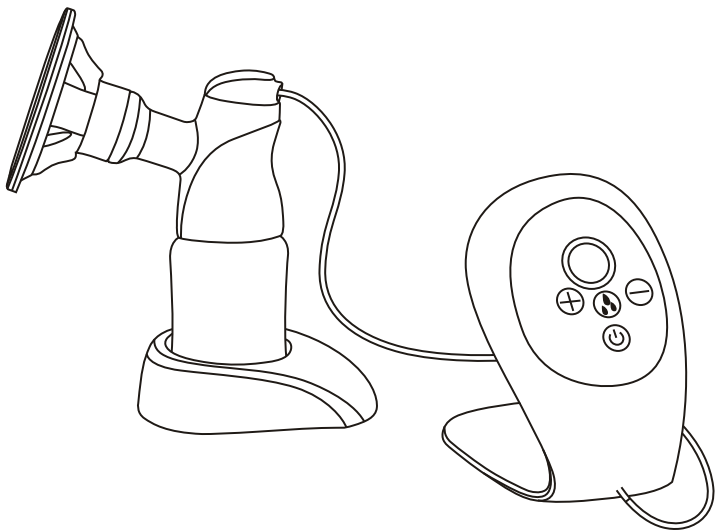
C/ Benigànim, 9

46892 MONTAVERNER (Valencia) Spain

Tel. 96229 76 70 Fax 96 229 76 76

# ELECTRIC BREAST PUMP

## INSTRUCTIONS

# Congratulations

Electric Breast Pump is perfect for many occasions.

- Collecting breast milk while you're away from your baby.
  - For example: when you're at work Relieving engorgement.
  - For example: when your baby's feeding habits change because they're eating more solids.
- Maintaining your milk supply if you have to temporarily stop breast feeding.
  - For example: to express unsuitable milk while you are undergoing medical treatment

The Breast Pump provides many of the features busy moms love.

- 2 cycles of expression to mimic babies sucking action
- Stimulation Level 1-4
- Expression Level 5-9
- Quiet motor for discretion
- Small and lightweight
- Two power options to fit your needs
- AC Adapter
- Rechargeable battery
- Easy to clean Anti-counter fluxion design can preserve the breast pump lasting for using, preventing milk from flowing into the electric pump.

|                                                      |           |
|------------------------------------------------------|-----------|
| <b>Indication for Use .....</b>                      | <b>22</b> |
| <b>Important Safeguards .....</b>                    | <b>22</b> |
| · DANGER .....                                       | 22        |
| · WARNING .....                                      | 22        |
| <b>Parts List.....</b>                               | <b>23</b> |
| · Breast Pump Part.....                              | 23        |
| · Feeding Bottle Part .....                          | 23        |
| <b>Description of the electrical properties.....</b> | <b>24</b> |
| <b>Specification .....</b>                           | <b>24</b> |
| · Clean and Sterilize your pump .....                | 24        |
| · Assemble your pump .....                           | 25        |
| · Powering your pump.....                            | 27        |
| · Using your pump.....                               | 27        |
| · When to use your pump .....                        | 28        |
| · Hints to help you succeed .....                    | 29        |
| · How to store the milk.....                         | 30        |
| · Troubleshooting Guide.....                         | 30        |
| <b>Warranty Card .....</b>                           | <b>32</b> |

# Important Safeguards

## **READ ALL INSTRUCTIONS BEFORE USING**

### **DANGER: To reduce the risk of electrocution**

- Do not use while bathing.
- Do not place or store this product where it can fall or be pulled into a tub or sink.
- Do not place or drop into water or other liquid.
- Do not reach for an electrical product that has fallen into water.

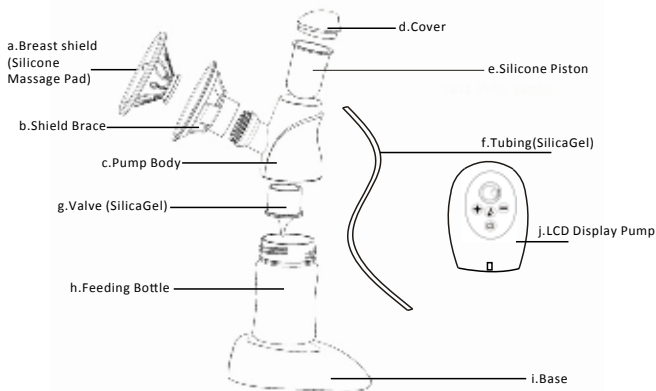
### **WARNING: To reduce the risk of burns, electrocution, fire, or injury to persons:**

- Close supervision is necessary when this product is used near children or invalids.
- Use this product only for its intended use as described in this manual. Do not use attachments not recommended by MyCarol.
- Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been dropped into water.
- Keep the cord away from the heated surfaces.
- Never use while sleeping or drowsy.
- Never drop or insert any other object into any opening or tubing.
- Make sure the voltage of the AC adaptor is compatible with the power source.
- Only use the AC adaptor that comes with your Split Type Electric Breast Pump.
- This is a single user product. Used by more than one person may present a healthy risk.
- Do not operate a motor vehicle while using the Electric Breast Pump.
- Do not attempt to modify the product.

## SAVE THESE INSTRUCTIONS

Along with your receipt for future reference.

### I Breast Pump Part

### II Feeding Bottle Part

k. Cap

l. Extra Soft Nipple

m. Spiral Lid

## Parameter

- I. Battery Voltage : 3.7 VDC
- II. Battery Capacity: 850 MAH
- III. Battery Charger Standards  
Input: AC115V-230V  
Output: 4.5VDC,0.4A
- IV. Charging Time: 2.5 hours

## Specification

### I Clean and Sterilize your pump.

Please clean and sterilize your pump for the first time and after use Sterilizing Way.

1. Pull the LCD Display Pump (j) to relocate it elsewhere. Separate all parts, a.b.c.d.e.f..g.h.i.k,l,m, then wash in warm soapy water and rinse. Do not use antibacterial or abrasive cleaners/detergents.

2. Sterilize all parts in steam sterilizer, or by boiling for 10 minutes. If using a steam sterilizer, the pump will remain sterile in the sterilizer for a minimum of 4 hours, provided the lid is not removed.

**Caution: NEVER WASH OR STERILIZE the LCD Display Pump (j). You can wipe it using a soft cloth.**

## Suggestion

1. Use steam sterilizer for sterilize. Not recommended boiling. Firstly, the product is easy to damage if improper sterilization. Secondly, this unscientific method of sterilization can not achieve the complete effect. If use boiling, note that be more water in pot, float the product, people do not leave. Not too long.

2. For the lactating mothers, breast relatively high frequency (typically 2-3 hours once), not suggest clean and sterilize each use. Excessive disinfection and cleaning accessories could easily lead to premature aging; reduce the life of a breast pump. Thoroughly clean the stain and steam sterilized once a day is recommended.

3. Store the pump parts in a clean container until next use.

**(Caution: If the breast pump is for the purpose of storage, be sure to disinfect all breast pump parts, otherwise, milk easy to go bad, difficult to store)**

## **II. Assemble your pump.**

1. Wash hands, insert the Breast shield (Silicone Massage Pad) (a) into the Shield Brace (b)

2. Insert the Silicone Piston (e) into the pump body (c)

3. Twist the Shield Brace (b) onto the pump body(c), rotating clockwise gently until securely in place.

4. Place the Valve (g) into the pump body(c), making sure it fits securely around the edge.(Caution: If the valve is lost or not properly installed, will not be able to use a breast pump).

5. Insert the feeding bottle (h) into the pump body (c), rotating it clockwise gently so that it sits securely with the pump body thread.

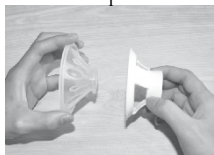
6. Put the cover (d) into the top of the pump body (c) properly. And twist the cover clockwise.

7. Insert one side of the tubing(f) into the pump body(c), another side into LCD Display Pump(j), then, breast pump can use.

8. To make the breast pump place more firmly, insert the feeding-bottle into the base (i).

# Mounting your pump

1

2

3

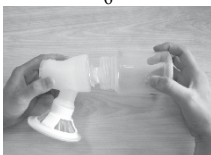
4

5

6

7.1

7.2

7.2

8

9

10

## III Powering your pump.

### Power Operation:

1. Plug the power adaptor cord into the LCD Display Pump.
2. Plug the power adaptor into the main socket.

## IV. Using your pump.

1. Wash hands thoroughly and make sure your breasts are clean. Squeeze a little breast milk from each breast to ensure no blockage.
2. Make sure your breast pump sterilized and installed.
3. Relax in a comfortable chair, leaning slightly forward (use cushionsto support your back).
4. Adjust suction, Frequency:

- Switch on the breast pump with , the LCD Display light and show stimulating phase indicator on right.

- Position the Breast shield (Silicone Massage Pad) firmly onto your breast so as to create a perfect seal(do not let air into), you will feel suction against your breast.

- The first to fourth level is stimulating. Using the Increase Button , you can find your exact comfort level  for massage your breast.

- Press Switch Button  to cycle expression, after massage your breast for few minutes. Then the LCD Display show suction phase indicator on left.

- The fifth to ninth level is expression. If you feel suction and frequency is not enough, press Increase Button , suction and frequency will increase a level when you press once. Press Reduce Button , suction and frequency will reduce a level.

(Level 1-4)

(Level 5-9)

# When to use your pump

1. Make sure you milk flow successfully into the feeding bottle and feel comfortable for you with finding or adjusting your Maximum Comfort Vacuums

2. Don't worry if your milk doesn't flow immediately, relax and continue pumping. If the process causing you breast pain, stop immediately and consult a doctor at once.

**(Do not continue pumping for more than 5 minutes at a time if NO result is achieved. Try expressing at another time during the day)**

3. Only fill the feeding bottle to the 120ml mark. On average you will need to pump for 8 minutes to express 120ml. However, this is just a guide and varies from woman to woman. If you find that you regularly express more than 120ml per session, you can use a bigger feeding bottle.

**(Do not overfill the milk bottle, otherwise it may leak).**

4. Switch off the breast pump with when you stop use.

5. When your pumping session is over unplugging tubing from the back of the breast shield before setting down the bottle.

6. Close the bottle with a Spiral lid.

## **V. When to use your pump**

1. If possible. After sufficient in milk secretion and breast-feeding time normally. Usually two or four weeks after birth the baby.

2. In the following cases, you can always use your breast pump:

3. If you are breastfeeding for the BB in the hospital

4. If your breasts are bulging (with swelling), you can suck out a part between two feeding time to relieve pain, this also make it easier to suck for BB.

5. If your nipple pain or cracking, you can suck out the milk for feeding until the nipple back to normal.

6. If you need to separate with your BB and hope to continue breastfeeding, when reunion, breast milk can be sucked out periodically to maintain milk secretion.

**VI. Hints to help you succeed.**

Using a breast pump takes practice. You may need to make several attempts before you succeed.

1. Choose a quiet time when you are not rushed and unlikely to be disturbed.
2. Breast position usually sitting back and perpendicular to the ground. Please put in your back a comfortable cushion before breastfeeding. Sitting could make all breastfeeding process more relaxed and more comfortable. Make sure you have a glass of water nearby as expressing can be thirsty work.
3. Placing a warm cloth on your breasts for a few minutes before you start pumping can also encourage your milk flow and soothe painful breasts.
4. Warmth and relaxation can help so you might like to try expressing after a warm bath or shower.
5. Try expressing from one breast while your baby is feeding from the other, or continue expressing just after a feed.
6. Repositioning the pump on your breast from time to time can help stimulate your milk ducts.

**Practice with your pump to find the best pumping techniques for you. If the process causing you breast pain, stop immediately and consult a doctor at once.**

**7. Breast Pump Use Errors.**

Breast pump is a tool to help postpartum women to ease the pain and a helper to extend the breastfeeding time. It uses the premise that there is sufficient milk and smooth mammary gland. Some women are the first time to be a mother, often appear mammary gland not smoothly. Some of them were caused when a congenital nipple inversion, or milk water shortage. So we suggest to do a Professional breast examination confirmed that the mammary gland really smooth and milk under normal conditions before buy and use a breast pump. Do not blindly to buy a breast pump as a device to open milk.

# How to store the milk

## VII. How to store the milk

1. Only milk collected with a sterile pump should be stored to be fed to your baby.
2. Breast milk can be stored in the baby bottle warmer in 40, can be fed your baby within 4 hours.
3. If you are storing freshly expressed milk, please seal the milk, breast milk can be stored 10 hours in room temperature in 19-25 °C.
4. Expressed breast milk can be stored in the refrigerator for up to 48 hours (not in the door) or in the freezer for up to 3 months.
5. Frozen breast milk must first thaw in the refrigerator naturally. If urgent, can soak and thaw in the kettle. After thaw, breast milk can be stored in the refrigerator for 24 hours.
6. NEVER refreeze breast milk.
7. Frozen breast milk can be heated gradually in warm water or heated with baby bottle warmer.

**(Never heat milk in a microwave as this may destroy valuable nutrients and antibodies. In addition, the uneven heating may cause hot spots in the liquid which could cause burns to the throat. Never immerse a frozen milk container in boiling water as this may cause it to crack/split. Never put boiling water directly into a feeding bottle, allow cooling for around 20 minutes before filling.)**

## VIII. Troubleshooting Guide

1. Lack of suction.
2. Check that all pump parts have been assembled correctly and that the valve is positioned correctly.
3. Ensure that the silicone diaphragm and let-down massage cushion are fitted securely to the pump body and a perfect seal is created.
4. No milk being expressed.
5. Ensure the pump is correctly assembled and that suction is being created.
6. Relax and try again.
7. Pain in the breast area when expressing.

8. You may be squeezing too hard. You do not need to use all the suction to pump can generate.
9. Try to adjust the suction and frequency.
10. Consult your health professional/breastfeeding advisor.
11. Milk is drawn up the let-down Massage Cushion.
12. Remove and re-fit the let-down Massage Cushion to ensure it is firmly attached and pushed far enough down the centre of the funnel.
13. Try leaning slightly forward.
14. Cracking or discoloration of pump/bottle parts.
15. Avoid contact with antibacterial or abrasive detergents/cleaners as they can damage the plastic. Combinations of detergents, cleaning products, sterilizing solution, softened water and temperature fluctuations may, under certain circumstances, cause the plastic to crack. If this occurs, do not use. The pump body and breast milk containers are dishwasher safe but food colorings may discolor them. Follow guidelines in 'How to clean and sterilize your pump'. Keep the product out of direct sunlight, as prolonged exposure may cause discoloration.
16. Lost or broken parts.
17. Contact Happy Family Customer Services for replacements. (See back cover for contact details).

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|                    |         |                    |      |
|--------------------|---------|--------------------|------|
| Customer Name      |         | Customer Tel       |      |
| Product Name       |         | Model No .         |      |
| Purchase Date      |         | Purchase Shop      |      |
| Address            |         |                    |      |
| Post Number        |         |                    |      |
| Maintenance record |         |                    |      |
| Maintenance date   | Failure | Maintenance Result | Note |
|                    |         |                    |      |
|                    |         |                    |      |
|                    |         |                    |      |
|                    |         |                    |      |
|                    |         |                    |      |
|                    |         |                    |      |
|                    |         |                    |      |

The contents of this document as it is shown in the following part will be effective since the product is bought.

Innovaciones M.S., S.L. the official distributor located in Pol. Ind La Cava, C/. Beniganim, 9, 46892 MONTAVERNER (Valencia - Spain) guarantees the consumer that this product is new and it has not got any fault in the materials, design and manufacturing process and that it has got the features declared by the manufacturer.

This formal guarantee is valid for every country in the European Union and in all of them the consumer has the same right according to legal measures which say that the shop which sells the product has to offer this guarantee.

To benefit from this formal guarantee, the Consumer has to take back the faulty product to the shop when this product was bought showing the shop assistant the buying ticket of the product with the stamp of the shop and a signature indicating which product it is being sold, or, failing that, the bill which must not be modified and it clearly has to show the same information we have mentioned before.

The formal guarantee is valid for twenty-four (24) months since the purchase of the product. During this period INNOVACIONES M.S., S.L. will repair and change any faulty product according to its judgement.

The formal guarantee is for the final buyer of the product (Consumer) and it does not exclude, neither limits the rights of the Consumer according to the law and/or the rights which the Consumer has in relation to the shop which is selling this product.

## **THE WARRANTY DOES NOT APPLY TO THE FOLLOWING.**

The formal guarantee is excluded in case of any damaged produced to the product for the following reasons: if the product is used in a different way from the information given in the manual book of instructions for its assembly and use, if the product has been hit or has fallen down, if the product has been exposed to extreme humidity or heat or any negative environmental conditions, or sudden weather changes, when the products corrodes itself, it rusts or the product is repaired or

## FORMAL GUARANTEE

modified without previous authorization, or if it is repaired using spare pieces which are not authorized, if the use and maintenance of the product are not correct, or it is not made any maintenance at all not following the instructions on the manual, it is not assembled properly, it has some accident, it is damaged because the food or drink, or any chemical product or major force.

In any case, INNOVACIONES M.S., S.L., rejects any responsibility for damages caused to the people or the goods, which are not the product itself, if the damaged is caused because the user has not followed the instructions, advices or warnings which are in the manual book of instructions by the owner or any person who is using the product. (Just an example, never leave the child alone or the child has to wear the harness all the time...).

It is more, INNOVACIONES M.S., S.L., rejects any responsibility of damaged caused to people or goods when the breakage of the product has been caused due to the fact that the components have been damaged because of the use of the product. Every component in this product made of plastic are worn out because the daily use.

Imported by **INNOVACIONES M.S. S.L.**

Pol. Ind. La Cava

C/. Beniganim, 9

46892 **MONTAVERNER** (Valencia) Spain

Tel. 96 229 76 70 - Fax 96 229 76 76

Dealer's stamp for warranty.

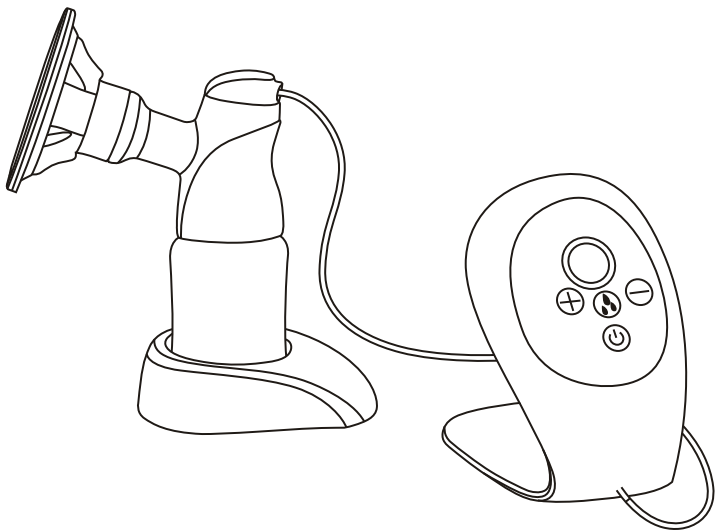
A large rectangular area containing horizontal lines for writing. Each line set consists of a solid top line, a dashed midline, and a solid bottom line. There are 10 such sets of lines filling the page.

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