Instructions for use Calma Feeding solution

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Dear parents, Congratulations on your decision to purchase Calma. For almost 50 years, Medela has pioneered innovative products which help to enhance babies' health through the life-giving benefits of breastmilk. With Calma, you can trust in a product that has been developed on a well-established research foundation. This research has shown that your baby is able to maintain with Calma the same learnt feeding behaviour that your baby exhibits when breastfeeding. Your baby will always prefer and benefit from breastfeeding. However, if you are separated from your baby, Calma provides an excellent alternative that stands to minimize any interference with the breastfeeding relationship between you and your baby.

Yes, it is the right length!

Even if you think that Calma looks a bit too long, don't worry, it is the perfect length. As you know, your nipple lengthens up to 2 times in your baby's mouth while breastfeeding. As such, Calma reaches a similar spot in your baby's mouth as the breast. It is, however, important that your baby has the right latch-on position (see picture 6-6a).

Do you know that your baby has to do some work? During breastfeeding, milk does not flow automatically. To facilitate milk flow, your baby has to create a vacuum. During breastfeeding milk flows when the vacuum in your baby's mouth is at its greatest level. At this time the tongue and your baby is at the lowest point and your nipple is drawn into your baby's mouth. This is the same with Calma. Your baby has to work to build up a vacuum for milk to flow (see picture 7-7a).

First time right

You might not believe it, but one size is sufficient. Just as your breast stay the same during the course of your lactation. The flow, shape and length of Calma are designed to suit your baby's needs as your baby grows. Calma will always meet your baby's needs during the lactation period.

Your baby has time to breathe

Your baby has a unique sucking pattern, with pauses to swallow and breathe. By using Calma, your baby is able to feed and pause naturally according to his personal rhythm. This helps to slow down the feed and results in a calm and relaxed baby during and after the feed.

Can Calma help avoid gassiness?

Calma helps avoid gassiness as any other vented teat bottles as the bottle is vented through our unique air control system (see picture 8). However, unlike many other vented bottles Calma vents air independent of how it is assembled.

Calma does not drip!

You can turn it upside down and it still won't drip. Your baby needs to apply a certain vacuum in order to get milk out. It is, however, possible that there is some dripping of milk out of the teat at the end of the feed if baby gets off the teat before drinking the portion of milk just drawn from the bottle.

Where does the milk come from?

The milk flows through the small hole (see picture 9) at a rate that is controlled by the interaction of your baby's vacuum and the milk-flow control system. Compressing the teat will not result in any milk flow. You can be confident your baby will be able to get enough milk.

Why is the hole at the tip of the silicone so wide?

Only the milk that passes through the milk-flow control system will flow into the baby's mouth. The wide hole makes the teat soft, flexible and adaptable to your baby (see picture 10). Hence your baby can drink with his own feeding action.

When feeding with Calma position the baby in a different way than when feeding at the breast (e.g. sit the baby in a baby chair facing the carer). This helps your baby to learn a new way to feed and makes it easier to switch from breast to Calma and back again (see picture 5).

Your baby will always prefer and benefit from breastfeeding so breastfeed your baby whenever you are together. For those times you are separated from your baby we do recommend you to use Calma exclusively and not in conjunction with other teats when feeding your baby from a bottle. If your baby has already used another teat, be aware, that with Calma, the milk flow is controlled by the interaction of your baby's sucking action and the milk-flow control system. This is different than with any other teat - you may need a bit of patience.

Medela wishes you and your baby all the best!

Made in Switzerland

International Sales
Medela AG, Medical Technology
Lättichstrasse 4b
6341 Baar/Switzerland
Phone +41 (0)41 769 51 51
Fax +41 (0)41 769 51 00
ism@medela.ch
www.medela.com

Breastfeeding must be well established before introducing a teat.

Intended use

Breastfeeding babies are able to feed calmly and breathe regularly and maintain high blood oxygen saturation levels. Babies feeding with Calma are also able to feed calmly and breathe regularly and maintain these same high blood oxygen saturation levels.

Important Safeguards

Read all instructions before use

- Follow these cleaning instructions.
- Inspect before each use and pull the teat and different parts in all directions. Throw away at the first signs of damage or weakness.
- Always use this product under adult supervision. All parts which are not in use must be kept out of children's reach.
- Never use a teat as a soother.
- Continuous and prolonged sucking of fluids can cause tooth decay.
- Always check the temperature of food before feeding it to your baby (risk of burning).
- Do not warm milk in a microwave oven as this may cause uneven heating and could scald your baby.

SAVE THIS INSTRUCTION MANUAL

Cleaning and Hygiene

- Use drinking water quality for cleaning.
- Clean all parts immediately after use to avoid dry up of milk residues and to prevent from growth of bacteria.
- Take care not to damage parts when using a brush.
- For hygiene reasons, we recommend replacing teats after 3 months.
- Note: Change in colours may occur. This will not affect performance.

Prior to first use and once daily

- Disassemble, wash and rinse all 3 parts.
- Place parts in a saucupen of sufficient capacity.
- Fill with cold water until the parts are well covered. Optionally, a tea spoon of citric acid may be added to avoid a build-up of lime scale.
- Heat and allow to boil for 5 minutes.
- Remove from stove and drain water.
- Dry parts with a clean towel.

After first use parts can be

- cleaned in top rack of dishwasher or
- using Medela Quick Clean Microwave Bags or steam sterilized according to manufacturer's instructions.

After each use

- Disassemble and wash all parts in warm, soapy water.
- Rinse in clear water. Dry parts with a clean towel.
- After cleaning, when parts are not in use, put in a clean zip lock bag or store in a container with lid.
- Alternatively, the parts can be wrapped in a clean paper or cloth towel.
- Do not leave feeding parts in direct sunlight or heat or leave in disinfectant ('sterilising solution') for longer than recommended by the supplier, as this may weaken the parts.
- Do not touch the inside of all parts, to avoid contamination.

BREASTMILK HANDLING

Freshly expressed breastmilk

Room Temperature	Do not leave at room temperature 3 - 5 days at approx. 4 °C
Refrigerator	3 - 5 days at approx. 4 °C
Refrigerator Freezer Compartment	6 months at approx. -16 °C
Deep Freezer	12 months at approx. -18 °C

Thawed breastmilk (previously frozen)

Room Temperature	Do not store
Refrigerator	10 hours
Refrigerator Freezer Compartment	Never refreeze
Deep Freezer	Never refreeze

- When freezing do not fill bottles/bags more than ¾ full to allow space for expansion.
- Label bottles/bags with the date of pumping.
- Do not store breastmilk in the door of the refrigerator.
- It is recommended to thaw breastmilk in the refrigerator overnight.
- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water (danger of burning).
- Gently shake the bottle/bag to blend any fat that has separated.

Material

Bottle/Cap/Multi-Lid	Polypropylene
Teat	Silicone
Base/Top	Polypropylene and thermoplastic elastomer

Disposal

According to local regulations

CE-mark according to 93/42/EEC
Conforms to EN 14350 and ISO 10993

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


Cleaning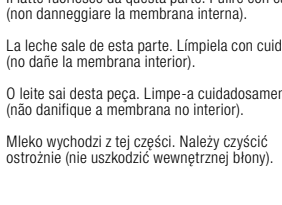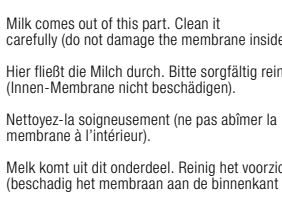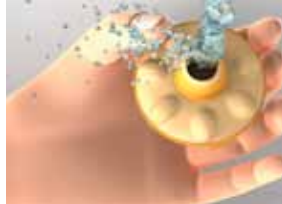

Tips

- We recommend to use Calma exclusively from the first bottle feeding and only for breastmilk.
- You may need a little patience until Calma will be accepted by your baby. It is the first time that your baby comes in contact with an artificial material instead of the familiar breast. Everything is new for your baby and he cannot smell you and your milk as with breastfeeding.
- To encourage your baby to open his mouth to take Calma, stimulate the lip as you would with breastfeeding.

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