Instruction Manual

Back Pain™ Relief System

03-020-000

Please read this detailed guidebook completely before operating this unit.

3-Year Warranty

Your Back Pain Relief System with TENS Technology is guaranteed to be free of manufacturing defects for a period of three years from the date of purchase under normal use. This limited warranty does not apply to use in any clinical or commercial applications. If the unit fails to operate during the warranty period, return it, postage prepaid, along with \$5.00 for return shipping and insurance to: MABIS Healthcare, Attn: Repair Department, 1931 Norman Drive South, Waukegan, IL 60085. If MABIS Healthcare determines that the unit failed to operate due to a manufacturing defect, the unit will be repaired or replaced at the option of MABIS Healthcare. Repair or replacement of the unit is the sole remedy under this limited warranty. This warranty gives you specific legal rights, which vary from state to state. As a condition of this warranty, the enclosed warranty registration card must be completed and sent to us within 10 days of the purchase date, or register online at www.mabisdmi.com.

This limited warranty constitutes MABIS Healthcare's only responsibility and obligation to repair and/or replace materials or components. We make no other express or implied warranties, arising by operation of law or otherwise, or any warranty of merchantability or fitness for a particular use or purpose whether or not the use or purpose has been disclosed to MABIS Healthcare in specifications, drawings or otherwise, and whether or not MABIS Healthcare's products are specifically designed and/or manufactured by MABIS Healthcare for the buyer's use or purposes, except for the limited warranty stated above. MABIS will not be responsible for any indirect, incidental, special, consequential, or punitive damages or other loss, including, but not limited to, damage to or loss of other property or equipment and personal injuries, whether to purchaser or others. MABIS shall in no event be liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit.

Toll-Free Customer Care Help Line:
1-800-622-4714

Monday - Friday 8:00 am - 4:30 pm CST

1931 Norman Drive South • Waukegan, IL 60085
847-680-6811 • FAX 847-680-9646 • www.mabisdmi.com

Index

Introduction.....	3
Important Safety Information	4-5
Product Identification.....	6
How to Use.....	7
Storage and Maintenance.....	8
Replacing the Battery.....	9
Troubleshooting.....	9
Product Specifications.....	9
Programs.....	10

Introduction

Thank you for purchasing a MABIS Back Pain™ Relief System. Please read the following instructions to achieve accurate usage and safe operation. With proper care and use, your Back Pain™ Relief System will provide you with many years of reliable usage.

What is TENS Technology?

Transcutaneous (through the skin) Electrical Nerve Stimulation sends mild electrical impulses to the muscles in the lower back needing treatment, causing the targeted muscles to exercise passively. This pocket-sized, battery-operated unit delivers pulses intermittently and may stimulate the body's production of endorphins (natural pain killers released by the brain). A TENS belt is a passive, non-invasive treatment for acute and chronic pain. Used properly, this TENS belt can provide pain relief lasting for several hours. TENS treatment can be combined with other therapy, such as exercise, to achieve optimum health benefits. TENS is widely used in hospitals and sports clinics for the treatment of muscular injuries and to improve muscle tone and blood circulation.

Important Safety Information

This system is intended for individual personal use for the temporary relief of lower back pain caused by muscle soreness, and should be placed only around muscles where pain originates in the lower back area.

The effectiveness of the system depends greatly on a person's individual physical condition and may not always be effective for every user.

You should know the reason and source of your pain before using this system. Do not rely solely on the treatment of this system for pain. Pain may indicate that you have other health problems.

Please read all of the following information carefully before using the MABIS BACK PAIN™ RELIEF SYSTEM. Be sure to comply with all "WARNINGS" and "CAUTIONS" in this manual. Failure to follow instructions can cause harm to user or the device. "WARNINGS" in this manual indicate what you cannot do with the device. "CAUTION" indicates matters in which bodily harm or material damage can occur.

WARNING:

Do not use this system if you have any of the following medical conditions:

- A cardiac pacemaker, implanted defibrillators, or any other implanted metallic or electronic device
- Suspected or diagnosed heart problem, epilepsy, a tendency to bleed internally following an injury, undiagnosed chronic pain, or pregnant

Or,

- Skin lacks normal sensations, such as tingling or numbness
- Swimming or close to water
- While driving, operating machinery or sleeping
- During menstration

Consult your physician prior to use if:

- You currently are under the care of a physician
- Have ever had medical or physical treatment for muscle pain
- Had back surgery or a recent surgical procedure (e.g. disk surgery, insertion of spinal cages, etc.) as stimulation may disrupt healing process

Important Safety Information

CAUTION:

Do not place pads on:

- Chest or close to your heart as the introduction of electrical current could cause heart rhythm disturbances
- Head or neck as severe muscle spasms may occur resulting in difficulty breathing, hearing or affect blood pressure
- Open wounds, rashes, swollen, red, infected, inflamed skin, or over or close to cancerous lesions

Discontinue use if skin irritation occurs.

Minor redness at the stimulation site is a normal skin reaction, and will normally go way within 30 minutes after removal of the electrodes. If the redness does not go way within 30 minutes do not use again until excessive redness has gone away.


Consult your physician if:

- You feel skin irritation or experience a very sensitive feeling in the skin or burns due to electrical stimulation
- If your pain does not improve or continues for four to six days

Skin irritation and burns under the pads have been reported during use. If the skin under one or more of the pads feels irritated after using the stimulator for a long period of time, the System usage time should be reduced. Skin irritation may be caused from the pad adhesive or when the electrode pads dry out. If you notice that the pads are drying out or no longer stick securely to the skin, the pads should be replaced.

Use only the electrodes, snap cables and accessories made for use with and recommended by the manufacturer.

Product Identification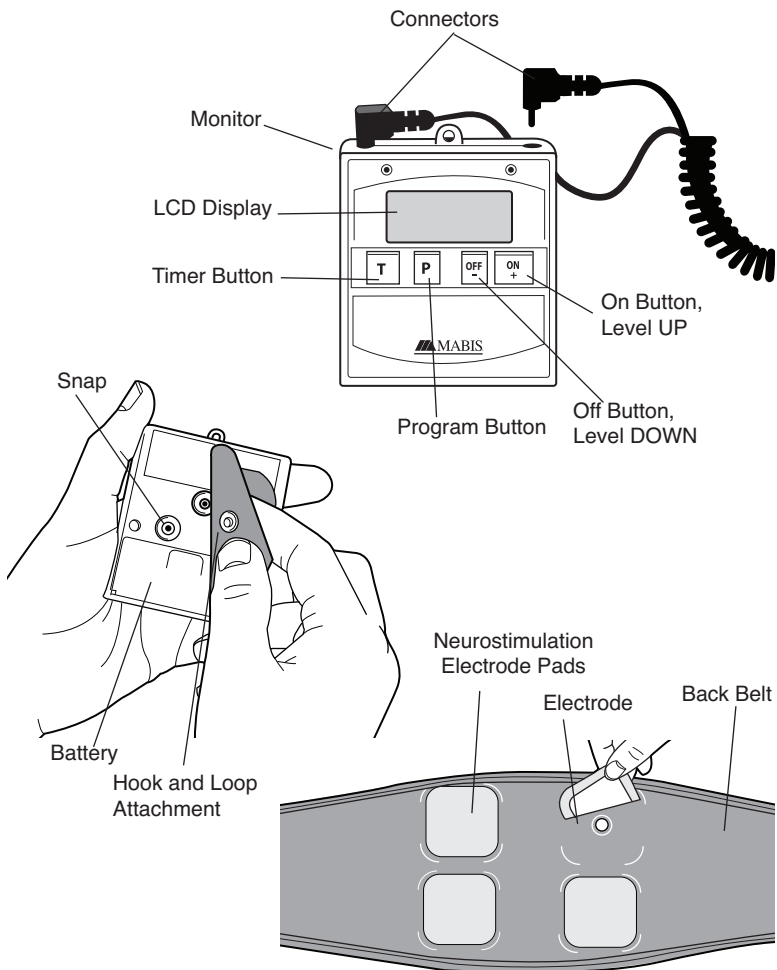

How To Use

Belt and Monitor Setup

1. Remove the plastic sheet from the side of electrode pad with the crisscross pattern. Firmly press the electrode pad (crisscross pattern down) over each electrode on the belt, *Fig. 1*. Repeat the steps for all of the electrodes.
2. Snap the hook and loop attachment piece on to the monitor, *Fig. 2*.

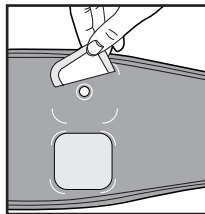
Fig. 1

Using the Back Pain Relief System

1. Take off the remaining plastic sheets on electrode pads and save for storage. Place the electrodes directly on your lower back and secure the belt around your waist with the hook and loop closure. Attach the belt extender if necessary.

NOTE: Do not try to adjust the belt once it is place; the electrode pads may roll up.

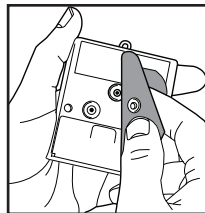
Fig. 2

2. Insert both connectors into either sockets on the top of the monitor, *Fig. 3*. Monitor will not stay on without being connected to the belt.
3. Hold the ON/+ button down for approximately **three seconds**.
4. Select the length of Time by pressing the “T” button until you reach desired time (10, 20, 30, 60).
5. Choose the Program by pressing the “P” button, until you reach desired level (P1-P8).
6. Choose the Level by pressing the ON/+ or the OFF/- button until you reach desired level (1-10).
7. Attach the monitor to the belt with the hook and loop closure, *Fig. 4*.
8. The monitor will automatically shut-off after the selected amount of time is completed.

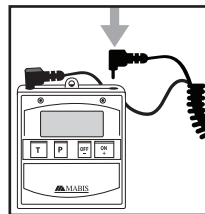
Fig. 3

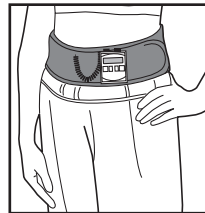
Fig. 4

NOTE: Should the electrode pads lose contact with the skin due to movement or re-positioning of the belt, the unit will reset and return to the lowest intensity setting.

Storage and Maintenance

Storage

Store your MABIS BACKPAIN™ RELIEF SYSTEM at room temperature in a dry place, out of reach of children. Detach the snap cables from the belt, and roll the belt up to store for next use.

Note: If the stimulator will not be used for more than a week, the battery should be removed from the unit.

Electrode Pads

The electrode pads will lift off the skin and stay attached to the belt when it is removed. Do not allow the pads to touch one another. Place the saved plastic liners on the pads, with the “ON” side covering the electrode pad surface. The electrode pads can be reused, but need to be replaced when they become loose or fail to adhere to the belt.

Any adhesive residue left on the skin from the electrodes can be removed with soap and water.

Simulator

The stimulator may be wiped clean with a damp cloth. DO NOT submerge the stimulator in liquids

Support Belt

Before washing the belt, remove the pads and put them on saved plastic liners. Remove the snap cables and the stimulator from the belt. Hand-wash your support belt in cool water and place flat or hang to air-dry. Do not wash the belt in the washing machine or place in a clothes dryer.

Snap Cables

Disconnect the snap cables from belt and the stimulator. The snap cables may be wiped with a clean, damp cloth. When re-connecting, take one snap cable and snap its two connectors onto the metal snaps on one side of the support belt.

Replacing the Batteries

Your Back Pain Relief System requires two AAA batteries. The batteries need to be replaced when the low battery symbol appears on the display. To replace the batteries:

1. Slide the battery cover off the back of the monitor.
2. Remove the used batteries.
3. Making sure polarity is correct, insert new batteries.
4. Replace the battery cover.

Note: Please properly dispose the batteries away from small children and heat.

Troubleshooting

<u>Problem</u>	<u>Solution</u>
Unit does not turn on	Depress ON button for a few seconds.
Unit does not turn off	Press OFF button until unit turns off.
Buttons do not appear to be functional	Remove batteries from unit and re-install making sure polarity is correct.
Unit turns on, then off	Pads need to be placed on belt and put directly on the skin of the back in order to work.
Blank display	Check battery polarity.

Product Specifications

Name and model number	Back Pain Relief System with TENS Technology, 03-020-000
Display System	Liquid Crystal Display
Operating Temperature	32°F to 104°F at relative air humidity of 20% - 90% 0°C to 40°C at relative air humidity of 20% - 90%
Storage Temperature	14°F - 140°F at relative air humidity of 20% - 90% -10°C - 60°C at relative air humidity of 20% - 90%
Automatic power-off	Automatic shut-off after 1 minute of non-use.
Battery	2 AAA batteries
Dimensions	2-7/10" X 2-1/3" X 3/5"
Weight	1.1 ounces
Accessories	Storage case, Belt extender, Instructions Guidebook, Quick Start Guide, 2 AAA batteries
Standards	Complies with US ASTM-NS4

Pre-Programmed Settings

Program	You Should Feel
P1	Continuous comfortable tingling. The underlying pain should decrease gradually after treatment.
P2	Comfortable pulsing sensation. The underlying pain should decrease immediately.
P3	Continuous comfortable pulsing. Pain should ease after treatment.
P4	Variable comfortable tingling and pulsing sensation (sensation will appear to come in waves). Pain should ease after treatment.
P5	Variable comfortable mild tingling sensation (sensation will appear to come in waves).
P6	Variable comfortable pulsing and pumping action (action will appear to come in waves).
P7	Variable comfortable tingling and pumping action (action will appear to come in waves).
P8	For pain relieve associated with muscle groups, prevents accommodation of habituation.

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