transcom

TRANSCENDENCIAS COMERCIALES S.L.

C.I.F. B-48833404

“HC – 1 NEW” MANUAL DE INSTRUCCIONES

CE0843

transcom
TRANSCENDENCIAS COMERCIALES S.L.
C.I.F. B-48833404

“HIDROTERAPIA DE COLON”

Estimado cliente:

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
HC - 1

MACHINE INSTALLATION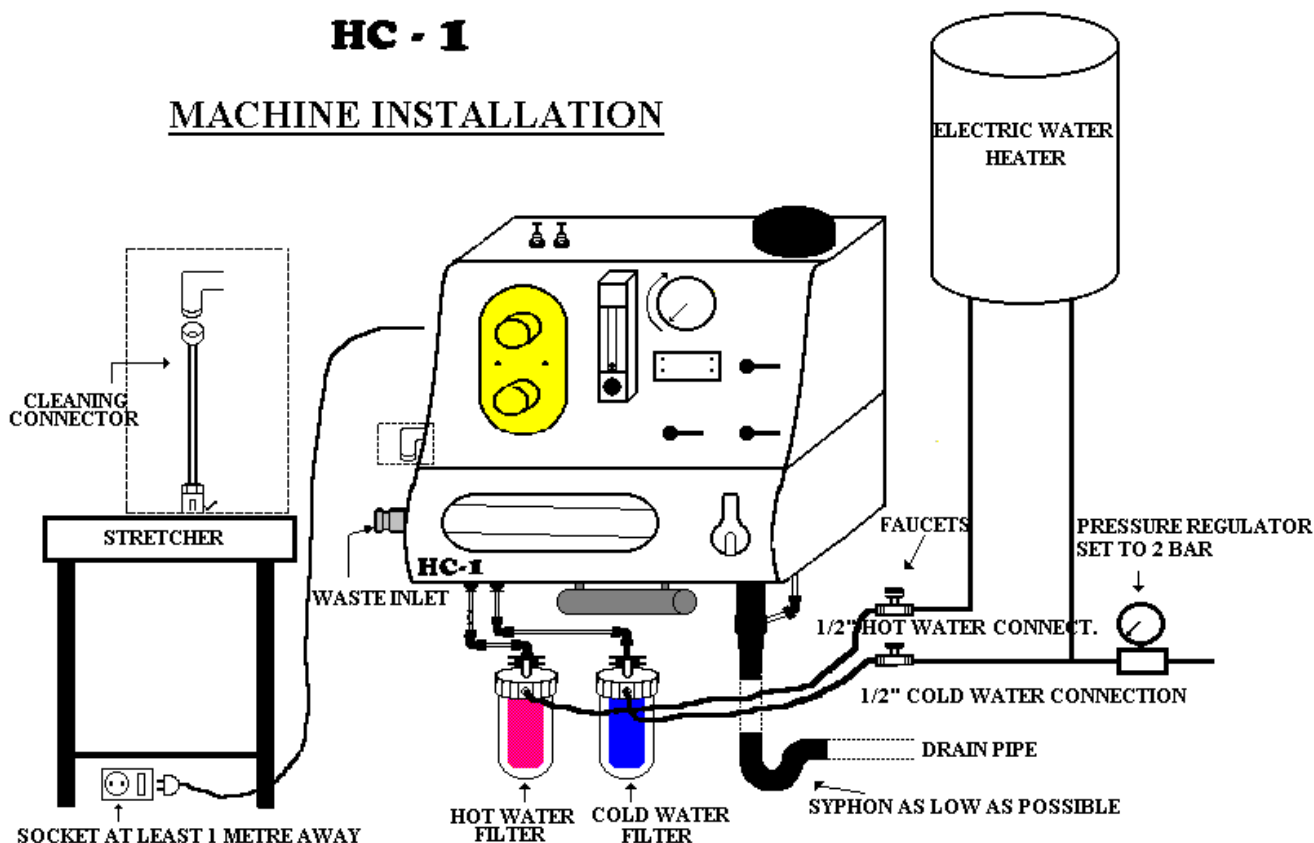

- 1.- Fit a faucets each at the hot and cold water inlets.
- 2.- Fit a pressure regulator set to 2 bar.
- 3.- Hang the unit on the wall, bearing in mind that the viewing tube should be at the same height as the stretcher, as otherwise the machine will not work.
- 4.- Connect the filters as shown on the template.
- 5.- Connect the tube marked in red to the hot water inlet on the machine and the left-hand filter.
- 6.- Connect the tube marked in blue to the cold water inlet of the machine and the right-hand filter.
- 7.- Fit the flexible hot water hose to the left-hand filter and the cold water hose to the right-hand one.
- 8.- Connect the drainage tube to the machine waste outlet; ensuring that the syphon is as low as possible.
- 9.- Use a 75 l electric water heater or, failing that, centrally heated hot water.
- 10.- Connect the flexible hose to the steel cleaning outlet connector.
- 11.- Plug in the machine, ensuring that it is switched off and that the socket is at least 1 m away.

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
transcom
TRANSCENDENCIAS COMERCIALES S.L.
C.I.F. B-48833404

COLON HYDROTHERAPY DEVICE MEASURES HC-1 NEW

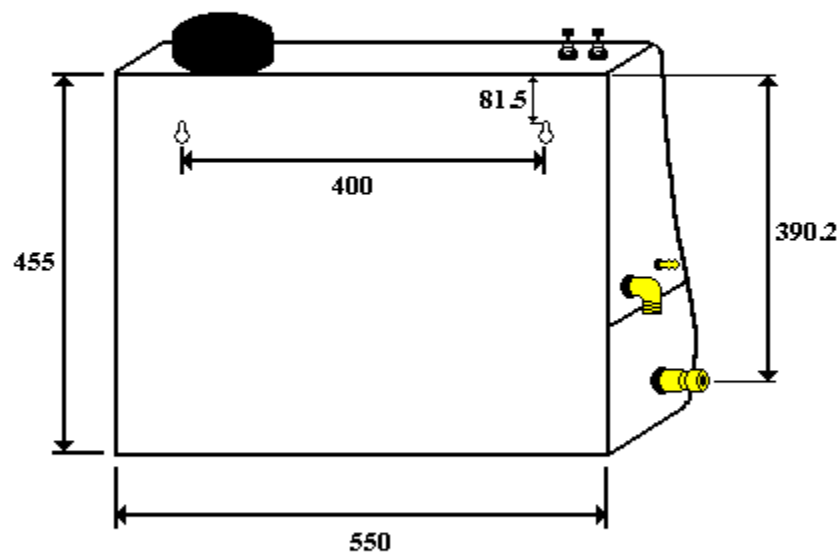MEASURES IN mm.


transcom

TRANSCENDENCIAS COMERCIALES S.L.

C.I.F. B-48833404

John Guest Super Speedfit fittings provide instant, secure and leak-proof connections, and offer considerable advantages over conventional pipe fittings and other styles of tube or pipe connectors. They allow complex tubing systems to be assembled more rapidly than with traditional fittings and facilitate fault-finding and maintenance operations. They are immensely useful for development, testing or training programmes, where systems may need to be constantly reconfigured.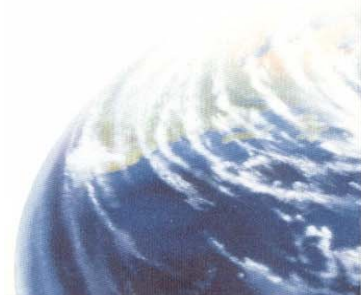

The Super Speedfit fittings featured in this brochure are made up of three basic components:


Bodies are produced in an acetal copolymer engineering plastic. (PM Range) or nickel plated brass (RM Range).

'O' Rings in food grade Nitrile rubber.


Collet in an acetal copolymer with integrally moulded stainless steel teeth.

How to make a connection

To make a connection, the tube is simply pushed in by hand; the unique patented John Guest collet locking system then holds the tube firmly in place without deforming it or restricting flow.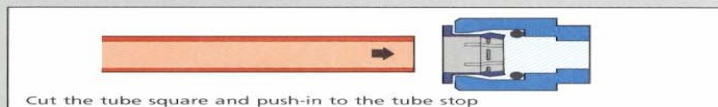

Disconnection is just as easy.

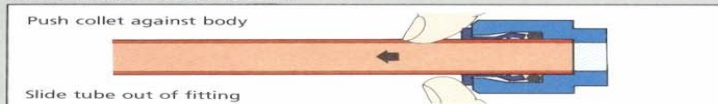Collets and 'O' Rings are readily replaceable - a useful feature in cases of misuse or accidental damage.

Fittings accommodate most standard types of tubing - plastics, nylon, brass, copper and steel. A special "Superseal" version is available for use with stainless steel or other polished metal tubes. See pages 19 to 21.

Pneumatics Applications

The brand new RM Range shown on pages 4 to 8 has been specially designed for miniature pneumatics applications.

Food Quality

PM fittings shown on pages 10 to 18 are recommended for applications involving contact with foodstuffs or potable liquids - in brewing and food processing for example.

Super Speedfit fittings shown in this brochure are in metric sizes 4mm to 22mm. John Guest also offer a range of inch size fittings, the PI Range, in sizes 5/32" to 1/2". See separate brochure for details.

If you have special requirement not covered by the standard fittings range, please contact our Customer Services Department who are always happy to discuss special needs.

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TRANSCENDENCIAS COMERCIALES S.L.

C.I.F. B-48833404

ULTRAVIOLET DISINFECTION OF WATER

History

It has been known since the beginning of mankind that radiation produces a certain effect. The human skin gets red or brown under the influence of the sun. The existence of invisible shortwave rays, however, was proved experimentally only in 1801 by J. W. Ritter. The start of this century saw a growing interest in the use of ultraviolet rays. They served a variety of applications: research in nuclear physics, analysis in organic chemistry, preparative photo chemistry, and finally disinfection of bacteriologically contaminated water.

The first UV disinfection unit was built in 1910 in Marseille but its quartz lamps were not yet reliable, and design as well as operation and maintenance of the unit were complicated and expensive. In the twenties, the production of chlorine became cheaper and its use was spreading —possible environmental risks had not yet been recognized. The disinfection of water by ultraviolet rays therefore did not enjoy a big success at that time. Things changed, however, at the beginning of the forties when the first gas discharge lamps, made on an industrial scale, came on the market: the mercury high pressure and the mercury low pressure lamps. By use of specific quartz glass types, these lamps were optimised for application in water disinfection. In 1955 the first pressure resistant UV plant was introduced in Switzerland. As a consequence of the negative influence of disinfectant chemicals on the environment, the ultraviolet system was gaining ground as a welcome alternative. Some 2000 communal waterworks in Europe now disinfect their drinking water with ultraviolet rays.

Fundamentals

The ultraviolet rays are part of the electromagnetic spectrum (Fig. 1). They are situated in the wavelength range of 10-400 nm, between X-rays and visible light. The following division of the UV spectrum is now in use:

Vacuum UV	10-200 nm
UV-C	200-280 nm
UV-B	280-315 nm
UV-A	315-400 nm

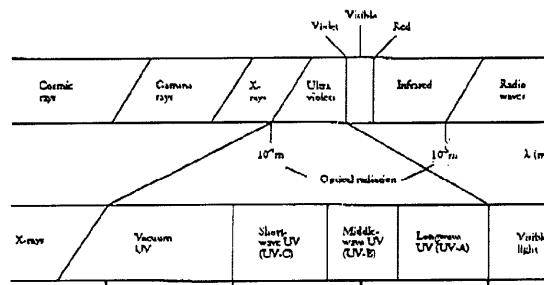
Fig. 1. Electromagnetic spectrum with expanded scale of ultraviolet radiation.

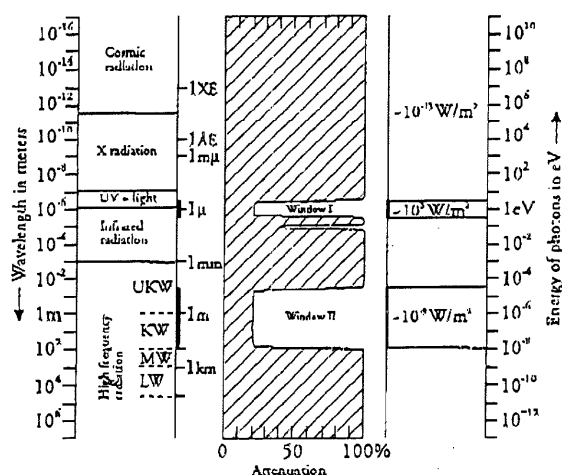
Fig. 2. Intensities of field radiation in biosphere and position of optical windows of atmosphere.

The optical "windows" (Fig. 2) in the atmosphere enable rays from 280 nm to the near-infrared range to reach the surface of the earth. The remaining rays of the sun and universe are absorbed: by the ozone layer as regards vacuum UV and UV-C. The expression "UV light" is strictly speaking not correct since UV is no optical radiation. But this expression is now in common use because UV rays also have certain optical behaviour patterns, which are similar to visible light. The most important physical quantum energies and frequencies as a function of wavelength can be seen in Fig. 3. In order to produce a photochemical reaction, it is necessary for the energy to be absorbed in the chromophores. This is different from the reaction with ionizing rays, where the whole object being irradiated takes in energy uniformly. This also distin-

1

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guishes the concept of the dose required for a reaction. In the case of ionizing rays, the dose is defined as the energy absorbed per mass element; in UV radiation, it is the energy applied on a surface element during a time period.

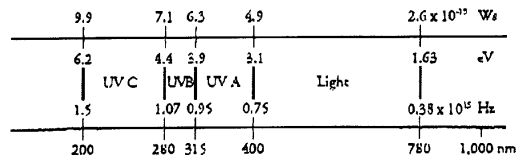
Fig. 3. Spectrum of electromagnetic radiation with corresponding quantum energies in J ($\sim$ Ws) and eV as well as frequencies and wave lengths.

To kill germs by radiation energy, the absorption spectrum and chemical make-up of various species have to be considered. Ultraviolet rays inactivate the deoxyribonucleic acid

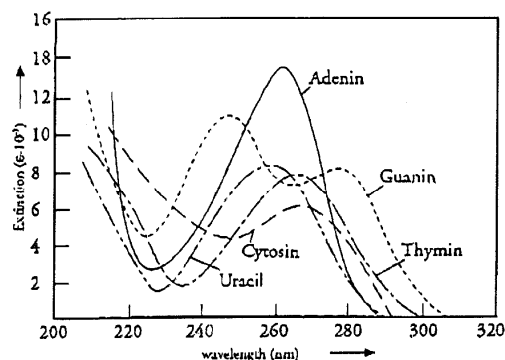
Fig. 4. Absorption spectrum of nucleic acid bases, pH 7.

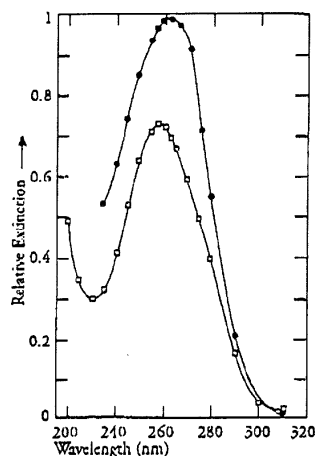
Fig. 5. Absorption spectrum of DNA in native (■) and hydrolyzed (○) state.

(DNA). A comparison of the absorption spectrum of the nucleic acid bases (Fig. 4) and DNA (Fig. 5) demonstrates that there is a distinct absorption maximum at 250-260 nm. Bacteria, as for example *E. Coli*, also show characteristic absorption curves (Fig. 6). A source of UV-C radiation is therefore in a position to kill bacteria. The damages that can be caused in the DNA are manifold, but they occur mostly on the pyrimidine bases thymine, cytosine and uracile. It is primarily the following changes that take place (Fig. 7):

—The *dimer formation* (a) occurs mostly with thymine. It takes place only between directly neighbouring thymine molecules on the same strand of the DNA. Also the other pyrimidine bases are prone to dimer formation under the impact of UV, but to a lesser extent.

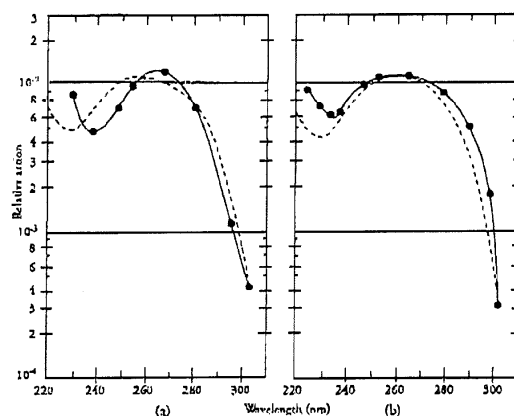
Fig. 6. Action spectra for killing of (a) *Escherichia coli* and (b) *Staphylococcus aureus*.

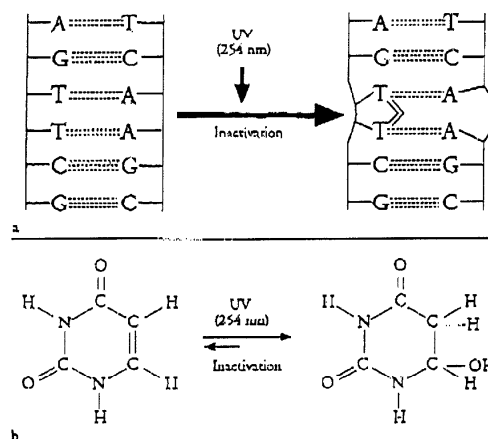
Fig. 7.

transcom

TRANSCENDENCIAS COMERCIALES S.L.

C.I.F. B-48833404

—The hydrate formation (b) in the DNA occurs only seldom because of the existing hydrogen bonds. Via hydration of the pyrimidine bases, a change in the genetic code is possible.

—Denaturations of the DNA double strand occur mostly as by-products of hydration and dimer formation.

Further effects to be found are the polymerization between nucleic acids and proteins, the polymerization within the nucleic acids, and the breakage of strands. The nucleic acids are thus the target of UV radiation, blocking the division of cells and causing their death. The pyrimidine dimer formation is mainly responsible for this effect. Already a few dimers can cause the blockage of cell division in most organisms. There are indications for further possible reactions (e.g. protein polymerization) that might also have an effect, but scientific evidence is still lacking.

Reactivations and repair processes are, if at all, possible only if deliberately induced under special laboratory conditions, for example by heightened temperature and long exposure to light of over 300 nm wavelength. Such conditions do not exist in practice.

Viruses, having a high DNA or RNA substance, also show a strong UV-ray absorption (Fig. 8). The inactivation of polio-

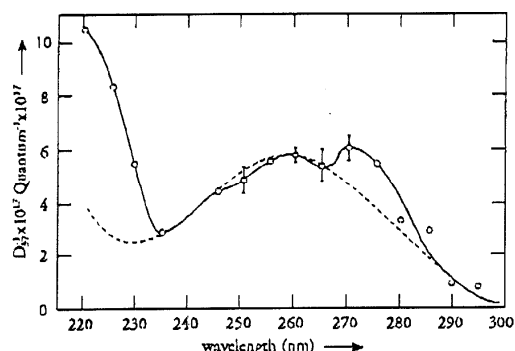
Fig. 8. Action spectrum for inactivation of *Vaccinia virus*. Broken line shows DNA absorption spectrum.

virus by UV is almost as easy as that of *E. Coli*. However, most of the knowledge and experience as well as practical application for UV radiation lie with bacteria. Not every strain and not every bacterium within a strain has the same sensitivity to UV radiation. Vegetative forms of bacteria are more sensitive to UV than spores (Table 1).

To produce a reaction in the bacterial cell, adequate radiation sources are needed. There are various ways to generate rays of 240-270 nm wavelength. A prerequisite will be to produce the desired radiation over a long period of time and at the lowest possible energy consumption. Figure 9 shows some possibilities of generating radiation. It is noteworthy that the mercury low pressure lamp emits an almost monochromatic

radiation of 253.7 nm. This type of lamp is in fact mostly used in today's UV plants for water disinfection.

In the low pressure gas discharge, the electric current is flowing between two electrodes through an inert gas, which is lightly loaded with mercury. The mercury ions are stimulated by the effect of the electric current and generate the characteristic radiation. The radiation intensity per cm lamp length is smaller in a low pressure lamp than in a high pressure lamp; but the relative radiation output of the low pressure lamp is 2-3 times higher. Figure 10 shows the relative spectrum energy distribution of the Hg low pressure lamp. A fundamental difference also exists in the temperatures of the two basic lamp types. A low pressure UV lamp is operated without cooling; its optimum temperature (on the lamp shell) is 40-50°C, as opposed to the operating temperature of the high pressure lamp of over 500°C.

Table 1

UV dose (mJ/cm²) required for 90% kill rate (D₁₀ values)

Microorganisms	
<i>Escherichia coli</i>	2.9
<i>Bacterium coli</i>	5.4
<i>Salmonella enteritidis</i>	4.0
<i>Salmonella paratyphi</i>	5.2
<i>Bacillus anthracis</i>	4.5
<i>Bacillus subtilis veg</i>	6.1
<i>Bacillus subtilis spr</i>	11.8
<i>Bacillus megatherium veg</i>	1.2
<i>Bacillus megatherium spr</i>	2.7
<i>Proteus vulgaris</i>	3.8
<i>Shigella flexneri</i>	1.7
<i>Streptococcus emolyticus</i>	2.2
<i>Streptococcus lactis</i>	6.2
<i>Pseudomonas aeruginosa</i>	5.5
<i>Pseudomonas fluorescens</i>	3.5
<i>Serratia marcescens</i>	2.4
<i>Spirillum rubrum</i>	1.1
<i>Staphylococcus albus</i>	1.8
<i>Vibrio comma</i>	3.4
<i>Leptospira</i>	3.2
Viruses	
Bacteriophage	2.6
Influenza	3.1
Polio	3.2
Yeasts	
Yeast (average)	6.0
Brewer's yeast	10.0
<i>Sacharomyces spr</i>	8.0
<i>Sacharomyces ellipsoides</i>	6.0
<i>Torula sphaerica</i>	2.3

Gas discharge lamps are subject to natural ageing —decrease of emission—by the deposit of burnt-off electrode material on the inner quartz glass surface. Figure 11 gives an indication of the active life span of the UV lamp types used today.

Mercury in its activated state also emits a resonance line at 185 nm. This line, however, can ionize oxygen of the air, thereby generating ozone and nitric oxides, which are not desired. Therefore, by use of special quartz glass types, all radiation below 200 nm is retained within the UV lamp (Fig. 12).

Mercury low pressure lamps yield a radiation output of up to 40% of the electric energy input. This is valid for a current consumption of 0.5 W/cm lamp length, which is today the case with 30-40 W UV lamps. If the electric load reaches values of for example 1.2-1.5 W/cm lamp length, the yield decreases to values below 30%.

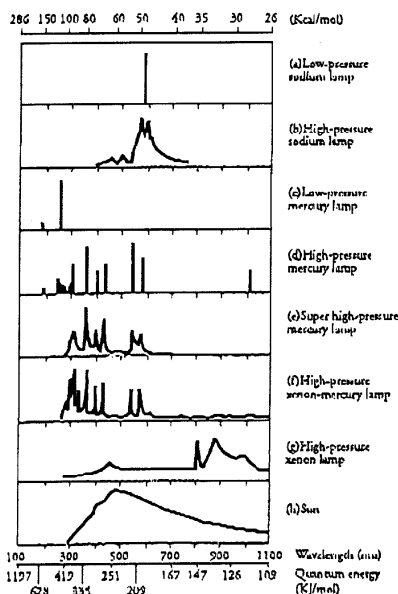
Fig. 9. Relative spectral energy distribution.

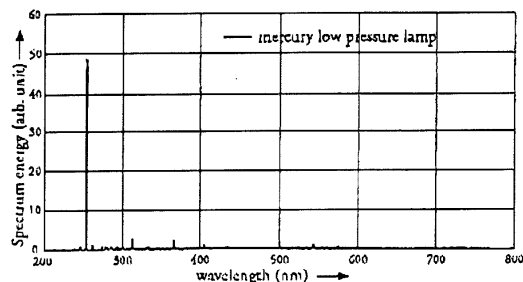
Fig. 10. Relative spectrum energy distribution of a mercury vapor low pressure lamp.

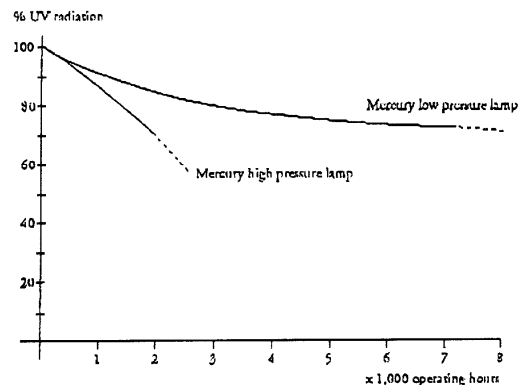
Fig. 11. Intensity reduction of Hg high and low pressure lamps, as a function of operation hours.

Ultraviolet and other disinfecting agents

Each disinfectant has its specific fields of application. The various influencing parameters can be gathered from Table 2. As of today, chlorine is still the most commonly used disinfectant. The possible problems of chlorination are known and need not be repeated here. It may just be mentioned that problems appear especially in connection with the increasing treatment of surface water as well as lake and river bank filtrates. With reference to ozone, it causes high costs of investment and operation especially for smaller waterworks and usually also entails after-treatment. The ultraviolet method requires no reaction tank and also distinguishes itself by its ease of operation and maintenance.

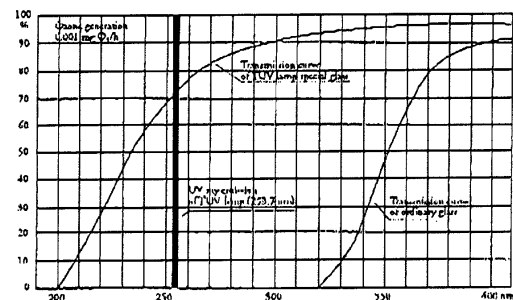
Fig. 12. Transmission curve of TUV special glass and of ordinary glass.

Table 2

Comparison of various water disinfection methods

Comparison points	UV	Ozone	Chlorine
Contact time	1-10 sec.	10-20 min.	30-50 min.
Reaction tank	none	required	required
Maintenance	minimum	considerable	average
Installation	simple	elaborate	elaborate
Influence of			
-suspended matter	strong	strong	strong
-temperature	none	strong	strong
-pH value	none	slight	strong
Residual in water	none	little	yes
Influence on water	none	yes	yes
Corrosion	none	yes	yes
Toxicity	none	yes	yes
Operating cost	low	high	low

As shown previously, UV radiation requires a smaller dose than chlorine or ozone for the same effect of disinfection (Fig. 13).

After-treatment of UV-radiated water is not necessary since re-infection in the distribution systems of communal waterworks can be excluded. This is demonstrated by the large practical experience of over 25 years as well as by a great many test results of government agencies; recent studies also document this fact. It should also be remembered that even today there are still many public water supply systems that get no treatment at all and yet do not encounter any infection problems in the distribution network. The condition is of course that the water entering the piping system be bacteriologically in order. The UV process fully satisfies this requirement.



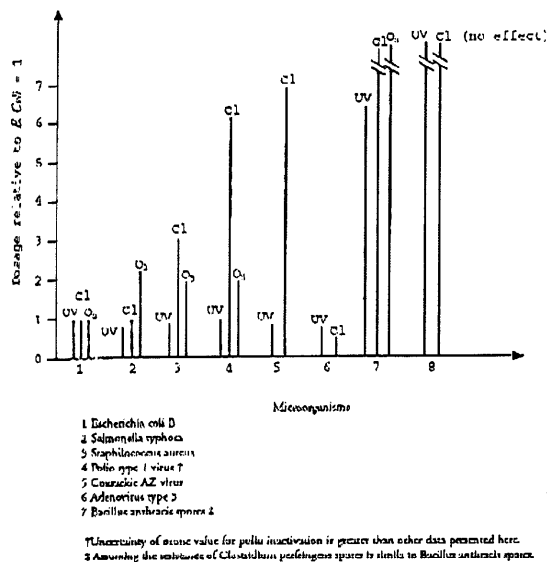
Fig. 13. Approximate dosage required for inactivation of various microorganisms relative to *Escherichia coli* dosage.

The ultraviolet method in practice

The practical application of ultraviolet rays asks for certain fundamentals to be considered and conditions to be met. The criteria shall be documented with the aid of an existing UV plant (Fig. 14). As referred to previously, the UV dose to be "dispensed" to a bacterium is defined as the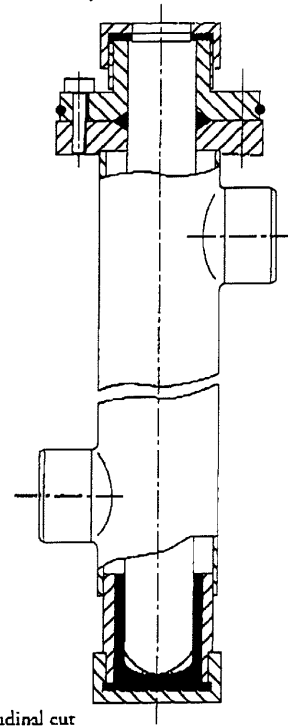

Longitudinal cut

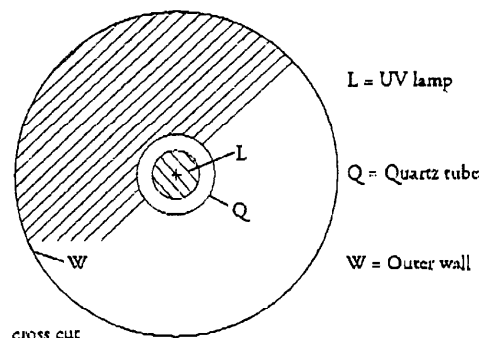CROSS CUT

Fig. 14. UV disinfection unit.

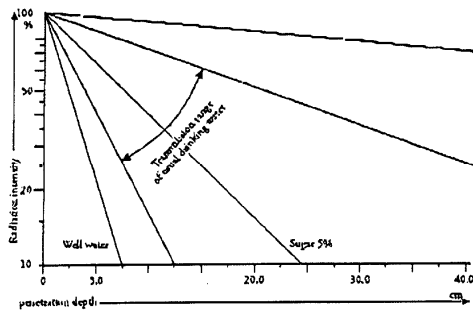
Fig. 15. Transmission of various liquids for bactericidal radiation 253.7 nm.

energy (of 240-280 nm) applied per surface unit during a certain time period.

Formula: $H = E \times t$

Units: $\text{mJ}/\text{cm}^2 \text{ mW}/\text{cm}^2 \text{ s}$

(H = UV dose; E = Intensity; t = Exposure time)

Intensity (E). The intensity is a function of the layer depth between UV lamp and wall of the irradiating chamber. It can be reduced by various factors:

—the pressure resistant quartz tube around the UV lamp. It causes a UV-ray absorption of 5-7%;

—the ageing of the UV lamp. This influence was already discussed previously;

—the quality of the water to be treated. Table 3 is a comparative compilation of the now most commonly used water quality data (since each country gives somewhat different indications). As a rule, UV treatment is possible for water in the transmission range of 15-85% at 10 cm layer depth, corresponding to an effective penetration depth of 12-140 cm. Water qualities below these minimum values usually need a pretreatment. Higher UV transmissions than 85% (at 10 cm) are present in ultra-pure water, which is needed for example in the pharmaceutical and electronics industry and which is of course also ideally suited for UV treatment (Fig. 15).

It follows from the above that the relevant value is the minimum intensity in the irradiating chamber i.e. the intensity at the most distant point from the UV source—all influencing factors having been allowed for—and at the end of the defined life span of the UV lamp.

Exposure time (t). It is the time period of irradiation. In the UV plant of Fig. 14 it means the time needed by a water particle in the irradiated volume to get from one end to the other end. The exposure time can be ascertained mathemati-

cally from the flow velocity and the diameter of the irradiating chamber.

All factors of this most basic UV unit (of Fig. 14) can thus be calculated. If several UV lamps are incorporated within one single irradiating chamber, or if UV lamps have a geometrically different shape than the straight round standard tube, the mathematical calculation is admissible with due reservations only. Above all, it is not possible to determine mathematically the flow profile since there are flow velocities as low as 10-20 cm/s. The only valid basis to establish the performance and treatment capacity of a UV plant is thus derived from bacteriological tests under defined conditions.

Example of UV unit and its operating cost

In addition to the actual investment cost, the cost for operation and maintenance of a water disinfecting method is also of importance.

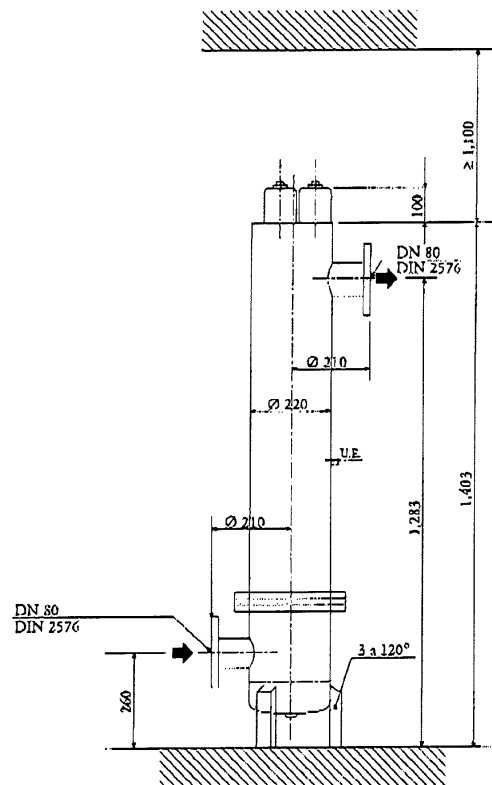
Fig. 16.

**transcom**TRANSCENDENCIAS COMERCIALES S.L.
C.I.F. B-48833404*Example:*

Flow rate:	25 m ³ /h
Water quality	85% T 100 mm
Investment cost	\$ 6,200
Unit type	3A250 (Fig. 16)
Current Consumption	228 W
Life span of UV lamp (guaranteed, allowing for 20% reduction of intensity)	9,000 h

The UV unit is suitable for an operating pressure up to 16 bar and can therefore be inserted direct into the water mains. A fail-safe system for automatic operation includes UV intensity monitor, operating hourcounter, optical alarm system on unit, remote alarm signal, control relays and monitoring for valves and pumps. The basic maintenance work is the replacement of UV lamps after 9,000 operating hours.

Operating cost	9,000 h of operation	per year (8760 h)
Water quantity	225,000 m ³	219,000 m ³
UV lamps (3 x \$60)	\$180	\$175
Current consumption (at \$0.10/kWh)	\$205.2	\$200
Operating cost	\$0.0013 m ³	
Capital cost per year (depreciation in 5 years)		\$1,240
Operating and capital cost p.a.	-	\$1,615
Operating and capital cost	-	\$0.0074 m ³

Safe and effective operation as well as low treatment costs make ultraviolet irradiation suitable, "ecology minded" and up-to-date water disinfecting method. It is a valid alternative to chlorination, without the use of chemicals.

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U.V. 245 nm (mWsec/cm²) required for a 90% killing of various micro-organisms.

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ENFERMEDADES DE PECES/FISH DISEASE/ MALADIES DES POISSONS

Fungi (Typical)	24
Ichthyophthirius (White Spot)	40
Infectious Pancreatic Necrosis (IPN)	60
Saprolegnia (Fungal Disease)	13
Viral Hemorrhagic Spticaemia (VHS)	10

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TRANSCENDENCIAS COMERCIALES S.L.
C.I.F. B-48833404

UL International (UK) Ltd

An affiliate of Underwriters Laboratories Inc.

EC Certificate - Production Quality Assurance System Approval Certificate

(Annex V section 3 of the Directive 93/42/EEC on Medical Devices)

Manufacturer

Transcendencias Comerciales, S.L.
Parque Empresarial Zuatzu
C/Zubiberri 31, Edificio Urumea Finca 1
Planta Baja, Local 1
20018 San Sebastian (SS)
Spain

Authorised Representative

N/A

Scope of Certificate: Manufacture, inspection and testing of colon cleaning machines and accessories for colon cleaning machines

Device Classifications: Annex V, Class IIa

Device Descriptions: Colon Cleaning Machines

Model/Type : Please refer to attachment 1

We hereby declare that an examination has been carried out following the requirements of the national legislation to which the undersigned is subject, transposing Annex V of the Directive 93/42/EEC on Medical Devices. We certify that the production quality system conforms with the relevant provisions of the aforementioned legislation.

Certificate issued by:

Certification Manager
For UL International (UK) Ltd

UL International (UK) Ltd
Womersley House
The Guildway
Old Portsmouth Road
Guildford, Surrey GU3 1LR
United Kingdom
+44 (0)1483 302130

Certificate no: 389
Original certificate: 05 August 2004
Current certificate: 07 November 2007
Certificate expiry: 01 November 2010

Page 1 of 2

transcom
TRANSCENDENCIAS COMERCIALES S.L.
C.I.F. B-48833404

U N D E R W R I T E R S L A B O R A T O R I E S I N C .
C E R T I F I C A T E O F R E G I S T R A T I O N

Transcendencias Comerciales S.L.

Parque Empresarial Zuatzu
C/ Zubiberri 31, Edificio Urumea Finca 1
Planta Baja, Local 1
20018 San Sebastian (SS)
Spain

Underwriters Laboratories Inc.® (UL) issues this certificate to the Firm named above, after assessing the Firm's quality system and finding it in compliance with

ISO 13485:2003

EN ISO 13485:2003, CAN/CSA 13485:2003 and ANSI/AAMI/ISO 13485:2003

for the following scope of registration

2845 (US) : Electromedical and Electrotherapeutic Apparatus

The manufacture, installation and servicing of colon cleaning machines and accessories.

This quality system registration is included in UL's Directory of Registered Firms and applies to the provision of goods and/or services as specified in the scope of registration from the address(es) shown above. By issuance of this certificate the firm represents that it will maintain its registration in accordance with the applicable requirements. This certificate is not transferable and remains the property of Underwriters Laboratories Inc.®.

File Number: A16403

Volume: 1 of 1

Original Certification Date: February 1, 2008

ISO 13485:2003 Issue Date: February 1, 2008

CMDCAS Issue Date: February 1, 2008

Effective Date: February 1, 2008

Expiry Date: January 31, 2011

John H. Schmidt
Senior Vice President, Chief Development Officer

U N D E R W R I T E R S L A B O R A T O R I E S I N C .
C E R T I F I C A T E O F R E G I S T R A T I O N

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Electromagnetic Compatibility Tests Summary

In view of the Electromagnetic Compatibility test results described in the test report 9701B31/44-1 dated 10 September 1997, and in the conditions expressed in this report, the tested sample of:

Product: Colon Hydrotherapy Equipment

Model: HC-1

Serial Number: 000430797

Manufacturer: TRANSCOM (Transcendencias Comerciales, S.L.)

- **CONFORMS** to **EN 60601-1-2 (1993)** in the paragraphs relating to electromagnetic emissions (36.201.1).

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CONTAINS

Small Speculum + Obturator + Water line (1500 mm/59") + Waste Hose (1300mm/51")

ONE USE ONLY

⊗ Under no circumstances should any item of this kit be reused. This kit is to be used on the TRANSCOM Colon Hydrotherapy devices and other standardised equipment.

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INSTRUCTIONS

△ Connect Water Line to Speculum, and insert Obturator in the speculum. Lubricate the Speculum and insert it gently into the rectum. Then remove the Obturator and attach the Waste Hose to the Speculum. After each treatment, dispose off the entire kit. You should use a plastic bag.

RESPONSABILITIES

The manufactures, TRANSCOM - Transcendencias Comerciales, S.L. or their Distributors, offer no explicit or express guarantee, nor do they accept any responsibility for either direct or consequential damages and losses beyond the purchase price. The seller cannot control handling or use of the product, or the effects of use of the product.

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CONTAINS

Straight Speculum + Obturator + Water line (1500 mm/59") + Waste Hose (1300mm/51")

ONE USE ONLY

Under no circumstances should any item of this kit be reused. This kit is to be used on the TRANSCOM Colon Hydrotherapy devices and other standardised equipment.

PRESCRIPTION STATEMENT

"CAUTION: (US FEDERAL LAW) This kit is to be used with a medical prescription".

INSTRUCTIONS

Connect Water Line to Speculum, and insert Obturator in the speculum. Lubricate the Speculum and insert it gently into the rectum. Then remove the Obturator and attach the Waste Hose to the Speculum. After each treatment, dispose off the entire kit. You should use a plastic bag.

RESPONSABILITIES

The manufactures, TRANSCOM - Transcendencias Comerciales, S.L. or their Distributors, offer no explicit or express guarantee, nor do they accept any responsibility for either direct or consequential damages and losses beyond the purchase price.

The seller cannot control handling or use of the product, or the effects of use of the product.

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