

Everest Jennings® EVEREST LITE POWER WHEELCHAIR **ASSEMBLY INSTRUCTIONS**

It is possible for one person to assemble the Everest Lite Power Wheelchair, but assembly is best performed by two people, since some parts are heavy and hard to maneuver alone.

 \triangle WARNING: Read these instructions before assembling the Everest Lite.

 \triangle WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.

Unpack the carton

▲ Caution: Never lift the Everest Lite by the front or rear cowl.

All components are in one carton. Remove the top packing material. Remove all components carefully and place them in the assembly area. Carefully position the base assembly on its wheels on the floor.

Assembly

- 1. Pick up the battery pack by the strap, as shown at right, and install it in the base, with the arrow on the top of the battery pack pointing toward the front of the wheelchair. Ensure that the battery pack is firmly seated in the base before proceeding.
- ▲ Caution: When installing the battery pack, press the strap all the way down.



2. Install the seat on the base as shown at right, while pulling the seat rotation lever to lock it in place. Ensure that the seat is locked into place and facing forward before continuing.



- 3. The hand control is usually installed on the right arm, as shown at right, but may be installed on either arm. If you would like to install the hand control on the left arm, please see that adjustment in the owner's manual.
 - Install the hand control: Slide the extension into the bracket. as shown at right; loosen the knob first, if necessary. When the hand control extension is in the desired position, tighten the knob.



4. Connect the hand control connector to the cowl connector, as shown at right.



⚠ WARNING: Ensure that the hand control extension is locked in place, and that all cables are away from moving parts of the wheelchair, before occupying or operating

the Everest Lite.

riangle WARNING: Ensure that the Everest Lite is assembled as described before operation, with

all warning labels and literature attached. Please refer to the owner's manual that accompanies the Everest Lite for all operation and adjustment instruc-

tions.



Everest & Jennings, GF, and Graham-Field are registered trademarks, and Everest is a trademark, of GF Health Products, Inc. ©GF Health Products, Inc. • 2935 Northeast Parkway • Atlanta, GA 30360 • tel 800-347-5678 • fax 800-726-0601 • www.grahamfield.com