Anti-tippers

Assembly, Installation and Operating Instructions

Kit Nos. 1036900, 1036901, 1064227, 1086189, and 1086190

Yes, you can!

SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. In case of shipping damage, DO NOT use. Contact Carrier/Dealer for further instruction.

This kit includes the following:

DESCRIPTION	QUANTITY
Anti-Tipper	2
Anti-Rattle*	2

**NOTE: Not included with Kit Nos. 1064227, 1086189, and 1086190.*

NOTE: To ensure the correct model anti-tipper is used, refer to Rear Anti-Tipper Application Chart on page 4.

Safety Summary

To ensure the safe and proper installation of the anti-tippers, these instructions **MUST** be followed:

⚠ WARNINGS

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions and instructions, contact a healthcare professional, dealer or technical personnel if applicable, before attempting to install this equipment - otherwise, injury or damage may occur.

⚠ INSTALLATION WARNINGS

Inasmuch as anti-tippers are an option on a particular wheelchair (you may order with or without the anti-tippers), Invacare strongly recommends ordering anti-tippers as an additional safeguard for the wheelchair user.

Anti-tippers are specific to the different rear wheels and/or seat-to-floor heights. Refer to this instruction sheet for correct usage and adjustment. If these requirements **CANNOT** be achieved, **DO NOT** use the wheelchair. Contact an Invacare dealer or qualified technician. Any changes to the seat-to-floor angle or seat-to-floor height may require different anti-tippers. The correct anti-tippers must be ordered to maintain a 1½ to 2-inch ground clearance.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

⚠️ INSTALLATION WARNINGS (CONTINUED)

Anti-tippers MUST be fully engaged. Ensure that the locking pin of the anti-tipper fully protrudes out of the hole on the wheelchair frame. Ensure both anti-tippers are adjusted to the same height.

Anti-tippers MUST be used at all times. When outdoors on wet, soft ground or on gravel surfaces, anti-tippers may not provide the same level of protection against tipover. Extra caution must be observed when traversing such surfaces.

Ensure the anti-tipper wheels are pointing towards the ground/floor BEFORE operating the wheelchair.

After ANY adjustments, repair or service and BEFORE use, make sure that all attaching component parts are secure.

Installing Anti-Tippers

NOTE: For this procedure, refer to FIGURE 1.

1. Remove the plug button or end cap from end of lower frame tube.
2. Visually inspect the top and bottom of lower frame tube for locking hole(s).

NOTE: Depending on your wheelchair frame style, one (1) of the following conditions will exist:

- One locking hole located on top of the lower frame tube.
 - One locking hole located on the bottom of the lower frame tube.
 - Locking holes located on both the top and the bottom of the lower frame tube.
3. If there are no hole locations for the anti-tippers to lock into position, drill a 9/32-inch diameter hole through one wall at the bottom of the lower frame tube. This hole should be placed 15/16-inch from the back edge of the lower frame tube.
 4. Place the anti-rattle over the end of the lower frame tube (if applicable).
 5. Depress the release buttons and insert the anti-tippers into the wheelchair frame tubing with the anti-tipper wheels pointing toward the ground/floor.
 6. Ensure that the release button of the anti-tipper fully protrudes out of the locking hole(s) in the lower wheelchair frame tubing.
 7. Repeat STEPS 1-6 for the opposite anti-tipper.

NOTE: A 1½ to 2-inch clearance between the bottom of the anti-tipper wheels and the ground/floor MUST be maintained at all times.

8. Measure the distance between the bottom of the anti-tipper wheels and the ground/floor.
- Non-Adjustable Height Anti-Tipper Assemblies: If the distance between the bottom of the anti-tipper wheels and the ground/floor is not 1½ to 2-inches, call Invacare Technical Support at the number listed on the back page.
 - Adjustable Height Anti-Tipper Assemblies: If the distance between the bottom of anti-tipper wheels and the ground/floor is not 1½ to 2-inches, adjust anti-tippers. Refer to [Adjusting Anti-Tippers](#) on page 3.

**NOTE: Non-adjustable anti-tipper shown.*

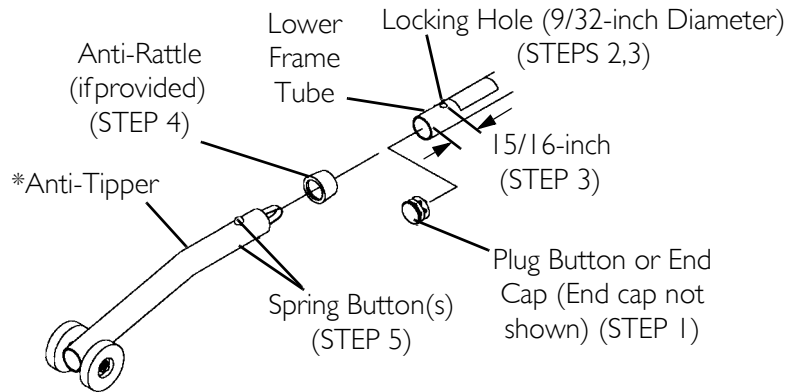
FIGURE 1 Installing Anti-Tippers

Adjusting Anti-Tippers

NOTE: For this procedure, refer to FIGURE 2.

NOTE: A 1½ to 2-inch clearance between the bottom of the anti-tipper wheels and the ground/floor MUST be maintained at all times.

1. Press the release buttons near the wheeled portion of the anti-tipper and slide the anti-tipper up or down to achieve the 1½ to 2-inch clearance.
2. Check to make sure that the release buttons are fully engaged in the adjustment holes.

⚠ WARNING

Ensure both anti-tippers are adjusted to the same height.

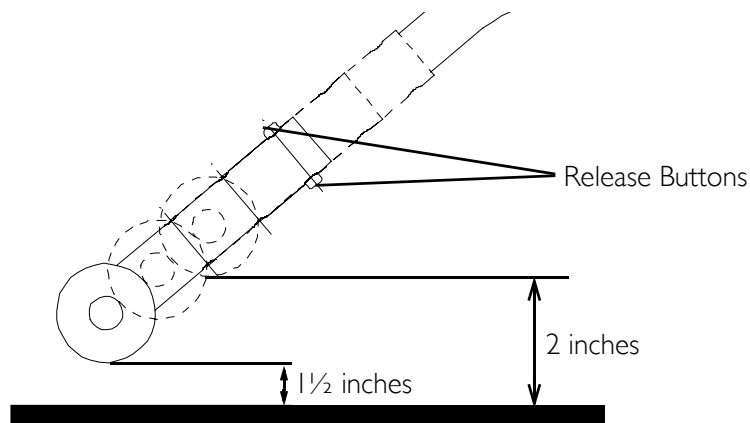
FIGURE 2 Adjusting Anti-Tippers

REAR ANTI-TIPPER APPLICATION CHART

SERIES	COMMENTS/ADJUSTMENTS	REAR TIPPER STATUS	OBSOLETE OPTION	CURRENT OPTION (MODEL #/ KIT #)
Tracer [®] /Tracer Plus	Swingaway Front Frame Models	Optional	N/A	1358M/1036900
Tracer/Tracer Plus	All Fixed Front Models	Optional	N/A	9859/1064227
Tracer LX	All Models (See following exceptions)	Optional	N/A	1358M/1036900
Tracer LX	1 - Arm Drive or Amputee Bracket	Optional	N/A	1358N/1036901
Tracer LX Recliner	All Models	Standard	N/A	9758/1086190
Tracer LT	All Models	Optional	N/A	1358M/1036900
Tracer IV	All Models	Optional	N/A	1358M/1036900
Tracer Titan, SX, DLX	All 19½ to 17½-inch Seat to Floor Height Models	Optional	N/A	9758/1086189
Tracer Titan, SX, DLX	All 17½ to 15½-inch Seat to Floor Height Models	Optional	N/A	9758/1086190
Tracer EX	All Models Before 2/99 (See following exceptions)	Optional	N/A	1358M/1036900
Tracer EX	1 - Arm drive Before 2/99	Optional	N/A	1358N/1036901
Tracer EX	All 24, 22-inch Rear Wheel Models After 1/99 (Including Chairs With Amputee and 1 - Arm Drive Options)	Optional	N/A	9758/1086190
Tracer EX	All 20-inch Rear Wheel Models After 1/99	Optional	N/A	9859/1064227
Invacare MG	All Swingaway Front Frame Models Before 2/99	Optional	N/A	1358M/1036900
Invacare MG	All Swingaway Front Frame Models After 1/99	Optional	N/A	9758/1086190
Invacare MG	All Fixed Front Models	Optional	N/A	9758/1086190
CareGuard™	All Models before 10/2000	Optional	N/A	9758/1086190
CareGuard Titan™	All Models	Optional	N/A	9759/1086189
3000	Child Size (12-inches Wide) or All Recliner Models.	Optional	I 358A	1358N/1036901
3000	Junior size (14-inches and 16-inches Wide) Adjustment Needed: Install with Wheels Up.	Standard	I 358A	1358M/1036900
900	All Models (See following exceptions)	Optional	I 358B	1358M/1036900
900	All Hemi Models	Optional	I 358A	1358M/1036900
900	All Recliner Models	Optional	I 358D	1358N/1036901
900	All Models with One Arm Drives or Amputee Options	Optional	I 358B	1358N/1036901

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for the lifetime of the product for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS: PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2005 Invacare Corporation
Invacare is a registered trademark of
trademarks of Invacare Corporation.