

# ANTI-THEFT DEVICE



## Assembly, Installation and Operating Instructions

Model No. I514

NOTE: Check all parts for shipping damage before using. In case of damage, do NOT use. Contact the Dealer/Carrier for further instructions.

### SAFETY SUMMARY

The following recommendations are made for the installation of the anti-theft device:

#### WARNING

DO NOT assemble or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

NEVER maneuver an occupied wheelchair by using ANY part of the anti-theft device. Serious injury or damage can occur. ALWAYS use the handles of the wheelchair when maneuvering an occupied wheelchair.

This kit contains the following:

DESCRIPTION	QTY
Anti-Theft Bar	1
Instruction Sheet	1

### INSTALLING THE ANTI-THEFT DEVICE (FIGURE 1)

1. Remove the upper four (4) phillips screws and washers that secure the back upholstery to the back canes.

NOTE: Phillips screws and washers will be reused.

2. Position the anti-theft device brackets with the back upholstery screw holes.
3. Install the phillips screws through the washers, anti-theft device brackets, back upholstery and into the back canes.
4. Tighten the four (4) phillips screws securely.

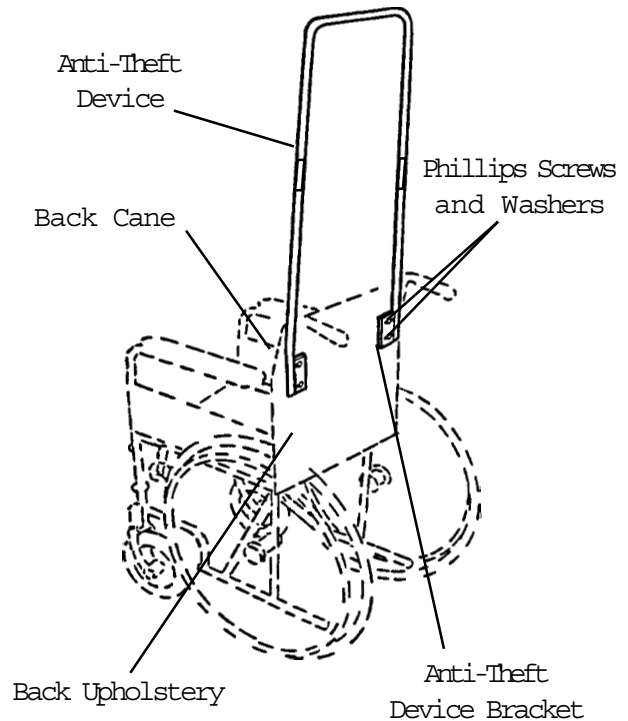


FIGURE 1 - INSTALLING THE ANTI-THEFT DEVICE

Invacare Corporation [www.invacare.com](http://www.invacare.com)



**USA**  
One Invacare Way  
Elyria, Ohio USA  
44036-2125  
800-333-6900

**Canada**  
5970 Chedworth Way  
Mississauga, Ontario  
L5R 3T9, Canada  
905-890-8838

Invacare is a registered trademark of Invacare Corporation.  
© 1999 Invacare Corporation

Form No.84-63 Part No. 1003365 Rev D (1) 1/00