SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact Carrier/Invacare for further instructions.

MODEL NO.	DESCRIPTION
95-1	One (1) Pack Shower Chair with Back Aluminum Legs
95-2	Two (2) Pack Shower Chair with Back Aluminum Legs
90-2	Two (2) Pack Shower Chair without Back Aluminum Legs

SAFETY SUMMARY

To ensure the safe and proper installation of the CareGuard Shower Chair, these instructions **MUST** be followed:

⚠ WARNING

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand these **Warnings, Cautions and Instructions**, contact a healthcare professional, dealer or technical personnel if applicable before attempting to install this equipment - otherwise, injury or damage may occur.

All four (4) leg tips **MUST** be in contact with shower/tub floor at all times.

Always inspect the shower chair to ensure that it is properly positioned and stable before using. **DO NOT** use if shower chair is wobbly or unstable.

Check leg tips for rips, tears, cracks or wear. If any of these conditions exist, replace leg tips immediately.

ALWAYS ensure that the snap buttons fully protrude through the same respective adjustment hole of each adjustable leg before using.

This product should only be used with tub floors wider than 16-inches.

Users with limited physical capabilities should be supervised or assisted when using the shower chair.

Shower chair is **NOT** to be used as a transfer bench, transfer device or climbing device.

These shower chairs (Models 95-1, 95-2, and 90-2) have a weight limitation of 300 lbs (136 kg).

Exercise caution when assembling the chair to avoid pinching.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

⚠️ INSTALLATION WARNING

After ANY adjustments, repair or service and BEFORE use, make sure that all attaching component parts are secure.

For Model Nos. 95-1, 95-2, and 90-2, ensure that the snap buttons fully protrude through the same respective adjustment hole of each adjustable leg before using. This ensures that the adjustable legs are securely locked in position and an even height adjustment is achieved.

ASSEMBLING THE SHOWER CHAIR


ATTACHING THE LEGS

NOTE: Refer to WARNINGS in the SAFETY SUMMARY of this instruction sheet.


NOTE: For this procedure, refer to FIGURE 1 and FIGURE 2.

1. Remove all contents from the package. For Model Nos. 95-1 and 95-2, put backrest to the side and refer to Installing/Removing the Backrest on page 4 after attaching and adjusting the legs.
2. Place seat upside down on a stable surface (floor, table, etc.).
3. Position one (1) leg over one (1) socket in the seat and align.
4. Angle the leg slightly so the rear corner enters the socket first as shown in FIGURE 1.
5. Push the leg down and rotate leg to front of socket until leg is fully seated in socket.


STEP 2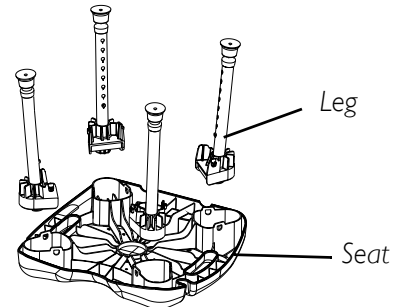

STEP 3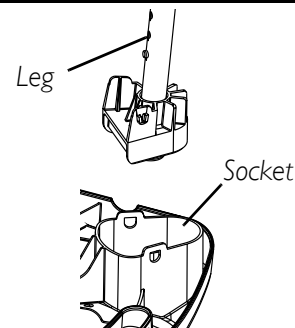

STEP 4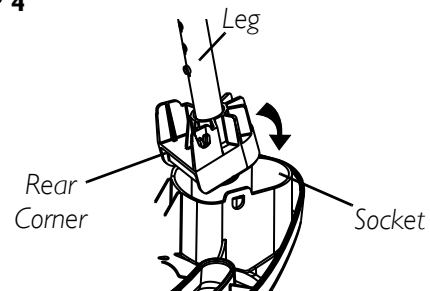

STEP 5

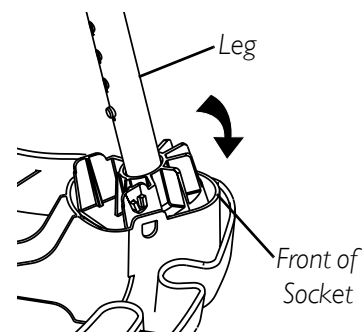
FIGURE 1 - ATTACHING THE LEGS

-
6. Ensure locking tabs are positioned through socket tab windows. Refer to FIGURE 2.

NOTE: Listen for two (2) audible clicks per leg to indicate tabs are properly engaged.

7. Pull up on the leg to ensure that it has “locked” into place.
8. Visually inspect tab windows to ensure locking tabs are properly engaged.
9. Repeat STEPS 3-5 for the remaining three (3) legs.

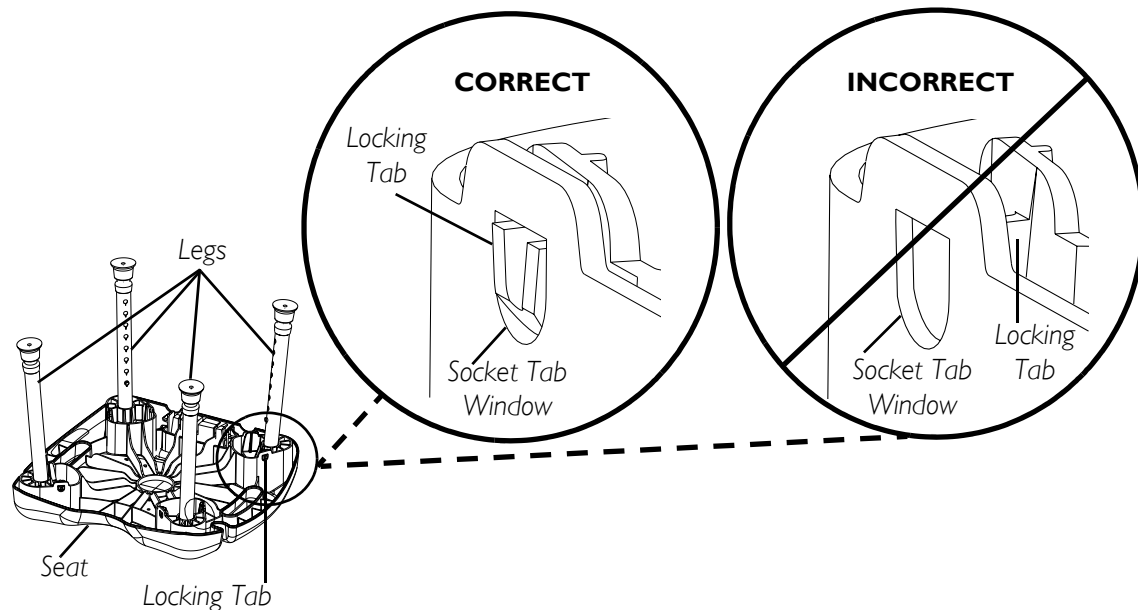
FIGURE 2 - LEGS INSERTED

ADJUSTING THE LEG EXTENSIONS

⚠ WARNING

Ensure that the shower chair is level and stable before using.

NOTE: Refer to WARNINGS in the SAFETY SUMMARY of this instruction sheet.

NOTE: For this procedure, refer to FIGURE 3.

1. Ensure the legs are attached. Refer to Attaching the Legs on page 2.
2. Place seat and legs upside down so legs are facing up.
3. Select one (1) of the eight (8) adjustment holes located on the leg frame.
4. Depress the snap button and slide the leg extension up or down to the desired height.
5. Ensure that the snap button protrudes fully through the adjustment hole of the leg frame. Repeat for the other three (3) legs.
6. Make sure all four (4) legs are adjusted to the same height so that the Shower Chair sits level.

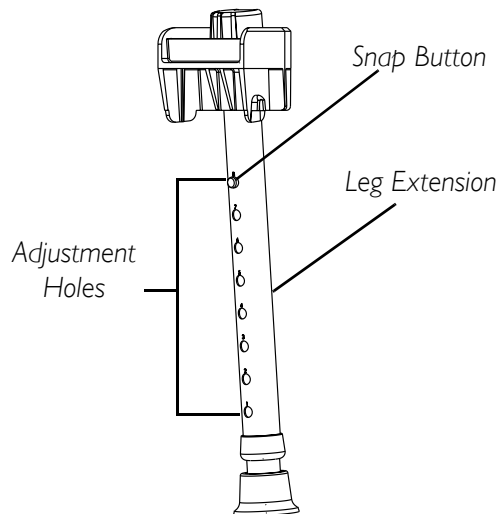
FIGURE 3 - ADJUSTING THE LEG EXTENSIONS

INSTALLING/REMOVING THE BACKREST

NOTE: Refer to WARNINGS in the SAFETY SUMMARY of this instruction sheet.

NOTE: For this procedure, refer to FIGURE 4.

INSTALLING

1. Before installing the backrest, attach and adjust the legs so the shower chair is level. Refer to Attaching the Legs on page 2 and Adjusting the Leg Extensions on page 3.
2. Position the backrest into the slot on the seat.
3. Push down on the backrest until it sits securely into the seat slot.
4. Check under the seat to ensure the compression tabs on the backrest protrude beyond the seat slot. If they are visible, the backrest is attached properly. If they are not, push the backrest further into the seat slot. Refer to Detail "A" in FIGURE 4.
5. Before using, pull upward on the backrest to ensure the backrest is securely attached and locked into place.

REMOVING

1. Locate the compression tabs underneath the seat.
2. Depress both compression tabs with one hand.
3. Using the other hand, hold the backrest handle and pull up and away from the seat.
4. Release the compression tabs to avoid pinching and continue pulling the backrest up until it is removed from the seat.

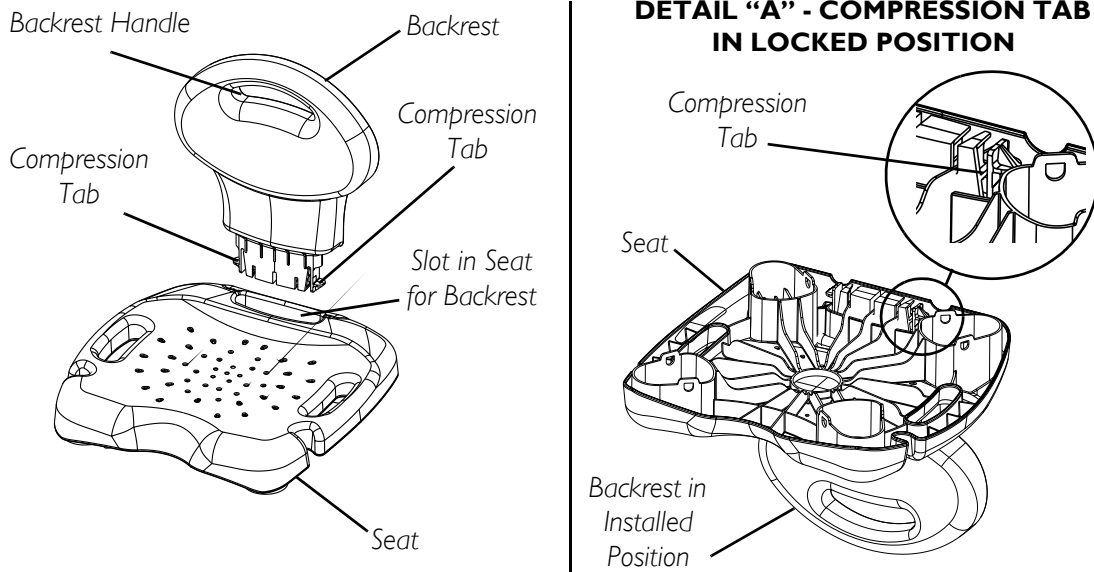
FIGURE 4 - INSTALLING/REMOVING THE BACKREST

CLEANING

NOTE: Before using any cleaning products, test an area on the underside of the seat for any discoloration or staining.

1. Use water and a mild soap or non-abrasive cleanser to clean the shower chair.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our product.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for the lifetime of use by original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, IMPROPER MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER. SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESSED OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/ PROVINCIAL LAWS AND REQUIREMENTS.

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© 2004 Invacare Corporation
Invacare is a registered trademark of
trademarks of Invacare Corporation.
Invacare Corporation www.invacare.com