SAVE THESE INSTRUCTIONS**Safety Summary**

To ensure the safe and proper installation of the WARNING/CAUTION labels, these instructions **MUST** be followed:

⚠ WARNING

DO NOT install or use this kit without first reading and understanding this instruction sheet. If you are unable to understand the **Warnings, Cautions or Instructions**, contact Invacare Technical Support before attempting to install this kit - otherwise, injury or damage may occur.

DO NOT use the wheelchair without installing these **Warning and Caution** labels.

This kit contains the following labels:

| KIT NUMBER | PART NUMBER | REPLACE LABEL |
|------------|-------------|---------------|
| 1145767 | 1123824 | 1118367 |
| | 1123821 | 1114826 |
| | 1123827 | 1114847 |
| | 1123822 | 1114825 |
| | 1123826 | 1114848 |
| 1145769 | 1143218 | 1082692 |
| | 1143217 | 1083199 |
| | 1145754 | 1134848 |

Installing Warning/Caution Labels

1. Determine the proper location for each label. Refer to FIGURE 1, FIGURE 2, FIGURE 3 and FIGURE 4.

NOTE: The English labels are included in FIGURE 1, FIGURE 2, FIGURE 3 and FIGURE 4 for ease in determining proper location.

2. Peel one label from the adhesive backing.
3. Position the appropriate warning or caution label directly over the English label at the location determined in STEP 1.
4. Repeat STEPS 1-3 for each label.

**For more information regarding Invacare products, parts, and services,
please visit www.invacare.com**

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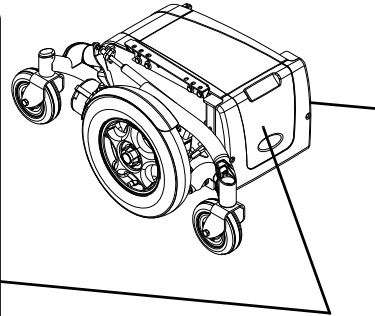
⚠ WARNING
 Wiring Diagram and Battery Install/Remove for GP24 Batteries
DO NOT REMOVE THIS LABEL

The POSITIVE (+) RED Battery Cable MUST connect to the POSITIVE (+) Battery Terminal(s)/ Post(s). The NEGATIVE (-) BLACK Battery Cable MUST connect to the NEGATIVE (-) Battery Terminal(s)/Post(s). DO NOT allow Battery Cable(s) to contact the opposite Battery Terminal(s)/Post(s). Install protective caps on positive and negative battery terminals. Replace cable(s) immediately if cable(s) insulation becomes damaged. Failure to observe these warnings may result in an electrical short with serious personal injury and/or damage to the electrical system. See Owner's Manual, part number 1114809.

DO NOT remove fuse or mounting hardware from POSITIVE (+) RED battery cable mounting screw.

GP24 Batteries
 P/N 1114825
 Rev B - 6/04

⚠ WARNING
 GP24 batteries with terminal configuration (negative on the left and positive on the right) as shown MUST be used. GP24 batteries that have the reverse terminal configuration MUST not be used. Terminals MUST have a cross hole located as shown for proper battery connection. See Owner's Manual, part number 1114809. These recommendations MUST be followed otherwise injury and damage may occur.

P/N 1114848 Rev A - 2/03

Cross Hole
 Negative Terminal Positive Terminal
USE this battery terminal configuration

Positive Terminal Negative Terminal
DO NOT use this battery terminal configuration

FIGURE 3 Installing Warning/Caution Labels - Installer les étiquettes avertissement/attention

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All rights reserved. Trademarks are identified by the symbols [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.
Invacare Corporation
© 2009 Invacare Corporation