

### SAFETY SUMMARY

The following recommendations are made for the safe installation of the O2 Holder Accessory Kit:

#### WARNING


DO NOT assemble or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact Carrier/Dealer for further instructions.

Ensure that ALL attaching hardware is securely assembled to the wheelchair before operating the chair with the O2 Cylinder Holder Accessory Kit.

Follow the Oxygen Cylinder manufacturers guidelines for proper usage of the oxygen cylinder.

2. Push the bracket of the O2 cylinder holder over the rear upright of the wheelchair and secure in place with the hex screw, spacer and locknut.
3. Install plug button on top of O2 cylinder holder.
4. Use the long adjustment knob on the O2 cylinder holder bracket to secure the oxygen cylinder in place.

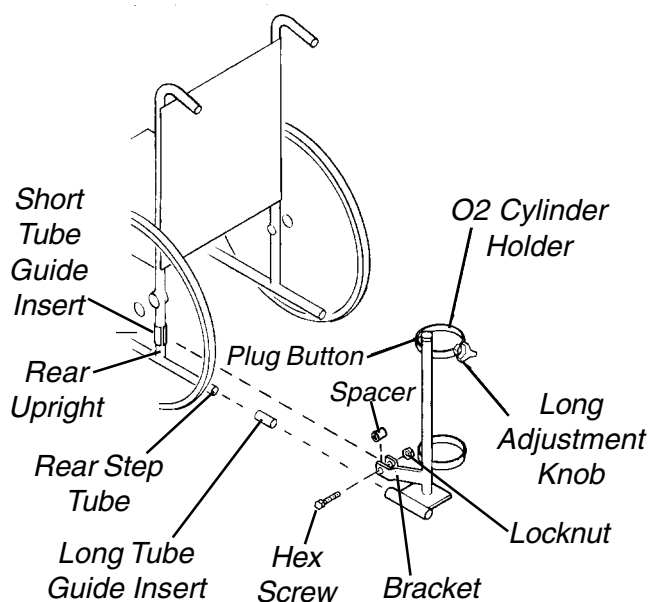

### INSTALLING O2 CYLINDER HOLDER (FIGURE 1)

*NOTE: If rear step tube is 7/8-inch diameter tubing, install the long tube guide insert into the rear step tube before installing the O2 cylinder holder.*

1. Place the O2 cylinder holder and slide it onto the rear step tube of the wheelchair.

*NOTE: IF rear upright tubing is 7/8-inch diameter tubing, install the short tube guide insert over the rear upright before installing and securing the O2 cylinder holder bracket.*

FIGURE 1 - INSTALLING O2 CYLINDER HOLDER



**Yes, you can.™**

Invacare Corporation

[www.invacare.com](http://www.invacare.com)

**USA**  
One Invacare Way  
Elyria, Ohio USA  
44036-2125  
800-333-6900

**Canada**  
5970 Chedworth Way  
Mississauga, Ontario  
L5R 3T9, Canada  
905-890-8838

Invacare® and Yes, you can. are trademarks of Invacare Corporation.

© 2000 Invacare Corporation

Form No. 91-200 Part No. 1027930 Rev E (2) 12/00