**ROLLITE™ ROLLATORS  
MARCHETTES ROLLITES™  
ANDADERAS RODANTES ROLLITE™**

Refer to page 2.  
Se référer à la page 13.  
Refiérase a la página 28.

**Invacare Rollators provide assistance for an individual weighing up to 300 lbs. (136 kg.)  
- Models 65100, 65100R, 65100-JR, 65100R-JR, 68100, and 68100-TA.**

**ROLLITE ROLLATORS**

Model No.	Width		Depth		Height Adjustment	Approximate Patient Height Min./Max.
	Inside Hand Grips	Inside Base Legs	Opened at Base	Folded		
65100	18" (46cm)	22" (56cm)	24" (61cm)	7" (18cm)	32"-36" (81-91cm)	5'5"-6'0" (165-183cm)
65100R	18" (46cm)	22" (56cm)	24" (61cm)	7" (18cm)	32"-36" (81-91cm)	5'5"-6'0" (165-183cm)
65100-JR	18" (46cm)	22" (56cm)	24" (61cm)	7" (18cm)	29"-33" (74-84cm)	5'0"-5'7" (152-170cm)
65100R-JR	18" (46cm)	22" (56cm)	24" (61cm)	7" (18cm)	29"-33" (74-84cm)	5'0"-5'7" (152-170cm)
68100	18" (46cm)	22" (56cm)	26" (66cm)	7" (18cm)	32"-36" (81-91cm)	5'5"-6'0" (165-183cm)
68100-TA	18" (46cm)	22" (56cm)	26" (66cm)	7" (18cm)	35"-39" (89-99cm)	5'11"-6'6" (180-198cm)

Specifications are subject to change without notice.

**For more information regarding  
Invacare products, parts, and services,  
please visit: [www.invacare.com](http://www.invacare.com)**

**SAVE THESE INSTRUCTIONS**

---

*NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact Dealer/Carrier for further instruction.*

## **SAFETY SUMMARY**

To ensure the safe operation of the **ROLLITE ROLLATORS**, these instructions **MUST** be followed:

### **GENERAL WARNING**

**DO NOT** install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment- otherwise, injury or damage may occur.

Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

Rollators are **NOT** intended to be self propelled while seated.

A physical/occupational therapist should assist in the height adjustment of the rollator for maximum support and correct brake activation.

Always ensure brake cables are flush along leg extensions. Brake cables should always be secured with the supplied tie wraps to the side braces. After any height adjustment, reposition brake cables so that any excess cable loops out the front of the rollator. Failure to follow these warnings can result in the brake prematurely gripping the wheel and may result in personal injury.

Care should be taken to ensure that **ALL** hand and height adjustments are secure, and that casters and moving parts are in good working order before using this or any mobility aid.

---

---

## **SAFETY SUMMARY (CONTINUED)**

### **GENERAL WARNING**

All wheels **MUST** be in contact with the floor at **ALL TIMES** during use. This will ensure the rollator is properly balanced.

The brakes **MUST** be in the locked position **BEFORE** using the seat.

For optimal stability, the seat should be in the **DOWN** position when using the walker for ambulatory assistance.

When using the rollator in a stationary position, the hand brakes **MUST** be locked.

### **STABILITY WARNINGS**

The rollator can provide ambulatory assistance to an individual weighing up to 300 lbs. (136 kg), **INCLUDING** the weight of the contents of the basket, if so equipped.

The rollator basket has a weight limitation of 10 lbs. (4.5 kg.).

### **FOLDING/ASSEMBLY WARNINGS**

After unfolding or assembling rollator, ensure that rollator is securely locked in **OPEN** position and level to the ground **BEFORE** using.

**DO NOT** hang anything on either the left or right side frames (area that includes dual release paddles). This will cause side frames to bend downward and prevent rollator from locking into place when fully opened. This may also cause rollator to tip, resulting in injury or damage.

### **INSTALLATION WARNINGS**

Ensure all snap buttons fully protrude through adjustment holes of each leg extension. This ensures that the leg extensions are securely locked in position.

The leg extensions should be adjusted so the rollator is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the rollator is no more than one (1) inch lower than the front.

---

## **SAFETY SUMMARY (CONTINUED)**

### **INSTALLATION WARNINGS**

**Always test to see that the rollator and attachments are properly and securely locked in place BEFORE using.**

---

## **OPENING/FOLDING THE ROLLATOR (FIGURE 1)**

*NOTE: Refer to the FOLDING/ASSEMBLY WARNINGS in the SAFETY SUMMARY of this manual.*

### **OPENING**

1. Swing the side frames outward until the release paddles activate **(engage upward)**.
2. Pull the handle **DOWN** and lower the seat.

*NOTE: Push DOWN until the seat supports fully engage the side braces as shown in OPEN of FIGURE 1.*


3. Before using, test rollator by grabbing the side frames and, with some force, attempt to fold them inward. The rollator will remain locked in the OPEN position if it is engaged properly.

### **FOLDING**

1. Raise the seat by grasping the handle and lifting UP.
2. Depress either release paddle and pivot the corresponding side frame inward toward the front crossbrace.
3. Repeat the same procedure for the remaining side frame.

---

*NOTE: Back rest not shown for clarity. Model No. 65100 shown.*

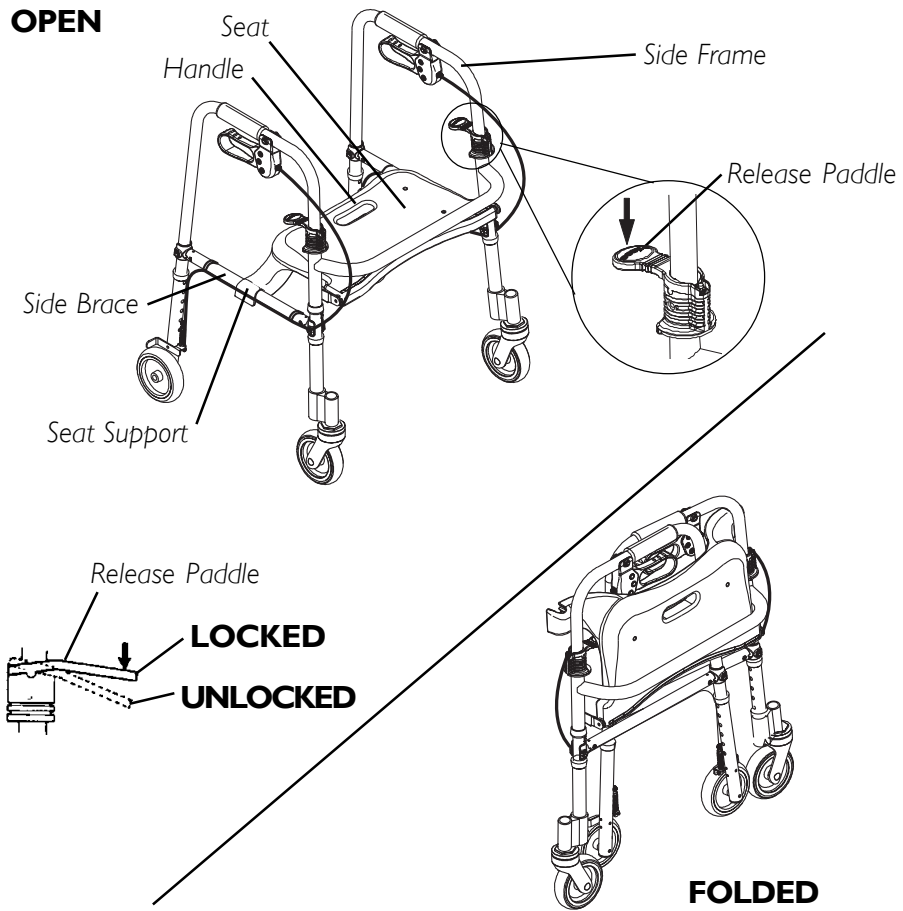**FIGURE I - OPENING/FOLDING THE ROLLATOR**

---

## **INSTALLING/ADJUSTING LEG EXTENSIONS/BRAKE CABLES (FIGURE 2)**

*NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.*




*NOTE: For the most comfortable position, make sure the adjustment is performed while the user is wearing the shoes he/she wears most often.*

*NOTE: Adjust the rollator height so that when the user's arm is down to their side, the hand grip is at wrist height. This will ensure the arms are at an approximate 20° - 30° bend when using the rollator.*

1. Depress the snap buttons on the front legs of the rollator and slide the leg extensions up or down to the desired height. Ensure that the snap button protrudes through the appropriate adjustment hole on the leg extension (FIGURE 2, DETAIL "A").
2. Push/pull the brake cables along the side braces so that excess brake cable is near the rear leg extensions (FIGURE 2, DETAIL "B").
3. Repeat STEP 1 for the rear leg extensions.
4. After adjusting the rear leg extensions, remove the excess brake cable (FIGURE 2, DETAIL "B") near the rear leg extensions and side braces by pulling brake cables through the cable tie wraps until the excess brake cables loop out the front of the rollator (FIGURE 2, DETAIL "C").

*NOTE: For the brakes to function properly, the brake cable should be flush along the leg extension and side brace and the excess brake cable should loop out the front of the rollator (FIGURE 2, DETAIL "C").*

5. Ensure all legs are adjusted to the same height.

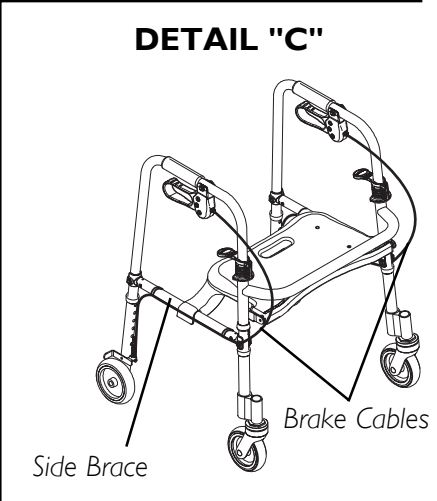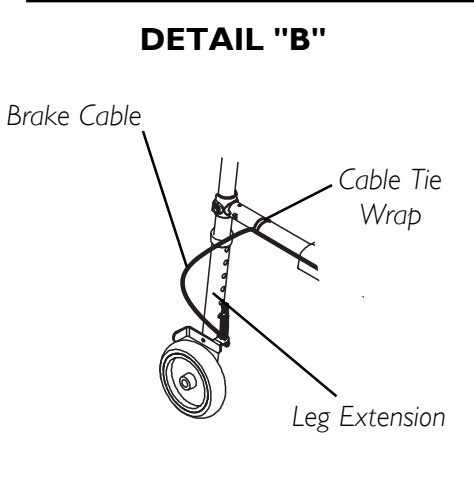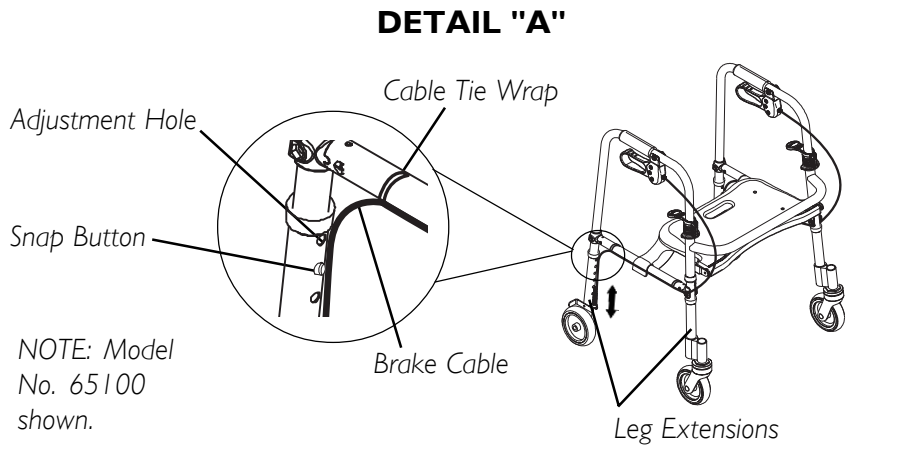**FIGURE 2 - INSTALLING/ADJUSTING LEG EXTENSIONS/BRAKE CABLE**

---

## **USING/LOCKING/UNLOCKING HAND BRAKES (FIGURE 3)**

*NOTE: Refer to INSTALLATION WARNINGS in the SAFETY SUMMARY in this instruction sheet.*

*NOTE: If brakes prematurely lock, refer to INSTALLING/ADJUSTING LEG EXTENSIONS/BRAKE CABLES in this instruction sheet.*

### **USING HAND BRAKES**


1. Pull the top portion of the brake handles UP towards the hand grips.
2. Do one (1) of the following:
  - A. REMAIN STATIONARY - hold the brake handle UP.
  - B. CONTINUE MOBILITY - release the brake handle.

### **LOCKING HAND BRAKES**

1. Push DOWN on the bottom portion of the brake handle as shown in FIGURE 3 until an audible “click” is heard.
2. Refer to UNLOCKING HAND BRAKES in this procedure of the instruction sheet.

### **UNLOCKING HAND BRAKES**

1. Pull UP on the top portion of the brake handle to release as shown in FIGURE 3.
2. Release brake handle.

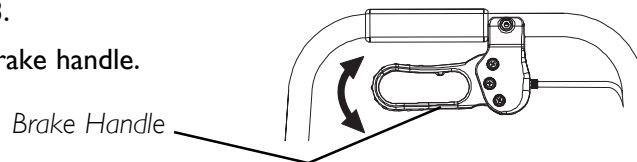**FIGURE 3 - USING/LOCKING/  
UNLOCKING THE HAND BRAKES**

---

## ADJUSTING THE HAND BRAKES


*NOTE: COUNTERCLOCKWISE/CLOCKWISE directions are determined by standing behind the rollator (user's position).*

*NOTE: Refer to INSTALLATION WARNINGS in the SAFETY SUMMARY in this instruction sheet.*

### BRAKE HANDLE (FIGURE 4)

Test the brake handle. Observe how the brake lever engages the wheel. If the tension of the brake is too loose or too tight, adjust using the following steps:

1. Loosen the brake handle adjustment nut by turning **CLOCKWISE**.
2. Perform one (1) of the following:
  - A. **LOOSEN BRAKE HANDLE TENSION** - turn brake handle nut **CLOCKWISE**.  
*NOTE: If the brake lever is scraping the rear wheel, the brake handle must be loosened.*
  - B. **TIGHTEN BRAKE HANDLE TENSION** - turn brake handle nut **COUNTERCLOCKWISE**.  
*NOTE: If the brake lever does not respond quickly, tighten brakes.*
3. Turn the brake handle adjustment nut **COUNTERCLOCKWISE** to secure in place.
4. Do one (1) of the following:
  - A. **ACCEPTABLE TENSION** - repeat STEPS 1 - 3 for opposite side, if necessary.
  - B. **UNACCEPTABLE TENSION** - refer to CABLE ADJUSTER UNIT in this procedure of the instruction sheet.

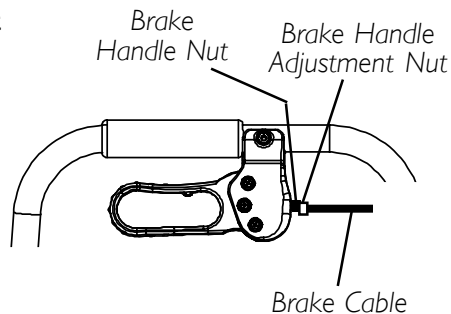**FIGURE 4 - BRAKE HANDLE**


---

## **CABLE ADJUSTER UNIT (FIGURE 5)**

1. Test the brake with the brake handle. Observe how the brake lever engages the wheel. If the tension of the brake handle is **STILL** too loose or too tight, adjust using the following steps:

- A. Ensure brake handle is not locked.
- B. **TO LOOSEN:** Brace the jam nut while turning the adjustment screw **CLOCKWISE**.
- C. **TO TIGHTEN:** Brace the jam nut while turning the adjustment screw **COUNTERCLOCKWISE**.

- D. Repeat with other brake.

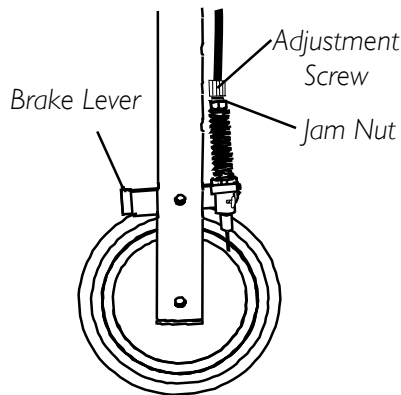**FIGURE 5 - CABLE ADJUSTER UNIT**

## **CARE AND MAINTENANCE**

1. Ensure that **ALL** attaching hardware is secure at **ALL** times.
2. Check screw securing seat to rollator for tightness. Tighten if necessary.
3. Make sure snap buttons release and re-engage properly.
4. Replace any broken, damaged or worn items immediately.

## **CARE OF WHEELED ACCESSORIES**

1. Clean wheels with warm water and mild cleanser. Dry with clean rag.
2. Lubricate with 3-in-1<sup>®</sup> oil periodically or if wheels begin to squeak.

---

## **LIFETIME LIMITED WARRANTY**

**NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our product.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the frame to be free from defects in materials and workmanship for the lifetime of the product, six (6) years on the hand brakes and one (1) year for all other remaining components, excluding tires/wheels for the original purchaser/user. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealers name, address, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

**LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, IMPROPER MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE; PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL. SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THESE INSTRUCTIONS.**

**THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSIONS AND LIMITATIONS MAY NOT APPLY TO YOU.**

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### **REGISTER YOUR PRODUCT!**

**The benefits of registering:**

1. Safeguard your investment.
2. Ensure long term maintenance and servicing of your purchase.
3. Receive updates with product information, maintenance tips, and industry news.
4. Invacare can contact you or your provider, if servicing is needed on your product.
5. It will enable Invacare to improve product designs based on your input and needs.

**Register ONLINE at [www.invacare.com](http://www.invacare.com)**

**- or -**

**Detach form, complete and mail.**

Any registration information you submit will be used by Invacare Corporation only, and protected as required by applicable laws and regulations.

---

**Name** \_\_\_\_\_

**Address** \_\_\_\_\_

**City** \_\_\_\_\_ **State/Province** \_\_\_\_\_

**Zip/Postal Code** \_\_\_\_\_

**Email** \_\_\_\_\_ **Phone No.** \_\_\_\_\_

**Invacare Model No.** \_\_\_\_\_

**Serial No.** \_\_\_\_\_

**Purchased From** \_\_\_\_\_

**Date of Purchase:** \_\_\_\_\_

---

1. Method of purchase: (check all that apply)

Medicare       Insurance       Medicaid       Other

2. This product was purchased for use by: (check one)

Self       Parent       Spouse       Other

3. Product was purchased for use at:

Home       Facility       Other

4. I purchased an Invacare product because:

Price       Features (list features)

5. Who referred you to Invacare products? (check all that apply)

Doctor       Therapist       Friend       Relative       Other

No referral       Advertisement (circle one): TV, Radio, Magazine, Newspaper

6. What additional features, if any, would you like to see on this product?

\_\_\_\_\_

7. Would you like information sent to you about Invacare products that may be available for a particular medical condition?  Yes  No

If yes, please list any condition(s) here and we will send you information by email and/or mail about any available Invacare products that may help treat, care for or manage such condition(s):

\_\_\_\_\_

8. Would you like to receive updated information via email or regular mail about the Invacare home medical products sold by Invacare's dealers?  Yes  No

\_\_\_\_\_

9. What would you like to see on the Invacare website?

\_\_\_\_\_

10. Would you like to be part of future online surveys for Invacare products?

Yes  No


11. User's Year of birth: \_\_\_\_\_

If at any time you wish not to receive future mailings from us, please contact us at Invacare Corporation, CRM Department, 39400 Taylor Parkway, Elyria, OH 44035, or fax to 877-619-7996 and we will remove you from our mailing list.

---

**To find more information about our products, visit  
[www.invacare.com](http://www.invacare.com).**

Please Seal with  
Tape Before Mailing

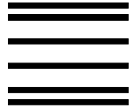**BUSINESS REPLY MAIL**

FIRST-CLASS MAIL PERMIT NO. 196 ELYRIA OH


POSTAGE WILL BE PAID BY ADDRESSEE


INVACARE CORPORATION  
CRM DEPARTMENT  
**39400 TAYLOR PARKWAY**  
**ELYRIA OH 44035-9836**

Invacare Product Registration Form

NO POSTAGE  
NECESSARY  
IF MAILED  
IN THE  
UNITED STATES

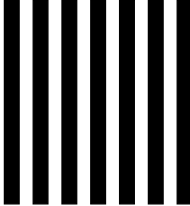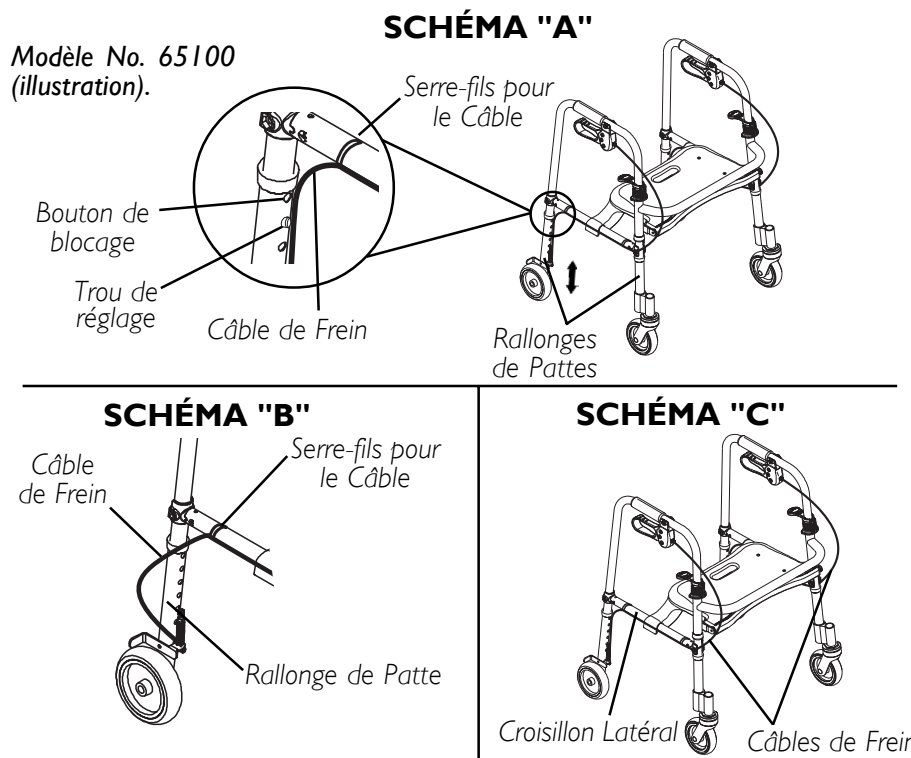**FIGURE 2 - INSTALLATION/RÉGLAGE DES EXTENSIONS DE PIED**

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© 2004 Invacare Corporation
Invacare is a registered trademark of
3-in-1 is a registered trademark of
Yes, you can and Rollite are trademarks
Invacare Corporation [www.invacare.com](http://www.invacare.com)
American Home Products Corporation.