Rubber Tips for Patient Lifts

Assembly, Installation and Operating Instructions

Model Nos. RHA450-I, RHL450-I, RPA450-I, RPA450-IE, RPL450-I, RPL450-IE,
RPA600-I, RPA600-IE, RPL600-I

Yes, you can.

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact Carrier/Invacare for further instructions.

Safety Summary

To ensure the safe and proper installation of the Rubber Tips, these instructions **MUST** be followed:

⚠ WARNING

DO NOT install this equipment without first reading and understanding this instruction sheet. If you are unable to understand the warnings, cautions and instructions, contact a healthcare professional, dealer or technical personnel if applicable, before attempting to install this equipment - otherwise, injury or damage may occur.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

Installing Rubber Tips on the Hanger Bar

NOTE: For this procedure, refer to FIGURE 1.

1. Align the rubber tip with the end of the hanger bar hook.
2. Firmly press the rubber tip onto the hanger bar hook.
3. Ensure the rubber tip is pressed all the way onto the hanger bar hook by flexing it.

NOTE: A blunt tool may be used to press the rubber tip on.

4. Repeat STEPS 1-3 for the remaining hanger bar hooks.

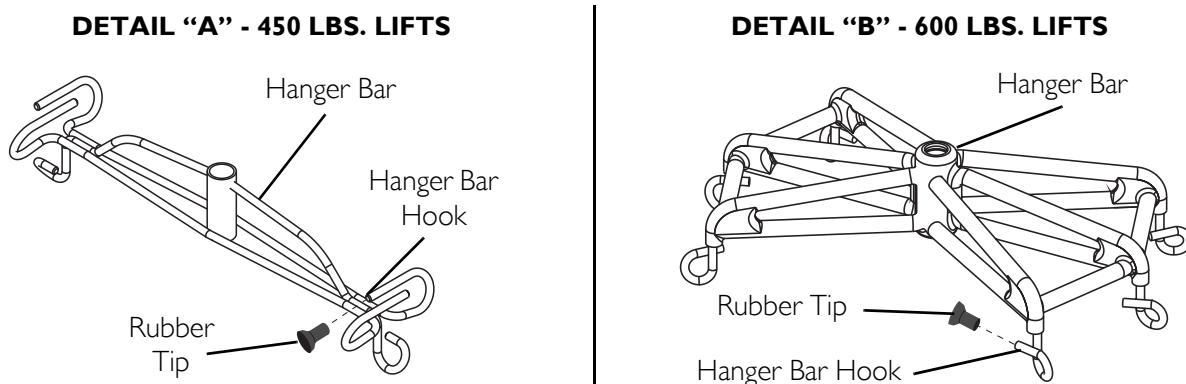
FIGURE 1 Installing Rubber Tips on the Hanger Bar

**For more information regarding Invacare products, parts, and services,
please visit www.invacare.com**

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© 2005 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of Invacare
Invacare Corporation
Corporation.