**MOPEDIA**

**SOLLEVATORI ELETTRICI E OLEODINAMICO  
LIFTERS ELECTRIC AND OLEODYNAMIC  
ÉLÉVATEURS ÉLETRIQUES ET OLÉODYNAMIQUE  
LEVANTADORES ELÉCTRICOS Y OLEODINÀMICO**

- **Manuale di istruzioni**
- **Instruction manual**
- **Manuel d'instruction**
- **Manual de instrucciones**

Mod. 71 - 08/2005

Moretti S.p.A.  
Via Bruxelles 3 - Melegnano  
52022 Cavriglia (Arezzo)  
Telefono +39 055 96 21 11

Fax +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)

**ITALIANO**

**ENGLISH**

**FRANCAIS**

**ESPAÑOL**

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**MOPEDIA**

# SOLLEVATORI

MODELLI: RP800 - RP805 - RP810

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# LIFTERS

MODELS: RP800 - RP805 - RP810

READ THIS INSTRUCTION  
MANUAL CAREFULLY BEFORE USE

## INDEX

<b>INTRODUCTION</b> .....	<b>13</b>
<b>DESTINATION OF USE</b> .....	<b>13</b>
<b>DECLARATION OF CE CONFORMITY</b> .....	<b>13</b>
<b>MODELS</b> .....	<b>13</b>
<b>DIMENSIONS/SIZES</b> .....	<b>14</b>
<b>ASSEMBLAGE</b> .....	<b>15</b>
<b>USE</b> .....	<b>16</b>
<b>CLEANING</b> .....	<b>17</b>
<b>MAINTENANCE</b> .....	<b>17</b>
<b>ACCESSORIES</b> .....	<b>18</b>
<b>GUARANTEE</b> .....	<b>18</b>

## INTRODUCTION

Thank you for having chosen a lifter from the **MOPEDIA** line by MORETTI. Its design and qualities guarantee the maximum comfort, the maximum safety and reliability. The **MOPEDIA** lifters by MORETTI have been designed and made to satisfy all your demands for a correct and practical use. This handbook gives you some little suggestions for a correct use of the device you have chosen and some useful advice for your safety. We suggest you to read carefully the whole handbook before using your lifter. In case of any doubt, please contact your retailer, who will be able to help you and give you some correct advice.

## DESTINATION OF USE

The lifters of the line **MOPEDIA** are made to lift and transfer people with difficulties in movement, with the help of an attendant who operates the commands. They cannot be used by a single person. MORETTI S.p.A. declines any responsibility for any damage caused by an improper use of the device or by a different use from that as suggested in the present handbook.

## DECLARATION OF EC CONFORMITY

MORETTI S.p.A. declares under its exclusive responsibility that the products made and put into the market by the same MORETTI S.p.A. , as part of the category of

### **LIFTERS**

are in conformity with the applicable dispositions of the **EEC DIRECTIVE 93/42** concerning **MEDICAL DEVICES**.

To such purpose MORETTI S.p.A. guarantees and declares under its own exclusive responsibility what follows:

1. The concerned devices meet the essential requisites as according to the 1° enclosure of the directive 93/42/CEE as prescribed by the enclosure VII° of the aforesaid directive.
2. A complete list of the concerned devices is mentioned in the enclosure A of the present declaration.
3. The concerned devices ARE NOT INSTRUMENTS OF MEASURE.
4. The concerned devices ARE NOT DESTINED FOR CLINICAL SURVEYS.
5. The concerned devices are sold in a NON STERILE packing.
6. The concerned devices are to be considered as belonging to the 1° class.
7. Any use or installation of the concerned devices that differs from the destination of use, as it is foreseen by MORETTI S.p.A., IS NOT ALLOWED.
8. MORETTI S.p.A. keeps and makes available to the Competent Authorities the technical documentation that proves the conformity to the EEC Directive 93/42.

## MODELS

The line of Lifters **MOPEDIA** by MORETTI is composed of three different products:

**RP800** OLEODYNAMIC LIFTER max. weight each patient 150kg

**RP805** ELECTRIC LIFTER max. weight each patient 150kg

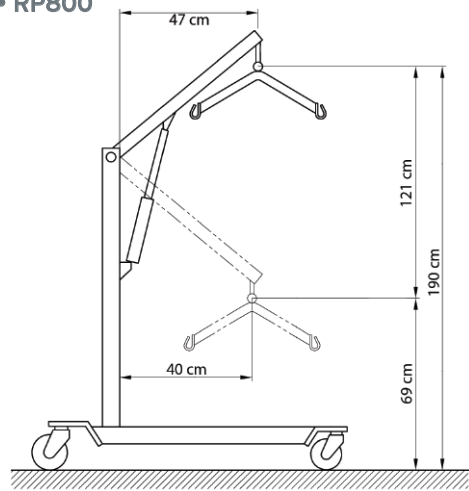
**RP810** ELECTRIC LIFTER max. weight each patient 180kg

# 14

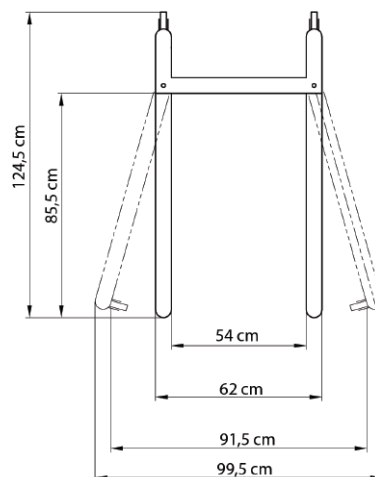
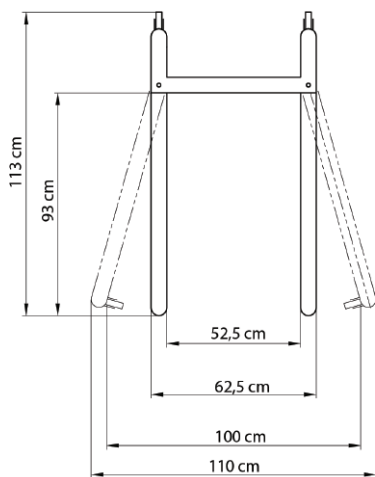
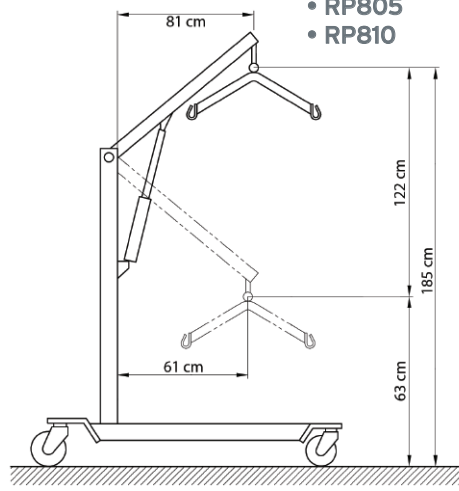
The present handbook refers to all models of the **MOPEDIA** line. Please, only read the instructions related to the model you have chosen.

## DIMENSIONS/SIZES

• **RP800**

• **RP805**  
• **RP810**

**ASSEMBLAGE****OLEODYNAMIC LIFTER.**

Remove the fastening spring from the check pin of the base. **(Fig. 1)**

Release the check pin from the base.

Insert the mast with the oleodynamic pump into the mast sleeve and carefully point the boom in the same direction as the base legs.

Insert the check pin and fix the screws. **(Fig. 2)**

Block the check pin with the fastening spring.

Screw the handles for movement into their proper place. **(Fig. 3)**

Insert the lever for the opening of the base into its correct place and make sure to block the inferior part.

Attach the cradle at the end of the boom in correspondence of the U-shaped hook. **(Fig. 4)** Hang the sling to the cradle.

FIGURA 3

FIGURA 4

FIGURA 1

FIGURA 2

FIGURA 5

**ELECTRIC LIFTERS.**

Release the fastening screws away from the base and from the mast. Insert the mast into the mast sleeve and make sure that the boom is pointed in the same direction as the base legs. **(Fig. 5)**

Screw the fixing screws of the base.

Install the handles in their correct place and block them with the fastening screws. **(Fig. 6)**

Release the central screw **(Fig. 7)** from the supporting bracket that is fixed on the external side of the mast.

Insert the LINAK - Jumbo control box on the inferior part of the bracket and ensure it correctly in the inferior slides.

Fix the control box with the screw previously removed.

Insert the LINAK - Jumbo battery on the superior part of the bracket, first fixing the inferior part and then blocking the superior one.

Connect the LINAK - Jumbo actuator and the control handset to the control box by using the connectors that are in the inferior part. **(Fig. 8)**

Insert the lever for the opening of the base into its correct place and make sure to block the inferior part. **(Fig. 9)**

Attach the cradle at the end of the boom in correspondence of the U-shaped hook. **(Fig. 10)**

Hang the sling to the cradle

FIGURA 8

FIGURA 9

FIGURA 6

FIGURA 7

FIGURA 10

# 16

## USE

### **Oleodynamic lifter.**

Spread the base legs to the widest position before lifting by slightly throwing the opening lever towards you and blocking it in one of the five possible positions. Lock rear casters. Keep the patient centred between the base legs - the patient has to face his attendant. Fasten with care the strings of the slings to the cradle. Make sure that the cylindrical valve is closed. Rotate the valve clockwise towards "Closed".

Operate the pumping lever to begin the lifting. Lift the patient up to the necessary height, keeping him in frontal position towards his attendant.

Release the rear casters and transfer the patient. Before beginning the descent of the patient lock the rear casters. Begin the descent by slightly rotating the valve anti-clockwise towards "Open". In order to make a slow descent slightly rotate the valve, while if you want to increase the speed of descent open more the valve.

During the descent, assist the patient to attain correct sitting posture.

### **Electric lifter.**

Spread the base legs to the widest position before lifting by slightly pulling the opening lever towards you and blocking it in the opening position (Fig. 11)

Lock rear casters.

Keep the patient centred between the base legs - the patient has to face his attendant.

Fasten with care the strings of the sling to the cradle

Operate the lifting by pressing the button (Arrow up) on the control handset (Fig. 12)

Lift the patient up to the necessary height, keeping him in a frontal position towards his attendant.

Release the rear casters and transfer the patient.

Before beginning the descent of the patient lock the rear casters.

Begin the descent by pressing the button (Arrow down) on the control handset.

During the descent, assist the patient to attain correct sitting posture.

### **Button of Emergency.**

The electric lifters are endowed with a button of emergency (red button) that, if pressed, immediately disarms all the commands. To restore the operation rotate the red button clockwise. (Fig. 13)

### **Descent of emergency (only for model RP810).**

When a rapid descent of the patient is necessary, operate a rapid release and unhook the mechanical boom by lifting it. In this way the patient will immediately begin the descent without operating the control handset. (Fig. 14)

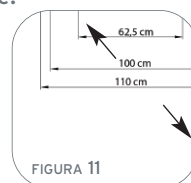
FIGURA 11

FIGURA 12

FIGURA 13

FIGURA 14

## ATTENTION

The electric lifter is provided with a safety device that stops the descent if there is any obstacle. We recommend you to let the patient go up again before removing the obstacle so as to avoid any fall.

### Recharge of the battery. (Only electric lifters.)

The batteries of the lifters of the **MOPEDIA** line can be recharged through the battery charger supplied with, whether the batteries are inserted in the lifter by using the connection that is under the control box, or by disconnecting the battery and using the present connector. While recharging the light "charge" is on. When the loading is ended, the light "charge" will automatically turn off.

## ATTENTION

During a normal functioning, when the battery power is 50% low, the control system will give "beep" warning. At this point the battery has to be immediately re-charged after having finished the transfer in progress.

### Warnings:

- The lifter is a device that allows short transfers of a patient with difficulties in movement, as for example from a bed to a wheelchair or to a commode or vice versa. The lifter must not be used to transport the patient.
- During the lifting of the patient, make sure that the base legs are in the most widely opened position and that the brakes of the rear casters are engaged.
- During the transfer do avoid to go against obstacles or the lifter might tilt over.
- Always check that the slings are in a good state.  
Don't use slings that are damaged or worn out.

## CLEANING

To clean slings use a solution of alcohol and neutral soap in equal parts mixed with water.

## MAINTENANCE

It is extremely important that the lifter is checked after every use:

- Particularly we recommend you to check that all screws and bolts are well tight.
- That the lifter can move freely with caster brakes off.
- Check the brakes of the rear casters for a proper operation
- Check that the "U"- shaped bracket at the end of the boom and the S-shaped hook on the hanging bar are secured tightly

The mechanical parts have to be wet with a drop of oil for a better lubrication, particularly take care of the following areas:

# 18

- Where the Mast and boom connect, the hanging bar hook , the pump handle hinge and the caster axles.
- Where pump and boom attach.
- Lubricate as suggested above every two or three months

## ACCESSORIES

All the lifters of the **MOPEDIA** line are compatible with the following accessories

Code	Description
<b>RP820</b>	STANDARD SLING WITH HEAD SUPPORT
<b>RP822</b>	DELUXE PADDED SLING
<b>RP824</b>	CRADLE FOR STANDARD SLING
<b>RP826</b>	CRADLE FOR PADDED SLING

**Only use original accessories by MORETTI. In a different case we decline any responsibility for any accident caused by an improper use.**

## GUARANTEE

This device has **2-YEAR** warranty from the purchase date which is certified by the receipt or invoice. The distributor reserves the right either to repair or replace necessary parts to correct defects in the materials or workmanship without any charge.

The warranty does not apply to damage resulting from failure to follow the operating instructions, accidents, abuse, alteration or disassembly by unauthorized individuals.

In the case you need to get your device repaired and you fulfil the conditions of warranty, send the device to the nearest retailer together with the purchase receipt.

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MORETTI S.p.A. Via Bruxelles 3