Leckey Corner Sitter
Leckey Ecksitz
Il Corner Sitter Leckey
Leckey Corner Sitter

LECKEY®

USER MANUAL
GEBRAUCHSANLEITUNG
USER GUIDA D'USO
GUIDE DE L'UTILISATEUR

Ollo Pook®
QUALITY FOR LIFE

0.1 Preface

The Leckey Corner Sitter has been designed to improve posture and offer a better quality of life. This manual shows you how you can quickly and easily make use of all of the functions. The instructions on the safety and maintenance of the product will ensure that you will enjoy the use of this product for a long time.

0.2 Intended Use

The Leckey Corner sitter is intended for children with disabilities from the age of 1½-14. It can be used for long sitting which aids the therapeutic stretching of the ham strings. The Corner Sitter also provides minimal support for the pelvis and lower spine to promote sitting balance and function for the trunk and extremities from a sitting position.

0.3 Declaration of Conformity

James Leckey Design Ltd., as manufacturer with sole responsibility, declares that the Leckey Corner Sitter conforms to the requirements of the 93/42/EEC Guidelines.

0.4 Terms of Warranty

The Warranty applies only when the product is used according to the specified conditions and for the intended purposes, following all manufacturer's recommendations (also see our general terms of sales, delivery and payment).

Getting Started

1.1

1.0 How to Unpack and Assemble

Congratulations on purchasing your Corner Sitter. Your Corner Sitter will be boxed and wrapped in bubble wrap packaging on receipt. The Corner Sitter will be supplied with a hip belt and pommel and with your choice of accessories.

PLEASE CHECK THAT YOU HAVE ALL THE PARTS YOU ORDERED

1.1 The Corner Sitter will arrive folded. To assemble the Corner Sitter lift up the backrest to the required position and secure all 4 knobs.


2.0 The Corner Sitter is a portable floor seating unit with a handle at the back to make carrying easier.

2.1 Lateral Adjustments


The sides of the Corner Sitter can be adjusted in towards the user to give additional lateral support. Simply release the two back adjustment knobs (a) (one on either side), select the position you require and re-tighten.

2.2 Pommel

To adjust the depth of the pommel, pull up the pommel adjustment pin (a), select the position you require and release the pin.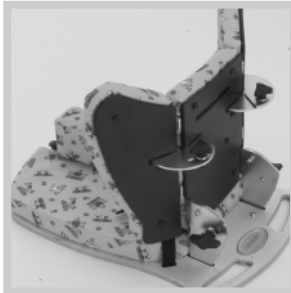

2.0

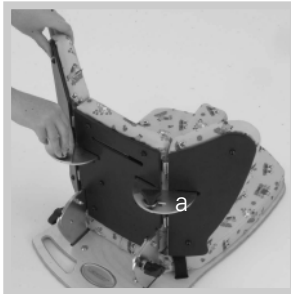2.1

2.2

2.3


2.3 Back Angle Adjustment

A degree of back recline and prone angle can be achieved by loosening the adjustments knob (a) (one at either side), selecting the position you require and re-tightening the knobs. Please note that the backrest should be supported by the carer when adjusting the angle while the user is seated in the chair.

3.0 Accessories


3.1 Tray Height and Angle Adjustment

The Corner Sitter Table is height and angle adjustable. To adjust the angle of the table, loosen the two table top adjustment knobs (a) (one on each side), select the position you require and re-tighten the knobs securely.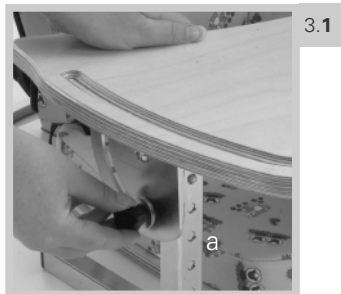


3.1

- 3.2 To adjust the height of the table, press in poppers and position the leg to the relevant slot to obtain required height.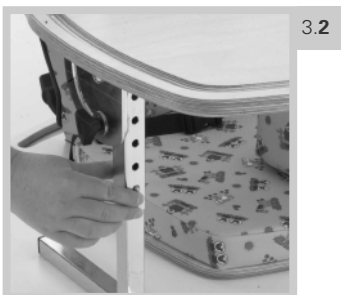


3.2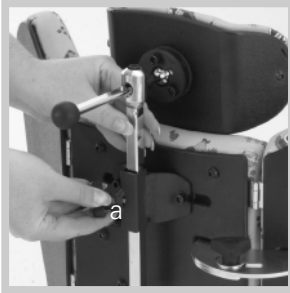

3.3

3.3 Multi Adjustable Contoured High Back

If a high back has been specified at the time of order, a bracket to fit the high back will already be attached. If the high back has been ordered separately then the high back bracket should be attached to the back of the seat by securing it with the two allen bolts supplied. The high back is height adjustable by loosening the knob (a). Once the required position is achieved knob (a) should be re-tightened securely. To adjust the depth, loosen the allen bolt (b), position as required and then re-tighten securely.

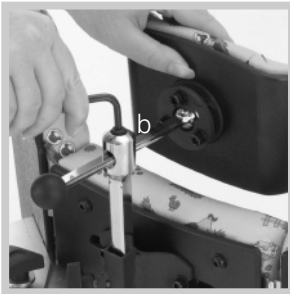Safety Instructions

1. Carry out all adjustments and ensure that they are securely fastened before the chair is used.
2. When placing your client in the Corner Sitter, both for positional and safety reasons, always secure the hip belt first.
3. We recommend that clients should not be left unattended at any time whilst using Leckey equipment.
4. When Corner Sitter parts are not in use, we would advise that you store them together in a safe place for future use.

How to Maintain

1. At least once a week, check that all fixings and fastening are in good condition.
2. When cleaning the Corner Sitter we would recommend that you use warm water and a non-abrasive detergent
3. If you are in any doubt as to the safe use of your Leckey Corner Sitter, or require any further assistance, please do not hesitate to contact your local dealer.

Re-issuing Leckey Products

1. Most Leckey products have been assessed and ordered to meet individual needs, therefore, we recommend that before a product is being re-issued, a compatibility check has been carried out.
2. Ensure that the product has been cleaned thoroughly.
3. A complete safety check of the product must be carried out prior to re-issuing, this must include the following checks:
 - a. Check that all locking mechanisms are functioning properly.
 - b. Check that all locking bolts are working properly and in place.
4. Where possible, please ensure that a copy of the relevant instruction manual is kept with the product and passed onto the new user.

Technical Data

Dimensions in (cm) and Weights in (kg)

Leckey Corner Sitter

	Size	1	2	3
Overall User Height		116.5	133	163
Seat Base				
Length	33.5	46	55	
Width	38	42.5	47.5	
Height in Folded Position	18	18	18	
Weight of Product	9kg	11kg	13kg	
Backrest Height	30	40	50	
High Back Height	12	14	16	
High Back Adjustment	17	17	17	
Pommel Adjustment	5	7.5	7.5	
Max User Weight	30kg	40kg	50kg	
Degree of Angular Adjustment on Backrest				
Prone	15°	15°	15°	
Recline	20°	20°	20°	
Tray Sizes	64x45	64x45	64x45	
Tray Height from Ground				
Min	22	22	22	
Max	32	32	32	

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Otto Bock HealthCare GmbH