ROLLITE™ ROLLATOR (MODEL 65100)
MARCHETTE ROLLITE™ (MODÉLE 65100)
ANDADERA RODANTE
ROLLITE™ (MODELO 65100)

Refer to page 2.
Se référer à la page 12.
Refiérase a la página 22.

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. In case of shipping damage, DO NOT use. Contact Dealer/Carrier for further instruction.

SAFETY SUMMARY

To ensure the safe operation of the **INVACARE ROLLATOR**, these instructions **MUST** be followed:

GENERAL WARNING

DO NOT install or use this equipment without first reading and understanding this instruction sheet. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Each individual should always consult with their physician or therapist to determine proper adjustment and usage.

Rollators are **NOT** intended to be self propelled while seated.

A physical/occupational therapist should assist in the height adjustment of the rollator for maximum support and correct brake activation.

Always ensure brake cables are flush along leg extensions and side braces otherwise brake may prematurely grip the wheel. Ensure excess brake cable loops out the front of the rollator.

Care should be taken to ensure that **ALL** hand and height adjustments are secure, and that casters and moving parts are in good working order before using this or any mobility aid.

All wheels **MUST** be in contact with the floor at **ALL TIMES** during use. This will ensure the rollator is properly balanced.

SAFETY SUMMARY (CONTINUED)

GENERAL WARNING

The brakes **MUST** be in the locked position **BEFORE** using the seat.

The seat must be in the **DOWN** position when using the rollator for ambulatory assistance.

When using the rollator in a stationary position, the hand brakes **MUST** be locked.

STABILITY WARNINGS

The Rollator can provide ambulatory assistance to an individual weighing up to 250 lbs. (113.4 kg), **INCLUDING** the weight of the contents of the basket, if so equipped.

The Rollator basket has a weight limitation of 10 lbs. (4.5 kg.).

FOLDING/ASSEMBLY WARNINGS

After unfolding or assembling rollator, ensure that rollator is securely locked in **OPEN** position and level to the ground **BEFORE** using.

DO NOT hang anything on either the left or right side frames (area that includes dual release paddles). This will cause side frames or slide tubes to bend downward and prevent rollator from locking into place when fully opened. This may also cause rollator to tip, resulting in injury or damage.

INSTALLATION WARNINGS

Ensure all snap buttons fully protrude through adjustment holes of each leg extension. This ensures that the leg extensions are securely locked in position.

The leg extensions should be adjusted so the rollator is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the rollator is no more than one (1) inch lower than the front.

SAFETY SUMMARY (CONTINUED)

INSTALLATION WARNINGS

Always test to see that the rollator and attachments are properly and securely locked in place BEFORE using.

OPENING/FOLDING THE ROLLATOR (FIGURE 1)

NOTE: Refer to the FOLDING/ASSEMBLY WARNINGS in the SAFETY SUMMARY of this instruction sheet.

OPENING

1. Swing the side frames outward until the release paddles activate **(engage upward)**.
2. Grasp the handle and lower the seat, pushing **DOWN** until the seat supports fully engage the side braces as shown in **OPEN** of **FIGURE 1**.
3. Before using, test rollator by grasping the side frames and, with some force, attempt to fold them inward. The rollator will remain locked in the **OPEN** position if it is engaged properly.

FOLDING

1. Raise the seat by grasping the handle and lifting **UP**.
2. Depress either release paddle and pivot the corresponding side frame inward toward the front crossbrace.
3. Repeat the same procedure for the remaining side frame.

NOTE: Back rest not shown for clarity.

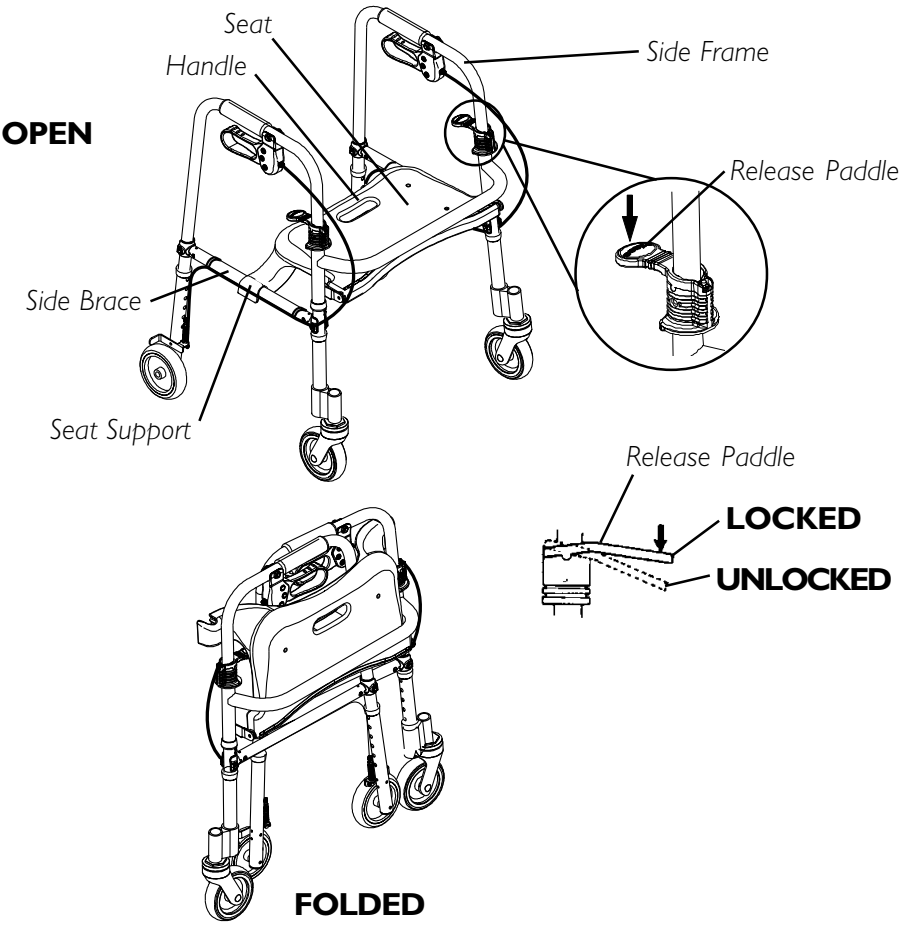
FIGURE I - OPENING/FOLDING THE ROLLATOR

INSTALLING/ADJUSTING LEG EXTENSIONS/BRAKE CABLES (FIGURE 2)

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

NOTE: For the most comfortable position, make sure the adjustment is performed while wearing the most frequently worn shoes.


NOTE: Adjust the rollator height so that when the user's arm is down to their side, the hand grip is at wrist height. This will ensure the arms are at an approximate 20° - 30° bend when using the rollator.

1. Depress the snap buttons on the front legs of the rollator and slide the leg extensions up or down to the desired height. Ensure that the snap button protrudes through the appropriate adjustment hole on the leg extension (FIGURE 2, DETAIL "A").
2. Push/pull the brake cables along the side braces so that excess brake cable is near the rear leg extensions (DETAIL "B").
3. Repeat STEP 1 for the rear leg extensions.
4. After adjusting the rear leg extensions, remove the excess brake cable (DETAIL "B") near the rear leg extensions and side braces by pulling brake cables through the cable tie wraps until the excess brake cables loop out the front of the rollator (DETAIL "C").


NOTE: For the brakes to function properly, the brake cable should be flush along the leg extension and side brace and the excess brake cable should loop out the front of the rollator (DETAIL "C").

5. Ensure all legs are adjusted to the same height.

DETAIL "A"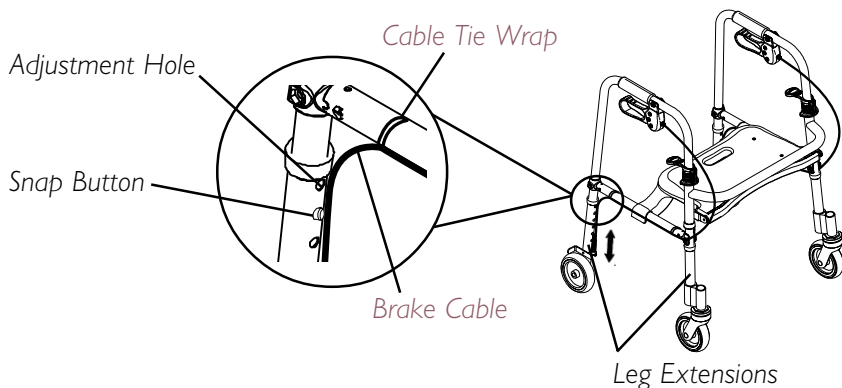

DETAIL "B"

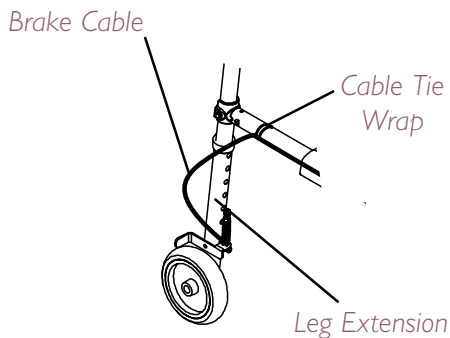DETAIL "C"

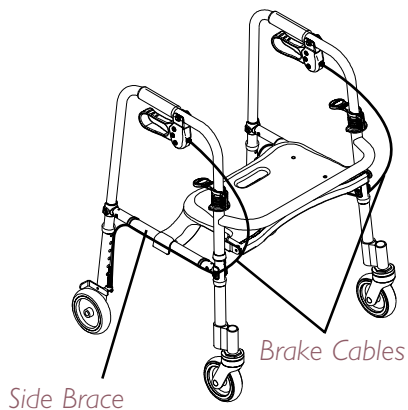
FIGURE 2 - INSTALLING/ADJUSTING LEG EXTENSIONS/BRAKE CABLE

USING/LOCKING/UNLOCKING HAND BRAKES (FIGURE 3)

NOTE: Refer to INSTALLATION WARNINGS in the SAFETY SUMMARY in this instruction sheet.

NOTE: If brakes prematurely lock, refer to INSTALLING/ADJUSTING LEG EXTENSIONS/BRAKE CABLES in this instruction sheet.

USING HAND BRAKES


1. Pull the top portion of the brake handles UP towards the hand grips.
2. Do one (1) of the following:
 - A. REMAIN STATIONARY - hold the brake handle UP.
 - B. CONTINUE MOBILITY - release the brake handle.

LOCKING HAND BRAKES

1. Push DOWN on the bottom portion of the brake handle as shown in FIGURE 3 until an audible “click” is heard.
2. Refer to UNLOCKING HAND BRAKES in this procedure of the instruction sheet.

UNLOCKING HAND BRAKES

1. Pull UP on the top portion of the brake handle to release as shown in FIGURE 3.
2. Release brake handle.

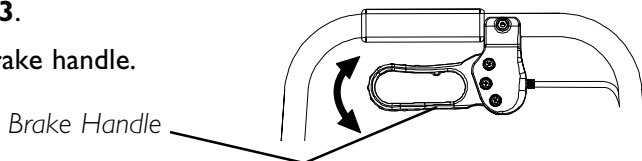**FIGURE 3 - USING/LOCKING/
UNLOCKING THE HAND BRAKES**

ADJUSTING THE HAND BRAKES

NOTE: COUNTERCLOCKWISE/CLOCKWISE directions are determined by standing behind the rollator (user's position).

NOTE: Refer to INSTALLATION WARNINGS in the SAFETY SUMMARY in this instruction sheet.

BRAKE HANDLE (FIGURE 4)

1. Loosen the brake handle adjustment nut by turning **CLOCKWISE**.
2. Do one (1) of the following:
 - A. **LOOSEN BRAKE HANDLE TENSION** - turn brake handle nut **COUNTERCLOCKWISE**.
 - B. **TIGHTEN BRAKE HANDLE TENSION** - turn brake handle nut **CLOCKWISE**.
3. Turn the brake handle adjustment nut **CLOCKWISE** to secure in place.
4. Do one (1) of the following:
 - A. **ACCEPTABLE TENSION** - repeat STEPS 1 - 3 for opposite side, if necessary.
 - B. **UNACCEPTABLE TENSION** - refer to CABLE ADJUSTER UNIT in this procedure of the instruction sheet.
5. Refer to LOCKING/UNLOCKING/USING HAND BRAKES in this instruction sheet.

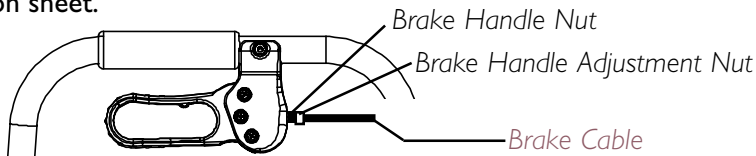
FIGURE 4 - BRAKE HANDLE

CABLE ADJUSTER UNIT (FIGURE 5)

1. Test the brake. If the tension of the brake handle is **STILL** too loose/tight, perform the following:

- A. Loosen the phillips screw on the brake clamp.
- B. Pull the brake wire further through the brake clamp.

NOTE: There will be additional excess wire. It may be necessary to fold wire up to keep wire from interfering with brake action.

2. Repeat STEP 1 for remaining brake.

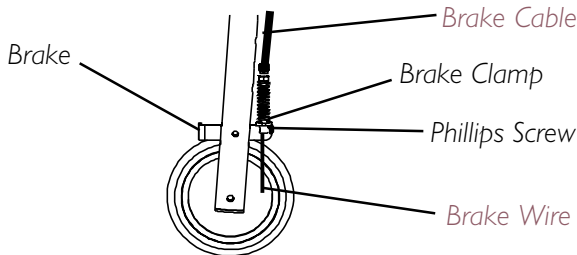
FIGURE 5 - CABLE ADJUSTER UNIT

CARE AND MAINTENANCE

- 1. Ensure that **ALL** attaching hardware is secure at **ALL** times.
- 2. Check screw securing seat to rollator for tightness. Tighten if necessary.
- 3. Make sure snap buttons release and re-engage properly.
- 4. Replace any broken, damaged or worn items immediately.

CARE OF WHEELED ACCESSORIES

- 1. Clean wheels with warm water and mild cleanser. Dry with clean rag.
- 2. Lubricate with 3-in-1[®] oil periodically or if wheels begin to squeak.

LIFETIME LIMITED WARRANTY

NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our product.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the frame to be free from defects in materials and workmanship for the lifetime of the product, six (6) years on the hand brakes and one (1) year for all other remaining components, excluding tires/wheels for the original purchaser/user. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover, provide dealers name, address, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, IMPROPER MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE; PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL. SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THESE INSTRUCTIONS.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES. IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION ON HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSIONS AND LIMITATIONS MAY NOT APPLY TO YOU.

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© 2001 Invacare Corporation
Invacare is a registered trademark of
3-in-1 is a registered trademark of
Yes, you can and Rollite are trademarks
Invacare Corporation
Corporation